

*0703, 0708, 0716, 0732, 0737	B7, B703	1											#
*0704, 0725	B7	1											#
*0709, 0711, 0717	B7	1											#
*0713, 0715	-	1											#
*0719	-												#
*0720, 0724	-	1		3									#
*0727	-	1											#
*0733, 0740, 0748, 0751, 0753	-												#
*0736	-	1											#
*0738	-	1											#
*0750	-	1											#
*0754	-	1											#
*0760	-			3									#
*080101-080105, 0805, 0808N, 0810-0812, 0815, 0816, 0818, 0819N, 0822-0824, 0827, 0829-0831, 0833, 0834	B8, Null			2									#
*0802	B8			2									#
*0803	B8			2									#
*0804	-			2									#
*0806, 0813, 0820	B8			2									#
*0807, 0814	-			2									#
*0809	-			2									#
*0817				2									#
*0821	-			2		3							#
*0825	-			2		3							#
*0826, 4207	-												#
*0828	-			2									#
*0832	-			2									#
*0835	-			2									#
*0836	-			2									#
*1301, 1317, 1320	B13					3		4					#
*130201-130204, 1308Q, 1315, 1318, 1319	B13							3		4			#
*1303	-					3		4					#
*1304, 1310	-					3		4					#
*1306	-					3		4					#
*1307N	Null					3		4					#
*1309	-							4					#
*1311	-					3		4					#
*1312, 1313	-					3		4					#
*1314	-					3							#
*1316	-					3		4					#
*1321	-					3		4					#

*1401, 1407N	B64			4	5										#	#
*140201-1404	B65			4		6									#	#
*1405-140602	-					6									#	#
*1408, 3925N, 3930, 3932-3934	-				5										#	#
*15010101-15010102N, 150103, 1526N, 152701, 1532, 1534, 1535, 1538, 1550, 1556, 1557, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509-9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529	B62, Null, B15					7			#						#	
*150102	B62					7		9		#					#	
*150104	B62									#					#	
*150106-150107, 1507, 9526, 9535, 9540, 9545, 9546	B62			3		7				#					#	
*150108, 152702	B62			3						#					#	
*1502, 1588, 9512, 9521	B75					7				#				#	#	
*1503, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532	B72						8		#	#					#	
*1504	B62, Null, B15					7				#					#	
*1505, 1506, 1533, 1539, 1540, 1565, 1578, 1584, 9507	B62, B15					7				#					#	
*1508, 151101-151103, 1515	B75					7				#					#	
*1509	B70						8	9	#						#	
*1510, 1537, 1590, 1599, 9533	B71						8		#						#	
*1512, 1514, 1519	B76					7				#	#				#	
*1513	B77					7				#	#			#	#	
*1516, 1567, 1595	B63							9		#	#				#	
*15170101-151702	B63					7		9			#				#	
*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534	B71						8		#	#					#	
*1520, 1525	B62					7				#				#	#	

*1521, 1544	B75					7		#	#		#	#				
*1523	-					8		#	#	#		#				
*1524, 1587	B62					7			#	#		#				
*1528, 9520	B15					7			#			#				
*1529	B15					8			#			#				
*1530, 1558, 1563	B62,					7						#				
*1531	B75					7			#			#				
*1536	-		3			7			#			#				
*1542	-					7						#				
*1543	-					7			#			#				
*1545	B62		3			7						#				
*1546	B72			4		7	8		#			#				
*1548	B62					7						#				
*1553	-			4		8			#			#				
*1554	-					8		#	#			#				
*1555	-		3			7			#			#				
*1562	-					8		#	#		#	#				
*1564	-					8		#	#			#				
*1566	-					7		#	#			#				
*1568	-		3			7	8		#	#		#				
*1569	-					8		#	#			#		#		
*1573	-					7						#				
*1574	-					8			#			#				
*1576, 9501	-					7			#			#				
*1577	-					7					#	#				
*1580	-					8		#	#			#	#			
*1583	-					7	8					#				
*1585	-					7			#			#	#			
*1586	-					7						#				
*1589	-		3			7			#			#	#			
*1591, 9531	-					8		#	#	#		#				
*9506	-			4		7	8		#			#	#			
*9515	-					8		#	#			#	#			
*9516	-					7			#			#				
*9524	-					8		#	#			#				
*9536	-		3			7			#			#		#		
*9537	-		3			7						#				
*9538	-	1				7			#			#				
*9539	-		3			7			#		#	#				
*9541	-		3			7			#			#				
*9542	-		2	3		7			#			#				
*9543	-					7	8		#			#				
*9544	-		3			7			#			#				
*180101-1803, 1805-1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825. 1827.	B18								#		#	#				
*1804	-								#		#	#				

*1809	-								#			#			#				
*1812	-								#			#			#				
*1819	-								#			#			#				
*1822	-								#			#	#	#					
*1826	-											#			#				
*1829, 3532, 3537, 3553N, 3564, 3566, 356801-356802	-								#						#				
*2701	B27					5								#					
*2702	B27					5								#					
*2703-270402, 270502- 2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737	B27					5								#					
*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738	B27													#					
*2708, 2726, 2740, 2742	B2708					5								#					
*2712	-					5								#					
*2716, 2739	-					5								#					
*2718	-					5			#					#					
*2719	-													#	#				
*2723	-					5								#					
*2724	-													#					
*2725	-													#					
*2729	-					5			#					#					
*2730	-													#	#				
*2733	-													#					
*2741	-					5			#					#					
*350101-350107, 3507- 350803, 3511, 3515, 352001-352002, 3523, 3529, 3540N-3542, 3548, 3550, 3552, 3554, 3557, 3561, 3577, 3578, 3582	B35								#						#	#			
*350201-350401, 3506, 3512, 3533, 3534, 3536, 3539, 3555, 3559, 3565Q, 3570, 3574, 3583-3585	B35														#	#			
*350402	-								9						#	#			
*3505, 3551, 3558, 3572	B35				3				#						#				
*350901-350902, 3518, 3531, 3575, 3588	B35														#				

*3510, 3528, 3569, 3580	-							#		#	#			
*3513	B35									#	#			
*351401-351402, 3562	B35							#		#	#			
*3516	-		3					#		#	#			
*3517, 3530	B35		3					#		#	#			
*3519	B35							#		#				
*3521, 3524	-							#		#	#			
*3522	-		3								#			
*3525, 3549	-							#		#				
*3526	-				6			#		#				
*3527	B35							#		#				
*3535	-							#		#	#		#	
*3538	-									#	#			
*3543, 3567, 3579, 3586	B35							#			#			
*3544	-										#			
*3545, 3571	-							#	#		#			
*3546	-							#			#			
*3547	-			4				#			#			
*3556	-										#			
*3560	-										#			
*3563	-			4				#			#			
3576	-							#			#	#		
*3581	-										#	#		
*3587	-					8					#			
*370101-370103, 3703N, 3706, 3713	B37, Null							#		#		#		
*370104	-							#		#		#		
*3702	-			5				#				#		
*3704	-							#		#		#		
*3705	-							#		#		W		
*3707	-	1						#				#		
*3708	-							#		#		#		
*3709	-							#				#		
*3710	-									#		#		
*3711	-									#	#			
*3712	-				8			#				#		
*380101-380102, 3809, 3811-3814, 3816	B38			5									#	#
*380201-380202, 3808, 3815	B38			5								#	#	#
*3803	B16			5				#				#	#	#
*3804	-											#	#	#
*3805	-			6								#	#	#
*3806	-											#		
*3807	-			5								#		
*3810	-			5									#	#
*3817	-			5				#				#	#	#

[illegible]

[illegible]

*4417, 4443	-			4			#	#		#		
*4418	-			4			#					
*4420	-			4								
*4425, 4450	-			4						#		
*4431	-			4								
*4437	-			4						#		
*4440	-						#		#	#		
*4441, 4453	-			4								
*4444	-						#		#	#		
*4446	-			4						#		
*4447	-			4								
*4449	-		2	4						#		
*4454	-			3 4								
*4457	-			3						#		
*4460	-			3 4						#		
*4501, 4505, 4507	B45			4			#					#
*4502, 4503	-			4								#
*4504	-			4								#
*4506	-			4			#					#
*4508	-			4								#
*460101-460102, 4604, 4605, 4607N, 4609	B46					7	#					
*4602	-					7	#					
*4603, 4608	-					7	#					
*4606	-					7	#					
*4610	-					7	#					
*4611	-			3		7						
*4612, 4614	-			3		7	#					
*4613	-			3		7	#					
*47010101-47010102	B47			4								
*4702	-			4								
*4703	-			4								
*4704, 4705	-			4	5							
*480101-480102, 4804, 4807, 4809, 4811,	B48						#					#
*4802	-						# #		#			#
*480301-480302, 4817	B48						#					#
*4805, 4808	B48	1					#					#
*4806	-											#
*4810, 4812, 4813	-						#					#
*4814	-						# #					#
*4815	-	1					#					#
*4816	-						#					#
*4818	-						#					
*4901	B49			4								
*4902	-			4								
*4903	-											
*4904, 4905	-			4			#					
*5001, 5004	B50			4								#

*5002	B45			4					#							#
*510101, 510103, 510105-510108, 510201, 510203, 5103, 5107, 510901, 5111N, 5112, 511302, 5114, 5117-5119, 5122, 5123, 5126-5130, 5132, 5133, 5138- 5141N, 5143, 5148, 5149, 5151-5153, 5155	B51, B5102, B5103, Null							9								
*510102, 510109, 510202, 5105, 510902	B51							9								
*510104, 511301, 5116, 5121, 512401- 512402, 5131, 5134, 5135	B51, B52															
*5104, 5146	B51							9					#			
*5106	B51							9								
*5108, 5120, 5144N	B51						8	9								
*5110	-															
*5115	-															
*5136	-						8									
*5137	-							9		#						
*5142	-											#				
*5145	-									#						
*5150, 5154	-							9								
*520101, 520103, 5204, 5205, 5207, 5210-5212	B52							9								
*520102, 520104-5203, 520601-520602, 5209	B52							9								
*5208	-															
*530101-530104, 5305, 5308, 5310, 5315,	B53									#			#			
*5302	B53									#			#			
*5303, 5309, 5311- 5313	-									#			#			
*5304, 5307	-											#				
*5306	B53									#			#			
*5314				3						#						
*5401, 5402, 5405N, 5407, 5408N, 5410, 5413	B54															
*5403	-															
*5404, 5411	-															
*5406	-									#						
*5409	-									#						

*5412	-																		
*550101-550104, 5503, 5505, 5511, 5515,	B55																		
*550201-550202, 5507, 5510, 5512, 5516, 5519, 5526, 5530	B54, B55																		
*5504, 5508, 5513, 5527, 5615, 5619N,	B55																		
*5509, 5517, 5528	-																		
*5514	-									#					#				
*5518	-									#									
*5520	-																		
*5521	-																		
*5522	-									#									
*5523, 5618	-											#							
*5524	-									#									
*5601, 5608, 5614, 5616, 5620	B56																		
*5602, 5604	B56																		
*5603	B22											#							
*560501	-								9										
*560502	-								9										
*5606	B78								9						#				
*5607	-																		
*5609	-										#				#				
*5610	-																		
*5611, 5612	-														#				
*5613	-							8											
*5617	-																		
*5621	-								9										
*5623	-																		
*570101-570103, 5706, 5708, 5710	B57										#				#				
*5702-5705	B57														#				
*5707	-														#				
*5709	-							8							#				
*5711	-										#								
*5712	-														#				
*5713	-										#								
*5714	-			3							#				#				
*5715	-			3							#				#				
*5716	-										#				#				
*580101, 5804, 5805, 5810N-5814	B58										#				#				
*580102, 5815											#				#				
*5802, 5806	B58										#								
*5807	-										#	#							
*5808	-								9										
*5809	-										#				#				

*5816	-								#									
*5817N	-								#				#					
*5901	B59																	
*5902	-																	
*670101	B67				5											#		#
*670102	B67															#		#
*6702	-															#		#
*7301	B73																	#
*7801, 780202, 7803	B78								9					#				
*780201, 7804	B78								9					#				
*7805	-								9					#				
*8101	B81																	
*8102	-																	#
*8103, 8104N	-																	
*8201, 8202	-							8		#					#			
*8301	-													#				
HLA-B allele ⁴	ser. ⁵																	
Well No.		1	2	3	4	5	6	7	8	9	#	#	#	#	#	#	#	#
Cw*0206										w								
Cw*0305, 0325, 0327				3														
Cw*0746								8										
Cw*150204																		
Well No.		1	2	3	4	5	6	7	8	9	#	#	#	#	#	#	#	#

INTERPRETATION TABLE

HLA-B low resolution SSP

Amplification patterns of the B*0702 to B*8301 alleles⁶

Well																													
#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#			
175	130	110	325	110	215	130	90	90	145	120	430	145	300	160	180	180	90	90	95	115	360	355							
				190		270	410	175		210				215	210		240			150									Length of spec. PCR product(s)
				225																									
				260																									
1070	1070	1070	1070	1070	800	1070	1070	1070	1070	800	1070	1070	1070	1070	1070	1070	800	1070	1070	1070	1070	1070	1070						Length of int. pos. control ¹
363	106	540	1 st I	355	141	209	41	209	206	357	49	206	1 st I	1 st I	141	420	209	209	206	165	1 st I	1 st I						5'-primer(s) ²	
5'-Agg ^{3'}	5'-CCg ^{3'}	5'-gAC ^{3'}	5'-CAG ^{3'}	5'-TCA ^{3'}	5'-ATT ^{3'}	5'-ggC ^{3'}	5'-CTg ^{3'}	5'-ggC ^{3'}	5'-gAA ^{3'}	5'-Tgg ^{3'}	5'-CAG ^{3'}	5'-gAA ^{3'}	5'-CAG ^{3'}	5'-CAG ^{3'}	5'-ATT ^{3'}	5'-TTA ^{3'}	5'-ggC ^{3'}	5'-ggA ^{3'}	5'-gAC ^{3'}	5'-ACC ^{3'}	5'-CAG ^{3'}	5'-CAG ^{3'}							
				418	357	209	368	363						418	420		362	369		463									
				5'-Agg ^{3'}	5'-Tgg ^{3'}	5'-ggg ^{3'}	5'-gTT ^{3'}	5'-AGC ^{3'}						5'-Agg ^{3'}	5'-TTA ^{3'}		5'-ggT ^{3'}	5'-TAC ^{3'}		5'-TgA ^{3'}									
				499		355																							
				5'-TCT ^{3'}		5'-TCA ^{3'}																							
499	193	603	282	538	309	299	282	259	309	435	309	311	259	175	311	559	256	256	259	272	317	311						3'-primer(s) ³	
5'-ggA ^{3'}	5'-CgT ^{3'}	5'-gTg ^{3'}	5'-gCC ^{3'}	5'-gTC ^{3'}	5'-ATC ^{3'}	5'-TCA ^{3'}	5'-GCC ^{3'}	5'-gTT ^{3'}	5'-ATC ^{3'}	5'-TCT ^{3'}	5'-ATC ^{3'}	5'-ggT ^{3'}	5'-CTC ^{3'}	5'-CCg ^{3'}	5'-ggT ^{3'}	5'-CAG ^{3'}	5'-CCC ^{3'}	5'-CCC ^{3'}	5'-gTT ^{3'}	5'-TgC ^{3'}	5'-ggA ^{3'}	5'-ggT ^{3'}							
		610		572	538	583	418	499		527				538	559		559	256		538									
		5'-CTC ^{3'}		5'-gCg ^{3'}	5'-gTC ^{3'}	5'-gTg ^{3'}	5'-gTC ^{3'}	5'-ggA ^{3'}		5'-CCT ^{3'}				5'-gTC ^{3'}	5'-CgT ^{3'}		5'-CgT ^{3'}	5'-CTC ^{3'}		5'-CCA ^{3'}									
																		420											
																		5'-gCT ^{3'}											
#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#	#						Well No.	
																							ser. ⁵					HL A-B allele ⁴	
			#																			#	B7					*070201-070207, 070501-0707, 0710, 0712, 0714, 071801- 071802, 0721-0723, 0726, 0728-0731, 0734, 0735, 0739, 0741-0747, 0749N, 0752, 0755-0759, 0761.	

					#	B7, B703	*0703, 0708, 0716, 0732, 0737
	#	#			#	B7	*0704, 0725
		#			#	B7	*0709, 0711, 0717
		W			#	-	*0713, 0715
	#	#			#	-	*0719
		#			#	-	*0720, 0724
					#	-	*0727
		#			#	-	*0733, 0740, 0748, 0751, 0753
		#			#	-	*0736
					#	-	*0738
					#	-	*0750
		#		#	#	-	*0754
		#			#	-	*0760
	#				#	B8, Null	*080101-080105, 0805, 0808N, 0810-0812, 0815, 0816, 0818, 0819N, 0822-0824, 0827, 0829-0831, 0833, 0834
	#				#	B8	*0802
	#				#	B8	*0803
	#				#	-	*0804
					#	B8	*0806, 0813, 0820
	#				#	-	*0807, 0814
	#	#			#	-	*0809
	#			#	#		*0817
	#				#	-	*0821
					#	-	*0825
	#				#	-	*0826, 4207
	#				#	-	*0828
	#				#	-	*0832
	#				#	-	*0835
	#				#	-	*0836
			#		#	B13	*1301, 1317, 1320
			#		#	B13	*130201-130204, 1308Q, 1315, 1318, 1319
			#	#	#	-	*1303
			#		#	-	*1304, 1310
			#	#	#	-	*1306
					#	Null	*1307N
			#		#	-	*1309
			#	#	#	-	*1311
			#		#	-	*1312, 1313
			#		#	-	*1314
		#	#		#	-	*1316
			#		#	-	*1321

					#	B64	*1401, 1407N
					#	B65	*140201-1404
					#	-	*1405-140602
					#	-	*1408, 3925N, 3930, 3932-3934
			#		#	B62, Null, B15	*15010101- 15010102N, 150103, 1526N, 152701, 1532, 1534, 1535, 1538, 1550, 1556, 1557, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509- 9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529
			#		#	B62	*150102
			#	#	#	B62	*150104
			#		#	B62	*150106-150107, 1507, 9526, 9535, 9540, 9545, 9546
			#		#	B62	*150108, 152702
		#			#	B75	*1502, 1588, 9512, 9521
			#		#	B72	*1503, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532
		W	#		#	B62, Null, B15	*1504
			#		#	B62, B15	*1505, 1506, 1533, 1539, 1540, 1565, 1578, 1584, 9507
		#			#	B75	*1508, 151101-151103, 1515
					#	B70	*1509
					#	B71	*1510, 1537, 1590, 1599, 9533
			#		#	B76	*1512, 1514, 1519
		#			#	B77	*1513
		#			#	B63	*1516, 1567, 1595
					#	B63	*15170101-151702
					#	B71	*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534
			#		#	B62	*1520, 1525

		#			#	B75	*1521, 1544
					#	-	*1523
			#		# #	B62	*1524, 1587
					#	B15	*1528, 9520
					#	B15	*1529
			#		#	B62,	*1530, 1558, 1563
		#			#	B75	*1531
			#		#	-	*1536
		#	#	#	#	-	*1542
			#		# #	-	*1543
			#		#	B62	*1545
			# #		#	B72	*1546
			#		#	B62	*1548
			# #		#	-	*1553
			#		#	-	*1554
		#			#	-	*1555
			#		#	-	*1562
					#	-	*1564
			#		#	-	*1566
			#		#	-	*1568
			#		#	-	*1569
			#	#	#	-	*1573
			#		#	-	*1574
	#				#	-	*1576, 9501
			#		#	-	*1577
					#	-	*1580
	#		#		#	-	*1583
			#		#	-	*1585
			#	#	#	-	*1586
		#			#	-	*1589
			#		#	-	*1591, 9531
			# #		#	-	*9506
					#	-	*9515
#			#		#	-	*9516
#					#	-	*9524
			#		#	-	*9536
		#	#		#	-	*9537
			#		#	-	*9538
		#			#	-	*9539
			#		#	-	*9541
			#		#	-	*9542
			#		#	-	*9543
		#			#	-	*9544
					#	B18	*180101-1803, 1805-1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825. 1827.
			#	#	#	-	*1804

										#		#		-	*1809
													#	-	*1812
										#	#		#	-	*1819
										#			#	-	*1822
										#			#	-	*1826
										#			#	-	*1829, 3532, 3537, 3553N, 3564, 3566, 356801-356802
			#							#		#		B27	*2701
			#				#			#		#		B27	*2702
													#	B27	*2703-270402, 270502- 2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737
			#				#			#				B27	*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738
			#				#						#	B2708	*2708, 2726, 2740, 2742
										#		#		-	*2712
							#					#		-	*2716, 2739
							#					#		-	*2718
			#				#					#		-	*2719
												#		-	*2723
			#			#	#			#		#		-	*2724
			#			#					#	#		-	*2725
							#					#		-	*2729
			#			#		#				#		-	*2730
			#			#		#					#	-	*2733
			#			#		#				#		-	*2741
										#			#	B35	*350101-350107, 3507- 350803, 3511, 3515, 352001-352002, 3523, 3529, 3540N-3542, 3548, 3550, 3552, 3554, 3557, 3561, 3577, 3578, 3582
										#			#	B35	*350201-350401, 3506, 3512, 3533, 3534, 3536, 3539, 3555, 3559, 3565Q, 3570, 3574, 3583-3585
										#			#	-	*350402
										#			#	B35	*3505, 3551, 3558, 3572
										#			#	B35	*350901-350902, 3518, 3531, 3575, 3588

			#		#	-	*3510, 3528, 3569, 3580
			#		#	B35	*3513
				#	#	B35	*351401-351402, 3562
			#		#	-	*3516
				#	#	B35	*3517, 3530
			#		#	B35	*3519
	#			#	#	-	*3521, 3524
				#	#	-	*3522
					#	-	*3525, 3549
					#	-	*3526
				#	#	B35	*3527
				#	#	-	*3535
	#			#	#	-	*3538
				#	#	B35	*3543, 3567, 3579, 3586
				#	#	-	*3544
	#			#	#	-	*3545, 3571
		#			#	-	*3546
			#		#	-	*3547
				#	#	-	*3556
				#	#	-	*3560
			#	#	#	-	*3563
	#			#	#	-	3576
	#			#	#	-	*3581
	#			#	#	-	*3587
	#		#		#	B37, Null	*370101-370103, 3703N, 3706, 3713
	#		#	#	#	-	*370104
			#		#	-	*3702
	#	#	#		#	-	*3704
	#		#		#	-	*3705
			#		#	-	*3707
	#			#	#	-	*3708
			#		#	-	*3709
	#		#		#	-	*3710
	#		#		#	-	*3711
	#		#		#	-	*3712
					#	B38	*380101-380102, 3809, 3811-3814, 3816
					#	B38	*380201-380202, 3808, 3815
			#		#	B16	*3803
			#		#	-	*3804
					#	-	*3805
				#	#	-	*3806
				#	#	-	*3807
					#	-	*3810
					#	-	*3817

					#	B3901, B39	*39010101-39010102L, 390103-390104, 3903, 3905-390602, 3909, 3911, 3914, 3915, 3924, 3926-3929, 3931, 3935, 3937, 3938Q, 3940N, 3941
			#		#	B3902,	*390201, 3908
			#		#	B3902, B39	*390202, 391301-391302, 3923
					#	B39	*3904
					#	-	*3907
					#	B39	*3910, 3916, 3920
					#	-	*3912
				#	#	-	*3917
					#	-	*3918
					#	-	*391901
					#	-	*391902
			#		#	-	*3922
					#	-	*3936
					#	-	*3939
					#	-	*3942
					#	-	*3943
#			# #		#	B60, B40	*400101-400106, 4007, 4051, 4079
#			# #		#	B61, B40	*400201-400203, 4004, 40060101-400602, 4039, 4070
#			# #		#	B40	*4003, 4020
#			# #		#	B4005	*4005
#			#		#	-	*4008
#			# #		#	B61, B40	*4009, 4011, 4018, 4024, 4027, 4029, 4035, 4037, 4040, 4044, 4050, 4056, 4057, 4064, 4075, 4078, 4082, 4085,
#			# #		#	Null, B40, B60	*4010, 4022N, 4033, 4036, 4042, 4043, 4049, 4054, 4055, 4061-4063, 4065-4067, 4073, 4081, 4084, 4087, 4088
#			#		#	-	*4012
#		# #	#		#	-	*4013, 4019
			# #		#	-	*401401-401403, 4030, 4034, 4053, 4072
			# #		#	B61	*4015, 4016, 4032

#							#					#	-	*4021
#							#	#				#	-	*4023
#							#					#	-	*4025, 4069
#	#						#	#				#	B21	*4026
#	#			#			#	#				#	-	*4028
					#		#	#				#	-	*4031, 4045, 4080
#							#	#				#	-	*4038, 4052
#								#				#	-	*4046
#								#				#	-	*4047
#							#	#		#		#	-	*4048
							#	#	#			#	-	*4058
							#	#				#	-	*4059
#							#	#				#	-	*4060
#	#						#					#	-	*4068
#	#						#	#		#	#	#	-	*4071
#	#						#	#				#	-	*4074
#								#					-	*4076
							#	#				#	-	*4077
							#	#				#	-	*4083
#				#			#	#				#	B41	*4101, 4106, 4107
#	#						#	#				#	B41	*4102-410302
#							#	#				#	-	*4104
#	#			#			?	#				#	-	*4105
#							#	#				#	-	*4108
#	#											#	B42	*4201, 4202, 4206
#	#			#								#	-	*4204
#	#											#	-	*420501-420502
#												#	-	*4208
#	#											#	-	*4209
				#				#	#			#	B44, B47, Null	*44020101-440204, 4411, 4416, 4419N, 4421-4424, 4427, 4433, 4434, 4448, 4451, 4452N, 4455, 4459
				#				#				#	B44	*440301-4405, 4413, 4426, 442801-4430, 4432, 4435, 4436, 4438, 4439, 4442, 4445, 4456N, 4458N
				#	#		#		#			#	B44	*4406
				#				#				#		*4407
				#				#	#			#	B44	*4408
				#			#	#	#			#	B12	*4409
				#				#		#		#	-	*4410
				#				#	#			#	-	*4412
				#				#				#	-	*4414
				#			#		#			#	-	*4415

	#			#				-	*4417, 4443
	#	#		#		#		-	*4418
	#	#		#	#			-	*4420
	#			#				-	*4425, 4450
#	#			#				-	*4431
				#				-	*4437
	#			#				-	*4440
	#			#	#			-	*4441, 4453
	#			#	#			-	*4444
	#			#	#			-	*4446
	#			#				-	*4447
	#			#	#			-	*4449
	#			#				-	*4454
	#			#				-	*4457
	#			#				-	*4460
	#			#	#			B45	*4501, 4505, 4507
	#			#	#			-	*4502, 4503
	#			#	#			-	*4504
#	#			#				-	*4506
	#			#	#	#		-	*4508
#	#						#	B46	*460101-460102, 4604, 4605, 4607N, 4609
	W		#				#	-	*4602
	#		#				#	-	*4603, 4608
#	#		#				#	-	*4606
	#		#		#		#	-	*4610
	#		#		#	#	#	-	*4611
	#		#		#		#	-	*4612, 4614
	#		#		#		#	-	*4613
		#		#			#	B47	*47010101-47010102
		#		#	#		#	-	*4702
		#		#			#	-	*4703
				#			#	-	*4704, 4705
		#		#			#	B48	*480101-480102, 4804, 4807, 4809, 4811,
#				#			#	-	*4802
				#			#	B48	*480301-480302, 4817
				#			#	B48	*4805, 4808
		#					#	-	*4806
				#			#	-	*4810, 4812, 4813
				#			#	-	*4814
		#		#			#	-	*4815
		#					#	-	*4816
		#		#			#	-	*4818
		#	#	#		#		B49	*4901
		#		#		#		-	*4902
		#		#		#		-	*4903
		#	#	#		#		-	*4904, 4905
		#	#	#		#		B50	*5001, 5004

								#		#	#			#				#	B45	*5002
								#	#					#	#				B51	*510101, 510103, 510105-510108, 510201, 510203, 5103, B51, 5107, 510901, 5111N, B5102, 5112, 511302, 5114, B5103, 5117-5119, 5122, Null 5123, 5126-5130. 5132, 5133, 5138- 5141N, 5143, 5148, 5149, 5151-5153, 5155
								#	#					#	#				B51	*510102, 510109, 510202, 5105, 510902 *510104, 511301,
								#	#					#	#				B51, B52	5116, 5121, 512401- 512402, 5131, 5134, 5135
					#	#			#					#	#				B51	*5104, 5146
					#				#					#	#				B51	*5106
					#			#	#					#	#				B51	*5108, 5120, 5144N
					#				#					#	#				-	*5110
									#				#	#	#				-	*5115
					#			#	#					#	#				-	*5136
					#			#	#					#	#				-	*5137
				#	#				#					#	#				-	*5142
				#					#					#	#				-	*5145
								#						#	#				-	*5150, 5154
								#	#		#				#				B52	*520101, 520103, 5204, 5205, 5207, 5210-5212
					#			#	#		#				#				B52	*520102, 520104-5203, 520601-520602, 5209
								#	#		#				#				-	*5208
								#						#	#				B53	*530101-530104, 5305, 5308, 5310, 5315,
					#				#					#	#				B53	*5302
														#	#				-	*5303, 5309, 5311- 5313
									#					#	#				-	*5304, 5307
				#	#				#					#	#				B53	*5306
									#					#	#					*5314
			#			#					#	#			#				B54	*5401, 5402, 5405N, 5407, 5408N, 5410, 5413
			#			#					#		#			#			-	*5403
			#			#					#				#				-	*5404, 5411
			#			#					#			#	#				-	*5406
			#			#					#				#				-	*5409

			#			#					#	#					#		-	*5412
			#			#					#	#					#	B55	*550101-550104, 5503, 5505, 5511, 5515,	
			#			#					#	#					#	B54, B55	*550201-550202, 5507, 5510, 5512, 5516, 5519, 5526, 5530	
			#			#					#	#					#	B55	*5504, 5508, 5513, 5527, 5615, 5619N,	
			#			#			#								#	-	*5509, 5517, 5528	
			#			#					#	#					#	-	*5514	
			#			#				#	#						#	-	*5518	
		#	#		#	#											#	-	*5520	
			#			#			#		#						#	-	*5521	
			#			#											#	-	*5522	
			#			#											#	-	*5523, 5618	
			#			#			#								#	-	*5524	
			2			3					4						4	B56	*5601, 5608, 5614, 5616, 5620	
			8			2					1						7	B56	*5602, 5604	
			2			3					4		4				4	B56	*5603	
			#			#									#		#	B22	*560501	
			#			#			#								#	-	*560502	
			#			#			#								#	B78	*5606	
			#			#									#		#	-	*5607	
			#			#											#	-	*5609	
			#			#							#				#	-	*5610	
			#			#											#	-	*5611, 5612	
			#		#	#					#						#	-	*5613	
			#			#					#						#	-	*5617	
			#			#			#								#	-	*5621	
									#								#	-	*5623	
									#					#			#	B57	*570101-570103, 5706, 5708, 5710	
									#					#			#	B57	*5702-5705	
				#					#					#			#	-	*5707	
			#						#					#			#	-	*5709	
									#					#			#	-	*5711	
									#					#			#	-	*5712	
						#			#					#			#	-	*5713	
					#				#					#			#	-	*5714	
						#			#					#			#	-	*5715	
						#			#					#			#	-	*5716	
									#				#				#	B58	*580101, 5804, 5805, 5810N-5814	
									#				#				#	B58	*580102, 5815	
									#				#				#	-	*5807	
					#			#	#				#				#	-	*5808	
					#			#					#				#	-	*5809	

[illegible]