

INTERPRETATION TABLE					
HLA-B*81 SSP subtyping					
Amplification patterns of the B*81 alleles					
	Well				
	1	2	3	4	5
Length of spec.	175	410	400	285	165
PCR product					
Length of int.	800	1070	1070	1070	1070
pos. control ¹					
5'-primer ²	363	44	44	363	652
	5'-AgC ^{3'}	5'-CTg ^{3'}	5'-ggC ^{3'}	5'-AgC ^{3'}	5'-CCA ^{3'}
3'-primer ³	499	282	272	605	774
	5'-ggA ^{3'}	5'-gCC ^{3'}	5'-TgT ^{3'}	5'-gCT ^{3'}	5'-ggT ^{3'}
Well No.	1	2	3	4	5
HLA-B allele					
*8101	1	2		4	4
*8102	1		3	4	4
*8103	1	?	?	4	
*070201-070207, 0704-0706, 0709, 0711, 0715, 0717, 0719-0726, 0728-0731, 0734-0736, 0739-0741, 0743-0749N, 0751, 0752, 0754-0759, 0761, 0762, 4201, 4202, 420501-4206, 4209	3 4 5				
*0703, 0708, 0710, 0713, 0716, 0727, 0732, 0737, 0738, 0750, 080101-0808N, 0810, 0811, 0813-0815, 0817-0830N, 0832-0835, 3707, 4015, 4016, 4032, 4102, 4104, 4207, 4805, 4808, 4810, 4812, 4814				4	5
*0707, 0712, 0714, 071801-071802, 0733, 0742, 0753, 0760, 4204, 4208, 4506, 670101-670102	3 5				
Well No.	1	2	3	4	5

Length of spec.	175	410	400	285	165
PCR product					
Well No.	1	2	3	4	5
*0809, 0812, 0816, 0831, 1301-1304, 1306-1321, 1401-1407N, 15010101-1519, 1521, 1523-1540, 1542-1558, 1560-1599, 9501, 9503-9529, 9531-9530, 180101-1815, 1817N-1828, 2712, 2716, 2718, 2723, 2729, 2739, 3531, 3543, 3546, 3557, 3560, 3579, 370101-3706, 3708-3713, 380101-3817, 39010101-39010102L, 390103-3920, 3922-3943, 400101-401403, 4018-4030, 4033-4040, 4042-4044, 4046-4079, 4101, 410301-410302, 4105-4108, 44020101-4459, 4501-4505, 4507, 460101-4612, 47010101-4705, 480301-480302, 4813, 4817, 4901-4905, 5001, 5002, 5004, 5207, 5518, 5617, 570101-5716, 5811, 5901, 5902, 6702					5
2702-270402, 270502-2711, 2713-2715, 2717, 2719-2721, 2725-2728, 2730-2738, 5401-5413, 550101-5505, 5507-5517, 5519-5529, 5601-5616, 5618-5620, 8201, 8202, 8301	2				5
*2724	1	2			5
*3566	4				
*3576, 5621		2			
*4031, 4045, 4080, 480101-480102, 4806, 4807, 4809, 4811, 4815, 4816, 4818, 4819	1	4			5
*4804	1				5
HLA-B allele					
Well No.	1	2	3	4	5