

INTERPRETATION TABLE								
HLA-B*57 SSP subtyping								
Amplification patterns of the B*5701 to 5714 alleles								
	Tube ⁴							
	1	2	3	4	5	6	7	8
Length of spec.	150	100	220	100	220	95	180	250
PCR product								
Length of int.	800	1070	1070	800	1070	1070	1070	1070
pos. control ¹								
5'-primer ²	355	362	362	355	362	362	362	362
	5'-TCA ^{3'}	5'-ggT ^{3'}	5'-ggT ^{3'}	5'-TCA ^{3'}	5'-ggT ^{3'}	5'-ggT ^{3'}	5'-ggT ^{3'}	5'-ggT ^{3'}
3'-primer(s) ³	463	419	539	412	539	418	500	572
	5'-gCT ^{3'}	5'-Cgg ^{3'}	5'-TCA ^{3'}	5'-gTT ^{3'}	5'-TCC ^{3'}	5'-gTC ^{3'}	5'-ggA ^{3'}	5'-gCg ^{3'}
		419						
		5'-CAg ^{3'}						
Tube No.	1	2	3	4	5	6	7	8
HLA-B allele								
*570101-570103	+	+	+					
*5702	+			+	+			
*570301-570302	+		+	+				
*5704	+				+	+		
*5705	+			+				
*5706	+	+	+				+	
*5707	+		+	+				+
*5708	+	+	+					
*5709	+			+				
*5710	+	+	+					
*5711								
*5712	+			+	+			
*5713		+			+			
*5714	+	+	+					
Tube No.	1	2	3	4	5	6	7	8

Length of spec.	150	100	220	100	220	95	180	250
PCR product								
Tube No.	1	2	3	4	5	6	7	8
*070201-0724, 0726-0741, 0743-0747, 0749N, 0750, 0752-0760, 0806, 0820, 1316, 1405-140602, 15010101-150104, 150106, 1503, 1504, 1506-1512, 1514-1519, 1523, 1524, 1526N-1530, 1532-1535, 1537-1540, 1542, 1543, 1545-1547, 1549, 1550, 1553, 1554, 1556-1558, 1560, 1561, 1563-1574, 1576, 1578, 1579N, 1581, 1582, 1587, 1590, 1592-1599, 9501-9504, 9508-9510, 9513, 9515-9520, 9522, 9525-9529, 9531-9535, 9537, 9538, 1815, 1819, 1821, 270401-270402, 2706, 2710, 2715, 2718, 2720, 2721, 2724, 2725, 3543, 3544, 3558, 3567, 3579, 3707, 3810, 3918, 3935, 3936, 4005, 4015, 4016, 4023, 4026, 4032, 4051, 460101-4605, 4607N, 4608, 4610, 4612, 4805, 4808, 4815, 4901-4905, 5001, 5002, 5004, 510101-5103, 5106, 5107, 5111N-5114, 5116-5118, 5121-5124, 5126-5130, 5132-5139, 5141N, 5143, 5145, 5148, 5149, 5150, 5151, 520101-5202, 5204-5209, 5211, 5406, 550101-550104, 5503, 5505, 5509, 5511, 5515, 5517, 5521, 5524, 5525, 5528, 5603, 560501-5606, 5621, 5808, 7801-7803, 7805								
*1301, 1306, 1307N, 1312, 1317, 1520, 1588, 1822, 2719, 2730, 350101-350106, 3503, 3507-350803, 3510, 3513, 3515-3517, 3519-352002, 3523-3530, 3533-3536, 3538-3542, 3545-3550, 3552-3557, 3561-3563, 3565Q, 3569-3571, 3574, 3576-3578, 3582, 44020101-4414, 4416, 4417, 4419N, 4421-4430, 4432-4440, 4442-4446, 4448-4452N, 4455-4457, 4802, 5142, 530101-5305, 5309-5313, 5609, 5611, 5801, 5804, 5805, 5809-5813, 5815, 8301	+							
*1313, 350201-350202, 350401-350402, 3512, 3583, 5612	+			+				
*1320, 1502, 1513, 1521, 1525, 1536, 1544, 1562, 1577, 1580, 1585, 1589, 9506, 9512, 9521, 9539, 3511, 351401-351402, 3521, 5306, 5308	+							
*1321, 350901-350902, 3518, 3712, 4004, 4064, 4068				+				
Tube No.	1	2	3	4	5	6	7	8

Length of spec.	150	100	220	100	220	95	180	250
PCR product								
Tube No.	1	2	3	4	5	6	7	8
*1401-1404, 1407N, 5806								
*3506, 3559, 370101-370104, 3703N-3706, 3708, 3710, 3711, 3942	+			W				
*3713	+			W				
*4028, 5104, 5146	+			+				
*4030, 4034			+	+				
*4817				W				
*5514	+	+	+					
*5802, 5807								
*5814	+	+	+					
HLA-B allele								
Tube No.	1	2	3	4	5	6	7	8
Cw*0510				W				
Tube No.	1	2	3	4	5	6	7	8

INTERPRETATION TABLE								
HLA-B*57 SSP subtyping								
Amplification patterns of the B*5701 to 5714 alleles								
Tube ⁴								
9	10	11	12	13	14	15	16	
170	210	165	90	90	195	145	85	Length of spec.
		215						PCR product
		250						
1070	1070	800	1070	1070	1070	1070	1070	Length of int.
								pos. control ¹
527	209	362	362	209	103	361	256	5'-primer ²
5'-TgA ^{3'}	5'-ggC ^{3'}	5'-ggT ^{3'}	5'-ggT ^{3'}	5'-ggC ^{3'}	5'-CCT ^{3'}	5'-AgT ^{3'}	Acg	
3 rd	2 nd	486	412	256	256	463	302	3'-primer(s) ³
5'-TAT ^{3'}	5'-TCg ^{3'}	5'-gCg ^{3'}	5'-gTT ^{3'}	5'-CCC ^{3'}	5'-CCC ^{3'}	5'-gCT ^{3'}	5'-ggC ^{3'}	
		538						
		5'-gTC ^{3'}						
		572						
		5'-gCg ^{3'}						
9	10	11	12	13	14	15	16	Tube No.
								HLA-B allele
				+				*570101-570103
			+	+				*5702
			+	+				*570301-570302
				+				*5704
				+				*5705
				+				*5706
		+	+	+				*5707
	+			+				*5708
+		+	+	+				*5709
				+	+			*5710
				+		+		*5711
			+	+			+	*5712
+				+				*5713
		+		+				*5714
9	10	11	12	13	14	15	16	Tube No.

170	210	165	90	90	195	145	85	Length of spec.
		215						PCR product
		250						
9	10	11	12	13	14	15	16	Tube No.
+								*070201-0724, 0726-0741, 0743-0747, 0749N, 0750, 0752-0760, 0806, 0820, 1316, 1405-140602, 15010101-150104, 150106, 1503, 1504, 1506- 1512, 1514-1519, 1523, 1524, 1526N-1530, 1532- 1535, 1537-1540, 1542, 1543, 1545-1547, 1549, 1550, 1553, 1554, 1556-1558, 1560, 1561, 1563- 1574, 1576, 1578, 1579N, 1581, 1582, 1587, 1590, 1592-1599, 9501-9504, 9508-9510, 9513, 9515- 9520, 9522, 9525-9529, 9531-9535, 9537, 9538, 1815, 1819, 1821, 270401-270402, 2706, 2710, 2715, 2718, 2720, 2721, 2724, 2725, 3543, 3544, 3558, 3567, 3579, 3707, 3810, 3918, 3935, 3936, 4005, 4015, 4016, 4023, 4026, 4032, 4051, 460101- 4605, 4607N, 4608, 4610, 4612, 4805, 4808, 4815, 4901-4905, 5001, 5002, 5004, 510101-5103, 5106, 5107, 5111N-5114, 5116-5118, 5121-5124, 5126- 5130, 5132-5139, 5141N, 5143, 5145, 5148, 5149, 5150, 5151, 520101-5202, 5204-5209, 5211, 5406, 550101-550104, 5503, 5505, 5509, 5511, 5515, 5517, 5521, 5524, 5525, 5528, 5603, 560501- 5606, 5621, 5808, 7801-7803, 7805
								*1301, 1306, 1307N, 1312, 1317, 1520, 1588, 1822, 2719, 2730, 350101-350106, 3503, 3507- 350803, 3510, 3513, 3515-3517, 3519-352002, 3523-3530, 3533-3536, 3538-3542, 3545-3550, 3552-3557, 3561-3563, 3565Q, 3569-3571, 3574, 3576-3578, 3582, 44020101-4414, 4416, 4417, 4419N, 4421-4430, 4432-4440, 4442-4446, 4448- 4452N, 4455-4457, 4802, 5142, 530101-5305, 5309-5313, 5609, 5611, 5801, 5804, 5805, 5809- 5813, 5815, 8301
								*1313, 350201-350202, 350401-350402, 3512, 3583, 5612
+								*1320, 1502, 1513, 1521, 1525, 1536, 1544, 1562, 1577, 1580, 1585, 1589, 9506, 9512, 9521, 9539, 3511, 351401-351402, 3521, 5306, 5308
								*1321, 350901-350902, 3518, 3712, 4004, 4064, 4068
9	10	11	12	13	14	15	16	Tube No.

170	210	165	90	90	195	145	85	Length of spec.
		215						PCR product
		250						
9	10	11	12	13	14	15	16	Tube No.
+						+		*1401-1404, 1407N, 5806
								*3506, 3559, 370101-370104, 3703N-3706, 3708, 3710, 3711, 3942
+								*3713
+								*4028, 5104, 5146
			+					*4030, 4034
								*4817
+		+						*5514
						+		*5802, 5807
		+						*5814
								HLA-B allele
9	10	11	12	13	14	15	16	Tube No.
								Cw*0510
9	10	11	12	13	14	15	16	Tube No.