

INTERPRETATION TABLE																								
HLA-Cw low resolution SSP typing																								
Amplification patterns of the Cw*0102 to Cw*1803 alleles																								
	Well																							
	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	
Length of spec.	155	270	280	155	280	125	390	130	245	115	340	100	130	220	130	95	110	190	225	220	325	135	160	
PCR product(s)								355	425	165		160	250		255	255	325	250	270	425	380		235	
										260		220												
										390														
Length of int.	800	800	800	1070	800	1070	1070	800	800	1070	1070	1070	800	800	1070	1070	1070	1070	800	800	1070	1070	1070	
pos. control ¹																								
5'-primer(s) ²	89	2 nd	105	477	105	112	1 st	28	47	1 st	1 st	361	201	361	98	98	201	201	112	47	355	105	97	
	5'-gAA ^{3'}	5'-CCA ^{3'}	5'-gCT ^{3'}	5'-gCC ^{3'}	5'-gCT ^{3'}	5'-CCT ^{3'}	5'-CgA ^{3'}	5'-TCA ^{3'}	5'-Agg ^{3'}	5'-CgA ^{3'}	5'-CgA ^{3'}	5'-AgT ^{3'}	5'-CCA ^{3'}	5'-AgT ^{3'}	5'-CTA ^{3'}	5'-CTC ^{3'}	5'-CCA ^{3'}	5'-CCA ^{3'}	5'-CCT ^{3'}	5'-Agg ^{3'}	5'-TCA ^{3'}	5'-gCT ^{3'}	5'-TCg ^{3'}	
						118		2 nd	648	176		419	2 nd		368	485	409	2 nd	2 nd	361	412	459	418	
						5'-CCA ^{3'}		5'-CCA ^{3'}	5'-CAC ^{3'}	5'-gCA ^{3'}		5'-gTC ^{3'}	5'-CCA ^{3'}		5'-gTg ^{3'}	5'-CAA ^{3'}	5'-ggC ^{3'}	5'-CCA ^{3'}	5'-CCA ^{3'}	5'-AgT ^{3'}	5'-ATA ^{3'}	5'-gAT ^{3'}	5'-Agg ^{3'}	
										527								361					419	
										5'-TAC ^{3'}								5'-AgT ^{3'}					5'-gTC ^{3'}	
3'-primer(s) ³	201	559	343	589	343	201	302	213	302	175	302	474	289	538	312	311	270	341	341	302	3 rd	201	289	
	5'-CTC ^{3'}	5'-CTC ^{3'}	5'-C ^{3'}	5'-CTT ^{3'}	5'-T ^{3'}	5'-CTT ^{3'}	5'-ggT ^{3'}	5'-Cgg ^{3'}	5'-ggC ^{3'}	5'-CCg ^{3'}	5'-ggT ^{3'}	5'-gCA ^{3'}	5'-AgC ^{3'}	5'-CCA ^{3'}	5'-AgT ^{3'}	5'-ggT ^{3'}	5'-TAG ^{3'}	5'-CgT ^{3'}	5'-Cgg ^{3'}	5'-ggT ^{3'}	5'-CTC ^{3'}	5'-CTC ^{3'}	5'-AgC ^{3'}	
	201							420	853	302		477	289	538	459	538	3 rd	341	512	538		559	289	
	5'-CTT ^{3'}							5'-gCT ^{3'}	5'-CAT ^{3'}	5'-ggC ^{3'}		5'-gCA ^{3'}	5'-AgC ^{3'}	5'-gCA ^{3'}	5'-AgA ^{3'}	5'-CAg ^{3'}	5'-CTC ^{3'}	5'-ggT ^{3'}	5'-CCA ^{3'}	5'-CCg ^{3'}	5'-CTC ^{3'}	5'-AgC ^{3'}		
										601		538	539					527					539	
										5'-CTT ^{3'}		5'-gCA ^{3'}	5'-TCT ^{3'}					5'-CCg ^{3'}					5'-TCT ^{3'}	
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	
HLA-Cw allele ⁴																								
*010201-010206, 0106-0108, 0111-0113, 0115-0120, 0123, 0125-0127	73																			92				
*0103, 0124	73																			92	93			
*0104	73											84		86										
*0105, 0122	73																							
*0109	73													86										
*0110	73	74																		92				
*0114	73										83									92				
*0121	73											84	85											
*020201		74	75			78					83				w									
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95
*020202-020203, 020205-0203, 0207-0211, 0215-021602, 0219, 0220	74	75			78				83				87										
*0204	74	w			78				83				87										
*0205, 0217	74	75			78				83				86	87									
*0206	74	75			78			80			83				87		89						
*0212	74	75			78								85		87								
*0213, 0218	74	75			78						83				87		90						
*0214	74	w			78				83														
*0221	74	75									83				87								
*0222	74	75			78				83						87						93		
*0223	74	75			78						83				87		89						
*0226	74								83						87								
*0227	74	75			78																		
*030201-030203, 0305, 0317, 0325, 0327, 0333, 0335, 0336	75	76																			94		
*030301-030305, 031101-0312, 0320N-0322Q, 0330, 0331, 034301-034302, 0349, 0350				76	77																93	94	
*030401-030406, 0306, 0308, 0309, 0314, 0319, 0323, 0324, 0326, 0328, 0332, 0334, 0337-033802, 0340, 0342, 0344, 0346-0348, 0357	75	76																			93	94	
*030407, 0341			75																		93	94	
*0307, 0345	75	76							83						87						93	94	
*0310, 0329			75	76											87						93	94	
*0313				76	77																	94	
*0315			75	76							83				87							94	
*0316	75																					94	
*0318, 0339				76																	93		
*0358					77														92		93	94	
*04010101-040106, 040401-0405, 0407-0409N, 0412-041502, 0417-0420, 0424-0428, 0430, 0431, 0434, 0435, 0438, 0440					78						83		85				90				93		
*0403, 0406	75				78				83				85		87		90				93		
*0410					78						83						90				93		
*0411					78											88		91			93		
*0416			75								83		85		87		90				93		
*0423					78						83		85				90						
*0429					78								85			88	90				93		
*0432	74				78						83		85				90				93	94	
*0433					78						83		85					91			93		

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95
*0436						78										88		90			93		
*0437						78					83		85		87			90		92	93		
*05010101-050104, 0503, 0505-0510, 0512-0515, 0518, 0519, 0521, 0524						79					83				87						93		
*0504, 0522							79				83				87								
*0511							79				83				87	88					93		
*0516							79				83				87					92	93		
*0517							79				83				87	88		90			93		
*0520							79				83										93		
*0523							79				83				87				91		93		
*0525							79				83				87			90			93		
*06020101-06020102, 060203, 0607, 0610, 0612, 0613, 0615, 0616N, 0618-0620, 0622, 0623								80			83				86								95
*0603			75					80			83			86	87							94	
*0604								80			83												95
*0605								80			83							90		92			
*0606								80			83									92			95
*0608		74						80			83			86									95
*0609								80			83			86							93		95
*0611								80						86									95
*0614								80			83										93		95
*0617											83			86						92			95
*070101-0706, 0708, 0711-0725, 072701-072702, 0729, 0730, 0732N, 0735-0738, 0742, 0744-0748, 0750-0763, 0766-0772											81												95
*0707, 0709, 0749											81	83								92			95
*0710, 0728, 0741, 0743											81										93		95
*0726, 0733N, 0739, 0740											81												
*0731											81							90					95
*0764											81					88							
*080101-080102, 0803, 0806, 0808, 0814, 0816, 0820, 0822											82							90			93		
*0802, 0815, 0817-0819, 0823											82										93		
*0804, 0813											82					88					93		
*0805											82			85							93		
*0807											82							91			93		
*0809, 0811											82							90					
*0810							79				82	83			87			90			93		
*0812											82									92	93		
*0821											82			85				90			93		

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95
*120201-120203, 1210, 1217, 1218, 1220												84	85										
*12030101-12030102, 120303, 120305-120306, 1206, 1207, 1211-1213, 1219												84	85	86									
*120302													85	86									
*120304												84		86									
*120401											83			86	87								
*120402											83	84	85	86	87								
*1205											83	84		86	87								
*1208												84	85				89						
*1209											83				87					92			
*121401													85					90	91				
*121402												84	85					90	91				
*1215								80				84	85	86			89						
*1216												84	85										95
*1221											83	84			87								
*140201-1403, 1405, 1407N, 1410, 1411																88				92			
*1404											83					88				92			
*1406, 1408, 1409																88							
*150201-150204, 1508, 151002, 1513, 1515, 1517, 1518			75					80				83			87		89					93	
*1503, 1516			75					80			83		85		87		89					93	
*1504-1506, 1509, 1519, 1520			75								83				87		89					93	
*1507			75					80									89					93	
*151001								80			83				87		89					93	
*1511			75			78		80			83				87		89					93	
*1512					77			80			83				87		89					93	
*1521			75					80							87		89					93	
*160101-160103, 1606-1608, 1610, 1611													85					90					95
*1602, 1609, 1612											83		85		87			90					95
*160401														86				90					
*1701-1705		74									83				87				91			93	
*1801, 1802											83									92		93	95
*1803		74									83									92		93	94
HLA-Cw allele ⁴																							
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95
B*1403																				92			
B*6702													85										
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95