

[illegible]

[illegible]

*05:17, 05:25								79						83					87					90					93		
*05:20, 05:43								79						83															93		
*05:23								79						83					87							91			93		
*05:26								78	79					83					87										93		
*05:36								79						83					87					89					93		
*05:42								79						83	84	85			87					90					93		
*05:46								79						83	84	85			87										93		
*06:02:01:01-06:02:01:02, 06:02:03-06:02:11, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18- 06:30, 06:33-06:34, 06:36-06:39, 06:41-06:51, 06:53-06:55													80					83													95
*06:03													75					80											94		
*06:04, 06:40														80				83												95	
*06:05														80				83							90		92			95	
*06:06														80				83									92			95	
*06:08													74				80		83											95	
*06:09														80				83											93	95	
*06:11, 06:52														80																95	
*06:14, 06:35														80				83											93	95	
*06:17																		83									92			95	
*06:31														80				83							90		92			95	
*06:32																		83												95	
*07:01:01-07:02:07, 07:02:10- 07:06, 07:08, 07:11-07:25, 07:27:01-07:27:02, 07:29-07:30, 07:32N, 07:35-07:38, 07:42, 07:44-07:48, 07:50-07:63, 07:65- 07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140- 07:147, 07:149-07:151, 07:153- 07:155, 07:157-07:160																		81													95
*07:02:08, 07:26, 07:33N, 07:39- 07:40, 07:92, 07:139, 07:152N, 07:156																		81													
*07:02:09														78				81							90					95	
*07:07, 07:09, 07:49, 07:76																		81		83							92			95	
*07:10, 07:28, 07:41, 07:43																		81										93		95	
*07:31																		81							90					95	
*07:64																		81									88				
*07:96														75				81											94		
*07:101, 07:148														74				81	82		84									95	
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96							
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96							
HLA-C allele ⁴																															
*07:123									81									89											95		
*07:125								78	81					85	86														95		

Negative Control

*08:01:01-08:01:03, 08:03:01-08:03:02, 08:06, 08:08, 08:14, 08:16, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44										82							90			93				
*08:02:01-08:02:05, 08:04, 08:13, 08:15:01, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43										82										93				
*08:05										82		84	85							93				
*08:07										82							91			93				
*08:09, 08:11										82							90							
*08:10							79			82	83				87		90			93				
*08:12										82									92	93				
*08:21										82		84	85				90			93				
*08:25										82			85							93				
*08:31		74								82										93				
*12:02:01-12:02:06, 12:10:01-12:10:02, 12:17-12:18, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40												84	85											
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:14, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50												84	85	86										
*12:03:02, 12:03:08, 12:26													85	86										
*12:03:04												84		86										
*12:03:09							80					84		86										
*12:04:01, 12:33											83			86	87									
*12:04:02											83	84	85	86	87									
*12:05											83	84		86	87									
*12:08												84	85				89							
*12:09											83				87				92					
*12:14:01													85				90	91						
*12:14:02												84	85				90	91						
*12:15							80					84	85	86			89							
*12:16												84	85									95		
*12:21											83	84			87									
*12:24												84	85						92					
*12:28												84	85	86			90							
*12:31												84	85	86						93				
*12:41											83	84	85		87									
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ⁴																								
*12:44												84	85							93				
*12:49												84												
*14:02:01-14:03, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:24																88				92				

Negative Control

Negative Control

*14:04									83								88								92																																							
*14:06, 14:08-14:09																	88																																															
*14:10																	88								90								92																															
*14:12									83																								92																															
*14:15																	88								90																																							
*14:16																	86								88																																							
*14:17																	88								91								92																															
*14:25	74																								88								92								94																							
*15:02:01-15:02:07, 15:08, 15:10:02, 15:13, 15:15, 15:17-15:18, 15:26, 15:28, 15:31-15:35, 15:38-15:39, 15:41, 15:44-15:45	75								80								83								87								89								93																							
*15:03, 15:16	75								80								83								85								87								89								93															
*15:04-15:06:03, 15:09, 15:19-15:20, 15:24, 15:27, 15:29-15:30, 15:40	75																83								87								89								93																							
*15:07	75								80																								89								93																							
*15:10:01									80								83								87								89								93																							
*15:11	75								78								80								83								87								89								93															
*15:12									77								80								83								87								89								90								93							
*15:21	75								80																87								89								93																							
*15:22	75																83								87								89																															
*15:23																	83																89								93																							
*15:25	75																85								90								91								93																							
*15:36									78								83																89								93																							
*15:37	75								80								83								87								89																															
*15:42	74								75								80								83								87								89								93															
*15:43	75								80																								89								93								94															
*16:01:01-16:01:05, 16:06-16:08, 16:10-16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32																	85								90								95																															
*16:02:01-16:02:05, 16:09, 16:12, 16:19, 16:25									83								85								87								90								95																							
*16:04:01																	86								90								92																															
*16:15:02																	84								85								90								95																							
*16:18																	85								87								90								95																							
*16:20																	85								89								90								95																							
*16:29	74																86								90								92																															
*17:01:01:01-17:01:03, 17:01:05-17:04, 17:06, 17:08	74																83								87								91								93																							
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96																																								
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96																																								
HLA-C allele ⁴																																																																
*17:01:04	74																83								85								87								91								93															
*17:05	74																83								84								87								91								93															
*17:07																	83								87								91								93																							
*18:01-18:02, 18:04																	83																92								93								95															
*18:03	74																83																92								93								94								95							
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96																																								

HLA-C allele ⁴																											Negat
A*24:106, B*46:25																										95	
B*14:03																			92								
B*67:02													85														
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96			