

[illegible]

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49
HLA-B allele ⁴	ser. ⁵																									
*07:02:01-07:02:20, 07:05:01-07:06, 07:10, 07:12-07:15, 07:18:01- 07:18:02, 07:21-07:23, 07:26, 07:29-07:31, 07:34- 07:35, 07:39, 07:41-07:46, 07:49N, 07:52, 07:55- 07:59, 07:61-07:64, 07:66- 07:68:02, 07:70-07:72, 07:74-07:76, 07:79-07:80, 07:82-07:83, 07:86-07:99, 07:101-07:117	B7, B42, Null, –	25																							48	
*07:03, 07:08, 07:16, 07:32, 07:37	B703, B7, –	25																							48	
*07:04, 07:25	B7	25													38										48	
*07:07, 07:77	B7, –	25																							48	
*07:09, 07:11, 07:17	B7	25											36												48	
*07:19, 07:33, 07:40, 07:53	B7, 42																								48	
*07:20, 07:24	B7, –	25		27																					48	
*07:27	–	25										35							44							
*07:28	B7	25					30																		48	
*07:36, 07:81	B7, –	25																								
*07:38	–	25																								
*07:47	B7	25																							48	
*07:48, 07:51	B7																								48	
*07:50	–	25										35													48	
*07:54	B7	25																							48	
*07:60	–			27																					48	
*07:65	–	25																	42							
*07:69, 07:85	–	25																						47	48	
*07:73	–	25															40								48	
*07:78	–	25									34														48	
*07:84	–	25																							48	
*07:100	–			27																					48	
*08:01:01-08:01:13, 08:05, 08:08N, 08:10- 08:11, 08:15, 08:18- 08:19N, 08:22-08:24, 08:27, 08:29-08:31, 08:33- 08:34, 08:39, 08:41-08:48, 08:51, 08:53, 08:56-08:59, 08:61, 08:63-08:64, 08:66	B8, Null, –		26						32																48	
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49

[illegible]

[illegible]

*15:23, 15:115	B70, B5, –								32				36	38			43									
*15:24, 15:87, 15:157	B62, B15, –								31				36	38			43									
*15:29	B70								32				36				43									
*15:30, 15:58, 15:63	B62								31								43									
*15:31	B75								31				36				43									
*15:36	–			w					31				36				41	43								
*15:42	B15								31								43									
*15:43	B15								31				36				43									
*15:45	B62			27					31								43									
*15:46	–				28				31	32			36				43									
*15:48, 15:150, 15:188	B62, –								31								43									
*15:53	–				28				32				36				43									
*15:54	B72								32			35	36				43									
*15:55	B15			27					31				36				43									
*15:57	B62								31				36				43									
*15:62	–								32			35	36				41	43								
*15:64	B71								32			35	36				43									
*15:66	B62								31				36	38			43									
*15:68	–			27								35	36													
*15:69	B72								32			35	36				43	45								
*15:73	B62								31								43									
*15:74	B72								32				36				43									
*15:76	–								31				36				43									
*15:77	B62						30	31									41	43								
*15:80	B70								32				36	38			41	43								
*15:83	B62								31	32							43									
*15:85, 15:154, 15:165	B62, –								31				36				41	43								
*15:86	–								31								43									
*15:89	B77			w					31				36				41	43								
*15:91, 15:131	–, –								32			35	36	37			43									
*15:101	–								31				36													
*15:106	–				28				31	32			36				41	43								
*15:116	B62								31				36				43								49	
*15:124	B71								32				36	38			43								49	
*15:137	–								31								43									
*15:138	–	25							31				36				43									
*15:142	–		26						31				36				43									
*15:143	–								31	32			36				43									
*15:144	–								31				36				41	43								
*15:161	–								32				36	37	38		43									
*15:180	–		26										36													
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49
HLA-B allele ⁴	ser. ⁵																									
*15:186	–								32				36	38				43	45							
*15:189	–						30	31					36	38				43								
*15:202	–							31					36					42	43							

[illegible]

[illegible]

[illegible]

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49
HLA-B allele ⁴	ser. ⁵																									
*37:04:01-37:04:02	B37, –													37				41			44					
*37:05	–													37				41			44					
*37:07	–	25												37							44					
*37:08	B37													37				41			44					
*37:09	B37													37							44					
*37:10, 37:22	B37, –																	41			44					
*37:11	–																	41	42		44					
*37:12	B37								32					37							44					
*37:14	–																	41			44					
*38:01:01-38:02:02, 38:08-38:09, 38:11-38:16, 38:18-38:19, 38:21, 38:23-38:24	B38, –					29																45	46	47		
*38:02:03	–																					45		47		
*38:03	B38					29						35										45	46			
*38:04, 38:25	B38, –																					45		47		
*38:05	B38						30															45	46	47		
*38:06	B38																					45				
*38:07	B38					29																45				
*38:10	B38					29																	46	47		
*38:17	–					29								37							44	45	46	47		
*38:20	–					29												41				45	46	47		
*38:22	–					29												40					46	47		
*38:26	–					29																45	46			
*39:01:01:01-39:01:01:02L, 39:01:03-39:01:08, 39:01:10-39:01:12, 39:03, 39:05:01-39:06:02, 39:09, 39:11, 39:14-39:15, 39:24:01, 39:26-39:29, 39:31, 39:35, 39:37-39:38Q, 39:40N-39:41, 39:44, 39:46, 39:48, 39:51-39:57, 39:59-39:62	B3901, B39, Low B39, Null, –					29																45		47	48	
*39:01:09, 39:12, 39:45	B3901, B39, –																					45		47	48	
*39:02:01, 39:08	B3902, B39					29						35										45			48	
*39:02:02, 39:13:01-39:13:02, 39:23, 39:49	B3902, B39, –											35										45			48	
*39:04	B39						30															45		47	48	
*39:07	B16					29							36									45		47	48	
*39:10, 39:16, 39:20	B39																					45			48	
*39:17	B39																					45			48	
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

*78:06	–									33																		
*81:01	B81																											
*81:02	B81																										48	
*81:03-81:05	Null, –																											
*82:01-82:03	B82, –								32			35							43									
*83:01	–																		41									
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49		
HLA-B allele ⁴	ser. ⁵																											
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49		
A*23:31, A*24:106, C*16:10									30																			
A*68:56, C*06:20, C*12:50																												
C*01:30	–				28																							
C*02:06																												
C*02:23, C*04:77		25																										
C*03:05, 03:25, 03:27				27																								
C*03:12, 03:19																												
C*03:102																			41									
C*07:46									32																			
C*15:02:04																												
C*15:25																												
C*15:39																												
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49		

50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																							ser. ⁵	HLA-B allele ⁴
		52																			71	72	B7, B42, Null, –	*07:02:01-07:02:20, 07:05:01-07:06, 07:10, 07:12-07:15, 07:18:01-07:18:02, 07:21-07:23, 07:26, 07:29-07:31, 07:34-07:35, 07:39, 07:41-07:46, 07:49N, 07:52, 07:55-07:59, 07:61-07:64, 07:66-07:68:02, 07:70-07:72, 07:74-07:76, 07:79-07:80, 07:82-07:83, 07:86-07:99, 07:101-07:117
																					71	72	B703, B7, –	*07:03, 07:08, 07:16, 07:32, 07:37
	51	52																			71	72	B7	*07:04, 07:25
		52																			71		B7, –	*07:07, 07:77
		52																			71	72	B7	*07:09, 07:11, 07:17
		52																			71		B7, 42	*07:19, 07:33, 07:40, 07:53
		52																			71	72	B7, –	*07:20, 07:24
																			70		72	–	*07:27	
		52																			71	72	B7	*07:28
		52																	70		72	B7, –	*07:36, 07:81	
																			70		72	–	*07:38	
		w																			71	72	B7	*07:47
		52																			71	72	B7	*07:48, 07:51
																					71	72	–	*07:50
		52									62										71	72	B7	*07:54
		52																			71		–	*07:60
		52				56							64					w			71	72	–	*07:65
																					71	72	–	*07:69, 07:85
		52																			71	72	–	*07:73
		52							59												71		–	*07:78
		52							59												71		–	*07:84
		52																	69		71		–	*07:100
		51																			71	72	B8, Null, –	*08:01:01-08:01:13, 08:05, 08:08N, 08:10-08:11, 08:15, 08:18-08:19N, 08:22-08:24, 08:27, 08:29-08:31, 08:33-08:34, 08:39, 08:41-08:48, 08:51, 08:53, 08:56-08:59, 08:61, 08:63-08:64, 08:66
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

																		ser. ⁵	HLA-B allele ⁴					
51															70	72	B8, –	*08:02-08:03, 08:52						
51																71 72	B8, –	*08:04						
51																71 72	B8	*08:07, 08:14						
51		54														71	B8	*08:09						
51																71	B8	*08:12:01-08:12:03, 08:16						
																71 72	B8, –	*08:13, 08:20, 08:40						
51								62								71 72	–, –	*08:17, 08:54						
51																71 72	B8	*08:21						
																71 72	B8	*08:25						
51																71 72	B8, –	*08:26, 08:50, 08:62, 42:07 ⁶						
51																71 72	B8, –	*08:28, 08:37						
51													68			71 72	B8	*08:32						
51																71 72	–	*08:35						
51																71 72	–	*08:36						
51								62								71 72	–	*08:38						
51		53														71 72	–	*08:49						
																71 72	–	*08:55						
51		53														71 72	–	*08:60						
51																71 72	–	*08:65						
										62					70		B13, –	*13:01:01-13:01:05, 13:17, 13:20, 13:22:01-13:23, 13:25, 13:28-13:29						
										62					70		B13, –	*13:02:01-13:02:09, 13:08Q, 13:15, 13:18-13:19, 13:27, 13:30, 13:32 13:34, 13:37-13:38, 13:40						
										62		65			70		B49, B15	*13:03						
										62					70		B49, B15, B13	*13:04, 13:10						
										62	64				70		B13	*13:06						
															70		Null	*13:07N						
										62					71		–	*13:09						
										62			67		70		B13	*13:11						
										62					70		B13	*13:12-13:13						
										62					70		B13	*13:14						
							59			62					70		B13	*13:16						
										62					70		–	*13:21, 13:35						
										62					70		–	*13:26						
							w			62				69	70		–	*13:31						
						57									70		–	*13:36						
										62					71		–	*13:39						
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																							ser. ⁵	HLA-B allele ⁴

																			71	B64, Null, –	*14:01:01-14:01:02, 14:07N, 14:12, 14:14, 14:19				
																			71	B65, B14, –	*14:02:01-14:02:02, 14:02:04, 14:03-14:04, 14:09, 14:11, 14:15-14:18				
																			71	B65, –	*14:02:03, 14:02:06				
																			71	B65	*14:02:05				
																			71	B14, –	*14:05-14:06:02, 14:13				
																			71	B39, B16, Null, –	*14:08, 39:25N, 39:30, 39:32-39:34, 39:47, 39:50 ⁷				
						55													71	–	*14:10				
																				B62, Null, –	*15:01:01:01- 15:01:01:02N, 15:01:03, 15:01:06-15:01:16, 15:01:18, 15:01:20, 15:26N-15:27:03, 15:32, 15:34-15:35, 15:38:01- 15:38:02, 15:50, 15:56, 15:60, 15:70-15:71, 15:75, 15:79N, 15:81-15:82, 15:92, 15:94N, 15:96- 15:97, 15:102, 15:104- 15:105, 15:109-15:111N, 15:113, 15:117-15:118, 15:122, 15:125, 15:128- 15:129, 15:135, 15:140, 15:145-15:147, 15:152, 15:159-15:160, 15:163- 15:164, 15:166-15:167, 15:169, 15:171-15:172, 15:174-15:175, 15:178, 15:181N-15:184, 15:187, 15:190N, 15:192-15:193, 15:201, 15:203, 15:206				
											62								69	71	B62	*15:01:02			
											62								69	71	B62	*15:01:04			
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																								ser. ⁵	HLA-B allele ⁴

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																							ser. ⁵	HLA-B allele ⁴
			53		55						62									70			B37, –	*37:04:01-37:04:02
			53								62										71		–	*37:05
											62									70		72	–	*37:07
			53															68		70			B37	*37:08
											62									70			B37	*37:09
			53								62									70			B37, –	*37:10, 37:22
			53								62										71		–	*37:11
			53								62									70			B37	*37:12
			53								62									70	w		–	*37:14
																				70			B38, –	*38:01:01-38:02:02, 38:08-38:09, 38:11-38:16, 38:18-38:19, 38:21, 38:23-38:24
																				70			–	*38:02:03
											62									70			B38	*38:03
											62									70			B38, –	*38:04, 38:25
																				70			B38	*38:05
																		68		70			B38	*38:06
																		68		70			B38	*38:07
																				70			B38	*38:10
																				70			–	*38:17
																				70			–	*38:20
																				70			–	*38:22
		52																		70			–	*38:26
																					71		B3901, B39, Low B39, Null, –	*39:01:01:01-39:01:01:02L, 39:01:03-39:01:08, 39:01:10-39:01:12, 39:03, 39:05:01-39:06:02, 39:09, 39:11, 39:14-39:15, 39:24:01, 39:26-39:29, 39:31, 39:35, 39:37-39:38Q, 39:40N-39:41, 39:44, 39:46, 39:48, 39:51-39:57, 39:59-39:62
																					71		B3901, B39, –	*39:01:09, 39:12, 39:45
											62										71		B3902, B39	*39:02:01, 39:08
											62										71		B3902, B39, –	*39:02:02, 39:13:01-39:13:02, 39:23, 39:49
																					71		B39	*39:04
																					71		B16	*39:07
																					71		B39	*39:10, 39:16, 39:20
												64					67				71		B39	*39:17
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

																		ser. ⁵	HLA-B allele ⁴						
															69	71		B39	*39:18						
															68		71	B39	*39:19:01						
															68		71	B39	*39:19:02						
										62							71	B39	*39:22						
																69	71	–	*39:36						
																	71	B39	*39:39						
																	71	B39	*39:42						
																	71	–	*39:43						
																	71	–	*39:58						
										61	62						71	B60, –, –, Null	*40:01:01-40:01:06, 40:01:08-40:01:10, 40:01:12-40:01:17, 40:07, 40:51, 40:66, 40:73, 40:79, 40:124:01, 40:139-40:140, 40:146-40:147, 40:150, 40:152-40:153, 40:155N						
										61	62						71	B60	*40:01:07, 40:72:01, 40:77						
										61	62	64					71	–	*40:01:11						
50										61	62						71	B61, B41, Null, –	*40:02:01-40:02:11, 40:04, 40:39, 40:89, 40:94, 40:97, 40:122, 40:142N-40:144N						
50										61	62						71	B61	*40:03, 40:20						
50										61	62						71	B4005, B50	*40:05						
50										61	62						71	B61, –	*40:06:01:01-40:06:03, 40:70						
50										61							71	B61, –	*40:08						
?										61	62						71	B61, –	*40:09, 40:11:01-40:11:02, 40:18, 40:24, 40:27:01, 40:29, 40:35, 40:37, 40:50, 40:56-40:57, 40:64, 40:78, 40:82, 40:85, 40:90-40:91, 40:99, 40:104, 40:107, 40:111, 40:115, 40:119-40:120, 40:133Q, 40:145						
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																								ser. ⁵	HLA-B allele ⁴

[illegible]

[illegible]

			54					61	62		65			71	B45, – *45:01, 45:05, 45:07, 45:11-45:13									
			54					61	62		65			71	B21	*45:04								
		52	54					61	62		65			71	B45	*45:06								
			54					61	62	64				71	–	*45:08								
			54					61	62					71	–	*45:09								
			54					61	62					71	–	*45:10								
		52											69	71	*46:01:01-46:01:05, 46:04 B46, 46:05, 46:07N, 46:09-Null, – 46:10, 46:14-46:16, 46:22-46:24									
		w											69	71	B46	*46:02								
		52												71	*46:03, 46:08, 46:13:01-B46, – 46:13:03, 46:19, 46:21:01-46:21:02, 46:25-46:26									
		52												71	–	*46:06								
		52									65			71	B22	*46:11								
		52											69	71	–	*46:12, 46:20								
		52											69	71	–	*46:17								
		52								64				71	–	*46:18								
50						56			62					70	*47:01:01:01-47:01:01:02, B47, – 47:06-47:07									
50						56			61	62				71	B47	*47:02								
50						56			w	62				70	w	B47 *47:03								
50									62					70		B17, – *47:04-47:05								
							57			62					71	72	*48:01:01-48:01:03, B48, – 48:07, 48:09, 48:11, 48:19 48:20, 48:22							
										62					71	B48, B70	*48:02:01-48:02:02							
										62					71	B48	*48:03:01-48:03:02							
							57			62					71	B48, –	*48:04, 48:24							
										62					71	72	B48, –	*48:05, 48:08						
							57								71	72	–	*48:06						
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																							ser. ⁵	HLA-B allele ⁴
											62										71	72	B48	*48:10, 48:12
											62										71		–	*48:13
											62										71	72	B48	*48:14
							57				62										71	72	B48	*48:15
							57														71	72	B48	*48:16
											62										71	72	–	*48:17
							57				62								70		72	B48	*48:18	
											62										71		–	*48:21
											62										71		–	*48:23
							58	59			62				65				70				B49, –	*49:01:01, 49:06, 49:08-49:10, 49:12-49:15

								58				62			65			70			B49, –	*49:01:02, 49:11		
								59				62			65			70			B49	*49:02		
								59				62			65			70			B49	*49:03		
								58	59			62						70			B49	*49:04-49:05		
								58	59			62						70			–	*49:07		
								59		61	62				65			71			B50, –	*50:01:01-50:01:02, 50:04 50:05, 50:08, 50:10-50:11		
								59		61	62				65			71			B45	*50:02		
								59			62				65			71			–	*50:06		
								59		61	62				65			71			–	*50:07		
								59		61	62							71			–	*50:09		
										61	62				65			71			–	*50:12		
																						*51:01:01, 51:01:03, 51:01:05-51:01:08, 51:01:10-51:02:01, 51:02:03, 51:03, 51:07:01, 51:09:01, 51:11N-51:12, 51:13:02-51:14, 51:16- B51, B5102, B5103, Null, – 51:19, 51:21-51:23, 51:26- 51:33, 51:38-51:41N, 51:43, 51:48-51:53, 51:55, 51:57-51:58, 51:60, 51:65- 51:67, 51:69-51:77, 51:79- 51:80, 51:82-51:88, 51:90- 51:91, 51:94-51:96, 51:98N, 51:100, 51:102, 51:105		
									59	60						68		70			B51, B5102, –	*51:01:02, 51:02:02, 51:05, 51:07:02, 51:09:02, 51:89, 51:103		
									59	60						68		70			B51, B5102, –	*51:01:04, 51:02:04, 51:13:01, 51:24:01, 51:34- 51:35, 51:92, 51:99		
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																							ser. ⁵	HLA-B allele ⁴
									59	60							68	70					B51	*51:01:09
				54	55					60							68	70					B51, –	*51:04, 51:46, 51:56:01
				54						60							68	70					B51, –	*51:06:01, 51:59
				54						59	60						68	70					B51, Null	*51:08, 51:20, 51:44N
				54						60							68	70					B51	*51:10
										60					65		68	70					–	*51:15
				54						59	60						68	70					B51	*51:24:02
				54						59	60						68	70					B51, –	*51:24:03-51:24:04
				54						59	60						68	70					B51	*51:36
				54						59	60						68	70					B51, –	*51:37, 51:63
				53	54					60							68	70					–	*51:42

			54						60						68	70			–	*51:45					
								59							68	70			–	*51:54, 51:78					
			54					w 60							68	69 70			–	*51:61					
			54					60				65	67	68	70				–	*51:62					
			54					60							68	70			–	*51:64					
			54					59 60							68	70			–	*51:68					
			54					60							68	70			–	*51:81					
								59 60							68	70			–	*51:93					
			54					59 60							68	70			–	*51:97					
			54					59 60							68	70			–	*51:101					
			54					59 60								70			–	*51:104					
			54					59 60				65		68	70				–	*51:106					
								59 60		62						70			B52, –	*52:01:01:01, 52:01:03, 52:01:06, 52:04-52:05, 52:07-52:08, 52:10-52:13, 52:15, 52:17-52:18, 52:22					
			54					59 60		62						70			B52	*52:01:02, 52:01:04, 52:02 52:03, 52:06:01-52:06:02, 52:09					
								59 60		62						70			B52, –	*52:01:05, 52:14					
								59 60		62						70			–	*52:16					
			54					59 60		62						70			–	*52:19					
								59		62						70			–	*52:20					
			54					59 60		62						69 70			–	*52:21					
								60							68	70			B53, –	*53:01:01-53:01:05, 53:05, 53:08:01-53:08:02, 53:10, 53:16, 53:18, 53:20 53:21, 53:23-53:24					
			55					60							68	70			B53	*53:02					
															68	70			B53, B35	*53:03, 53:09, 53:11-53:13					
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																							ser. ⁵	HLA-B allele ⁴	
										60								68	70				B53, –	*53:04, 53:07, 53:19	
			54	55						60								68	70				B53, B51	*53:06	
										60								68	70				–	*53:14	
										60								68	70	72			–	*53:15	
										60	62								70				–	*53:17:01	
			53							60								68	70				–	*53:22	
		52				56						63	64								71			B54, Null, –	*54:01:01-54:02, 54:05N, 54:07-54:08N, 54:10, 54:13, 54:16-54:19, 54:21-54:23
		52				56						63		65							71		B22	*54:03	
		52				56						63									71		B54, –	*54:04, 54:15	
		52				56						63						69	71				–	*54:06	

	52		56				63				71	– *54:09, 54:14												
	52		56				63				71	B54	*54:11											
	52		56	58			63 64			70		– *54:12												
	52		56		59		63 64				71	–	*54:20											
	52		56		59		64				71	B55, – *55:01:01-55:01:06, 55:03, 55:05, 55:11, 55:15, 55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:44-55:45												
	52		56				64				71	B55, B54, –	*55:02:01-55:02:06, 55:07, 55:10, 55:12, 55:16, 55:19, 55:26, 55:30, 55:35, 55:37, 55:39, 55:41-55:43, 55:47-55:48											
	52		56								71	B55, B56, Null, –	*55:04, 55:08, 55:13, 55:27, 55:46, 56:15, 56:19N, 56:22 ¹⁰											
	52		56		59						71	B22, B55	*55:09, 55:24											
	52		56						66		71	– *55:14												
	52		56		59						71	B55, –	*55:17, 55:28											
							62 64				71	– *55:18												
51	52	54	56								71	B22	*55:20											
	52		56		w		64			69	71	B55	*55:21											
	52		56								71	B22	*55:22											
	52		56								71	B55, –	*55:23, 55:32, 56:18 ¹¹											
	52		56				62 64				71	–	*55:34											
	52		56		59		62 64				71	–	*55:40											
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																							ser. ⁵	HLA-B allele ⁴
	52		56												65						71		B56, Null, –	*56:01:01-56:01:04, 56:08, 56:14, 56:16, 56:20, 56:24, 56:26-56:30
	52		56												65	67					71		B56	*56:02, 56:04
	52		56															69			71		B22, 15	*56:03
	52		56				59					64									71		B56	*56:05:01
	52		56				59														71		B56	*56:05:02
	52		56				59					64					w				71		B78	*56:06
	52		56												65					70			B56	*56:07
	52		56																		71		B56	*56:09
	52		56									64				67					71		B55	*56:10
	52		56																		71		B56, B22	*56:11-56:12
	52	54	56												65						71		B56	*56:13
															65						71		B56	*56:17
	52		56				58 59													70			–	*56:21

[illegible]

[illegible]