

[illegible]

[illegible]

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
HLA-B allele ⁴	ser. ⁵																							
*08:02-08:03, 08:52	B8, –		26						32															
*08:04	B8, –		26						32			35												
*08:07, 08:14	B8		26																					
*08:09	B8		26						32															
*08:12:01-08:12:03, 08:16	B8		26						32															
*08:13, 08:20, 08:40	B8, –		26																					
*08:17, 08:54	–, –		26						32			35												
*08:21	B8		26	27					32															
*08:25	B8		26	27																				
*08:26, 08:50, 08:62, 42:07 ⁶	B8, –								32															
*08:28, 08:37	B8, –		26									36												
*08:32	B8		26						32									42						
*08:35	–		26						32			36												
*08:36	–		26						32															
*08:38	–		26						32															
*08:49	–		26															41		44				
*08:55	–		26																		45			
*08:60	–		26						32															
*08:65	–								32														47	
*13:01:01-13:01:05, 13:17, 13:20, 13:22:01-13:23, 13:25, 13:28-13:29	B13, –			27	28						34							41						
*13:02:01-13:02:09, 13:08Q, 13:15, 13:18-13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40	B13, –			27	28						34													
*13:03	B49, B15			27	28																			
*13:04, 13:10	B49, B15, B13			27	28							36												
*13:06	B13			27	28													41						
*13:07N	Null			27	28						34							41						
*13:09	–			28							34													
*13:11	B13			27	28						34													
*13:12-13:13	B13			27	28													41						
*13:14	B13			27							34													
*13:16	B13			27	28						34													
*13:21, 13:35	–			27	28																			
*13:26	–			27	28							36						41						
*13:31	–			27	28						34													
*13:36	–			27	28						34							41						

*13:39	–				28						34						41										
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47			
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47			
HLA-B allele ⁴	ser. ⁵																										
*14:01:01-14:01:02, 14:07N, 14:12, 14:14, 14:19	B64, Null, –				28	29																					47
*14:02:01-14:02:02, 14:02:04, 14:03-14:04, 14:09, 14:11, 14:15-14:18	B65, B14, –				28		30																				47
*14:02:03, 14:02:06	B65, –				28																						47
*14:02:05	B65				28		30																				47
*14:05-14:06:02, 14:13	B14, –						30																				47
*14:08, 39:25N, 39:30, 39:32-39:34, 39:47, 39:50 ⁷	B39, B16, Null, –					29																					47
*14:10	–					29												41									47
*15:01:01:01- 15:01:01:02N, 15:01:03, 15:01:06-15:01:16, 15:01:18, 15:01:20, 15:26N-15:27:03, 15:32, 15:34-15:35, 15:38:01- 15:38:02, 15:50, 15:56, 15:60, 15:70-15:71, 15:75, 15:79N, 15:81-15:82, 15:92, 15:94N, 15:96- 15:97, 15:102, 15:104- 15:105, 15:109-15:111N, 15:113, 15:117-15:118, 15:122, 15:125, 15:128- 15:129, 15:135, 15:140, 15:145-15:147, 15:152, 15:159-15:160, 15:163- 15:164, 15:166-15:167, 15:169, 15:171-15:172, 15:174-15:175, 15:178, 15:181N-15:184, 15:187, 15:190N, 15:192-15:193, 15:201, 15:203, 15:206	B62, Null, –							31				36							43								
*15:01:02	B62							31		33			36							43							
*15:01:04	B62												36							43							
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47			
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47			

[illegible]

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
HLA-B allele ⁴	ser. ⁵																							
*15:21, 15:44	B75							31					36		38			41		43				
*15:23, 15:115	B70, B5, –								32				36		38					43				
*15:24, 15:87, 15:157	B62, B15, –							31					36		38					43				
*15:29	B70								32				36							43				
*15:30, 15:58, 15:63	B62							31												43				
*15:31	B75							31					36							43				
*15:36	–			w				31					36					41		43				
*15:42	B15							31												43				
*15:43	B15							31					36							43				
*15:45	B62			27				31												43				
*15:46	–				28			31	32				36							43				
*15:48, 15:150, 15:188	B62, –							31												43				
*15:53	–				28				32				36							43				
*15:54	B72								32			35	36							43				
*15:55	B15			27				31					36							43				
*15:57	B62							31					36							43				
*15:62	–								32			35	36					41		43				
*15:64	B71								32			35	36							43				
*15:66	B62							31					36		38					43				
*15:68	–			27								35	36											
*15:69	B72								32			35	36							43		45		
*15:73	B62							31												43				
*15:74	B72								32				36							43				
*15:76	–							31					36							43				
*15:77	B62						30	31										41		43				
*15:80	B70								32				36		38			41		43				
*15:83	B62							31	32											43				
*15:85, 15:154, 15:165	B62, –							31					36					41		43				
*15:86	–							31												43				
*15:89	B77			w				31					36					41		43				
*15:91, 15:131	–, –								32			35	36	37						43				
*15:101	–							31					36											
*15:106	–				28			31	32				36					41		43				
*15:116	B62							31					36							43				
*15:124	B71								32				36		38					43				
*15:137	–							31												43				
*15:138	–	25						31					36							43				
*15:142	–		26					31					36							43				
*15:143	–							31	32				36							43				
*15:144	–							31					36					41		43				
*15:161	–								32				36	37	38					43				

*15:180	–	26								36															
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	
HLA-B allele ⁴	ser. ⁵																								
*15:186	–								32				36		38					43		45			
*15:189	–						30	31					36		38					43					
*15:202	–							31					36							42	43				
*18:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30-18:36, 18:38-18:40, 18:42-18:43, 18:45-18:47, 18:50-18:52	B18, Null, –												36		39				42						
*18:04	B18												36		39				42						
*18:09	B18												36		39										
*18:12	B18												36		39				42						
*18:19	B18												36		39				42						
*18:22	B18												36		39			41	42						
*18:26, 18:41	B18, –														39				42						
*18:29, 35:32, 35:37, 35:53N, 35:64, 35:68:01-35:68:02, 35:99, 35:118-35:119 ⁸	B35, Null, –												36						42						
*18:37	–											35	36			39									
*18:44	–				28								36			39			42						
*18:48	–												36			39									
*18:49	–												36			39									
*27:01	B27					29											40								
*27:02, 27:53, 27:57, 27:65N	B27, Null, –					29											40								
*27:03-27:06, 27:09-27:10, 27:13, 27:17, 27:20, 27:27, 27:31, 27:35-27:37, 27:45-27:46, 27:48-27:51, 27:54-27:56, 27:58, 27:60-27:61, 27:64N, 27:66N, 27:68-27:69, 27:72-27:73	B27, Null, –					29											40								
*27:07, 27:11, 27:15, 27:21, 27:28, 27:32, 27:34, 27:38, 27:43, 27:47, 27:63, 27:70-27:71	B27, –																40								

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

*44:02:01-44:02:14, 44:11, 44:19N, 44:21- 44:24, 44:27:01, 44:33- 44:34, 44:48, 44:52N, 44:55, 44:59, 44:63, 44:66- 44:68, 44:71-44:74, 44:80, 44:84, 44:86-44:87, 44:89, 44:93, 44:101-44:102, 44:104, 44:112-44:113	B44, Null, –					28													41							
*44:03:01-44:03:06, 44:03:08-44:05:02, 44:13, 44:26, 44:28:01-44:30, 44:32, 44:35-44:36, 44:38- 44:39, 44:42, 44:45, 44:51, 44:56N, 44:58N, 44:61N, 44:69-44:70, 44:76-44:79, 44:82, 44:85, 44:88, 44:92, 44:94, 44:96, 44:98, 44:105, 44:107-44:109, 44:114- 44:115	B44, Null, –					28													41							
*44:03:07, 44:07, 44:81, 44:103, 44:111	B44, –																		41							
*44:06	B44																		41							
*44:08	B44			w															41							
*44:09	B45				28														41							
*44:10	B44				28							35							41		43					
*44:12	B44				28														41							
*44:14	B44				28				32										41							
*44:15	B12				28				32			35														
*44:16, 44:91	B47, B44, –				28		30												41							
*44:17, 44:43:01-44:43:02	B44, –				28								36	37					41							
*44:18	B44				28				32			35														
*44:20, 44:100	B44, –				28				32																	
*44:25, 44:50	B44				28														41							
*44:31	B44				28																					
*44:37, 44:64:01-44:64:02	B44, –				28		30												41							
*44:40	B44											35					40		41							
*44:41:01, 44:53, 44:99	B44, –				28																					
*44:44	B44											35					40		41							
*44:46, 44:75	B12, –				28														41							
*44:47, 44:65	B44, –				28																					
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47		
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47		

[illegible]

[illegible]

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
HLA-B allele ⁴	ser. ⁵																							
*51:01:09	B51									33														
*51:04, 51:46, 51:56:01	B51, –									33								41						
*51:06:01, 51:59	B51, –									33														
*51:08, 51:20, 51:44N	B51, Null								32	33														
*51:10	B51																							
*51:15	–																							
*51:24:02	B51																							
*51:24:03-51:24:04	B51, –									33														
*51:36	B51								32															
*51:37, 51:63	B51, –									33			36											
*51:42	–																	41						
*51:45	–												36											
*51:54, 51:78	–									33														
*51:61	–									33														
*51:62	–																							
*51:64	–			27						33														
*51:68	–		26							33														
*51:81	–																							
*51:93	–																							
*51:97	–								32	33			36											
*51:101	–									33												45		
*51:104	–									33														
*51:106	–									33														
*52:01:01:01, 52:01:03, 52:01:06, 52:04-52:05, 52:07-52:08, 52:10-52:13, 52:15, 52:17-52:18, 52:22	B52, –									33														
*52:01:02, 52:01:04, 52:02-52:03, 52:06:01- 52:06:02, 52:09	B52									33														
*52:01:05, 52:14	B52, –																							
*52:16	–									33		35												
*52:19	–								32	33														
*52:20	–									33														
*52:21	–									33														
*53:01:01-53:01:05, 53:05, 53:08:01-53:08:02, 53:10, 53:16, 53:18, 53:20- 53:21, 53:23-53:24	B53, –												36					41						
*53:02	B53												36					41						
*53:03, 53:09, 53:11- 53:13	B53, B35												36					41						

[illegible]

[illegible]

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
HLA-B allele ⁴	ser. ⁵																							
*58:07	B58												36	37										
*58:08	B17, B5									33														
*58:09	B17												36					41						
*58:17N	Null												36					41						
*58:18	–			27									36											
*58:20	–																					45		
*58:27	–																							
*58:28	–																	41						
*59:01:01:01, 59:05	B59, –																							
*59:02:59:03	B59, –																							
*59:04	–																							
*67:01:01, 67:03	B67, –					29																45		
*67:01:02:67:02	B67, –																					45		
*73:01-73:02	B73, –																							
*78:01, 78:02:02-78:03, 78:07	B78, –									33									42					
*78:02:01, 78:04	B78, B35									33									42					
*78:05	–									33									42					
*78:06	–									33														
*81:01	B81																							
*81:02	B81																							
*81:03-81:05	Null, –																							
*82:01-82:03	B82, –								32			35							43					
*83:01	–																	41						
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
HLA-B allele ⁴	ser. ⁵																							
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
A*23:31, A*24:106, C*16:10							30																	
A*68:56, C*06:20, C*12:50																								
C*01:30	–				28																			
C*02:06																								
C*02:23, C*04:77		25																						
C*03:05, 03:25, 03:27				27																				
C*03:12, 03:19																								
C*03:102																		41						
C*07:46									32															
C*15:02:04																								
C*15:25																								
C*15:39																								

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47
----------	--	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----

[illegible]

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72					

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
48				52																			71	72	B7, B42, Null, –
48																							71	72	B703, B7, –
48			51	52																			71	72	B7
48				52																			71		B7, –
48				52																			71	72	B7
48				52																			71		B7, 42
48				52																			71	72	B7, –
																					70		72		–
48				52																			71	72	B7
				52																		70		72	B7, –
																					70		72		–
48				w																			71	72	B7
48				52																			71	72	B7
48																							71	72	–
48				52									62										71	72	B7
48				52																			71		–
				52			56							64			w						71	72	–
48																							71	72	–
48				52																			71	72	–
48				52						59													71		–
48				52						59													71		–
48				52															69				71		–
48																							71	72	B8, Null, –

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		
																									ser. ⁵	
			51																			70		72	B8, –	
48			51																				71	72	B8, –	
48			51																				71	72	B8	
48			51			54																	71		B8	
48			51																				71		B8	
48																							71	72	B8, –	
48			51											62									71	72	–, –	
48			51																				71	72	B8	
48																							71	72	B8	
48			51																				71	72	B8, –	
48			51																				71	72	B8, –	
48			51																68				71	72	B8	
48			51																				71	72	–	
			51																				71	72	–	
48			51											62									71	72	–	
48			51			53																	71	72	–	
48																							71	72	–	
48			51			53																	71	72	–	
48			51																				71	72	–	
														62								70			B13, –	
														62								70			B13, –	
														62			65					70			B49, B15	
														62								70			B49, B15, B13	
														62		64						70			B13	
																						70			Null	
														62									71			–
														62					67			70			B13	
														62								70			B13	
														62								70			B13	
											59			62								70			B13	
														62								70			–	
														62								70			–	
											w			62							69	70			–	
									57													70			–	

														62									71		–
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
48																							71		B64, Null, –
48																							71		B65, B14, –
48																							71		B65, –
																							71		B65
48																							71		B14, –
48																							71		B39, B16, Null, –
48							55																71		–
														62									69	71	B62, Null, –
														62									69	71	B62
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
									57														71		B75
																						70			B70, B5, –
														62							69	70			B62, B15, –
																							71		B70
														62							69		71		B62
									57														71		B75
														62								70			–
										59				62		64							71		B15
														62							69	70			B15
														62							69		71		B62
													61	62							69		71		–
														62									71		B62, –
													61	62							69		71		–
														62							69		71		B72
									57														71		B15
						w								62							69		71		B62
														62									71		–
																							71		B71
														62							69		71		B62
														62							69		71		–
														62									71		B72
														62			65		67		69		71		B62
														62									71		B72
				52					w												69		71		–
														62							69		71		B62
																							71		B70
					54									62									71		B62
														62							69		71		B62, –
														62		64							71		–
									57													70			B77
														62									71		–, –
				52										62							69		71		–
													61	62									71		–
	49													62									71		B62
	49																						71		B71
										w				62							69		71		–
														62									71		–
														62							69		71		–
													61								69		71		–
									57												69		71		–
																							71		–

																					69	71			–
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
																							71		–
									57												69		71		–
													62								69		71		–
																				68			71		B18, Null, –
															64				68			71			B18
																			68		70				B18
													62										71		B18
																			68	69		71			B18
							55												68			71			B18
																			68			71			B18, –
																				68			71		B35, Null, –
																							71		–
																			68			71			–
		50											61										71		–
																			68			71			–
				52				w					62									70			B27
				52				56			60		62									70			B27, Null, –
				52				56					62									70			B27, Null, –
				52				56					62												
				52				56					62									70			B27, –

	52				56								62								71				B2708, B27, B7, –
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
													62									71			–
48				52				56					62									70			B27
													62									70			B27, Null, –
													62									71			–
				52				56					62									70			B27
																						70			–
				52				56	57				62									70			B27
				52				56					62								69	70			B27
													62									70			B27
				52				56				60	62									70			B27
				52				56					62									71			–
				52				56					62									70			–
				52				56					62									70			–
				52				56				60	62									70			–
																					68		71		B35, Null, –
																					68		71		B35

[illegible]

														64				68			71		B35		
													61	62							71		–		
																		68			71	72	B35		
				52														w			71		B35, 22		
								55										68			71		–		
				51														68			71	72	–		
								53										68			71		–		
								53						62							70		B37, Null, –		
														62							70		B37, B27		
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
					53		55							62								70			B37, –
					53									62									71		–
														62								70		72	–
					53															68		70			B37
														62								70			B37
					53									62								70			B37, –
					53									62									71		–
					53									62								70			B37
					53									62								70	w		–
																						70			B38, –
																						70			–
														62								70			B38
														62								70			B38, –
																						70			B38
																				68		70			B38
																				68		70			B38
																						70			B38
																						70			–
																						70			–
																						70			–
					52																	70			–
48																							71		B3901, B39, Low B39, Null, –
48																							71		B3901, B39, –
48														62									71		B3902, B39
48														62									71		B3902, B39, –
48																							71		B39
48																							71		B16
48																							71		B39
48															64				67				71		B39

[illegible]

[illegible]

[illegible]

					53								62	63						70				B44, Null, –	
					53								62							70				B44, Null, –	
					53								62							70				B44, –	
					53	54					60		63				68			70				B44	
					53								62	63						70				B44	
					53							61	62	63						71				B45	
					53								62			65				70				B44	
					53									63						70				B44	
					53								62							70				B44	
						54							62			65				70				B12	
					53								62	63						70				B47, B44, –	
					53								62							70				B44, –	
						54				58			62			65				70				B44	
					53	54							62	63						70				B44, –	
					53					58	60		62							70				B44	
	49				53								62							70				B44	
													62							70				B44, –	
					53								62							70				B44	
					53								62	63						70				B44, –	
					53								62	63						70				B44	
					53							61	62								71			B12, –	
					53								62							70				B44, –	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	

[illegible]

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
48													62										71	72	B48
48													62										71		–
48													62										71	72	B48
48									57				62										71	72	B48
48									57														71	72	B48
48	49												62										71	72	–
									57				62								70		72		B48
48													62										71		–
48	49												62										71		–
										58	59		62				65					70			B49, –
										58			62				65					70			B49, –
											59		62				65					70			B49
											59		62				65					70			B49
										58	59		62									70			B49
										58	59		62									70			–
48											59		61	62			65						71		B50, –
48											59		61	62			65						71		B45
48											59			62			65						71		–
48											59		61	62			65						71		–
48											59		61	62									71		–
48													61	62			65						71		–
						54						59	60						68			70			B51, B5102, B5103, Null, –
												59	60						68			70			B51, B5102, –
						54						59	60						68			70			B51, B5102, –

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
48											59	60								68		70			B51
						54	55					60								68		70			B51, –
						54						60								68		70			B51, –
						54					59	60								68		70			B51, Null
48						54						60								68		70			B51
												60					65			68		70			–
48						54					59	60								68		70			B51
48						54					59	60								68		70			B51, –
						54					59	60								68		70			B51
						54					59	60								68		70			B51, –
					53	54						60								68		70			–
						54						60								68		70			–
											59									68		70			–
						54					w	60								68	69	70			–
						54						60					65		67	68		70			–
						54						60								68		70			–
						54					59	60								68		70			–
						54						60								68		70			–
											59	60								68		70			–
						54					59	60								68		70			–
						54					59	60								68		70			–
						54					59	60										70			–
						54					59	60					65			68		70			–
											59	60		62								70			B52, –
						54					59	60		62								70			B52
											59	60		62								70			B52, –
											59	60		62								70			–
						54					59	60		62								70			–
											59			62								70			–
						54					59	60		62							69	70			–
												60								68		70			B53, –
							55					60								68		70			B53
																				68		70			B53, B35

[illegible]

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
																									ser. ⁵
				52				56									65						71		B56, Null, –
				52				56									65		67				71		B56
				52				56													69		71		B22, 15
				52				56			59				64								71		B56
				52				56			59												71		B56
				52				56			59				64				w				71		B78
				52				56									65					70			B56
				52				56															71		B56
				52				56							64			67					71		B55
				52				56															71		B56, B22
				52		54		56									65						71		B56
																	65						71		B56
				52				56			58	59										70			–
															64								71		–
				52				56			59						65						71		–
												60						66				70			B57, –
												60						66				70			B57, –
												60						66				70			B57, Null, –
												60						66				70			B57
					53							60						66				70			B57
					53							60						66				70			B57
												60						66					71		B57
							55					60						66				70			–
					53							60						66				70			–
												60							67			70			B58, Null, –
												60							67			70			B58, –

[illegible]

48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	
----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	--

[illegible]

Well No.

Well No.
HLA-B allele ⁴
*07:02:01-07:02:20, 07:05:01-07:06, 07:10, 07:12-07:15, 07:18:01- 07:18:02, 07:21-07:23, 07:26, 07:29-07:31, 07:34 07:35, 07:39, 07:41-07:46, 07:49N, 07:52, 07:55- 07:59, 07:61-07:64, 07:66 07:68:02, 07:70-07:72, 07:74-07:76, 07:79-07:80, 07:82-07:83, 07:86-07:99, 07:101-07:117
*07:03, 07:08, 07:16, 07:32, 07:37
*07:04, 07:25
*07:07, 07:77
*07:09, 07:11, 07:17
*07:19, 07:33, 07:40, 07:53
*07:20, 07:24
*07:27
*07:28
*07:36, 07:81
*07:38
*07:47
*07:48, 07:51
*07:50
*07:54
*07:60
*07:65
*07:69, 07:85
*07:73
*07:78
*07:84
*07:100
*08:01:01-08:01:13, 08:05, 08:08N, 08:10- 08:11, 08:15, 08:18- 08:19N, 08:22-08:24, 08:27, 08:29-08:31, 08:33 08:34, 08:39, 08:41-08:48, 08:51, 08:53, 08:56-08:59, 08:61, 08:63-08:64, 08:66

Well No.
Well No.
HLA-B allele⁴
*08:02-08:03, 08:52
*08:04
*08:07, 08:14
*08:09
*08:12:01-08:12:03, 08:16
*08:13, 08:20, 08:40
*08:17, 08:54
*08:21
*08:25
*08:26, 08:50, 08:62, 42:07⁶
*08:28, 08:37
*08:32
*08:35
*08:36
*08:38
*08:49
*08:55
*08:60
*08:65
*13:01:01-13:01:05, 13:17, 13:20, 13:22:01- 13:23, 13:25, 13:28-13:29
*13:02:01-13:02:09, 13:08Q, 13:15, 13:18- 13:19, 13:27, 13:30, 13:32 13:34, 13:37-13:38, 13:40
*13:03
*13:04, 13:10
*13:06
*13:07N
*13:09
*13:11
*13:12-13:13
*13:14
*13:16
*13:21, 13:35
*13:26
*13:31
*13:36

*13:39
Well No.
Well No.
HLA-B allele⁴
*14:01:01-14:01:02, 14:07N, 14:12, 14:14, 14:19
*14:02:01-14:02:02, 14:02:04, 14:03-14:04, 14:09, 14:11, 14:15-14:18
*14:02:03, 14:02:06
*14:02:05
*14:05-14:06:02, 14:13
*14:08, 39:25N, 39:30, 39:32-39:34, 39:47, 39:50⁷
*14:10
*15:01:01:01- 15:01:01:02N, 15:01:03, 15:01:06-15:01:16, 15:01:18, 15:01:20, 15:26N-15:27:03, 15:32, 15:34-15:35, 15:38:01- 15:38:02, 15:50, 15:56, 15:60, 15:70-15:71, 15:75, 15:79N, 15:81-15:82, 15:92, 15:94N, 15:96- 15:97, 15:102, 15:104- 15:105, 15:109-15:111N, 15:113, 15:117-15:118, 15:122, 15:125, 15:128- 15:129, 15:135, 15:140, 15:145-15:147, 15:152, 15:159-15:160, 15:163- 15:164, 15:166-15:167, 15:169, 15:171-15:172, 15:174-15:175, 15:178, 15:181N-15:184, 15:187, 15:190N, 15:192-15:193, 15:201, 15:203, 15:206
*15:01:02
*15:01:04
Well No.
Well No.

HLA-B allele ⁴
*15:01:17, 15:05:01-15:06, 15:33, 15:39:01-15:40, 15:65, 15:78:01-15:78:02, 15:84, 15:107, 15:136, 15:141, 15:155, 15:179, 15:185, 15:195, 15:199
*15:01:19, 15:28, 15:120, 15:149N
*15:02:01-15:02:05, 15:88, 15:112, 15:121, 15:139, 15:170, 15:194
*15:03:01-15:03:02, 15:47, 15:49, 15:61, 15:98, 15:103, 15:123, 15:127, 15:132, 15:151, 15:156, 15:158, 15:173
*15:03:03
*15:04
*15:07:01-15:07:02, 15:126, 15:205, 15:207
*15:08, 15:11:01-15:11:03, 15:11:05, 15:15, 15:148, 15:191, 15:209N
*15:09
*15:10:01, 15:37, 15:90, 15:99, 15:133
*15:10:02
*15:11:04
*15:12, 15:14, 15:19
*15:13
*15:16:01, 15:67, 15:95
*15:17:01:01-15:17:02, 15:162, 15:168, 15:177, 15:196, 15:208
*15:18:01-15:18:02, 15:18:04, 15:51-15:52, 15:72, 15:93, 15:108, 15:114, 15:119, 15:134, 15:153, 15:176, 15:197-15:198, 15:200
*15:18:03
*15:20, 15:25:01-15:25:03, 15:204

Well No.
Well No.
HLA-B allele ⁴
*15:21, 15:44
*15:23, 15:115
*15:24, 15:87, 15:157
*15:29
*15:30, 15:58, 15:63
*15:31
*15:36
*15:42
*15:43
*15:45
*15:46
*15:48, 15:150, 15:188
*15:53
*15:54
*15:55
*15:57
*15:62
*15:64
*15:66
*15:68
*15:69
*15:73
*15:74
*15:76
*15:77
*15:80
*15:83
*15:85, 15:154, 15:165
*15:86
*15:89
*15:91, 15:131
*15:101
*15:106
*15:116
*15:124
*15:137
*15:138
*15:142
*15:143
*15:144
*15:161

*15:180
Well No.
Well No.
HLA-B allele⁴
*15:186
*15:189
*15:202
*18:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13 18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30 18:36, 18:38-18:40, 18:42 18:43, 18:45-18:47, 18:50 18:52
*18:04
*18:09
*18:12
*18:19
*18:22
*18:26, 18:41
*18:29, 35:32, 35:37, 35:53N, 35:64, 35:68:01-35:68:02, 35:99, 35:118-35:119⁸
*18:37
*18:44
*18:48
*18:49
*27:01
*27:02, 27:53, 27:57, 27:65N
*27:03-27:06, 27:09-27:10, 27:13, 27:17, 27:20, 27:27, 27:31, 27:35 27:37, 27:45-27:46, 27:48 27:51, 27:54-27:56, 27:58, 27:60-27:61, 27:64N, 27:66N, 27:68-27:69, 27:72-27:73
*27:07, 27:11, 27:15, 27:21, 27:28, 27:32, 27:34, 27:38, 27:43, 27:47, 27:63, 27:70-27:71

*27:08, 27:26, 27:40, 27:42, 27:44
Well No.
Well No.
HLA-B allele⁴
*27:12
*27:14
*27:16, 27:39, 27:59N
*27:18
*27:19
*27:23
*27:24
*27:25
*27:29
*27:30
*27:33
*27:41
*27:52, 27:67
*27:62
*35:01:01:01-35:01:09, 35:01:11-35:01:22, 35:07- 35:08:04, 35:11:01- 35:11:02, 35:15, 35:20:01 35:20:02, 35:23, 35:29, 35:40N-35:42:01, 35:48, 35:50, 35:52, 35:54, 35:57, 35:61, 35:77-35:78, 35:82, 35:90-35:94, 35:100-35:101:02, 35:103- 35:105, 35:107-35:108:02, 35:110-35:112, 35:116, 35:120-35:124, 35:126, 35:130N-35:134N, 35:137- 35:148
*35:01:10

*35:02:01-35:03:02, 35:03:04-35:04:01, 35:04:03, 35:06, 35:12:01 35:12:02, 35:33-35:34, 35:36, 35:39, 35:55, 35:59, 35:65Q, 35:70, 35:74, 35:83-35:85, 35:95, 35:98, 35:106, 35:128- 35:129N, 35:136, 35:149- 35:150
*35:03:03, 35:56
*35:04:02
Well No.
Well No.
HLA-B allele ⁴
*35:05:01-35:05:02, 35:51, 35:58, 35:72, 35:89, 35:97, 35:114
*35:09:01-35:09:02, 35:18, 35:31, 35:75, 35:88, 35:127, 35:151
*35:10, 35:28, 35:69, 35:80
*35:13
*35:14:01-35:14:02, 35:62
*35:16
*35:17, 35:30, 35:113, 35:125
*35:19
*35:21, 35:24:01-35:24:02
*35:22
*35:25, 35:49
*35:26
*35:27
*35:35
*35:38
*35:42:02
*35:43:01, 35:67, 35:79, 35:86, 35:102, 35:117, 35:135
*35:43:02
*35:44
*35:45, 35:71
*35:46
*35:47

*35:60
*35:63
*35:66
*35:76
*35:81, 35:96, 35:109
*35:87
*35:115
*37:01:01-37:01:07, 37:03N, 37:06, 37:13, 37:15-37:21, 37:23-37:25
*37:02
Well No.

Well No.
HLA-B allele ⁴
*37:04:01-37:04:02
*37:05
*37:07
*37:08
*37:09
*37:10, 37:22
*37:11
*37:12
*37:14
*38:01:01-38:02:02, 38:08
38:09, 38:11-38:16, 38:18
38:19, 38:21, 38:23-38:24
*38:02:03
*38:03
*38:04, 38:25
*38:05
*38:06
*38:07
*38:10
*38:17
*38:20
*38:22
*38:26
*39:01:01:01-
39:01:01:02L, 39:01:03-
39:01:08, 39:01:10-
39:01:12, 39:03, 39:05:01
39:06:02, 39:09, 39:11,
39:14-39:15, 39:24:01,
39:26-39:29, 39:31, 39:35,
39:37-39:38Q, 39:40N-
39:41, 39:44, 39:46,
39:48, 39:51-39:57, 39:59
39:62
*39:01:09, 39:12, 39:45
*39:02:01, 39:08
*39:02:02, 39:13:01-
39:13:02, 39:23, 39:49
*39:04
*39:07
*39:10, 39:16, 39:20
*39:17

Well No.
Well No.
HLA-B allele ⁴
*39:18
*39:19:01
*39:19:02
*39:22
*39:36
*39:39
*39:42
*39:43
*39:58
*40:01:01-40:01:06, 40:01:08-40:01:10, 40:01:12-40:01:17, 40:07, 40:51, 40:66, 40:73, 40:79, 40:124:01, 40:139- 40:140, 40:146-40:147, 40:150, 40:152-40:153, 40:155N
*40:01:07, 40:72:01, 40:77
*40:01:11
*40:02:01-40:02:11, 40:04, 40:39, 40:89, 40:94, 40:97, 40:122, 40:142N-40:144N
*40:03, 40:20
*40:05
*40:06:01:01-40:06:03, 40:70
*40:08
*40:09, 40:11:01- 40:11:02, 40:18, 40:24, 40:27:01, 40:29, 40:35, 40:37, 40:50, 40:56-40:57, 40:64, 40:78, 40:82, 40:85, 40:90-40:91, 40:99, 40:104, 40:107, 40:111, 40:115, 40:119-40:120, 40:133Q, 40:145
Well No.
Well No.
HLA-B allele ⁴

*40:10:01-40:10:02, 40:22N, 40:33, 40:36, 40:42-40:43, 40:49, 40:54 40:55, 40:61-40:63, 40:65, 40:67, 40:81, 40:84, 40:87:01-40:88, 40:92, 40:100-40:102, 40:108, 40:112-40:114, 40:116, 40:118N, 40:123, 40:124:02-40:126, 40:128, 40:130, 40:134, 40:141, 40:151, 40:154, 40:156
*40:12
*40:13
*40:14:01-40:14:03, 40:121, 40:129
*40:15, 40:32
*40:16
*40:19
*40:21
*40:23
*40:25, 40:69, 40:106
*40:26
*40:28
*40:30, 40:34
*40:31, 40:45, 40:80
*40:38, 40:52
*40:40
*40:44, 40:75, 40:86, 40:103, 40:127, 40:131
*40:46
*40:47
*40:48
*40:53, 40:72:02
*40:58
*40:59
*40:60
*40:68
*40:71
*40:74, 40:149
Well No.
Well No.

HLA-B allele ⁴
*40:76
*40:83
*40:93
*40:95
*40:96
*40:98
*40:105
*40:109
*40:110
*40:117
*40:132
*40:135
*40:136
*40:137
*40:138
*40:148
*40:157
*40:158
*41:01, 41:06-41:07, 41:12, 41:14, 41:16
*41:02:01-41:02:03, 41:11, 41:13
*41:03:01-41:03:02, 41:15
*41:04, 41:10
*41:05
*41:08
*41:09, 45:02-45:03 ⁹
*42:01:01-42:02, 42:06, 42:10, 42:12
*42:04
*42:05:01-42:05:02
*42:08
*42:09
*42:11
*42:13
*42:14
Well No.
Well No.
HLA-B allele ⁴

*44:02:01:01-44:02:14, 44:11, 44:19N, 44:21- 44:24, 44:27:01, 44:33- 44:34, 44:48, 44:52N, 44:55, 44:59, 44:63, 44:66 44:68, 44:71-44:74, 44:80, 44:84, 44:86-44:87, 44:89, 44:93, 44:101-44:102, 44:104, 44:112-44:113
*44:03:01-44:03:06, 44:03:08-44:05:02, 44:13, 44:26, 44:28:01-44:30, 44:32, 44:35-44:36, 44:38 44:39, 44:42, 44:45, 44:51, 44:56N, 44:58N, 44:61N, 44:69-44:70, 44:76-44:79, 44:82, 44:85, 44:88, 44:92, 44:94, 44:96, 44:98, 44:105, 44:107-44:109, 44:114- 44:115
*44:03:07, 44:07, 44:81, 44:103, 44:111
*44:06
*44:08
*44:09
*44:10
*44:12
*44:14
*44:15
*44:16, 44:91
*44:17, 44:43:01-44:43:02
*44:18
*44:20, 44:100
*44:25, 44:50
*44:31
*44:37, 44:64:01-44:64:02
*44:40
*44:41:01, 44:53, 44:99
*44:44
*44:46, 44:75
*44:47, 44:65
Well No.
Well No.

HLA-B allele ⁴
*44:49
*44:54
*44:57
*44:60, 44:110
*44:62
*44:83
*44:90
*44:95
*44:97
*44:106
*45:01, 45:05, 45:07, 45:11-45:13
*45:04
*45:06
*45:08
*45:09
*45:10
*46:01:01-46:01:05, 46:04 46:05, 46:07N, 46:09- 46:10, 46:14-46:16, 46:22 46:24
*46:02
*46:03, 46:08, 46:13:01- 46:13:03, 46:19, 46:21:01 46:21:02, 46:25-46:26
*46:06
*46:11
*46:12, 46:20
*46:17
*46:18
*47:01:01:01-47:01:01:02, 47:06-47:07
*47:02
*47:03
*47:04-47:05
*48:01:01-48:01:03, 48:07, 48:09, 48:11, 48:19 48:20, 48:22
*48:02:01-48:02:02
*48:03:01-48:03:02
*48:04, 48:24
*48:05, 48:08
*48:06
Well No.

Well No.
HLA-B allele ⁴
*48:10, 48:12
*48:13
*48:14
*48:15
*48:16
*48:17
*48:18
*48:21
*48:23
*49:01:01, 49:06, 49:08-49:10, 49:12-49:15
*49:01:02, 49:11
*49:02
*49:03
*49:04-49:05
*49:07
*50:01:01-50:01:02, 50:04-50:05, 50:08, 50:10-50:11
*50:02
*50:06
*50:07
*50:09
*50:12
*51:01:01, 51:01:03, 51:01:05-51:01:08, 51:01:10-51:02:01, 51:02:03, 51:03, 51:07:01, 51:09:01, 51:11N-51:12, 51:13:02-51:14, 51:16-51:19, 51:21-51:23, 51:26-51:33, 51:38-51:41N, 51:43, 51:48-51:53, 51:55, 51:57-51:58, 51:60, 51:65-51:67, 51:69-51:77, 51:79-51:80, 51:82-51:88, 51:90-51:91, 51:94-51:96, 51:98N, 51:100, 51:102, 51:105
*51:01:02, 51:02:02, 51:05, 51:07:02, 51:09:02, 51:89, 51:103
*51:01:04, 51:02:04, 51:13:01, 51:24:01, 51:34-51:35, 51:92, 51:99

Well No.
Well No.
HLA-B allele⁴
*51:01:09
*51:04, 51:46, 51:56:01
*51:06:01, 51:59
*51:08, 51:20, 51:44N
*51:10
*51:15
*51:24:02
*51:24:03-51:24:04
*51:36
*51:37, 51:63
*51:42
*51:45
*51:54, 51:78
*51:61
*51:62
*51:64
*51:68
*51:81
*51:93
*51:97
*51:101
*51:104
*51:106
*52:01:01:01, 52:01:03, 52:01:06, 52:04-52:05, 52:07-52:08, 52:10-52:13, 52:15, 52:17-52:18, 52:22
*52:01:02, 52:01:04, 52:02-52:03, 52:06:01- 52:06:02, 52:09
*52:01:05, 52:14
*52:16
*52:19
*52:20
*52:21
*53:01:01-53:01:05, 53:05, 53:08:01-53:08:02, 53:10, 53:16, 53:18, 53:20 53:21, 53:23-53:24
*53:02
*53:03, 53:09, 53:11- 53:13

Well No.
Well No.
HLA-B allele ⁴
*53:04, 53:07, 53:19
*53:06
*53:14
*53:15
*53:17:01
*53:22
*54:01:01-54:02, 54:05N, 54:07-54:08N, 54:10, 54:13, 54:16-54:19, 54:21 54:23
*54:03
*54:04, 54:15
*54:06
*54:09, 54:14
*54:11
*54:12
*54:20
*55:01:01-55:01:06, 55:03, 55:05, 55:11, 55:15, 55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:44-55:45
*55:02:01-55:02:06, 55:07, 55:10, 55:12, 55:16, 55:19, 55:26, 55:30, 55:35, 55:37, 55:39, 55:41-55:43, 55:47 55:48
*55:04, 55:08, 55:13, 55:27, 55:46, 56:15, 56:19N, 56:22 ¹⁰
*55:09, 55:24
*55:14
*55:17, 55:28
*55:18
*55:20
*55:21
*55:22
*55:23, 55:32, 56:18 ¹¹
*55:34
*55:40

Well No.
Well No.
HLA-B allele⁴
*56:01:01-56:01:04, 56:08, 56:14, 56:16, 56:20, 56:24, 56:26-56:30
*56:02, 56:04
*56:03
*56:05:01
*56:05:02
*56:06
*56:07
*56:09
*56:10
*56:11-56:12
*56:13
*56:17
*56:21
*56:23
*56:25
*57:01:01-57:01:04, 57:01:06-57:01:10, 57:06, 57:08, 57:10, 57:15-57:16, 57:18-57:20, 57:22-57:23, 57:25-57:27, 57:29-57:30, 57:33-57:37
*57:01:05, 57:11, 57:13, 57:21, 57:31
*57:02:01-57:03:02, 57:05, 57:17, 57:28N, 57:32
*57:04
*57:07
*57:09
*57:12
*57:14
*57:24
*58:01:01-58:01:02, 58:01:04-58:01:08, 58:04- 58:05, 58:10N-58:15, 58:19, 58:21-58:24, 58:29 58:31N
*58:01:03, 58:02, 58:06, 58:16, 58:25-58:26

Well No.
Well No.
HLA-B allele ⁴
*58:07
*58:08
*58:09
*58:17N
*58:18
*58:20
*58:27
*58:28
*59:01:01:01, 59:05
*59:02-59:03
*59:04
*67:01:01, 67:03
*67:01:02-67:02
*73:01-73:02
*78:01, 78:02:02-78:03, 78:07
*78:02:01, 78:04
*78:05
*78:06
*81:01
*81:02
*81:03-81:05
*82:01-82:03
*83:01
Well No.
HLA-B allele ⁴
Well No.
A*23:31, A*24:106, C*16:10
A*68:56, C*06:20, C*12:50
C*01:30
C*02:06
C*02:23, C*04:77
C*03:05, 03:25, 03:27
C*03:12, 03:19
C*03:102
C*07:46
C*15:02:04
C*15:25
C*15:39

Well No.
