

																		5' -CCA ^{3'}										
																		5' -CCg ^{3'}	527									
																		5' -CCg ^{3'}										
																		530										
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96				

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ⁴																								
*01:02:01-01:02:15, 01:06-01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66	73																			92				
*01:03, 01:24, 01:34	73																			92	93			
*01:04	73											84		86										
*01:05, 01:22, 01:35-01:36, B*54:18 ⁶	73																							
*01:09	73													86										
*01:10	73	74																		92				
*01:14, 01:59	73											83								92				
*01:21	73											84	85											
*01:43	73	74									82									92				
*01:49-01:50, 01:55	73																				93			
*01:60	73																		91	92				
*02:02:01		74	75			78						83			w									
*02:02:02-02:02:03, 02:02:05-02:02:12, 02:02:14-02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:23-02:25Q, 02:28-02:32, 02:34-02:38N, 02:40, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56		74	75			78						83			87									
*02:02:13, 02:21, 02:26:02-02:26:03, 02:39, 02:45		74	75									83			87									
*02:04		74	w			78						83			87									
*02:05, 02:17		74	75			78						83		86	87									
*02:06		74	75			78	80					83			87		89							
*02:12		74	75			78							85		87									
*02:13, 02:18, 02:33		74	75			78						83			87		90							
*02:14		74	w			78						83												
*02:22		74	75			78						83			87						93			
*02:26:01		74										83			87									
*02:27:01-02:27:02		74	75			78																		
*02:47		74	75			78	80					83			87									
*02:49		74	75			78						83	85	87			90							
*02:55		74	75			78						83	85	87										
*03:02:01-03:02:09, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146			75	76																	94			
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ⁴																								

Negative Control

[illegible]

[illegible]

*06:02:01:01-06:02:01:02, 06:02:03-06:02:11, 06:02:13- 06:02:15, 06:02:17-06:02:19, 06:07, 06:10, 06:12-06:13, 06:15- 06:16N, 06:18-06:30, 06:33- 06:34, 06:36-06:39, 06:41-06:51, 06:53-06:71, 06:73-06:75								80				83												95	Negative Control
*06:02:12, 06:11, 06:52								80						86										95	
*06:02:16, 06:04, 06:40								80				83												95	
*06:03:01				75				80				83			86	87							94		
*06:03:02, 06:76:01								80				83			86	87									
*06:05								80				83						90		92				95	
*06:06								80				83								92				95	
*06:08				74				80				83			86									95	
*06:09								80				83			86							93		95	
*06:14, 06:35, 06:72								80				83										93		95	
*06:17												83			86					92				95	
*06:31								80				83			86			90		92				95	
*06:32												83			86									95	
*07:01:01:01-07:02:07, 07:02:10- 07:02:28, 07:02:30-07:06, 07:08, 07:11-07:25, 07:27:01-07:27:02, 07:29-07:30, 07:32N, 07:35- 07:38:02, 07:42, 07:44-07:48, 07:50-07:63, 07:65-07:75, 07:77- 07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126- 07:138, 07:140-07:147, 07:149- 07:151, 07:153-07:155, 07:157- 07:160, 07:162-07:172, 07:174- 07:176, 07:178-07:183, 07:185- 07:195, 07:197-07:209, 07:211- 07:222, 07:224-07:235												81												95	
*07:02:08, 07:02:29, 07:26, 07:33N, 07:39-07:40, 07:92, 07:139, 07:152N, 07:156												81													
*07:02:09							78					81			85			90						95	
*07:07, 07:09, 07:49, 07:76												81	83							92				95	
*07:10, 07:28, 07:41, 07:43, 07:184, 07:196												81										93		95	
*07:31												81			86			90						95	
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96	
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96	
HLA-C allele ⁴																									
*07:64									81							88									
*07:96				75					81													94			
*07:101, 07:148, 07:161				74					81	82														95	
*07:123, 07:173									81								89							95	
*07:125							78		81				85	86										95	
*07:177									81				86				90								

*07:210									81	83						92								
*07:223									81										w					
*08:01:01-08:01:05, 08:03:01-08:03:02, 08:06, 08:08, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56-08:60									82							90		93						
*08:02:01-08:02:07, 08:04:01-08:04:02, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N									82									93						
*08:05, 08:25									82			85						93						
*08:07, 08:47									82							91		93						
*08:09, 08:11									82							90								
*08:10						79			82	83			87			90		93						
*08:12									82								92	93						
*08:21									82			85				90		93						
*08:31		74							82									93						
*08:48									82															
*12:02:01-12:02:09, 12:10:01-12:10:02, 12:17-12:18:01, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74											84	85												
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:17-12:03:19, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:76											84	85	86											
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ⁴																								
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:26, 12:59													85	86										
*12:03:04												84		86										
*12:03:09							80					84		86										
*12:03:13												84	w	86										
*12:04:01, 12:33												83			86	87								
*12:04:02, 12:60												83	84	85	86	87								
*12:05												83	84		86	87								
*12:08												84	85				89							
*12:09												83			87				92					

Negative Control

Negative Control

*12:14:01													85					90	91					Negative Control
*12:14:02												84	85					90	91					
*12:15							80					84	85	86			89							
*12:16												84	85									95		
*12:18:02, B*67:02 ⁶													85											
*12:21										83	84				87									
*12:24												84	85							92				
*12:28, 12:58												84	85	86				90						
*12:31												84	85	86								93		
*12:41										83	84	85			87									
*12:44												84	85									93		
*12:49												84												
*12:54										83	84	85	86											
*12:55												84	85	86		88								
*12:72												84	85		87									
*14:02:01-14:02:07, 14:02:09-14:03, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:24:01, 14:26-14:27, 14:29-14:37																88				92				
*14:02:08												84				88				92				
*14:04										83						88				92				
*14:06, 14:08-14:09, 14:28, A*30:62 ⁷																88								
*14:10																88		90		92				
*14:12										83										92				
*14:15														86		88		90						
*14:16														86		88								
*14:17																88			91	92				
*14:25		74														88				92		94		
*15:02:01-15:02:09, 15:08, 15:10:02, 15:13, 15:15, 15:17-15:18, 15:26, 15:28, 15:31-15:35, 15:38-15:39, 15:41, 15:44-15:45, 15:47-15:53, 15:56-15:57, 15:60			75					80			83				87		89				93			
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ⁴																								
*15:02:10			75					80			83				87		89				93	94		
*15:03, 15:16			75					80			83		85		87		89				93			
*15:04-15:06:03, 15:09, 15:19-15:20, 15:24, 15:27, 15:29-15:30, 15:40, 15:46, 15:54, 15:59			75								83				87		89				93			
*15:07			75					80									89				93			
*15:10:01								80			83				87		89				93			
*15:11			75			78		80			83				87		89				93			
*15:12					77			80			83				87		89	90			93			
*15:21			75					80							87		89				93			
*15:22, 15:55			75								83				87		89							
*15:23											83						89				93			
*15:25			75										85					90	91		93			

Negative Control

*15:36						78					83					89				93				
*15:37, 15:58			75					80			83			87		89								
*15:42		74	75					80			83			87		89				93				
*15:43			75					80								89				93	94			
*16:01:01-16:01:07, 16:06-16:08, 16:10-16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39, 16:41, 16:44													85				90					95		
*16:02:01-16:02:09, 16:09, 16:12, 16:19, 16:25										83		85		87			90					95		
*16:04:01, 16:33, 16:42													86				90		92					
*16:15:02											84	85					90					95		
*16:18, 16:37												85		87			90					95		
*16:20													85				89	90					95	
*16:29		74												86				90		92				
*16:34			75			78							85					90					95	
*16:35														86				90						
*16:40						79										88								
*16:43													85										95	
*16:45													85					90			93		95	
*17:01:01-17:01:03, 17:01:05-17:06, 17:08-17:11		74									83			87				91		93				
*17:01:04		74									83		85	87				91		93				
*17:07											83			87				91		93				
*18:01-18:02, 18:04-18:05											83								92	93			95	
*18:03		74									83								92	93	94		95	
A*24:96, A*24:106, A*24:174, B*08:16, B*46:25																							95	
B*14:03																				92				
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96