

[illegible]

*15:08, 15:11:01-15:11:03, 15:11:05, 15:15, 15:148, 15:191, 15:209N, 15:215	B75(15) , B62(15) , -							31				36					43								
*15:09	B70							32	33					38			43								
*15:10:01, 15:37, 15:90, 15:99, 15:133, 15:229, 15:252	B71(70) , B70, -							32						38			43								
*15:10:02	B71(70)							31	32					38			43								
*15:11:04	B75(15)											36					43								
*15:12, 15:14, 15:19	B76(15)							31				36	37				43								
*15:13:01-15:13:02	B77(15) , -							31				36		38		41	43								
*15:16:01-15:16:02, 15:67, 15:95, 15:254	B63(15) , -									33		36		38			43								
*15:16:03	-									33		36		38			43								
*15:17:01:01-15:17:02, 15:162, 15:168, 15:177, 15:196, 15:208, 15:216	B63(15) , -							31		33				38			43								
*15:18:01-15:18:02, 15:18:04- 15:18:05, 15:51-15:52, 15:72, 15:93, 15:108, 15:114, 15:119, 15:134, 15:153, 15:176, 15:197- 15:198, 15:200, 15:221, 15:226N, 15:238, 15:263	B71(70) , B70, B15, Null, -								32			36		38			43								
*15:18:03	B71(70)								32			36		38			43								
*15:20	B62(15)							31				36				41	43								
*15:21, 15:44	B75(15) , -							31				36		38		41	43								
*15:23, 15:115	-								32			36		38			43								
*15:24, 15:87, 15:157	B62(15) , -							31				36		38			43								
*15:25:01-15:25:03, 15:204, 15:240, 15:250	B62(15) , -							31				36				41	43								
*15:29	B15								32			36					43								
*15:30, 15:58, 15:63	B62(15) , -							31									43								
*15:31	B75(15)							31				36					43								
*15:36	-			w				31				36				41	43								
*15:42	-							31									43								
*15:43, 15:256	-							31				36					43								
*15:45	B62(15)			27				31									43								
*15:46	B72(70)				28			31	32			36					43								
*15:48, 15:150, 15:188, 15:248	B62(15) , -							31									43								
*15:53, 15:212	-				28				32			36					43								
*15:54	-								32			35	36				43								
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
HLA-B allele ⁴	ser. ⁵																								
*15:55	B15			27				31				36						43							
*15:57	-							31				36						43							
*15:62	-								32			35	36				41	43							
*15:64:01-15:64:02	-								32			35	36					43							

*15:66	-							31				36	38			43									
*15:68	B35		27									35	36												
*15:69	-								32			35	36			43	45								
*15:73	B62(15)							31								43									
*15:74	-								32			36				43									
*15:76	-							31				36				43									
*15:77	-						30	31							41	43									
*15:80	B70								32			36	38		41	43									
*15:83	-							31	32							43									
*15:85	-							31				36			41	43									
*15:86	-							31								43									
*15:89	-		w					31				36			41	43									
*15:91, 15:131	-								32			35	36	37		43									
*15:101	-							31				36													
*15:106	-			28				31	32			36			41	43									
*15:116	-							31				36				43									
*15:124	-								32			36	38			43									
*15:137	-							31								43									
*15:138	-	25						31				36				43									
*15:142	-		26					31				36				43									
*15:143	-							31	32			36				43									
*15:144	-							31				36			41	43									
*15:154, 15:165	-							31				36			41	43									
*15:161	-								32			36	37	38		43									
*15:180	-		26									36													
*15:186	-								32			36	38			43	45								
*15:189	-						30	31				36	38			43									
*15:194	-							31				36			41	43									
*15:202, 15:239	-							31				36			42	43									
*15:222	-									33		36	38			43									
*15:224	-							31								43									
*15:230	-	25						31	33				38			43									
*15:233	-						30	31								43									
*15:236	-							31				36													
*15:241	-	25						31				36				43									
*15:243	-								32	33			38			43									
*15:253	-								32		35					43									
*15:255	-								32			36													
*15:259	-							31	32			35	36			43									
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
HLA-B allele ⁴	ser. ⁵																								
*18:01:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30-18:36, 18:38-18:40, 18:42-18:43, 18:45-18:47, 18:50-18:53, 18:55, 18:57-18:60, 18:62-18:65, 18:68, 18:70-18:71, 18:73-18:78, 18:80-18:83	B18, Null, -												36		39		42								

*27:29	B27					29						35				40									
*27:30	B27															40	41								
*27:33	-															40									
*27:41	-					29							36			40									
*27:52, 27:67	-					29																			
*27:62	-															40									
*27:75	-					29																			
*27:77	-					29										40									
*35:01:01:01-35:01:09, 35:01:11-35:01:36, 35:07-35:08:04, 35:08:06, 35:11:01-35:11:03, 35:15, 35:20:01-35:20:02, 35:23, 35:29:01-35:29:02, 35:40N-35:42:01, 35:48, 35:50, 35:52, 35:54, 35:57, 35:61:01-35:61:02, 35:77-35:78, 35:82, 35:90-35:94, 35:100-35:101:02, 35:103-35:105, 35:107-35:108:02, 35:110-35:112, 35:116, 35:120-35:124, 35:126, 35:130N-35:134N, 35:137-35:148, 35:158-35:159, 35:161, 35:165N-35:166, 35:168, 35:170-35:171, 35:173N, 35:175-35:178, 35:180, 35:187, 35:189, 35:192, 35:194, 35:196-35:197, 35:200, 35:202, 35:206, 35:208-35:209, 35:212, 35:215-35:216N, 35:219	B35, Null, -												36				41	42							
*35:01:10	B35										33		36				41	42							
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
HLA-B allele ⁴	ser. ⁵																								
*35:02:01-35:03:02, 35:03:04-35:04:01, 35:04:03, 35:06, 35:12:01-35:12:03, 35:33-35:34, 35:36, 35:39, 35:55, 35:59, 35:65Q, 35:70, 35:74, 35:83-35:85, 35:95, 35:98, 35:106, 35:128-35:129N, 35:136, 35:149-35:150, 35:152-35:153, 35:155-35:156, 35:160, 35:162, 35:167, 35:172, 35:179, 35:181-35:184, 35:193, 35:195, 35:201, 35:204, 35:211, 35:214, 35:220	B35, Null, -																	41	42						
*35:03:03, 35:56	B35, -																	41							
*35:04:02	B35									33								41	42						
*35:05:01-35:05:03, 35:51, 35:58, 35:72, 35:89, 35:97, 35:114	B35, -			27									36						42						
*35:08:05	-												36					41							

[illegible]

*37:07	-	25												37						44					
*37:08	B37													37				41		44					
*37:10, 37:22, 37:29	B37, -																	41		44					
*37:11, 37:37	-																	41	42	44					
*37:12	-								32					37						44					
*37:14	-																	41		44					
*37:28	-												35	37				41		44					
*37:34	-																	41		44					
*37:35	-													36	37			41		44					
*38:01:01-38:02:02, 38:02:04, 38:08-38:09, 38:11-38:16, 38:18-38:19, 38:21, 38:23-38:24, 38:27-38:29, 38:31-38:32, 38:34N, 38:38-38:42	B38(16), B16, Null, -							29													45	46	47		
*38:02:03	-																			45		47			
*38:03	B16							29					35							45	46				
*38:04, 38:25	-																			45		47			
*38:05, 38:33	B38(16), -							30												45	46	47			
*38:06	-																			45					
*38:07	-							29												45					
*38:10, 38:36-38:37	-							29														46	47		
*38:17	-							29						37						44	45	46	47		
*38:20	-							29										41			45	46	47		
*38:22	-							29									40					46	47		
*38:26	-							29													45	46			
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
HLA-B allele ⁴	ser. ⁵																								
*38:30	-					29			32													46	47		
*38:35	-					29																45		47	
*39:01:01:01-39:01:01:03, 39:01:03-39:01:08, 39:01:10-39:01:14, 39:03, 39:05:01-39:06:02, 39:09, 39:11, 39:14-39:15, 39:24:01, 39:26-39:29, 39:31, 39:35, 39:37-39:38Q, 39:40:01N-39:41, 39:44, 39:46, 39:48, 39:51-39:57, 39:59-39:62, 39:64-39:68, 39:70-39:71, 39:73, 39:75, 39:77, 39:79	B3901, B39(16), Low B39, Null, -							29														45		47	48
*39:01:09, 39:12, 39:45, 39:72, 39:76, 39:78	B3901, B39(16), -																					45		47	48
*39:01:15	-					29																45		47	48
*39:02:01, 39:08	B3902, B39(16)					29						35										45			48
*39:02:02, 39:13:01-39:13:02, 39:23, 39:49	B3902, B39(16), -											35										45			48
*39:04	B39(16)							30														45		47	48
*39:07	-					29							36									45		47	48

[illegible]

[illegible]

[illegible]

[illegible]

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
HLA-B allele ⁴	ser. ⁵																								
*44:31	B44(12)				28																				
*44:37:01-44:37:02, 44:64:01-44:64:02	-				28		30											41							
*44:40, 44:130	B44(12), -											35					40	41							
*44:41:01-44:41:02, 44:53, 44:99, 44:138Q	B44(12), -				28																				
*44:44	-											35					40	41							
*44:46, 44:75, 44:129	-				28													41							
*44:47, 44:65	-				28																				
*44:49	-		26		28													41							
*44:54	B44(12)			27	28																				
*44:57	-			w														41							
*44:60, 44:110	-			27	28													41							
*44:62	-				28				32	33								41							
*44:83, 44:145	-																	41							
*44:90	-				28													41							
*44:95	-				28													41							
*44:97	-				28													41							
*44:106	-			27	28																				
*44:127	-				28													41							
*44:131	-																	41							
*44:140	-				28							35						41							
*44:146	-				28													41							
*44:150	-	25			28		30											41							
*44:156	-		26									35						41							
*45:01, 45:05, 45:07, 45:11-45:14	B45(12), -				28				32			35													48
*45:04	-				28				32																48
*45:06	-				28				32			35													48
*45:08	-				28				32																48
*45:09	-				28				32				36	37											48
*45:10	-				28				32		34														48
*46:01:01-46:01:06, 46:04-46:05, 46:07N, 46:09-46:10, 46:14-46:16, 46:20, 46:22-46:24, 46:26-46:31, 46:34-46:36, 46:38	B46, Null, -								31				36												
*46:01:07	-								31				36												
*46:02	B46								31				36												
*46:03, 46:08, 46:13:01-46:13:03, 46:19, 46:21:01-46:21:02, 46:25	B46, -								31				36												
*46:06	-								31				36												
*46:11	-								31																
*46:12	-			27					31				36												
*46:17	-								31				36	37											
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

																		B8, Null, -	*08:01-01-08:01:20, 08:05, 08:08N, 08:10-08:11, 08:15, 08:18-08:19N, 08:22-08:24, 08:27, 08:29-08:31, 08:33-08:34, 08:39, 08:41-08:48, 08:51, 08:53:01-08:53:02, 08:56:01-08:59:01, 08:61, 08:63-08:64, 08:66-08:69, 08:71-08:73, 08:75, 08:80-08:83, 08:86N, 08:90-08:93, 08:95-08:100, 08:104-08:105						
																		B8, -	*08:02-08:03, 08:52, 08:78						
																		-	*08:04						
																		B8, -	*08:07, 08:14, 08:74, 08:77						
																		B8, -	*08:09, 08:84						
																		B8, -	*08:12:01-08:12:03, 08:16						
																		-	*08:13, 08:20, 08:40, 08:70						
																		-	*08:17, 08:54						
																		-	*08:21						
																		-	*08:25						
																		-	*08:26, 08:50, 08:62, 08:85, 08:94, 42:07 ⁶						
																		-	*08:28, 08:37						
																		-	*08:32						
																		-	*08:35						
																		-	*08:36						
																		-	*08:38, 08:101						
																		-	*08:49						
																		-	*08:55						
																		-	*08:60, 08:76						
																		-	*08:65						
																		-	*08:79						
																		-	*08:87						
																		-	*08:88, 08:102						
																		-	*08:89						
																		-	*08:103						
																		B13, Null, -	*13:01:01-13:01:06, 13:17, 13:20, 13:22:01-13:23, 13:25, 13:28-13:29, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N						
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																								ser. ⁵	HLA-B allele ⁴
																								B13, Null, -	*13:02:01-13:02:05, 13:02:07-13:02:15, 13:08Q, 13:15, 13:18-13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:47, 13:49N, 13:54-13:56N

50													62							70			B13	*13:02:06	
													62		65					70			-	*13:03	
													62							70			B15, B21, -	*13:04, 13:10	
													62	64						70			-	*13:06	
																				70			Null	*13:07N	
													62							71			-	*13:09	
													62			67				70			-	*13:11	
													62							70			B13, -	*13:12:01-13:13, 13:51	
													62							70			-	*13:14	
								59					62							70			-	*13:16	
													62							70			-	*13:21, 13:35, 13:59, 44:135, 44:158 ⁷	
50													62							70			-	*13:26	
								w					62						69	70			-	*13:31	
								57												70			-	*13:36	
													62							71			-	*13:39	
													62						69	70			-	*13:41	
				54									62							70			-	*13:46	
								59					62		65					70			-	*13:48	
													62	64						70			-	*13:53	
													62							70			-	*13:58	
								59					62							70			-	*13:62	
																				71		B64(14), Null, -	*14:01:01-14:01:02, 14:07N, 14:12, 14:14, 14:19, 14:26, 14:32		
																				71		B65(14), B14, -	*14:02:01-14:02:02, 14:02:04, 14:02:07-14:04, 14:09, 14:11, 14:15-14:18, 14:20, 14:22-14:23, 14:25, 14:27, 14:29, 14:31, 14:33		
																				71		B65(14), -	*14:02:03, 14:02:06, 14:24, 14:28		
																				71		B65(14), -	*14:02:05, 14:30		
																				71		B14, -	*14:05-14:06:02, 14:13		
																				71		B39(16), Null, -	*14:08, 39:25N, 39:30, 39:32-39:34, 39:47, 39:50, 39:74 ⁸		
								55												71			-	*14:10	
			52																	71			-	*14:21	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																								ser. ⁵	HLA-B allele ⁴

[illegible]

[illegible]

												62						69	71	-	*15:66				
												62						69	71	B35	*15:68				
												62							71	-	*15:69				
												62		65	67			69	71	B62(15)	*15:73				
												62							71	-	*15:74				
		52						w										69	71	-	*15:76				
												62						69	71	-	*15:77				
																			71	B70	*15:80				
				54								62							71	-	*15:83				
50												62						69	71	-	*15:85				
												62	64						71	-	*15:86				
								57										70		-	*15:89				
												62							71	-	*15:91, 15:131				
		52										62						69	71	-	*15:101				
											61	62							71	-	*15:106				
49												62							71	-	*15:116				
49																			71	-	*15:124				
								w				62						69	71	-	*15:137				
												62							71	-	*15:138				
												62						69	71	-	*15:142				
											61							69	71	-	*15:143				
								57										69	71	-	*15:144				
												62						69	71	-	*15:154, 15:165				
																			71	-	*15:161				
																		69	71	-	*15:180				
																			71	-	*15:186				
								57										69	71	-	*15:189				
50								57											71	-	*15:194				
												62						69	71	-	*15:202, 15:239				
																		70		-	*15:222				
												62	64		67				71	-	*15:224				
																		70		-	*15:230				
												62						69	71	-	*15:233				
												62						69	71	-	*15:236				
												62						69	71	-	*15:241				
												62							71	-	*15:243				
												62							71	-	*15:253				
		52										62							71	-	*15:255				
												62						69	71	-	*15:259				
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																								ser. ⁵	HLA-B allele ⁴
																		68			71			B18, Null, -	*18:01:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:18, 18:20-18:21, 18:23N-18:25, 18:27-18:28, 18:30-18:36, 18:38-18:40, 18:42-18:43, 18:45-18:47, 18:50-18:53, 18:55, 18:57-18:60, 18:62-18:65, 18:68, 18:70-18:71, 18:73-18:78, 18:80-18:83

--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--

[illegible]

[illegible]

																			71		B39(16)	*39:10:01-39:10:02, 39:16, 39:20				
													64			67			71		-	*39:17				
																		69	71		-	*39:18				
																			68	71	B39(16)	*39:19:01				
																			68	71	B39(16)	*39:19:02				
													62							71	-	*39:22				
																				71	-	*39:24:02				
																			69	71	-	*39:36				
																				71	-	*39:39:01				
																				71	-	*39:39:02				
																				71	-	*39:42				
																				71	-	*39:43				
																				71	-	*39:58				
																				71	-	*39:63				
													64							71	-	*39:69				
													61							71	-	*39:69				
49													61	62							71	B60(40) , Null, -	*40:01:01-40:01:06, 40:01:08-40:01:10, 40:01:12-40:01:17, 40:01:19-40:01:29, 40:07, 40:51, 40:66, 40:73, 40:79, 40:124:01, 40:139-40:140, 40:146-40:147, 40:150, 40:152-40:153, 40:155N, 40:172, 40:186			
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	HLA-B allele ⁴
													61	62								71	B60(40) , -	*40:01:07, 40:01:18, 40:72:01, 40:77		
49													61	62		64						71	-	*40:01:11		
	50												61	62								71	B61(40) , B41, Null, -	*40:02:01-40:02:17, 40:04, 40:06:05, 40:39, 40:89, 40:94, 40:97, 40:122, 40:142N-40:144N, 40:214		
	50												61	62								71	B61(40)	*40:03, 40:20		
	50												61	62								71	B4005	*40:05		
	50												61	62								71	B61(40) , -	*40:06:01:01-40:06:04, 40:06:06, 40:70		
	50												61									71	-	*40:08		
													61	62								71	B61(40) , B40, -	*40:09, 40:11:01-40:11:02, 40:18, 40:24, 40:27:01-40:27:02, 40:29, 40:35, 40:37, 40:50, 40:56-40:57, 40:64, 40:78, 40:82, 40:85, 40:90-40:91, 40:99, 40:104, 40:107, 40:111, 40:115, 40:119-40:120, 40:133Q, 40:145, 40:169, 40:173, 40:176, 40:180-40:181, 40:189, 40:200, 40:202-40:203, 40:205-40:206, 40:211, 40:220, 40:224-40:226		

49											61	62						71	B60(40)	*40:10:01						
49	?										61	62						71	-	*40:10:02, 40:135						
49												62						71	-	*40:12						
	50							58	60			62						70	-	*40:13						
	?										61	62						71	B60(40), -	*40:14:01-40:14:03, 40:121, 40:129, 40:164, 40:192, 40:197, 40:219						
	?										61	62						71	72	-	*40:15, 40:32					
											61	62						71	72	B61(40)	*40:16					
	?							58	60			62						70	-	*40:19						
49												62						71	-	*40:21						
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	HLA-B allele ⁴
49	?											61	62									71	B60(40), Null, -	*40:22N, 40:33, 40:36, 40:42-40:43, 40:49, 40:54-40:55, 40:61, 40:63, 40:65, 40:67, 40:81, 40:84, 40:87:01-40:88, 40:92, 40:100-40:102, 40:108, 40:112-40:114, 40:116, 40:118N, 40:123, 40:124:02-40:126, 40:128, 40:130, 40:134, 40:141, 40:151, 40:154, 40:156, 40:163, 40:168, 40:171, 40:175, 40:179, 40:182, 40:187, 40:191, 40:193-40:196, 40:199, 40:204, 40:207, 40:212-40:213, 40:216N-40:218, 40:221, 40:223		
49												61	62									71	-	*40:23		
49	?											61										71	-	*40:25, 40:69, 40:106		
	?											61	62									71	B21	*40:26		
	?				55							61	62									71	-	*40:28		
49	?											61	62									71	72	B60(40), -	*40:30, 40:34, 40:160:01-40:160:02	
	?							57				61	62									71	72	B60(40), -	*40:31, 40:45, 40:80	
49	?											61	62									71	B60(40), -	*40:38, 40:52, 40:222		
												61	62									71	-	*40:40		
	?											61	62									71	-	*40:44, 40:75, 40:86, 40:103, 40:127, 40:131, 40:162, 40:165, 40:167, 40:190		
49												62										71	-	*40:46		
49	?											62								70		B40, -	*40:47, 40:188			
49	?											61	62					67				71	B60(40)	*40:48		
	?											61	62									71	B61(40), -	*40:53, 40:72:02		
	?											61	62		64							71	-	*40:58		
49												61	62									71	72	-	*40:59	
49												61	62									71	-	*40:60		
	?											61										71	-	*40:68		
	?											61	62			65	67					71	-	*40:71		

49	?									61	62						71	-	*40:74, 40:149, 40:210						
49	?										62							-	*40:76						
										61	62						71	-	*40:83, 40:177						
											62						71	-	*40:93						
	?							59		61	62						71	-	*40:95						
	?										62						70	-	*40:96						
	?									61	62						71	72	*40:98						
	?									61	62						71	-	*40:105						
	?							58	60		62						70	-	*40:109						
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																								ser. ⁵	HLA-B allele ⁴
	?												62								70			-	*40:110
49	?								58	60			62								70			-	*40:117
49	?			53								61	62								71			-	*40:132, 40:178
	?	51										61	62								71			-	*40:136
49													62								71	72		-	*40:137
49												61	62								71			-	*40:138
	?									59		61	62								71			-	*40:148, 40:161
	?												62								70			-	*40:157, 40:201
	?											61	62								71			-	*40:158
	?											61	62								71			-	*40:159
49	?		52									61									71			-	*40:166
49												61	62								71			-	*40:170
	?											61	62								71			-	*40:174
49	?											61	62								71			-	*40:183, 40:198
	?											61	62								71			-	*40:184
49													62								71			-	*40:185
49	?												62								71			-	*40:208
													62								71			-	*40:209
49												61	62								71			-	*40:215
		51		54								61	62								71		B41, -	*41:01, 41:06-41:07, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25	
		51										61	62								71	72	B41, -	*41:02:01-41:02:04, 41:11, 41:13, 41:19, 41:23	
		51										61	62								71		B41, -	*41:03:01-41:03:02, 41:15	
		51										61	62								71	72	-	*41:04, 41:10, 41:18	
		51		54								?	62								71		-	*41:05	
		51										61	62								71		-	*41:08	
				54								61	62								71		-	*41:09, 45:02-45:03 ¹⁰	
		51										61	62								71	72	-	*41:24	
		51	52																		71	72	B42, -	*42:01:01-42:02, 42:06, 42:10, 42:12, 42:15, 42:17	
		51	52		54																71		-	*42:04	
		51	52																		71	72	-	*42:05:01-42:05:02	
			52																		71		-	*42:08	
		51	52																		71	72	-	*42:09	
		51											62								71	72	-	*42:11	
		51	52	53																	71	72	-	*42:13	

		51	52															71	-	*42:14							
		51	52															71	72	-	*42:16						
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.		
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.		
																									ser. ⁵	HLA-B allele ⁴	
				53								62	63							70				B44(12) , Null, -	*44:02:01:01-44:02:21, 44:02:23-44:02:26, 44:11, 44:19N, 44:21-44:24, 44:27:01-44:27:02, 44:33-44:34:02, 44:48, 44:52N, 44:55, 44:59:01-44:59:02, 44:63, 44:66-44:68, 44:71-44:74, 44:80, 44:84, 44:86-44:87, 44:89, 44:93, 44:101-44:102, 44:104, 44:112-44:113, 44:116, 44:118-44:119, 44:121, 44:126:01-44:126:02, 44:137, 44:142Q, 44:148-44:149N, 44:151-44:152, 44:162		
				53								63								70				B44(12) , -	*44:02:22, 44:12		
				53								62								70				B44(12) , B21, Null, -	*44:03:01-44:03:06, 44:03:08-44:03:15, 44:03:18, 44:04-44:05:04, 44:13, 44:26, 44:28:01-44:30, 44:32, 44:35-44:36, 44:38-44:39, 44:42, 44:45, 44:51, 44:56N, 44:58N, 44:61N, 44:69-44:70, 44:76-44:79, 44:82, 44:85, 44:88, 44:92, 44:94, 44:96, 44:98, 44:105, 44:107-44:109, 44:114-44:115, 44:117, 44:120, 44:122-44:125, 44:128, 44:133, 44:136, 44:139, 44:141, 44:143, 44:147, 44:153-44:155, 44:157, 44:159-44:161, 44:163-44:165		
				53								62								70				B44(12) , -	*44:03:07, 44:03:16, 44:03:19, 44:07, 44:81, 44:103, 44:111, 44:134		
												62								70				-	*44:03:17		
				53	54					60			63					68		70				B44(12)	*44:06		
				53									62	63						70				B44(12)	*44:08		
				53								61	62	63							71			B45(12)	*44:09		
				53									62			65				70				B44(12)	*44:10		
				53									62							70				B44(12)	*44:14		
					54								62			65				70				B12	*44:15		
				53									62	63						70				B47, -	*44:16, 44:91, 44:132		
				53									62							70				B44(12) , -	*44:17, 44:43:01-44:43:02, 44:144		
					54				58				62			65				70				-	*44:18		
				53	54								62	63						70				-	*44:20, 44:100		
				53					58	60			62							70				-	*44:25, 44:50		

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																								ser. ⁵	HLA-B allele ⁴
49				53								62									70			B44(12)	*44:31
												62									70			-	*44:37:01-44:37:02, 44:64:01-44:64:02
				53								62									70			B44(12)	*44:40, 44:130
				53								62	63								70			B44(12)	*44:41:01-44:41:02, 44:53, 44:99, 44:138Q
				53								62	63								70			-	*44:44
				53								61	62									71		-	*44:46, 44:75, 44:129
				53								62									70			-	*44:47, 44:65
				53								62	63								70			-	*44:49
				53								62									70			B44(12)	*44:54
				53								62									70			-	*44:57
				53								62									70			-	*44:60, 44:110
				53								62									70			-	*44:62
				53								62	63								70			-	*44:83, 44:145
			52	53			56					61	62	63								71		-	*44:90
				53					58		60		62	63							70			-	*44:95
			52	53			56					62	63								70			-	*44:97
				53								62	63								70			-	*44:106
				53								62	63											-	*44:127
				53								61	62	63								71		-	*44:131
				53								62				65					70			-	*44:140
				53								62								69	70			-	*44:146
												62									70		72	-	*44:150
				53								62									70			-	*44:156
					54							61	62			65						71		B45(12)	*45:01, 45:05, 45:07, 45:11-45:14
					54							61	62			65						71		-	*45:04
			52		54							61	62			65						71		-	*45:06
					54							61	62		64							71		-	*45:08
					54							61	62									71		-	*45:09
					54							61	62									71		-	*45:10
			52			55														69		71		B46, Null, -	*46:01:01-46:01:06, 46:04-46:05, 46:07N, 46:09-46:10, 46:14-46:16, 46:20, 46:22-46:24, 46:26-46:31, 46:34-46:36, 46:38
	50		52			55														69		71		-	*46:01:07
			w			55														69		71		B46	*46:02
				52			55															71		B46, -	*46:03, 46:08, 46:13:01-46:13:03, 46:19, 46:21:01-46:21:02, 46:25
49			52			55																71		-	*46:06
			52			55										65						71		-	*46:11
			52			55														69		71		-	*46:12
			52			55														69		71		-	*46:17
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

[illegible]

[illegible]

				54						60						68	70			-	*51:81				
										59 60						68	70			-	*51:93, 51:144				
				54						59 60						68	70			-	*51:97				
				54						59 60						68	70			-	*51:101				
				54						59 60							70			Null, -	*51:104, 51:118N, 51:147				
				54						59 60				65		68	70			-	*51:106				
				54				58	59 60								70			-	*51:112				
										60						68	70			-	*51:148				
										59 60			62				70			B52(5), -	*52:01:01-52:01:01:02, 52:01:03, 52:01:06-52:01:08, 52:01:11-52:01:14, 52:01:16, 52:04-52:05, 52:07-52:08, 52:10:01-52:13, 52:15, 52:17-52:18, 52:22-52:24, 52:27-52:31				
				54						59 60			62				70			B52(5), -	*52:01:02, 52:01:04, 52:01:09, 52:02-52:03, 52:06:01-52:06:02, 52:09				
										59 60			62				70			B52(5), -	*52:01:05, 52:01:15, 52:14				
										59 60			62				70			-	*52:01:10				
										59 60			62				70			-	*52:16				
				54						59 60			62				70			-	*52:19				
										59			62				70			-	*52:20				
				54						59 60			62				69 70			-	*52:21				
										59 60			62		65		70			-	*52:25				
										59 60			62				70			-	*52:26				
										60						68	70			B53, -	*53:01:01-53:01:07, 53:05, 53:08:01-53:08:02, 53:10, 53:16, 53:18, 53:20-53:21, 53:23-53:27, 53:29				
				55						60						68	70			-	*53:02				
																68	70			B53, -	*53:03, 53:09, 53:11-53:13				
										60						68	70			B53, -	*53:04, 53:07, 53:19				
				54 55						60						68	70			-	*53:06				
										60						68	70			-	*53:14				
										60						68	70	72		-	*53:15				
										60		62					70			-	*53:17:01-53:17:02				
				53						60						68	70			-	*53:22				
						55				60		62					70			-	*53:28				
			52				56						63 64					71		B54(22), Null, -	*54:01:01, 54:01:03-54:02, 54:05N, 54:07-54:08N, 54:10, 54:13, 54:16-54:19, 54:21-54:25, 54:27-54:28				
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																								ser. ⁵	HLA-B allele ⁴

[illegible]

			52			56											71	B56(22)	*56:11-56:12							
			52		54	56							65				71	B56(22)	*56:13							
													65				71	-	*56:17, 56:33							
			52			56							65				71	-	*56:20:02							
			52			56		58	59								70	-	*56:21							
												64					71	-	*56:23							
			52			56			59				65				71	-	*56:25, 56:37							
											60						70	B57(17)	*57:01:01-57:01:04, 57:01:06-57:01:14, 57:06, 57:08, 57:10, 57:15-57:16, 57:18-57:20, 57:22-57:23, 57:25-57:27, 57:29-57:30, 57:33-57:38, 57:40-57:41, 57:43-57:44, 57:48-57:50, 57:52-57:56, 57:58, 57:60							
											60						70	B57(17)	*57:01:05, 57:11, 57:13, 57:21, 57:31, 57:47							
											60						70	B57(17)	*57:02:01-57:03:02, 57:17, 57:28N, 57:32, 57:39, 57:42, 57:46, 57:57							
											60						70	B57(17)	*57:04							
50											60						70	-	*57:05							
				53							60						70	-	*57:07							
				53							60						70	-	*57:09							
																	71	-	*57:12							
						55					60						70	-	*57:14							
				53							60						70	-	*57:24							
								58			60						70	-	*57:45, 57:51							
49											60						70	-	*57:59							
											60						70	B58(17)	*58:01:01-58:01:02, 58:01:04-58:01:12, 58:04-58:05, 58:10N-58:15, 58:19, 58:21-58:24, 58:29, 58:31N-58:33, 58:35, 58:37, 58:39N							
											60						70	B58(17)	*58:01:03, 58:02, 58:06, 58:16, 58:25-58:26, 58:38							
											60						70	-	*58:07							
					54				59	60							70	B17	*58:08:01							
									59	60							70	-	*58:08:02							
						55				60							70	-	*58:09							
											60						70	Null	*58:17N							
											60						70	-	*58:18							
											60						70	-	*58:20							
											60						70	-	*58:27							
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.	
																									ser. ⁵	HLA-B allele ⁴
											60						67				70				-	*58:28, 58:34
	50										60						66				70				-	*58:36
									58					64							70				B59, -	*59:01:01:01-59:01:01:02, 59:05

[illegible]