

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:04N, 01:06-01:07, 01:09-01:11N, 01:16N-01:18N, 01:20, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:08, 01:14-01:15N, 01:30, 01:79, 01:99, 01:145	A1, Null, -	1																							
*01:01:56	-	1			4									w											
*01:12, 01:19	A1	1		3																					
*01:13	A1	1			4							11		13		15									
*01:21, 01:126	A1, -	1		3	4																				
*01:28	-	1			4							11				15									
*01:43	-	1			4						10														
*01:51	-	1			4				8		10														
*01:95	-	1			4													17			19				
*01:121	-	1			4																				
*01:136, 11:94, 11:112 ⁴	-	1			4							11													
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64, 02:66-02:69, 02:71-02:77, 02:79:01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:128, 02:130-02:134, 02:136-02:148, 02:150-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:478	A2, Low A2, A203/A2, A210/A2, A2/blank, Null,-	2																							
*02:17:01-02:17:03	A2, -	2						w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2	2												13						20		22			
*02:35:02	A2													13						20		22			
*02:46, 02:70	A2	2																		20		22			
*02:48, 02:129	A2, -																			20		22			
*02:55	A2/A28	2							8													21		23	
*02:65, 02:407, 02:449	A2/A19, -	2																	19						
*02:78	-		w		4															20		22			
*02:135	-	2												13										24	
*02:149	-	2																			21				
*02:169	-	2			4																				
*02:237	-	2													14		16						23		
*02:243:01-02:243:02	-	2																18			21				
*02:309	-	2											12		14						21			24	
*02:338	-	2		3																					
*02:454	-	2											12		14									24	
*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:07, 03:09-03:11N, 03:13-03:14, 03:16-03:17:02, 03:20-03:23:02, 03:25-03:29, 03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44:01-03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:171, 03:174-03:175, 03:179-03:182	A3, A3/blank, Null, -	3												13											
*03:01:19	-			3									12	13										24	
*03:01:23, 03:08, 03:32, 03:36N, 03:57, 03:59, 03:72, 03:89, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N	A3, Null, -	3																							
*03:12	A3			3	4																				
*03:15, 03:19, 03:30	A3			3		5								13											
*03:18, 03:135	-	1			4									13											
*03:24, 03:50	A3, -			3					8					13											
*03:41	-			3																			23		
*03:43, 03:82	-			3										13		15		17							
*03:63	-			3								11		13									23		
*03:75, 03:177	-													13									23		
*03:88, 11:130 ⁵	-			3	4								11										23		
*03:95	-													13	14		16						23		
*03:152, 24:92 ⁵	-			3		5																			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*11:01:01:11-02:06, 11:04-11:09, 11:12-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:129, 11:131-11:138, 11:140-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176	A11, Null, -				4							11													
*11:03, 11:20, 11:25, 11:60, 11:158, 11:175	A11, -			3	4							11													
*11:10	A11				4				8			11													

[illegible]

[illegible]