

[illegible]

*11:113	-				4							11			15											
*11:139	-				4	5		7				11														
*11:155	-				4							11											22			
*23:01:01-23:13, 23:15-23:56, 23:58-23:60	A23, Null, -					5	6																			
*23:14:01-23:14:02, 24:24, 24:71 ⁵	A23, -, A9, A24					5	6	7																		
*23:57	-					5																	21			
*24:02:01:01-24:11N, 24:13:01-24:13:02, 24:17, 24:20-24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56, 24:58-24:63, 24:66-24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:113, 24:115-24:137, 24:139-24:187, 24:189-24:203, 24:205-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:252N	A24, Low A24, A2403/A24, A9, A24/A9, Null, -					5		7																		
*24:14-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, C*04:01:03 ⁵	A24, Cw4, Null, -					5																				
*24:18, 24:204, 24:213	A24/A3, -					5		7																23		
*24:19, 24:44	A9					5		7			11															
*24:82	-					5		7															21			
*24:243	-					5		7									17									
*25:01:01-25:04, 25:07-25:12N, 25:14-25:16, 25:18-25:23	A25, Null, -							8	9				13													
*25:05	A25							8	9			12	13							20						
*25:06	A25							8	9			12														
*25:13	-							8	9				13											24		
*25:17	-								9				13													
*26:01:01-26:01:20, 26:01:22-26:01:29, 26:08, 26:10-26:15, 26:17-26:18, 26:20, 26:23-26:28, 26:31-26:33, 26:35-26:43:02, 26:45-26:48, 26:50-26:53, 26:55-26:63, 26:66-26:71N, 26:74-26:77, 26:79-26:88, 26:90, 26:93-26:94	A26, A10, Null, -							8		10			13													
*26:01:21, 26:02:02, 26:04, 26:34, 26:73	A26, -							8		10																
*26:02:01, 26:29, 26:49	A26, -							8		10			w													
*26:03:01-26:03:02, 26:06, 26:21	A26, -							8			11		13											23		
*26:05	A26							8					13											23		
*26:07:01-26:07:02, 26:64, 26:89	A26, -									10			13													
*26:09, 26:91	A26, -							8		10		12														
*26:16	A26							7		10			13													
*26:19	-			4						10			13	14												
*26:22	A26							8		10			13	14								21				
*26:30	A26							8					13										23	24		
*26:54	-							8		10		12	13					20								
*26:65	-							8		10			13											24		
*26:72	-			4						10			13													
*26:78	-							8			11													23		
*26:92	-										11		13													
*29:01:01-29:06, 29:08N-29:12, 29:15-29:27, 29:29-29:38, 29:40-29:47, 29:50-29:55	A29, Null, -													14												
*29:07, 29:49	A29, -					5	6							14												
*29:13, 29:39	-													14			17									
*29:14	-													14			16									
*29:28	-							8						14												
*29:48	-													14				18								
*30:01:01-30:04:02, 30:06-30:07, 30:09-30:20, 30:22-30:54, 30:56-30:78N	A30, Null, -														15											
*30:08	A30				4										15											
*30:55	-													13	15											
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*31:01:02-31:02, 31:05-31:07, 31:09-31:28, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:79	A31, Null, -																16									
*31:03	A31													12		14	16									
*31:04	A31													12		14	16								24	
*31:08, 33:53 ⁵	A31/A24, -						5										16									
*31:29	-						5	6									16									
*31:35	-																15	16	17							
*31:41	-																16						21			
*31:62	-																16							22		
*32:01:01-32:01:13, 32:01:15-32:02, 32:07-32:14, 32:16-32:37, 32:39-32:41, 32:43-32:59	A32, Null, -									9									17							
*32:01:14, 32:03, 32:38, 74:07 ⁵	A32, -																		17							
*32:04	A32/A3				3						9															
*32:05	A32						5										16	17								
*32:06	A32										9							17					21			
*32:15	A32								8	9								17	18							

*32:42	-									9					14		17								
*33:01:01-33:01:07, 33:03:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:49-33:50, 33:52, 33:54-33:74N	A33, Null, -																18								
*33:13, 33:48	-									10							18								
*33:19	-					5		7									18								
*33:22	-																18			21					
*33:24	-																18						23		
*33:51	-								8								18								
*34:01:01-34:01:02, 34:05-34:06, 34:11	A34, A10, -								8		11	12													
*34:02:01-34:03, 34:07, 34:10N	A34, Null, -			3					8		11	12													
*34:04	A34			3					8		11	12		14											
*34:08	A34			3					8		11	12	13												
*34:09	-			3					8			12												24	
*36:01, 36:03, 36:05	A36, -	1																				22			
*36:02	A36	1		3																		22	23		
*36:04	A36	1			4																	22			
*43:01	A43									10	12	13													
*66:01:01, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:19	A66, A26, -								8		11	13													
*66:02-66:03, 66:16	A66, A10, -								8			12												24	
*66:06	A66								8		11	13								21					
*66:09	-								8		11	13	14							21					
*66:10	-								8		11	w													
*66:12	-								8			13												24	
*66:15	-								8			13													
*68:01:01:01-68:04, 68:06-68:12, 68:13:02-68:14, 68:16-68:19, 68:21:01-68:24, 68:26-68:28, 68:30-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:104, 68:106	A68, A28, Null, -								8									20							
*68:05, 68:15, 68:20	A68								8									20			23				
*68:13:01, 68:66	A68, -			4					8									20							
*68:25	A68								8									19	20						
*68:29	A68								8									18	20	21					
*68:45	-							7	8										20						
*68:71	-								8				13						20						
*68:84	-								8		10								20						
*68:105	-								8										20	21					
*69:01	A69								8											21					
*69:02	-								8		11									21					
*74:01-74:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18	A74, Null, -																	19							
*74:04	A74																	18	19						
*74:13	-												13						19						
*74:19	-				4														19						
*80:01:01:01-80:01:01:02, 80:03	A80, -					6																	23		
*80:02	-					6					11														
B*07:81, B*08:52, B*18:67, B*38:41, B*53:05, B*53:16	B53, -									9															
B*18:27	-					6																			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24