

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ^{1,2}																								
*01:02:01-01:02:17, 01:02:19-01:02:23, 01:06-01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:80-01:84	73																			92				
*01:02:18	73											84								92				
*01:03, 01:24, 01:34, 01:78	73																			92	93			
*01:04	73											84		86										
*01:05, 01:22, 01:35-01:36, 01:79:01-01:79:02, B*39:76, B*54:18 ³	73																							
*01:09	73													86										
*01:10	73	74																		92				
*01:14, 01:59	73										83									92				
*01:21	73											84	85											
*01:43	73	74									82									92				
*01:49-01:50, 01:55	73																					93		
*01:60	73																		91	92				
*01:67, 14:58 ³	73															88				92				
*02:02:01		74	75			78					83				w									
*02:02:02:01-02:02:03, 02:02:05-02:02:12, 02:02:14-02:02:25, 02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38N, 02:40, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:70-02:72		74	75			78					83				87									
*02:02:13, 02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45		74	75								83				87									
*02:04		74	w			78					83				87									
*02:05:01-02:05:02, 02:17		74	75			78					83			86	87									
*02:06		74	75			78	80				83				87		89							
*02:12		74	75			78							85		87									
*02:13, 02:18, 02:33		74	75			78					83				87			90						
*02:14		74	w			78					83													
*02:22, 02:62		74	75			78					83				87							93		
*02:23, 02:36, 02:68		74	75			78					83				87		89							
*02:26:01		74									83				87									
*02:27:01-02:27:02		74	75			78																		
*02:47		74	75			78	80				83				87									
*02:49		74	75			78					83		85		87			90						
*02:55		74	75			78					83		85		87									
*02:58			75			78					83				87									
*02:65		74	75			78					83													
*03:02:01-03:02:09, 03:02:11, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194, 03:197-03:198, 03:200-03:201N, 03:216				75	76																	94		
*03:02:10				76																				

Negative Control

*04:10, 04:153						78						83								90			93		
*04:11						78													88				93		
*04:14, 04:68						78						83		85							90		93		95
*04:16			75									83		85		87					90		93		
*04:29						78								85		88					90		93		
*04:32, 04:77			74			78						83		85							90		93	94	
*04:36, 04:55						78													88		90		93		
*04:37						78						83		85		87					90	92	93		
*04:42												83		85	86						90		93		95
*04:58						78						83		85							90	91	93		
*04:71			73			78						83		85							90		93		
*04:103												83		85		87					90		93		
*04:107			75			78						83		85		87							93		
*04:108						78						83		85						89	90				
*04:112						78						83		85						89	90		93		
*04:114						78						83		85		88					90		93		
*04:129						78	79					83		85							90		93		
*04:160			75			78						83		85		87					90	91	93		
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96	
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96	
*05:01:01:01-05:01:27, 05:03, 05:05-05:15, 05:18:01-05:19, 05:21, 05:24, 05:27-05:35, 05:37-05:41, 05:44:01-05:45, 05:47-05:54, 05:56-05:57, 05:58:02-05:61, 05:63-05:67, 05:69-05:77, 05:80-05:84, 05:86-05:97							79				83				87							93			
*05:04:01-05:04:02, 05:22:01-05:22:02							79				83				87										
*05:16, 05:85							79				83				87						92	93			
*05:17, 05:25, 05:55, 05:68, 05:79							79				83				87			90				93			
*05:20, 05:43							79				83											93			
*05:23, 05:62							79				83				87				91			93			
*05:26						78	79				83				87							93			
*05:36							79				83				87		89					93			
*05:42							79				83		85		87			90				93			
*05:46							79				83		85		87							93			
*05:58:01			75				79				83				87							93			
*05:78						78	79				83											93			
*06:02:01:01-06:02:01:03, 06:02:03-06:02:11, 06:02:13-06:02:15, 06:02:17-06:02:34, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18-06:30, 06:33-06:34, 06:36-06:39, 06:41-06:51, 06:53:01-06:66, 06:68-06:71, 06:73-06:75, 06:78, 06:80, 06:82-06:88, 06:90-06:93, 06:96-06:98, 06:100, 06:102-06:117								80			83			86									95		
*06:02:12, 06:11, 06:52								80						86										95	
*06:02:16, 06:04, 06:40, 06:79N, 06:81, 06:101								80			83													95	
*06:03:01			75					80			83			86	87								94		
*06:03:02, 06:76:01								80			83			86	87										
*06:05							79	80			83						90		92				95		
*06:06								80			83										92			95	
*06:08			74					80			83			86										95	
*06:09								80			83			86								93		95	
*06:14, 06:35, 06:72								80			83											93		95	
*06:17											83			86							92			95	
*06:31								80			83			86			90		92					95	
*06:32, 06:94-06:95, 06:99											83			86										95	
*06:67							79	80			83			86										95	
*06:76:02											83	84	85	86	87			90							
*06:77								80			83			86											
*06:89								80			83			86			89							95	

Negative Control

