

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:04N, 01:06-01:07, 01:09-01:11N, 01:16N-01:18N, 01:20, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:170	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:08, 01:14-01:15N, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167	A1, Null, -	1																							
*01:01:56	-	1		4									w												
*01:12, 01:19	A1	1	3																						
*01:13	A1	1		4							11		13		15										
*01:21, 01:126	A1, -	1	3	4																					
*01:28	-	1		4							11				15										
*01:43	-	1		4						10															
*01:51	-	1		4				8		10															
*01:95	-	1		4														17							
*01:121	-	1		4															19						
*01:136, 11:94, 11:112, 11:211 ⁴	-	1		4							11														
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64, 02:66-02:69, 02:71-02:77, 02:79:01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:107, 02:109, 02:111-02:128, 02:130-02:134, 02:136-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:267, 02:269-02:299, 02:301N-02:302, 02:304-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-02:555	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																							
*02:17:01-02:17:03	A2, -		2				w															21			
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2		2										13								20	22			
*02:35:02	A2												13								20	22			
*02:46, 02:70	A2		2																		20	22			
*02:48, 02:129	A2, -																				20	22			
*02:55	A2/A28		2					8														21	23		
*02:65, 02:407, 02:449	A2/A19, -		2															19							
*02:78	-		w		4																20	22			
*02:108, 02:110, 02:268, 02:300, 02:303	A2, -		2																			21			
*02:135	-		2										13											24	
*02:169	-		2		4																				
*02:237	-		2											14		16							23		
*02:243:01-02:243:02	-		2															18			21			24	
*02:309, 02:454	-		2									12		14											
*02:338	-		2	3																					
*02:527	-		2					8																	
*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:07, 03:09-03:11N, 03:13-03:14, 03:16-03:17:02, 03:20-03:23:02, 03:25-03:29, 03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44:01-03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:171, 03:174-03:175, 03:179-03:185, 03:188-03:197N, 03:200Q-03:202, 03:206-03:209	A3, A3/blank, Null, -			3									13												
*03:01:19	-			3								12	13											24	
*03:01:23, 03:08, 03:32, 03:36N, 03:57, 03:59, 03:72, 03:89, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:199, 03:203, 03:205	A3, Null, -			3																					
*03:12	A3			3	4																				
*03:15, 03:19, 03:30	A3			3		5							13												
*03:18, 03:135	-	1		4									13												
*03:24, 03:50	A3, -			3				8					13												
*03:41	-			3																			23		
*03:43, 03:82, 03:186	-			3									13		15		17								
*03:63	-			3							11		13										23		
*03:75, 03:177	-												13										23		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*03:88, 11:130 ⁵	-			3	4							11											23		
*03:95	-													13	14		16						23		
*03:152, 24:92 ⁵	-			3		5																			
*03:187	-													13								22			
*03:204	-												13												

[illegible]

*30:08	A30			4										15											
*30:55	-											13		15											
*30:89	-													15		17									
*31:01:02-01:31:02, 31:05-31:07, 31:09-31:28, 31:30-31:34, 31:36-31:61, 31:63-31:84, 31:86-31:89	A31, Null, -														16										
*31:03	A31									12		14		16											
*31:04	A31									12		14		16					24						
*31:08, 33:53 ⁵	A31/A24, -				5										16										
*31:29	-				5	6									16										
*31:35	-													15	16	17									
*31:62	-														16				22						
*31:85	-					6									16										
*32:01:01-32:01:13, 32:01:15-32:01:17, 32:01:19-32:02, 32:06-32:14, 32:16-32:37, 32:39-32:41, 32:43-32:59, 32:61, 32:63, 32:65-32:69	A32, Null, -								9							17									
*32:01:14, 32:03, 32:38, 32:60	A32, -															17									
*32:01:18, B*07:81, B*08:52, B*18:67, B*38:41, B*51:185, B*53:05, B*53:16, B*53:33 ⁵	B53, -								9																
*32:04	A32/A3			3					9																
*32:05	A32				5										16	17									
*32:15	A32							8	9							17	18								
*32:42	-								9				14			17									
*32:62	-								9							17	19								
*32:64	-			4					9							17									
*33:01:01-33:01:07, 33:03:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:49-33:50, 33:52, 33:54-33:95	A33, Null, -																18								
*33:13, 33:48	-								10								18								
*33:19	-				5		7										18								
*33:22	-																18		21						
*33:24	-																18								
*33:51	-							8									18		23						
*34:01:01-34:01:02, 34:05-34:06, 34:11-34:12	A34, A10, -							8		11	12														
*34:02:01-34:02:03, 34:03, 34:07, 34:10N	A34, Null, -			3				8		11	12														
*34:02:04	-			3				8		11	12						20		22						
*34:04	A34			3				8		11	12		14												
*34:08	A34			3				8		11	12	13													
*34:09	-			3				8			12								24						
*36:01, 36:03, 36:05	A36, -	1																	22						
*36:02	A36	1		3															22 23						
*36:04	A36	1			4														22						
*43:01	A43								10	12	13														
*66:01:01, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:20	A66, A26, -							8		11	13														
*66:01:02	-							8		11															
*66:02-66:03, 66:16, 66:21	A66, A10, -							8			12								24						
*66:06	A66							8		11	13							21							
*66:09	-							8		11	13	14						21							
*66:10	-							8		11	w														
*66:12	-							8			13								24						
*66:15	-							8			13														
*68:01:01-68:04, 68:06-68:08:01, 68:09-68:12, 68:13:02-68:14, 68:16-68:19, 68:21:01-68:24, 68:26-68:28, 68:30-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:127	A68, A28, Null, -							8										20							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*68:05, 68:15, 68:20, 68:109	A68, -								8												20				
*68:08:02, 68:105	A68, -								8												20	21			
*68:13:01, 68:66	A68, -				4				8												20				
*68:25	A68								8												19	20			
*68:29	A68								8										18		20	21			
*68:45, 68:117	-							7	8												20				
*68:71	-								8				13								20				
*68:84	-								8		10										20				
*69:01, 69:03	A69, -								8														21		
*69:02	-								8			11											21		
*74:01-74:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22	A74, Null, -																				19				
*74:04, 74:21	A74, -																				18	19			
*74:07	A74																				17	19			
*74:13	-												13			16						19			
*74:19	-			4																		19			

[illegible]