

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ^{1,2}																								
*01:02:01-01:02:17, 01:02:19-01:02:35, 01:06-01:07:01, 01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:80-01:89N, 01:91-01:100, 01:103-01:107	73																			92				
*01:02:18	73											84								92				
*01:03, 01:24, 01:34, 01:78	73																			92	93			
*01:04	73											84		86										
*01:05, 01:07:02, 01:22, 01:35-01:36, 01:79:01-01:79:02, B*39:76, B*54:18 ³	73																							
*01:09	73													86										
*01:10	73	74																		92				
*01:14, 01:59	73											83								92				
*01:21	73											84	85											
*01:43	73	74									82									92				
*01:49-01:50, 01:55	73																				93			
*01:60	73																		91	92				
*01:67, 14:58 ³	73															88				92				
*01:90	73							80									89			92				
*01:101-01:102	73																89			92				
*02:02:01		74	75			78					83				w									
*02:02:02:01-02:02:03, 02:02:05-02:02:12, 02:02:14-02:02:25, 02:02:27-02:02:28, 02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38N, 02:40:01-02:40:02, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:69-02:74, 02:76-02:86, 02:88-02:93, 02:95		74	75			78					83				87									
*02:02:13, 02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45		74	75								83				87									
*02:02:29		74	75								83				87						94			
*02:02:30		74				78					83				87									
*02:04		74	w			78					83				87									
*02:05:01-02:05:03, 02:17		74	75			78					83			86	87									
*02:06		74	75			78	80				83				87		89							
*02:12		74	75			78							85		87									
*02:13, 02:18, 02:33, 02:75		74	75			78					83				87		90							
*02:14		74	w			78					83													
*02:22, 02:62		74	75			78					83				87						93			
*02:23, 02:36, 02:68		74	75			78					83				87		89							
*02:26:01		74									83				87									
*02:27:01-02:27:02		74	75			78																		
*02:47		74	75			78	80				83				87									
*02:49		74	75			78					83		85		87		90							
*02:55		74	75			78					83		85		87									
*02:58			75			78					83				87									
*02:65		74	75			78					83													
*02:87		74									82													
*02:94		74	75			78	79				83				87									
*03:02:01-03:02:09, 03:02:11-03:02:12, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194, 03:197-03:198, 03:200-03:201N, 03:216, 03:221-03:222, 03:224N-03:226, 03:238, 03:249, 03:258			75	76																	94			
*03:02:10, 03:264				76																				
*03:02:13, 03:16, 03:110			75																			94		
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96

Negative Control

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*03:03:01-03:03:20, 03:03:22-03:03:28, 03:11:01-03:11:02, 03:18:02, 03:20N-03:22Q, 03:30-03:31, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68-03:69, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116:01-03:116:02, 03:119-03:120, 03:122, 03:124, 03:126-03:127, 03:133, 03:141-03:142, 03:150, 03:152, 03:158, 03:160-03:161, 03:168, 03:171, 03:176, 03:185, 03:187-03:189N, 03:192, 03:196, 03:202-03:207, 03:214, 03:217, 03:220, 03:223, 03:227-03:230, 03:237, 03:241-03:242, 03:253-03:254, 03:272-03:276				76	77																93	94		
*03:03:21, 03:04:26, 03:18:01, 03:39, 03:64:02, 03:137, 03:144, 03:156, 03:166, 03:170, 03:182, 03:195				76																	93			
*03:04:01:01-03:04:06, 03:04:08-03:04:24, 03:04:27-03:04:46, 03:06:01-03:06:02, 03:08-03:09, 03:14, 03:19, 03:23-03:24, 03:26, 03:28, 03:32, 03:34, 03:37:01-03:38:02, 03:40:01-03:40:04, 03:42, 03:44, 03:46-03:48, 03:51, 03:54, 03:57, 03:63-03:64:01, 03:65, 03:70, 03:72-03:74, 03:77-03:78, 03:82, 03:87:01-03:87:02, 03:90-03:93, 03:98, 03:100-03:101, 03:104-03:107, 03:109, 03:111, 03:114-03:115, 03:117-03:118, 03:123, 03:125, 03:128-03:129, 03:131, 03:134, 03:136, 03:138, 03:143, 03:147-03:149, 03:153, 03:157, 03:159, 03:162, 03:164, 03:173-03:174, 03:179-03:181, 03:183-03:184, 03:186:01-03:186:02, 03:191, 03:193, 03:208N, 03:210-03:213, 03:215, 03:218-03:219, 03:232-03:236, 03:239-03:240, 03:244Q, 03:246-03:247, 03:250, 03:252, 03:255-03:257, 03:259, 03:261, 03:263, 03:265N-03:266, 03:269-03:270, 03:277N			75	76																	93	94		
*03:04:07, 03:41:01-03:41:02, 03:80, 03:113, 03:260			75																		93	94		
*03:04:25, 03:145, 03:172			75	76																	93			
*03:07, 03:45, 03:163			75	76							83				87						93	94		
*03:10, 03:29, 03:209			75	76											87						93	94		
*03:13:01-03:13:02, 03:97, 03:132, 03:167, 03:177, 03:271				76	77																	94		
*03:15			75	76							83				87							94		
*03:58					77										87					92	93	94		
*03:67				76	w																93	w		
*03:81, 03:175				76	77												89				93	94		
*03:86					77										87					92		94		
*03:94			75												87					92		94		
*03:99															87					92				
*03:130, 03:140			75	76							83										93	94		
*03:151, 03:165, 03:251, 03:267					77																93	94		
*03:155			75	76													88				93	94		
*03:199, 03:245			75	76													89					94		
*03:231				76		78										88					93			
*03:243				76	77						83										93	94		
*03:248			75	76																				
*03:262				76	77																?	94		
*03:268				76	77						83				87						93	94		
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96

Negative Control

[illegible]

[illegible]

*07:125, 07:356										81				85	86									95
*07:154										81								90						95
*07:177										81					86			90						
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*07:210, 07:238									81		83									92				
*07:242				76					81															95
*07:316, 07:338	73								81															95
*08:01:01-08:01:17, 08:03:01-08:03:03, 08:06, 08:08:01-08:08:02, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72:01-08:72:02, 08:78-08:82, 08:84-08:89N, 08:91, 08:96-08:99, 08:101-08:102, 08:105-08:106, 08:109										82								90			93			
*08:01:18-08:02:12, 08:04:01-08:04:03, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N, 08:62-08:63, 08:66-08:71, 08:73-08:77, 08:90, 08:92-08:94, 08:100, 08:103, 08:107, 08:110-08:112, 08:114-08:115										82											93			
*08:05, 08:25										82			85								93			
*08:07, 08:104										82									91		93			
*08:09, 08:11, 08:83										82								90						
*08:10							79			82	83				87			90			93			
*08:12										82										92	93			
*08:21										82			85					90			93			
*08:31		74								82											93			
*08:47										82								90	91		93			
*08:48, 08:108										82														
*08:95										82						88		90			93			
*08:113										82							89							
*12:02:01-12:02:10, 12:02:12-12:02:13, 12:10:01-12:10:02, 12:17-12:18:01, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:83-12:86, 12:96, 12:103-12:106, 12:112, 12:114, 12:117-12:118, 12:124, 12:126, 12:128, 12:130, 12:134, 12:136-12:137, 12:142, 12:145, 12:148N												84	85											
*12:02:11								80				84												
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:17-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:03:27-12:03:29, 12:03:31-12:03:33, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:95, 12:97, 12:99-12:102, 12:107-12:111, 12:115-12:116, 12:120-12:122, 12:125, 12:129, 12:131, 12:133, 12:138-12:141, 12:143												84	85	86										
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:03:30, 12:26, 12:59														85	86									
*12:03:04, 12:98												84		86										
*12:03:09								80				84		86										
*12:03:13, 12:03:24												84	w	86										
*12:03:23			75									84	85	86										
*12:03:26								80				84	85	86										
*12:03:34														86										
*12:04:01											83			86	87									

Negative Control

Negative Control

*12:04:02, 12:60											83	84	85	86	87											
*12:05											83	84		86	87											
*12:08												84	85				89									
*12:09											83				87					92						
*12:14:01													85					90	91							
*12:14:02												84	85					90	91							
*12:15								80				84	85	86			89									
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96		
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96		
*12:16												84	85											95		
*12:18:02, B*67:02 ³													85													
*12:21							79				83	84			87											
*12:24												84	85							92						
*12:28, 12:58												84	85	86				90								
*12:31												84	85	86							93					
*12:33							79				83			86	87											
*12:41											83	84	85		87											
*12:44												84	85								93					
*12:49, 12:123												84														
*12:54											83	84	85	86												
*12:55												84	85	86		88										
*12:72												84	85		87											
*12:81, 12:113												84	85	86			89									
*12:119		74										84	85	86												
*12:127										82		84	85													
*12:132												84	85					90								
*12:135												84	85	86	87			90								
*12:144													85	86							93					
*12:146											83	84	85		87		90									
*12:147												84	85	86									95			
*14:02:01-14:02:07, 14:02:09-14:03, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:24:02, 14:26-14:27, 14:29-14:44, 14:46-14:48, 14:50-14:52, 14:55-14:57, 14:59-14:62, 14:65-14:69																88				92						
*14:02:08												84				88				92						
*14:04, 14:64											83					88				92						
*14:06, 14:08-14:09, 14:28:01-14:28:02, 14:53, 14:63, A*30:62, B*35:252 ³																88										
*14:10																88		90		92						
*14:12, 14:49											83									92						
*14:15														86		88		90								
*14:16														86		88										
*14:17																88		90	91	92						
*14:25		74														88				92		94				
*14:45															87	88				92						
*14:54																88					93					
*15:02:01:01-15:02:09, 15:02:11-15:02:16, 15:02:18-15:02:20, 15:08, 15:10:02-15:10:03, 15:13, 15:15, 15:17-15:18, 15:26, 15:28, 15:32Q-15:35, 15:38-15:39, 15:41, 15:44-15:45, 15:47-15:53, 15:56-15:57, 15:60, 15:64, 15:67-15:68, 15:73-15:74, 15:78-15:84Q, 15:86-15:89, 15:91-15:101, 15:103, 15:106			75					80			83				87		89				93					
*15:02:10, 15:02:17			75					80			83				87		89				93	94				
*15:03, 15:16			75					80			83		85		87		89				93					
*15:04-15:05:08, 15:05:10, 15:06:01-15:06:03, 15:09, 15:19, 15:24, 15:27, 15:29-15:30, 15:40, 15:46, 15:54, 15:59, 15:66, 15:69-15:70, 15:76-15:77, 15:90, 15:105Q			75								83				87		89				93					
*15:05:09, 15:61											83				87		89				93					
*15:05:11			75												87		89				93					
*15:07			75					80									89				93					
*15:10:01								80			83				87		89				93					

Negative Control

*15:11			75			78		80			83			87		89				93				
*15:12					77			80			83			87		89	90			93				
*15:21, 15:71			75					80						87		89				93				
*15:22, 15:55			75								83			87		89								
*15:23											83					89				93				
*15:25			75										85				90	91		93				
*15:31, 15:62			75					80			83			87		89	90			93				
*15:37, 15:58, 15:72			75					80			83			87		89								
*15:42			74	75				80			83			87		89				93				
*15:43			75					80								89				93	94			
*15:63								80			83					89				93				
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*15:65			75								83				87		89	90	91					
*15:75			75					80			83		85		87		89	90			93		95	
*15:85			75					80			83						89				93			
*15:102			75								83				87		89			92				
*15:104	73		75								83				87		89				93			
*16:01:01-16:01:15, 16:01:18-16:01:19, 16:06-16:08, 16:10-16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39:02, 16:41, 16:44, 16:49, 16:51-16:52, 16:54, 16:56, 16:58-16:59, 16:62, 16:65, 16:67, 16:71-16:73, 16:75-16:76, 16:79													85					90					95	
*16:01:16								80					85				90						95	
*16:01:17, 16:15:02												84	85				90						95	
*16:02:01-16:02:12, 16:09, 16:12, 16:19, 16:25, 16:46-16:47, 16:60, 16:63, 16:69, 16:74, 16:77N											83		85		87		90						95	
*16:02:13											83	84	85		87		90						95	
*16:04:01, 16:04:03, 16:33, 16:42, 16:55, 16:61, 16:66, 16:78														86			90		92					
*16:18, 16:37													85		87		90						95	
*16:20, 16:64													85				89	90					95	
*16:29		74												86			90		92					
*16:34			75			78							85				90						95	
*16:35														86			90							
*16:40							79									88								
*16:43													85										95	
*16:45													85				90			93			95	
*16:48											83			86	87		90							
*16:50		74											85				90						95	
*16:53							79									88			92					
*16:57											83		85				90						95	
*16:68																	90		92					
*16:70			75								83		85		87		89	90					95	
*17:01:01-17:01:03, 17:01:05-17:04, 17:06, 17:08-17:16, 17:18-17:19, 17:21, 17:24-17:28		74									83				87			91		93				
*17:01:04		74									83		85		87			91		93				
*17:05		74					79				83				87			91		93				
*17:07, 17:17, 17:23											83				87			91		93				
*17:20		74									83							91		93			95	
*17:22		74																91		93				
*18:01-18:02, 18:04-18:08											83									92	93		95	
*18:03		74									83									92	93	94	95	
*18:09											83				87					92	93			
A*24:64, A*24:96, A*24:106, A*24:174, B*08:16, B*08:123																							95	
B*07:239, B*14:03																				92				
B*15:277, B*35:222															87									
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	

Negative Control