

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:72, 01:02-01:04N, 01:06-01:07, 01:09-01:11N, 01:16N-01:18N, 01:20, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N, 01:53N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:180-01:191, 01:193, 01:195-01:199	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:08, 01:14-01:15N, 01:30, 01:79, 01:99, 01:147Q, 01:149, 01:159, 01:167, 01:178N-01:179N	A1, Null, -	1																							
*01:01:56	-	1			4									w											
*01:01:73, 01:52:02N	Null, -	1			w																				
*01:12, 01:19	A1, -	1		3																					
*01:13, 01:176	A1, -	1			4							11		13		15									
*01:21, 01:126	A1, -	1		3	4																				
*01:28	-	1			4							11				15									
*01:43	-	1			4						10														
*01:51	-	1			4				8		10														
*01:95	-	1			4													17							
*01:121	-	1			4															19					
*01:136, 01:192, 11:94, 11:112, 11:211 ⁴	-	1			4							11													
*01:145	-	1												w											
*01:194	-	1			4									13		15									
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79-01-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:107, 02:109, 02:111-02:128, 02:130-02:134, 02:136-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:267, 02:269-02:299, 02:301N-02:302, 02:304-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-02:570, 02:572-02:579, 02:581, 02:583-02:610	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																							
*02:17:01-02:17:03	A2, -		2					w														21			
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -		2											13						20		22			
*02:35:02	A2													13						20		22			
*02:46, 02:70	A2		2																	20		22			
*02:48, 02:129, 02:571	A2, -																			20		22			
*02:55	A2/A28	2						8														21		23	
*02:65, 02:407, 02:449	A2/A19, -	2																	19						
*02:78	-	w		4																20		22			
*02:108, 02:110, 02:268, 02:300, 02:303	A2, -	2																			21				
*02:135	-	2						8						13										24	
*02:169	-	2		4																					
*02:237	-	2													14		16							23	
*02:243:01-02:243:02	-	2																	18		21				
*02:309, 02:454	-	2											12		14										24
*02:338	-	2	3																						
*02:527, 02:582	-	2						8																	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:07, 03:09-03:11N, 03:13-03:14, 03:16-03:17:02, 03:20-03:23:02, 03:25-03:29, 03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44-01-03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:123:01, 03:124-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197N, 03:200Q-03:202, 03:204, 03:206-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:232-03:242	A3, A3/blank				3									13											
*03:01:19	-			3									12	13											24
*03:01:23, 03:08, 03:36N, 03:57, 03:59, 03:72, 03:89:01-03:89:02, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:199, 03:203, 03:205, 03:211	A3, Null, -			3																					
*03:01:38, 03:123:02, 03:171, 03:231	-			3										13	14										
*03:12	A3			3	4																				
*03:15, 03:19, 03:30	A3			3		5								13											
*03:18, 03:135	-	1			4									13											
*03:24, 03:50	A3, -			3				8						13											

*03:32, 03:219	-					3											17									
*03:41	-					3																23				
*03:43, 03:82, 03:186	-					3								13		15	17									
*03:63	-					3						11		13								23				
*03:75, 03:177	-													13								23				
*03:88	-					3	4					11										23				
*03:95	-													13	14		16					23				
*03:152	-					3		5									17									
*03:187	-													13								22				
*03:215, 03:225	-													13												
*11:01:01:01-11:02:06, 11:04-11:09, 11:12-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:230, 11:232-11:235Q	A11, Null, -						4																			
*11:03, 11:20, 11:25:01-11:25:02, 11:60, 11:158, 11:175, 11:209	A11, -					3	4							11												
*11:10, 11:191	A11, -						4				8			11												
*11:11	-						4							11	12											
*11:17, 11:40, 11:223	-						4						10	11												
*11:113, 11:162	-						4							11				15								
*11:116, 11:140, 11:199:01, 11:222	-						4							11		13										
*11:130	-						3	4						11			14						23			
*11:139	-						4	5		7				11												
*11:155	-						4							11								22				
*11:166	-						4		6					11												
*11:183	-						3	4			8			11												
*11:199:02	-						4							11		13				20		22				
*11:226	-			1			4							11								22				
*11:231	-						4							11									?			
*23:01:01-23:01:12, 23:01:14-23:01:18, 23:03:01-23:13, 23:15-23:56, 23:58-23:63, 23:65, 23:67-23:69	A23, Null, -							5	6																	
*23:01:13	-							5	6											20		22				
*23:02	A23							5	6													21				
*23:14:01-23:14:02, 24:24, 24:71, 24:315 ⁵	A23, A9, A24, -							5	6	7																
*23:57, 23:66, 24:94, 24:138, 24:188, 24:228 ⁵	-							5														21				
*23:64	-							5	6									17								
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*24:02:01:01-24:05, 24:07:01-24:11N, 24:13:02, 24:17, 24:20-24:21:03, 24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56, 24:58-24:63, 24:66-24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:103, 24:105-24:113, 24:115-24:137, 24:139-24:166, 24:168-24:187, 24:189-24:203, 24:205-24:206, 24:208-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:290, 24:292-24:295, 24:297-24:303N, 24:305-24:314, 24:317-24:323N, 24:325-24:334	A24, Low A24, A2403/A24, A9, A24/A9, Null, -						5			7																
*24:06, 24:13:01, 24:22, 24:82, 24:167, 24:207	A24, -						5			7												21				
*24:14-24:15, 24:51-24:53, 24:57, 24:64, 24:114, 24:222N, 24:291, 24:296, 24:304, 24:316, 24:324, C*04:01:03 ^d	A24, Cw4, Null, -						5																			
*24:18, 24:204, 24:213	A24/A3, -						5		7															23		
*24:19, 24:44	A9						5		7				11													
*24:92	-				3		5																			
*24:104, 24:243	-						5		7									17								
*25:01:01-25:02, 25:04, 25:07-25:12N, 25:14-25:29, 25:31-25:33	A25, Null, -									8	9				13											
*25:03	A25									8	9				13			17								
*25:05	A25									8	9			12	13						20					
*25:06	A25									8	9			12												
*25:13	-									8	9				13										24	
*25:30	-									8	9				13			17				21				
*26:01:01:01-26:01:20, 26:01:22-26:02:02, 26:08, 26:10-26:15, 26:17-26:18, 26:20, 26:23-26:28, 26:31-26:33, 26:35-26:43:02, 26:45-26:48, 26:50-26:53, 26:55-26:64, 26:66-26:71N, 26:74-26:77, 26:79-26:90, 26:93-26:110, 26:112-26:120	A26, A10, Null, -									8			10		13											
*26:01:21, 26:04, 26:34, 26:73	A26,-									8			10													
*26:03:01, 26:06, 26:21, 26:111	A26, -									8				11	13								23			
*26:05	A26									8					13								23			
*26:07:01-26:07:02, 26:65	A26, -									8			10		13										24	
*26:09, 26:91	A26, -									8			10		12											

[illegible]

*66:12	-								8					13											24
*66:15	-								8					13											
*66:23	-				4				8			11		13											
*68:01:01-68:01:31, 68:02:01:01-68:04, 68:06-68:08:01, 68:09-68:10, 68:12, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:142N	A68, A28, Null, -								8												20				
*68:01:32, 68:11N	Null, -								8												20		22		
*68:05, 68:15, 68:20, 68:109, 68:136	A68, -								8												20			23	
*68:08:02, 68:105	A68, -								8												20	21			
*68:13:01, 68:66, 68:134	A68, -				4				8												20				
*68:19	A68								8					14							20				
*68:25	A68								8											19	20				
*68:29	A68								8										18		20	21			
*68:45, 68:117	-						7		8												20				
*68:71	-								8					13							20				
*68:84	-								8		10										20				
*68:130	-								8				12								20				
*69:01, 69:03	A69, -								8													21			
*69:02	-								8			11										21			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*74:01:01-74:01:02, 74:02:01:01-74:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:27	A74, Null, -																				19				
*74:01:03	-												12								19				24
*74:04, 74:21	A74, -																			18	19				
*74:07	A74																		17		19				
*74:13	-													13			16				19				
*74:19	-				4																19				
*74:23	-				3																19				
*80:01:01-80:01:01:02, 80:03	A80, -							6																23	
*80:02	-							6				11													
B*07:81, B*08:52, B*18:67, B*38:41, B*51:185, B*53:05, B*53:16, B*53:33	B53, -									9															
B*18:27	-					6																			
C*07:404	-											11													
C*07:481	-																						?		
C*12:131	-				4																				
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24