

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04N, 01:06, 01:09:01-01:11N, 01:15N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:240N	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236	A1, Null, -	1																							
*01:01:56	-	1			4							12													
*01:07, 01:95	A1, -	1			4													17							
*01:12, 01:19, 01:200	A1, -	1		3																					
*01:13, 01:176	A1, -	1			4						11		13		15										
*01:21, 01:126	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211 ⁴	A1, -	1			4						11														
*01:28, 01:229	-	1			4						11				15										
*01:43	-	1			4						10														
*01:51	-	1			4				8		10														
*01:121	-	1			4														19						
*01:194	-	1			4									13		15									
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570, 02:572-02:575, 02:577-02:579, 02:581, 02:583-02:643N, 02:645-02:650, 02:652-02:661, 02:663-02:670, 02:672Q-02:681, 02:683-02:694	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																							
*02:17:01-02:17:04	A2, -		2					w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -		2											13						20		22			
*02:35:02	A2													13						20		22			
*02:46, 02:70	A2		2																	20		22			
*02:48, 02:129, 02:571	A2, -																			20		22			
*02:55	A2/A28	2						8													21		23		
*02:65, 02:407, 02:449	A2/A19, -	2																	19						
*02:78	-	w		4																20		22			
*02:81, 02:124	A2	2							9																
*02:135	-	2											13											24	
*02:149	-	2																			21				
*02:169	-	2		4																					
*02:237	-	2													14		16							23	
*02:243:01-02:243:02	-	2																18			21				
*02:309	-	2										12		14							21			24	
*02:338	-	2	3																						
*02:454	-	2										12		14											24
*02:527, 02:582	-	2						8																	
*02:576, 02:682	-	2																				22			
*02:644	-							8		10											21				
*02:651	-	2																		20					
*02:662	-	2									11														
*02:671	-	2																						23	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

*03:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:07:02, 03:09-03:11N, 03:13-03:14, 03:16-03:17:02, 03:20-03:23:02, 03:25-03:29, 03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44:01-03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:123:01, 03:124-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197N, 03:201-03:202, 03:204, 03:206-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:231:02-03:245, 03:247-03:248, 03:250-03:251, 03:253-03:259, 03:261-03:266N, 03:268-03:272, 03:274-03:281	A3, Null, -, A3/blank				3							13														
*03:01:19	-				3							12	13						24							
*03:01:23, 03:08, 03:36N, 03:57, 03:59, 03:72, 03:89:01-03:89:02, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:200Q, 03:203, 03:205, 03:211, 03:252, 03:267	A3, Null, -				3																					
*03:01:38, 03:123:02, 03:171, 03:231:01	-				3								13	14												
*03:12	A3				3	4																				
*03:15, 03:19, 03:30	A3				3		5						13													
*03:18, 03:135	-		1			4							13													
*03:24, 03:50	A3, -				3				8				13													
*03:32, 03:219	-				3											17										
*03:41	-				3														23							
*03:43, 03:82, 03:186	-				3								13	15		17										
*03:63	-				3						11		13						23							
*03:75, 03:177	-												13						23							
*03:88	-				3	4					11								23							
*03:95	-												13	14		16			23							
*03:152, 03:273	-				3		5									17										
*03:187	-												13						22							
*03:215, 03:225, 03:249	-												13													
*03:246, 03:282	-				3								13					19								
*03:260	-		1		3										15											
*11:01:01-01-11:02:06, 11:04-11:09, 11:12-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:115N, 11:117-11:129, 11:131-11:138, 11:141-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176-11:182Q, 11:184-11:190, 11:192-11:198, 11:200-11:208N, 11:210N, 11:212-11:221, 11:224-11:225, 11:227-11:264	A11, Null, -						4					11														
*11:03, 11:20, 11:25:01-11:25:02, 11:60, 11:158, 11:175, 11:209	A11, -				3	4						11														
*11:10, 11:191	A11, -					4			8			11														
*11:11	-					4						11	12													
*11:17, 11:40, 11:223	-					4					10	11														
*11:113, 11:162	-					4						11			15											
*11:116, 11:140, 11:199:01, 11:222, 66:23 ⁵	-					4						11	13													
*11:130	-				3	4						11		14					23							
*11:139	-					4	5		7			11														
*11:155	-					4						11							22							
*11:166	-							6				11														
*11:183	-				3	4			8			11														
*11:199:02	-					4						11	13					20								
*11:226	-		1			4						11							22							
*23:01:01-01-23:01:12, 23:01:14-23:13, 23:15-23:56, 23:58-23:63, 23:65, 23:67-23:68, 23:70-23:83	A23, Null, -						5	6																		
*23:01:13	-						5	6										20								
*23:14:01-23:14:02, 24:24, 24:71, 24:315 ⁵	A23, A9, A24, -						5	6	7																	
*23:57	-						5												21							
*23:64	-						5	6								17										
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*23:66, 24:14:01-01-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, 24:291, 24:296, 24:304, 24:316, 24:324, C*04:01:03 ⁵	A24, Cw4, Null, -						5																			

[illegible]

*74:01:03	-											12						19				24				
*74:04, 74:21	A74, -																	18	19							
*74:07	A74																	17	19							
*74:13	-												13			16			19							
*74:19	-				4														19							
*74:23	-			3															19							
*80:01:01-01-80:01:01:02, 80:03	A80, -						6															23				
*80:02	-						6				11															
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:60	B53, -								9																	
B*18:27	-						6																			
B*40:359, B*57:65, C*04:31, C*06:137, C*07:569	-																				22					
C*03:365, C*12:131	-				4																					
C*06:187	-																					23				
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24