

Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
HLA-C allele ^{1,2}																								
*01:02:01:01-01:02:17, 01:02:19-01:02:48, 01:02:50-01:02:65, 01:05-01:08, 01:11-01:13, 01:15-01-01:20, 01:22-01:23, 01:25-01:33, 01:35-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:79-01-01:89N, 01:91-01:100, 01:103-01:112, 01:114-01-01:115, 01:117N, 01:119-01:121Q, 01:123-01:124, 01:126-01:130, 01:132-01:135, 01:137N-01:144, 01:147-01:159, 01:161-01:167, 01:170-01:175, 01:178-01:185Q, 01:187-01:190	73																			92				
*01:02:18	73											84								92				
*01:02:49, 01:136, 01:145N-01:146:02	73							80												92				
*01:03, 01:24, 01:34, 01:49-01-01:50, 01:55, 01:78, 01:176	73																			92	93			
*01:04	73											84		86						92				
*01:09:01-01:09:02	73													86						92				
*01:10	73	74																		92				
*01:14, 01:59, 01:118, 01:186	73											83								92				
*01:21	73											84	85											
*01:43	73	74									82									92				
*01:60	73																		91	92				
*01:67, 01:122	73			76												88				92				
*01:90, 01:101-01:102, 01:113	73							80									89			92				
*01:116	73																89			92				
*01:125, 14:58 ³	73															88				92				
*01:131	73			76																92	93			
*01:160	73													86				90		92				
*01:168-01:169:01	73														87					92				
*01:169:02, 03:94 ³	73		75												87					92		94		
*01:177	73															88				92		94		
*02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:14-02:02:25, 02:02:27-02:02:28, 02:02:31-02:02:37, 02:02:39, 02:02:41-02:02:43, 02:02:45-02:02:52, 02:02:54-02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38:02N, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:69-02:74, 02:76-02:86, 02:88-02:93, 02:95-02:100, 02:102-02:106, 02:108-02:110, 02:112-02:114, 02:116-02:117, 02:119-02:130, 02:132-02:133, 02:135N-02:151, 02:153-02:163, 02:165N-02:166, 02:168-02:179, 02:181-02:186		74	75			78					83				87									
*02:02:13, 02:02:29		74	75								83				87							94		
*02:02:26, 02:21, 02:26-02-02:26:03, 02:39, 02:45		74	75								83				87									
*02:02:30, 02:02:40		74				78					83				87									
*02:02:38, 02:02:53		74	75			78					83				87							94		
*02:02:44		74	75			78					83	84			87									
*02:04		74	w			78					83				87									
*02:05:01-02:05:03, 02:17		74	75			78					83			86	87									
*02:06:01-02:06:02, 02:23, 02:36:01-02:36:02		74	75			78		80			83				87		89							
*02:12, 02:55:01-02:55:02		74	75			78					83			85	87									
*02:13, 02:18, 02:33, 02:75		74	75			78					83				87			90						
*02:14:01		74	w			78					83													
*02:14:02, 02:164		74				78					83													
*02:22, 02:62, 02:118		74	75			78					83				87							93		
*02:26:01, 02:134, 02:152, 02:167		74									83				87									
*02:27:01-02:27:02, 02:65, 02:131, 02:187		74	75			78					83													
*02:40:01-02:40:02		74	75			78					83				87				91					
*02:47, 02:111		74	75			78		80			83				87									
*02:49		74	75			78					83		85		87				90					
*02:58			75			78					83				87									
*02:68		74	75			78					83				87		89							
*02:87		74									82													
*02:94		74	75			78	79				83				87									
*02:101		74	75			w					83				87									
*02:107		74									83													

Negative Control

[illegible]

[illegible]

[illegible]

*04:383					75								78												83								85								90								93							
*04:406					74								78												83				85								90								93											
*05:01:01:01-05:01:50, 05:03, 05:05:01-05:15, 05:18:01-05:19, 05:21, 05:24, 05:27-05:35, 05:37-05:41, 05:44:01-05:45, 05:47-05:54, 05:56-05:57, 05:58:02, 05:58:04-05:61, 05:63-05:67, 05:69-05:75, 05:77, 05:80-05:84, 05:86-05:102, 05:104, 05:106:01-05:106:02, 05:108-05:111, 05:114, 05:116-05:122, 05:124-05:128N, 05:130-05:131, 05:133, 05:136-05:142, 05:144-05:145, 05:148-05:150, 05:152-05:155, 05:157-05:162, 05:164-05:190, 05:192-05:199, 05:201, 05:203-05:205, 05:208N-05:214, *05:01:51, 05:04:01-05:04:02, 05:103:01-05:103:02, 05:123, 05:135, 05:147, 05:191, 05:207									79								83								87												93																			
*05:16									79								83								87								92				93																			
*05:17, 05:25, 05:55, 05:68, 05:76, 05:79, 05:115									79								83								87				90								93																			
*05:20, 05:43, 05:113N, 05:156, 05:202Q									79								83																				93																			
*05:22:01-05:22:02									79				80								83								87																											
*05:23, 05:62, 05:146, 05:151, 05:163									79								83								87				91								93																			
*05:26					78				79								83								87												93																			
*05:36									79								83								87				89								93																			
*05:42									79								83				85				87				90								93																			
*05:46:01:01-05:46:01:02									79								83				85				87												93																			
*05:58:01, 05:58:03					75								79								83								87												93															
*05:78:01-05:78:02									78				79								83																				93															
*05:85	73												79								83								87								92				93															
*05:105					74												79								83								87												93											
*05:107	73												79								83								87								92																			
*05:112													79								83				85				87								93				95															
*05:129													79																87								93																			
*05:132, 05:215													79				80								83								87				89																			
*05:134, 05:143													79								83								87				90				91				93															
*05:200	73				74												79								83																93															
*05:206, 17:05 ³					74												79								83								87				91				93															
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96																																
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96																																
*06:02:01:01-06:02:01:43, 06:02:03-06:02:04, 06:02:06-06:02:11, 06:02:13-06:02:15, 06:02:17-06:02:37, 06:02:40-06:02:42, 06:02:44-06:02:64, 06:02:66-06:02:71, 06:02:73-06:02:82, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18-06:30, 06:33-06:34:02, 06:36-06:39, 06:41-06:51, 06:53:01-06:66, 06:68-06:71, 06:73-06:75, 06:78, 06:80, 06:82-06:88, 06:90-06:93, 06:96-06:98, 06:100, 06:102:01-06:117, 06:119-06:121, 06:125-06:126, 06:128N-06:131, 06:133-06:135, 06:137-06:142, 06:145-06:146, 06:148-06:152N, 06:154-06:179, 06:181-06:196, 06:198-06:202, 06:205-06:216, 06:218-06:223, 06:225-06:235, 06:237-06:243, 06:245-06:247, 06:249-06:251. 06:253-06:263N. 06:265-06:270. *06:02:05, 06:02:39, 06:32, 06:94-06:95, 06:99, 06:180, 06:236, 06:244													80								83								86												95															
*06:02:12, 06:02:65, 06:11, 06:52, 06:122, 06:124, 06:147, 06:217, 06:248, 06:252, 06:280													80																86												95															
*06:02:16, 06:02:43, 06:04:01-06:04:02, 06:40, 06:79N, 06:81, 06:101, 06:123, 06:127:01-01-06:127:02, 06:136, 06:153, 06:197, 06:203, 06:264													80								83																				95															
*06:02:38													80								83				84				86												95															
*06:02:72																					83				85				86				90								95															
*06:03:01, 06:132:01					75												80								83								86				87								94											
*06:03:02, 06:76:01													80								83								86				87																							
*06:05													79				80								83												90				92								95							
*06:06, 06:271																	80								83																92								95							
*06:08	74																80								83								86																95							
*06:09:01-06:09:02																	80								83								86												93				95							
*06:14, 06:35, 06:72, 06:143-06:144																	80								83																93				95											
*06:17																					83								86												92				95											
*06:31																	80								83								86				90				92								95							

*06:67								79	80			83		86											95	Neg
*06:76:02												83	84	85	86	87		90								
*06:77									80			83		86												
*06:89, 06:224									80			83		86			89								95	
*06:118									80			83		86				90	91						95	
*06:132:02			75									83		86	87								94			
*06:204	73								80			83								92					95	
*07:01:01:01-07:01:02:08, 07:01:04-07:01:40, 07:01:42-07:01:73, 07:01:75-07:01:92, 07:02:01:01-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:40, 07:02:42-07:02:90, 07:02:92-07:06:03, 07:08, 07:11-07:25, 07:27:01-07:27:02, 07:29:01:01-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93:01-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:132, 07:133:02-07:138, 07:140-07:147, 07:149-07:151, 07:153, 07:155, 07:157-07:160, 07:162-07:172:02, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:209, 07:211-07:215, 07:217-07:222, 07:224-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:266, 07:268-07:292, 07:297-07:313, 07:318-07:321, 07:323-07:327, 07:329N, 07:330:02-07:334, 07:336-07:337, 07:339-07:350N, 07:352-07:355, 07:357-07:366, 07:368:01-07:401, 07:404, 07:407-07:414, 07:417-07:448, 07:450-07:492, 07:494Q-07:513Q, 07:515-07:516, 07:518, 07:520-07:530, 07:532-07:535, 07:537-07:562, 07:564-07:567, 07:569-07:577, 07:580-07:582Q, 07:584-07:597, 07:599-07:625, 07:627-07:713Q, 07:715-07:722, 07:726N-07:747N									81															95		
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96		
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96		
*07:01:41, 07:02:41, 07:223, 07:415-07:416									81															w		
*07:01:74						78			81				85				89	90						95		
*07:01:93, 07:02:08, 07:02:29, 07:02:91, 07:26:01:01-07:26:03, 07:33N, 07:92, 07:139, 07:152N, 07:156, 07:246:01-07:246:02, 07:267, 07:296, 07:314:01, 07:314:03, 07:317, 07:322, 07:351, 07:405, 07:449, 07:493, 07:517, 07:714, 07:724, 07:821N, 07:845									81																	
*07:02:09						78			81				85				90							95		
*07:07, 07:09, 07:49, 07:76:01-07:76:02, 07:247, 07:315, 07:328, 07:403, 07:406, 07:598									81		83								92					95		
*07:10, 07:28, 07:41, 07:43:01-07:43:02, 07:184, 07:196, 07:367, 07:536, 07:568									81												93			95		
*07:31:01-07:31:02, 07:514									81				86				90							95		
*07:39-07:40, 07:335									81										92							
*07:53, 07:216									81		83													95		
*07:64, 07:402									81						88							94				
*07:96:01-07:96:02, 07:314:02, 07:578, 07:756			75						81													94				
*07:101, 07:148, 07:161		74							81	82														95		
*07:123, 07:173, 07:294, 07:626, 07:725, 07:775									81							89								95		
*07:125, 07:356, 07:531						78			81		83		85	86										95		
*07:133:01, 07:242, 07:330:01, 07:748				76					81															95		
*07:154									81								90							95		
*07:177									81				86				90		92							
*07:210, 07:238									81		83								92							
*07:293, 07:519									81						88									95		
*07:316, 07:338, 07:579	73								81															95		
*07:563									81										92					95		
*07:583		74				78			81	82			85				90									
*07:723						78			81				85													

[illegible]

[illegible]

*12:208									80					85				89										Negativ
*12:215	73												84	85	86													
*12:233														85											93			
*12:269									78					84	85	86		88										
*12:287														84	85				89	90								
*12:298									80						85													
*14:02:01:01-14:02:07, 14:02:09-14:02:20, 14:02:22-14:02:31, 14:02:33-14:03:03, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:24:02, 14:27, 14:29-14:44, 14:46:01-14:48, 14:50-14:52, 14:55-14:57, 14:59-14:62, 14:65-14:76, 14:78-14:86, 14:88-14:89, 14:91, 14:93N-14:104, 14:106-14:107, 14:109, 14:111-14:115																												
*14:02:08, 14:02:21																												
*14:02:32, 14:06, 14:08-14:09, 14:28:01-14:28:02, 14:53, 14:63, 14:105Q, A*29:86, A*30:62, A*33:63, B*08:249, B*18:64, B*35:183, B*35:252, B*39:114, B*39:140, B*40:138, B*41:59 ³																												
*14:04, 14:64																												
*14:10, 14:116																												
*14:12, 14:49, 14:108																												
*14:15, 14:87																												
*14:16																												
*14:17																												
*14:25																												
*14:26																												
*14:45																												
*14:54																												
*14:77																												
*14:90																												
*14:92																												
*14:110, B*07:239, B*14:03 ³																												
Well No.																												
Well No.																												
*15:02:01:01-15:02:09, 15:02:11-15:02:16, 15:02:18-15:02:20, 15:02:22-15:02:24, 15:02:26-15:02:44, 15:05:01:01-15:05:08, 15:05:10, 15:05:13-15:05:14, 15:05:16-15:06:03, 15:08:01-15:09:01:02, 15:10:02-15:10:03, 15:13:01:01-15:13:01:02, 15:15, 15:17-15:19, 15:26-15:29, 15:32Q-15:35, 15:38-15:39, 15:41, 15:44:01-15:54, 15:56-15:57, 15:59-15:60, 15:64, 15:67-15:70, 15:73-15:74, 15:78:01-15:84Q, 15:86-15:101, 15:103, 15:105Q-15:106, 15:108-15:110, 15:112-15:115N, 15:117, 15:119-15:124, 15:126-15:137, 15:139-15:143, 15:145N-15:148, 15:150-15:157, 15:159-15:161, 15:163-15:168, 15:170-15:176, 15:179-15:180, 15:182-15:189N, 15:191-15:200, 15:202-15:207																												
*15:02:10, 15:02:17																												
*15:02:21, 15:02:25, 15:05:09, 15:05:15, 15:10:01, 15:10:04, 15:61, 15:208																												
*15:03, 15:16																												
*15:04:01:01-15:04:03, 15:24, 15:30, 15:40, 15:66, 15:76-15:77, 15:177N																												
*15:05:11-15:05:12, 15:21, 15:71, 15:116																												
*15:07:01:01-15:07:01:02																												
*15:11, 15:118																												
*15:12																												
*15:22, 15:37, 15:55, 15:58, 15:72, 15:125, 15:149																												
*15:23:01-15:23:02, 15:63, 15:138, 15:158																												
*15:25																												
*15:31, 15:62, 15:169																												
*15:36																												
*15:42, 15:111																												
*15:43																												
*15:65																												
*15:75																												
*15:85, 15:162, 15:181																												

*15:102	73	75							83			87	89		92									
*15:104	73	75						80	83			87	89			93								
*15:107							79	80	83			87	89			93								
*15:144					77			80					89	90		93								
*15:178			75				79	80	83			87	89			93								
*15:190			75						83			87	89											
*15:201		74	75					80	83			87	89		91	93								
*16:01:01:01-16:01:15, 16:01:18-16:01:36, 16:06-16:08, 16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39:02, 16:41, 16:44, 16:49, 16:51-16:52, 16:54, 16:56-16:59, 16:62, 16:65, 16:71-16:73, 16:75-16:76, 16:79-16:81, 16:83, 16:85-16:86, 16:92-16:98, 16:100, 16:105-16:106, 16:111-16:112, 16:114, 16:116-16:119, 16:122, 16:125-16:131, 16:134-16:135, 16:137-16:138, 16:141-16:142, 16:146-16:148, 16:151-16:152, 16:154, 16:157-16:162, 16:164-16:165									83		85			90			95							
*16:01:16								80	83		85			90			95							
*16:01:17, 16:15:02									83	84	85			90			95							
*16:02:01:01-16:02:12, 16:02:14-16:02:17, 16:09, 16:12, 16:18-16:19, 16:25, 16:37, 16:46-16:47, 16:60, 16:63, 16:69, 16:74, 16:77N, 16:84, 16:89N-16:90, 16:99, 16:101-16:102, 16:104, 16:107-16:108, 16:115, 16:120, 16:123N, 16:132N-16:133, 16:136, 16:143-16:145, 16:153, 16:155-16:156, 16:163									83		85		87	90			95							
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*16:02:13											83	84	85		87		90						95	
*16:04:01:01-16:04:01:03, 16:04:03-16:04:05, 16:33, 16:42, 16:55, 16:61, 16:66, 16:78, 16:82, 16:124, 16:149														86			90		92					
*16:10, 16:67													85				90						95	
*16:20								80			83		85				89	90					95	
*16:29		74												86			90		92					
*16:34			75			78					83		85				90						95	
*16:35														86			90							
*16:40, 16:110							79									88								
*16:43											83		85										95	
*16:45													85				90			93			95	
*16:48											83			86	87		90							
*16:50		74									83		85				90						95	
*16:53							79									88				92				
*16:64, 16:87											83		85				89	90					95	
*16:68																	90		92					
*16:70			75								83		85		87		89	90					95	
*16:88											83				87		90		92					
*16:91											83			86	87		90		92					
*16:103											83				87		90							
*16:109								80						86			89	90		92				
*16:113							79				83		85			88							95	
*16:121			75			78					83		85		87		90						95	
*16:139										82	83		85				90						95	
*16:140											83				87									
*16:150						78								86		88		90		92				
*17:01:01:02-17:01:03, 17:01:05-17:04, 17:06, 17:08-17:16:02, 17:18-17:19, 17:21, 17:24-17:32, 17:34-17:51:01:02		74									83				87			91		93				
*17:01:04		74									83		85		87			91		93				
*17:07, 17:17, 17:23, 17:33											83				87			91		93				
*17:20		74									83							91		93			95	
*17:22		74																91		93				
*18:01:01:01-18:02:04, 18:04-18:08, 18:10-18:13											83								92	93			95	
*18:03		74									83								92	93	94		95	
*18:09											83				87				92	93				
*18:14											83								92				95	
A*02:42:01, B*15:539, B*39:76, B*54:18	73																							
A*24:64, A*24:96, A*24:106, B*08:16, B*08:123, B*08:145																								95

Negative Control

Negative Control

[illegible]