

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01:01:22, 01:01:24:01:01:47, 01:01:49:01:01:55, 01:01:57:01:01:64, 01:01:67:01:01:78, 01:01:80:01:04:01:02N, 01:06, 01:09:01:01:01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-01:343, 01:345-01:359, 01:362-01:375, 01:377-01:379N, 01:382-01:390, 01:392-01:395, 01:397-01:399	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314, 01:376, 01:391, 01:396Q	A1, Null, -	1																							
*01:01:56, 01:361N	Null, -	1			4								12												
*01:07, 01:95, 01:289	A1, -	1			4													17							
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																					
*01:13, 01:176	A1, -	1			4							11		13		15									
*01:15N	Null	1			w																				
*01:21, 01:126, 01:360	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 01:380:01:01:01:380:01:02, 11:94, 11:112, 11:211, 11:290, 11:326 ⁴	A1, -	1			4							11													
*01:28, 01:229, 01:299, 11:282 ⁴	-	1			4							11				15									
*01:43, 11:271 ⁴	-	1			4							10													
*01:51	-	1			4				8			10													
*01:121	-	1			4															19					
*01:194	-	1			4									13		15									
*01:298	-	1			4						9							17							
*01:301Q	-	1			4	5		7																	
*01:344, 01:381	-	1			4				8																
*02:01:01:01:02:01:15, 02:01:17:02:01:19, 02:01:21:02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01:02:33, 02:36-02:45, 02:47, 02:49:01:02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01:02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01:02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01:02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01:02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830:01:02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:951, 02:953-02:1034	A2, Low A2, A203, A210, Null, -	2																							
*02:17:02:01:02:17:04	A2, -	2						w																	
*02:34-02:35:01, 02:35:03, 02:56:01:02:56:02, 02:62, 02:103, 02:580, 02:907	A2, -	2												13						20		22			
*02:35:02	A2													13						20		22			
*02:46, 02:70	A2	2																		20		22			
*02:48, 02:129, 02:571	A2, -																			20		22			
*02:55	A2, A28	2						8													21		23		
*02:65, 02:407, 02:449, 02:932	A2, A19, -	2																	19						
*02:78	-	w			4															20		22			
*02:81, 02:124	A2	2								9															
*02:135	-	2												13										24	
*02:149:01:02:149:02, 02:842, 02:947	-	2																			21				
*02:169	-	2			4																				
*02:237, 02:809, 02:927	-	2												14		16							23		
*02:243:01:02:243:03	-	2																	18		21				
*02:309	-	2											12		14						21			24	
*02:338, 02:952	-	2	3																						
*02:454	-	2											12		14									24	
*02:527, 02:582	-	2						8																	
*02:576, 02:682	-	2																				22			
*02:644	-							8		10											21				
*02:651	-	2																		20					
*02:671	-	2																					23		
*02:741	-	2						8						13						20	21	22			
*02:804	-	2						7																	
*02:815	-	2						8													21				
*02:829	-				4					9															
*02:929	-	2																	18						
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*23:01:01:01-23:13, 23:15-23:56, 23:58-23:63, 23:65, 23:67-23:68, 23:70-23:98, 23:100-23:103N, 23:106N-23:112, 23:114-23:119	A23, Null, -					5	6																		
*23:14:01:23:14:02, 23:105, 24:24, 24:71, 24:315, 24:392, 24:527 ⁴	A23, A9, A24, -					5	6	7																	
*23:57	-					5																21			
*23:64	-					5	6											17							
*23:66, 23:99, 24:14:01:01-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, 24:291, 24:296, 24:304, 24:316, 24:324, 24:412, 24:481, 24:515, C*04:01:03 ⁴	A24, Null, -					5																			
*23:104, 23:113N, 24:02:01:01-24:11N, 24:13:01-24:13:02, 24:17:01:01-24:17:01:02, 24:20:01:01-24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56:02, 24:58-24:63, 24:66-24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:103, 24:105-24:113, 24:115-24:137, 24:139-24:187, 24:189-24:203, 24:205-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:290, 24:292-24:295, 24:297-24:303N, 24:305-24:314, 24:317-24:323N, 24:325-24:391, 24:393-24:411, 24:413-24:457, 24:459-24:470, 24:472-24:480, 24:482-24:514N, 24:516-24:526, 24:528-24:550, 24:552-24:554, 24:556N, 24:558-24:563 ⁴	A24, Low A24, A2403, A24, A9, Null					5		7																	
*24:18, 24:204, 24:213	A24, A3, -					5		7																23	
*24:19, 24:44	A9					5		7			11														
*24:82	-					5		7											18			21			
*24:92, 24:458, 24:551, 24:557	-				3	5																			
*24:104, 24:243	-					5		7										17							
*25:01:01:01-25:02, 25:04, 25:07-25:12N, 25:14-25:16, 25:18-25:29, 25:31-25:33, 25:35, 25:37-25:42N, 25:44-25:45, 25:47-25:49N, 25:51, 25:53, 25:55-25:58, 25:60-25:64, 25:66-25:70, 25:72-25:76, 25:79-25:80	A25, Null, -							8	9				13												
*25:03	A25							8	9				13				17								
*25:05	A25							8	9			12	13							20					
*25:06	A25							8	9			12													
*25:13	-							8	9				13											24	
*25:17, 25:34, 25:54, 25:71	-							8	9				13												
*25:30	-							8	9				13					17				21			
*25:36, 25:50, 25:52, 25:65	-							8	9																
*25:43	-				4			8	9																
*25:46	-							8	9					13										24	
*25:59	-			1	4						10										20				
*25:78, 26:01:63, 26:57, 66:15 ⁴	-							8						13											
*26:01:01:01-26:01:47, 26:01:49-26:01:62, 26:01:64-26:01:72, 26:08:01:01-26:08:03, 26:10-26:15, 26:17-26:18, 26:20:01-26:20:02, 26:23-26:28, 26:31-26:43:02, 26:45-26:48, 26:50-26:53, 26:55-26:56, 26:58-26:63, 26:66-26:71N, 26:73-26:74, 26:76-26:77, 26:79-26:88, 26:90, 26:93-26:110, 26:112-26:119, 26:121, 26:123-26:151, 26:153-26:176, 26:178-26:183, 26:185-26:192, 26:194-26:197, 26:199-26:203, 26:206N, 26:208-26:216, 26:218-26:219	A26, -, Null, A10							8		10				13											
*26:01:48, 26:64, 26:89	-									10				13											
*26:02:01-26:02:02, 26:29, 26:49	A26, -							8		10				w											
*26:03:01:01-26:03:01:02, 26:06, 26:21, 26:111	A26, -							8			11			13										23	
*26:04, 26:75, 26:205	A26, -							8		10															
*26:05	A26							8						13										23	
*26:07:01-26:07:02	A26									10				13											24
*26:09, 26:91	A26, -							8		10		12													
*26:16	A26							7		10				13											
*26:19	-				4					10				13	14										
*26:22	A26							8		10				13	14							21			
*26:30	A26							8						13									23	24	
*26:54	-							8		10		12		13						20					
*26:65	-							8		10				13											24
*26:72, 26:184	-				4					10				13											
*26:78	-							8			11													23	
*26:92, 66:42 ⁴	-										11			13											24
*26:120	-			1				8		10				13											
*26:122	-							8		10				13						20					
*26:152, 26:193	-							8		10		11		13											
*26:177	-											11		13				18			21		23		
*26:198	-							8		10				13	14										
*26:204:01:01-26:204:01:02	-				4			8		10															
*26:207	-									10				13				18							
*26:217	-							8		10		12	13												
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*29:01:01:01-29:06, 29:08N-29:12, 29:15-29:27, 29:29-29:38, 29:40-29:47, 29:50-29:66, 29:68-29:79, 29:81-29:97, 29:99-29:104, 29:106-29:154, 29:157-29:163, 29:165	A29, Null, -														14										
*29:07, 29:49	A29, -					5	6								14										
*29:13, 29:39, 29:98	-														14			17							
*29:14	-														14		16								
*29:28	-							8							14										
*29:48	-														14				18						
*29:67, 29:156	-				4										14										
*29:80, 29:155	-														14							21			
*29:105, 29:164	-									10					14				18						
*30:01:01:01-30:04:03, 30:06-30:07, 30:09:01:01-30:20, 30:22-30:54, 30:56-30:88, 30:90-30:124, 30:126-30:139, 30:141-30:149, 30:151-30:162, 30:164-30:176, 30:178N-30:185, 30:187-30:199, B*07:260, C*12:328 ⁴	A30, Null, -															15									
*30:08:01-30:08:02, 30:150	A30, -				4											15									
*30:55, 30:186	-													13		15									
*30:89, 30:177	-															15		17							
*30:125, 30:140	-										11					15									
*30:163	-						7									15									
*31:01:02:01-31:02:02, 31:05-31:07, 31:09-31:20, 31:22-31:23, 31:26-31:28, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:84, 31:86-31:108, 31:110-31:122, 31:124-31:175, 31:177-31:178, 31:180-31:199, 31:201N-31:206Q	A31, Null, -															16									
*31:03	A31			w								12		14		16									
*31:04:01:01-31:04:02	A31, -			w								12		14		16								24	
*31:08, 33:53 ⁴	A31, A24, -					5										16			18						
*31:21	-															16		17							
*31:24-31:25	-													13		16									
*31:29	-					5	6									16									
*31:35	-															15	16	17							
*31:41	-															16						21			
*31:62	-															16							22		
*31:85	-						6									16									
*31:109, 33:125, 33:131 ⁴	-															16		18							
*31:123	-											12		14		16								24	
*31:176, 31:179	-			3												16									
*31:200	-		1													16									
*32:01:01:01-32:02, 32:07-32:14, 32:16-32:25, 32:27N-32:37, 32:39-32:41, 32:43:01-32:59, 32:61-32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:127, 32:129-32:141, 32:143-32:151, 32:153-32:155, 32:157-32:158	A32, Null, -									9								17							
*32:03:01:01-32:03:01:02, 32:38, 32:60, 32:71, 32:78, 32:80, 32:107	A32, -																	17							
*32:04, 32:142	A32, A3, -			3					9								17								
*32:05, 32:79	A32, -					5										16	17	18							
*32:06	A32							8	9							17					21				
*32:15	A32							8	9							17	18								
*32:26:01-32:26:02, 32:152	-							9					13			17									
*32:42	-							9						14		17									
*32:64	-				4			9								17									
*32:72	-						6	9								17									
*32:93	-							9								17	18								
*32:128	-							9								16	17								
*32:156	-		1					9								17									
*33:01:01:01-33:01:15, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:178, 33:180-33:183, 33:185-33:186, 33:188-33:189, 33:191-33:200, 33:202-33:224	A33, Null, -																	18							
*33:13, 33:48	-									10								18							
*33:19	-					5	7											18							
*33:22	-																	18			21				
*33:24	-																	18				23			
*33:49, 33:144, 33:179, 33:201	-			w														18							
*33:51	-							8										18							
*33:61, 33:190	-													13				18							
*33:119	-						7											18							
*33:184	-										11							18					23		
*33:187	-					5	6											18							
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*34:01:01:01-34:01:07, 34:05-34:06, 34:11-34:12, 34:14, 34:16-34:18, 34:23, 34:27, 66:40 ^d	A34, A10, -								8			11	12													
*34:02:01:01-34:03, 34:07, 34:10N, 34:13, 34:15, 34:20-34:21, 34:24-34:26N	A34, Null, -			3					8			11	12													
*34:04	A34			3					8			11	12		14											
*34:08	A34			3					8			11	12	13												
*34:09	-			3					8			12													24	
*34:22	-			3					8			11	12										22			
*36:01:01:01-36:01:02, 36:03, 36:05-36:12	A36, -	1																				22				
*36:02	A36	1		3											14								22	23		
*36:04	A36	1			4																		22			
*43:01-43:02N	A43, Null										10	12	13													
*66:01:01:01-66:01:06, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:41	A66, A26, Null, -								8			11		13												
*66:02:01:01-66:03:01:03, 66:16, 66:21, 66:25-66:26Q, 66:28N, 66:34, 66:39N, 66:43	A66, A10, Null, -								8			12													24	
*66:06	A66								8			11		13								21				
*66:09	-								8			11		13	14							21				
*66:10	-								8			11		w												
*66:12	-								8					13											24	
*66:19	-								8			11	12	13												
*68:01:01:01-68:04:02, 68:06-68:08:01:02, 68:09-68:10, 68:12:01-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:40, 68:42-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179-68:217, 68:219-68:226, 68:228-68:229, 68:231-68:248, 68:251N-68:254, 68:256-68:257, 68:259-68:261, 68:263Q-68:270, 68:272-68:275, 68:277-68:283	A68, A28, Null, -								8												20					
*68:05, 68:15, 68:20, 68:109, 68:136, 68:255	A68, -								8												20			23		
*68:08:02, 68:105, 68:157, 68:276Q	A68, -								8												20	21				
*68:11N, 68:41, 68:230	Null, -								8												20		22			
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4				8												20					
*68:19, 68:130:02	A68, -								8					14							20					
*68:25, 68:262	A68, -								8											19	20					
*68:29	A68								8									18		20	21					
*68:45, 68:117	-							7	8												20					
*68:71	-								8					13							20					
*68:84, 68:227	-								8		10										20					
*68:103:01-68:103:02	-			3					8												20					
*68:130:01	-								8				12								20					
*68:178	-				4																20					
*68:218, 68:258	-								8												20					
*68:249	-					5			8												20					
*68:250	-																	18		20	21					
*68:271	-								8			11									20					
*69:01:01:01-69:01:04, 69:03-69:06, 69:08	A69, -								8													21				
*69:02	-								8			11										21				
*69:07	-								8					13								21				
*74:01:01:01-74:01:02, 74:01:04-74:03:02, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36, 74:38-74:41	A74, Null, -																			19						
*74:01:03	-											12								19					24	
*74:04, 74:21	A74, -																			18	19					
*74:07, 74:33	A74, -																		17		19					
*74:13	-													13		16				19						
*74:19	-				4															19						
*74:23	-				3															19						
*74:30	-											11								19						
*74:35	-																			19	21					
*74:37	-													13						19						
*80:01:01:01-80:01:02, 80:03, 80:05-80:09N	A80, Null, -					6																	23			
*80:02	-					6					11															
*80:04	-					6													17				23			
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -									9															
B*07:439, B*35:491, B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569	-																						22		
B*18:27	-						6																		
C*03:365, C*12:131	-				4																				
C*06:187	-																							23	
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24