

Tube No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
HLA-A allele1	ser2																				
*01010101-0104N, 0106, 0109-0111N, 0116N-0118N, 0120, 0122N-0125	A1, null	1			4																
*0107	-	1			4																
*0108, 0114, 0115N	-	1																			
*0112, 0119	-	1		3								11									
*0113	-	1			4							11		13							
*0121	-	1		3	4									13							
*02010101-0209, 0211-0216, 0218-0222, 0224-0233, 0236-0238, 0240-0245, 0247, 0249, 0251, 0253N, 0254, 0257-0261, 0263-0269, 0271, 0272, 0274-0277, 0279-0292, 0294N, 0296, 0297, 0299, 9201, 9202, 9204-9209, 9211-9213N, 9215	A2, A203, null		2																		
*0210, 0239, 0250, 0252, 0273, 0293, 0295, 9210, 9214	-		2																		20
*021701-021702	A2		2					w													
*0234, 023501, 0256, 0262, 9203	-		2											13							20
*023502	-													13							20
*0246, 0270	-		2																		20
*0248	-																				20
*0255	-		2						8												
*0278	-		2		4																20
*03010101-0307, 0309, 0311N, 0313-0317, 0319-0323, 0325-0329	A3			3										13							
*0310	-			3								11		13							
*0308	-			3																	
*0312	-			3	4																
*0318, 2619	-				4									13							
*0324	-			3					8					13							
*110101-1109, 1113-1117, 1119-1124, 1126-1130	A11				4							11									
*1110	-				4				8			11									
*1111	-				4							11	12								
*1112, 1118	-				4							11									
*1125	-			3	4							11									
*2301-2313, 2315	A23					5	6														
*2314, 2405, 2424, 2465	-					5	6	7													
*24020101-2404, 2406-2411N, 2413, 2417, 2418, 2420-2423, 2425-2443, 2445N-2450, 2454-2456, 2458-2463, 2466-2476	A24					5		7													
*2414, 2415, 2452, 2453, 2457, 2464	A24					5															
*2419, 2444	-					5		7				11									
*2451	-					5						11									
*250101-2504	A25								8	9				13							
*2505	-								8	9				13							20
*2506	-								8	9			12								
*260101-260105, 2608, 2610-2615, 2617, 2618, 2620, 2623-2629, 2632	A26, null								8		10			13							
*2602	A26								8		10			w							
*2603, 2606, 2621	A26								8			11		13							
*2604, 2634	A26								8		10										

[illegible]

[illegible]

[illegible]

[illegible]
