

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*070201-070207, 070501-0706, 0710, 0712, 0714, 071801- 071802, 0721-0723, 0726, 0728-0731, 0734, 0735, 0739, 0741, 0743, 0747, 0749N, 0752, 0755- 0759	B7	25																							48
*0703, 0708, 0716, 0732, 0737	B7, B703	25																							48
*0704, 0725	B7	25													38										48
*0707, 0742	B7	25																							48
*0709, 0711, 0717	B7	25											36												48
*0713, 0715	-	25																							48
*0719	-																								48
*0720, 0724	-	25		27																					48
*0727	-	25										35								44					
*0733	-																								48
*0736	-	25																							
*0738	-	25																							
*0740, 0748, 0751	-																								48
*0753	-																								48
*0750	-	25										35													48
*0754	-	25																							48
*0760	-			27																					48
*080101-080105, 0805, 0808N, 0810, 0811, 0815, 0818, 0819N, 0822-0824, 0827, 0829, 0830, 0833, 0834	B8, Null		26						32																48
*0802	B8		26						32											44					
*0803	B8		26						32																
*0804	-		26						32			35													48
*0806, 0813, 0820	B8		26																						48
*0807, 0814	-		26																						48
*0809	-		26						32																48
*0812, 0816, 0831	-		26						32																48
*0817			26						32			35													48
*0821	-		26	27					32																48
*0825	-		26	27																					48
*0826, 4207	-								32																48
*0828	-		26									36													48
*0832	-		26						32										42						48

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
----------	--	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*1301, 1317, 1320	B13			27	28						34							41							
*130201-130204, 1308Q, 1315, 1318, 1319	B13			27	28						34														
*1303	-			27	28																				
*1304, 1310	-			27	28							36													
*1306	-			27	28													41							
*1307N	Null			27	28						34							41							
*1309	-				28						34														
*1311	-			27	28						34														
*1312, 1313	-			27	28													41							
*1314	-			27							34														
*1316	-			27	28						34														
*1321	-			27	28																				
*1401, 1407N	B64				28	29																		47	48
*140201-1404	B65				28		30																	47	48
*1405-140602	-						30																	47	48
*15010101- 15010102N, 150103, 1504, 1526N, 152701, 1532, 1534, 1535, 1538, 1550, 1556, 1557, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509-9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529	B62, Null, B15							31				36							43						
*150102	B62							31		33			36						43						
*150104	B62											36							43						
*150106, 1507	B62			27				31					36						43						
*1502, 1588, 9512, 9521	B75							31				36						41	43						
*1503, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532	B72								32			35	36						43						
*1505, 1506, 1533, 1539, 1540, 1565, 1578, 1584, 9507	B62, B15							31				36							43						
*1508, 151101-151103, 1515	B75							31					36						43						

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*1509	B70								32	33	34									43					
*1510, 1537, 1590, 1599, 9533	B71								32		34									43					
*1512, 1514, 1519	B76							31					36	37						43					
*1513	B77							31					36		38			41		43					
*1516, 1567, 1595	B63									33			36		38					43					
*15170101-151702	B63							31		33					38					43					
*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534	B71								32		34		36							43					
*1520, 1525	B62							31					36					41		43					
*1521, 1544	B75							31			34		36					41		43					
*1523	-								32		34		36		38					43					
*1524, 1587	B62							31					36		38					43					
*152702	-			27									36												
*1528	B15							31					36							43					
*1529	B15								32				36							43					
*1530, 1558, 1563	B62, B15							31												43					
*1531	B75							31					36							43					
*1536	-			27				31					36					41		43					
*1542	-							31												43					
*1543	-							31					36							43					
*1545	B62			27				31												43					
*1546	B72				28			31	32				36							43					
*1548	B62							31												43					
*1553	-				28				32				36							43					
*1554	-								32			35	36							43					
*1555	-			27				31					36							43					
*1562	-								32			35	36					41		43					
*1564	-								32			35	36							43					
*1566	-							31			34		36							43					
*1568	-			27				31	32			35	36							43					
*1569	-								32			35	36							43		45			
*1573	-							31												43					
*1574	-								32				36							43					
*1576, 9501	-							31					36							43					
*1577	-							31										41		43					
*1580	-								32		34		36					41		43					
*1583	-							31	32											43					
*1585	-							31					36					41		43					
*1586	-							31												43					
*1589	-			27				31					36					41		43					
*1591, 9531	-								32			35	36	37						43					

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*9506	-				28			31	32				36					41		43					
*9515	-								32		34		36							43	44				
*9516	-							31					36							43					
*9520	-							31					36							43					
*9524	-								32		34		36							43					
*9526, 9535	-			27				31					36							43					
*9536	-			27				31					36							43		45			
*9537	-			27				31												43					
*9538	-	25						31					36							43					
*9539	-			27				31					36					41		43					
*180101-1803, 1805-1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825, 1827	B18												36			39			42						
*1804	-												36			39			42						
*1809	-												36			39				44					
*1812	-												36			39			42						
*1819	-												36			39			42						
*1822	-												36			39		41	42						
*1826	-															39			42						
*2701	B27					29											40								
*2702	B27					29											40								
*2703-270402, 270502-2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737	B27					29											40								
*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738	B27																40								
*2708, 2726	B2708					29											40								
*2712	-					29											40								
*2716	-					29											40								
*2718	-					29					35						40								
*2719	-																40	41							
*2723	-					29											40								
*2724	-																40								
*2725	-																40								
*2729	-					29					35						40								
*2730	-																40	41							
*2733	-																40								
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*350101-350106, 3507-350803, 3511, 3515, 352001-352002, 3523, 3529, 3540N-3542, 3548, 3550, 3552, 3554, 3557, 3561, 3577, 3578, 3582	B35												36					41	42						
*350201-350401, 3506, 3512, 3533, 3534, 3536, 3539, 3555, 3559, 3565Q, 3570, 3574, 3583	B35																	41	42						
*350402	-									33								41	42						
*3505, 3551, 3558, 3572	B35			27									36						42						
*350901-350902, 3518, 3531, 3575	B35																	42							
*3510, 3528, 3569	-												36					41	42						
*3513	B35																	41	42						
*351401-351402, 3562	B35												36					41	42						
*3516	-			27									36					41	42						
*3517, 3530	B35			27									36					41	42						
*3519	B35												36					41							
*3521, 3524	-												36					41	42						
*3522	-			27														42							
*3525, 3549	-												36					41							
*3526	-						30						36					41							
*3527	B35												36					41							
*3532, 3537, 3553N, 3564, 3568	-												36					42							
*3535	-												36					41	42			45			
*3538	-																	41	42						
*3543, 3567, 3579	B35												36						42						
*3544	-																	42							
*3545, 3571	-												36	37				41	42						
*3546	-												36					41							
*3547	-				28								36					41							
*3556	-																	41							
*3560	-																		42						
*3563	-				28								36					41							
*3566	-												36						42						
3576	-												36					41	42						
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*370101-370103, 3703N, 3706, 3713	B37, Null													37				41			44				
*370104	-													37				41			44				
*3702	-					29								37							44				
*3704	-													37				41			44				
*3705	-													37				41			w				
*3707	-	25												37							44				
*3708	-													37				41			44				
*3709	-													37							44				
*3710	-																	41			44				
*3711	-																	41	42						
*3712	-								32					37							44				
*380101-380102, 3809, 3811-3814, 3816	B38					29																45	46	47	
*380201-380202, 3808, 3815	B38					29														44		45	46	47	
*3803	B16					29						35								44		45	46		
*3804	-																			44		45		47	
*3805	-					30																45	46	47	
*3806	-																					45			
*3807	-					29																45			
*3810	-					29																	46	47	
*3817	-					29								37						44		45	46	47	
*39010101-39010102L, 390103-390104, 3903, 3905-390602, 3909, 3911, 3914, 3915, 3924, 3926-3929, 3931, 3935, 3937, 3938Q, 3940N, 3941	B3901, B39					29																45		47	48
*390201, 3908	B3902, B39					29						35										45			48
*390202, 391301- 391302, 3923	B3902, B39											35										45			48
*3904	B39					30																45		47	48
*3907	-					? ?						36										45		47	48
*3910, 3916, 3920	B39																					45			48
*3912	-																					45		47	48
*3917	-																					45			48
*3918	-					29																45		47	48
*3919	-																	42				45		47	48
*3922	-					29																45		47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*3925N, 3930, 3932-3934	-					29																		47	48
*3936	-					29																		47	48
*3939	-					29					35											45			48
*3942	-					29												41				45			
*3943	-					29					36														
*400101-400105, 4007	B60, B40				28																				48
*400201-400203, 4004, 40060101-400602, 4039, 4075, 4078	B61, B40																								
*4003, 4020	B40											36													
*4005	B4005		27																						
*4008	-																								
*4009, 4011, 4018, 4024, 4027, 4029, 4035, 4037, 4040, 4044, 4050, 4056, 4057, 4064	B61, B40																								?
*4010, 4022N, 4033, 4036, 4042, 4043, 4049, 4054, 4055, 4061-4063, 4065-4067, 4073	Null, B40, B60				28																				?
*4012	-								32		35								43						
*4013, 4019	-																								
*401401-401403, 4053, 4072	-				28																				?
*4015, 4016, 4032	B61	25			28																				?
*4021	-																								?
*4023	-	25			28																				48
*4025	-				28																				?
*4026	B21									33															?
*4028	-									33								41							?
*4030, 4034	-				28																				?
*4031, 4045	-				28																				?
*4038, 4052	-				28							36													?
*4046	-										35						40								?
*4047	-				28																				
*4048	-				28						34														?
*4051	-				28																				48
*4058	-				28																				?
*4059	-				28							36													48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*4060	-				28								36												48
*4068	-																								?
*4069	-				28																				?
*4070	-																								
*4071	-				27																				?
*4074	-																								?
*4076	-				28																				
*4101, 4106, 4107	B41				28				32																48
*4102	B41				28				32																48
*410301-410302	B41				28				32																48
*4104	-				28																				48
*4105	-				28				32																48
*4108	-	25			28				32						38										48
*4201, 4202, 4206	B42								32																48
*4204	-								32																48
*420501-420502	-	25							32						38										48
*4208	-								32																48
*4209	-												36												48
*44020101-440204, 4411, 4416, 4419N, 4421-4424, 4427, 4433, 4434, 4448, 4451, 4452N, 4455	B44, B47, Null				28													41							
*440301-4405, 4413, 4426, 4428-4430, 4432, 4435, 4436, 4438, 4439, 4442, 4445, 4456N	B44				28													41							
*4406	B44																	41							
*4407																		41							
*4408	B44				27													41							
*4409	B12				28													41							
*4410	-				28								35					41		43					
*4412	-				28													41							
*4414	-				28				32									41							
*4415	-				28				32				35												
*4417, 4443	-				28								36	37				41							
*4418	-				28				32				35												
*4420	-				28				32																
*4425	-				28													41							
*4431	-				28																				
*4437	-				28													41							
*4440	-												35				40	41							
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*4441	-				28																				
*4444	-											35					40	41							
*4446	-				28													41							
*4447	-				28																				
*4449	-		26		28													41							
*4450	-				28													41							
*4453	-				28																				
*4454	-				27	28																			
*4457	-			27														41							
*4501, 4505, 4507	B45				28				32			35													48
*4502, 4503	-				28				32																48
*4504	-				28				32																48
*4506	-				28				32			35													48
*460101-460102, 4604, 4605, 4607N, 4609	B46								31				36												
*4602	-								31				36												
*4603, 4608	-								31				36												
*4606	-								31				36												
*4610	-								31				36												
*4611	-			27					31																
*4612	-			27					31				36												
*47010101-47010102	B47				28																				
*4702	-				28																				
*4703	-				28																				
*4704, 4705	-				28	29																			
*480101-480102, 4807, 4809, 4811, 4819	B48											35													48
*4802	-											35	36					41							48
*480301-480302	B48											35													48
*4804	B48											35													48
*4805, 4808	B48	25										35													48
*4806	-																								48
*4810, 4812	-											35													48
*4813	-											35													48
*4814	-											35	36												48
*4815	-	25										35													48
*4816	-											35													48
*4817	-											35													48
*4818	-											35													
*4901	B49				28																				
*4902	-				28																				
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*4903	-																								
*4904, 4905	-				28								36												
*5001, 5004	B50				28																				48
*5002	B45				28							35													48
*510101, 510103, 510105-510201, 510203, 5103, 5107, 510901, 5111N, 5112, 511302, 5114, 5117- 5119, 5122, 5123, 5126-5130, 5132, 5133, 5138-5141N, 5143, 5148, 5149, 5151	B51, B5102, B5103, Null									33															
*510102, 510202, 5105, 510902	B51									33															
*510104, 511301, 5116, 5121, 5124, 5131, 5134, 5135	B51, B52																								
*5104, 5146	B51									33								41							
*5106	B51									33															
*5108, 5120	B51								32	33															
*5110	-																								
*5115	-																								
*5136	-								32																
*5137	-									33			36												
*5142	-																	41							
*5144N	Null								32	33															
*5145	-												36												
*5150										33															
*520101, 520103, 5204, 5205, 5207, 5210, 5211	B52									33															
*520102, 520104-5203, 520601-520602, 5209	B52									33															
*5208	-																								
*530101-530104, 5305, 5308, 5310	B53												36					41							
*5302	B53												36					41							
*5303, 5309, 5311- 5313	-												36					41							
*5304, 5307	-																	41							
*5306	B53												36					41							

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
----------	--	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*5401, 5402, 5405N, 5407, 5408N, 5410, 5413	B54																								
*5403	-																								
*5404, 5411	-																								
*5406	-												36												
*5409	-												36												
*5412	-																								
*550101-550104, 5503, 5505, 5511, 5515, 5525	B55																								
*550201-550202, 5507, 5510, 5512, 5516, 5519, 5526	B54, B55																								
*5504, 5508, 5513, 5527, 5615, 5619N	B55																								
*5509, 5517, 5528	-																								
*5514	-												36					41							
*5518	-											35													
*5520	-																								
*5521	-																								
*5522	-											34													
*5523, 5618	-												36												
*5524	-											34													
*5601, 5608, 5614, 5616, 5620	B56																								
*5602, 5604	B56																								
*5603	B22												36												
*560501	-									33															
*560502	-									33															
*5606	B78									33								42							
*5607	-																								
*5609	-											36						41							
*5610	-																								
*5611, 5612	-																	41							
*5613	-								32																
*5617	-																								
*5621	-									33															
*570101-570103, 5706, 5708, 5710	B57											36						41							
*5702-5705	B57																	41							
*5707	-																	41							
*5709	-								32									41							
*5711	-											36													

Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*5712	-																	41							
*5713													36												
*5714				27									36					41							
*5801, 5804, 5805, 5810N-5814	B58												36					41							
*5802, 5806	B58												36												
*5807	-												36	37											
*5808	-									33															
*5809	-												36					41							
*5815	-												36					41							
*5901	B59																								
*5902	-																								
*670101	B67					29																45			48
*670102	B67																					45			48
*6702	-																					45			48
*7301	B73																								48
*7801, 780202, 7803	B78									33								42							
*780201, 7804	B78									33								42							
*7805	-									33								42							
*8101	B81																								
*8102	-																								48
*8103	-																								
*8201, 8202	-								32			35						43							
*8301	-																	41							
HLA-B allele ⁴	ser. ⁵																								
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Cw*0206												?													
Cw*0305, 0325, 0327				27																					
Cw*0746									32																
Well No.		25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
			52																			71	72	B7	*070201-070207, 070501-0706, 0710, 0712, 0714, 071801- 071802, 0721-0723, 0726, 0728-0731, 0734, 0735, 0739, 0741, 0743-0747, 0749N, 0752, 0755- 0759
																						71	72	B7, B703	*0703, 0708, 0716, 0732, 0737
		51	52																			71	72	B7	*0704, 0725
			52																			71		B7	*0707, 0742
			52																			71	72	B7	*0709, 0711, 0717
			w																			71	72	-	*0713, 0715
		51	52																			71	72	-	*0719
			52																			71	72	-	*0720, 0724
																				70		72		-	*0727
			52																			71		-	*0733
			52																	70		72		-	*0736
																				70		72		-	*0738
			52																			71	72	-	*0740, 0748, 0751
			52																			71		-	*0753
																						71	72	-	*0750
			52									62										71	72	-	*0754
			52																			71		-	*0760
		51																				71	72	B8, Null	*080101-080105, 0805, 0808N, 0810, 0811, 0815, 0818, 0819N, 0822-0824, 0827, 0829, 0830, 0833, 0834
		51																			70		72	B8	*0802
		51																			70		72	B8	*0803
		51																				71	72	-	*0804
																						71	72	B8	*0806, 0813, 0820
		51																				71	72	-	*0807, 0814
		51			54																	71		-	*0809
		51																				71		-	*0812, 0816, 0831
		51										62										71	72		*0817
		51																				71	72	-	*0821
																						71	72	-	*0825
		51																				71	72	-	*0826, 4207
		51																				71	72	-	*0828
		51																	68			71	72	-	*0832

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72			Well No.
----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	--	--	----------

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
												62								70				B13	*1301, 1317, 1320
												62								70				B13	*130201-130204, 1308Q, 1315, 1318, 1319
												62				65				70				-	*1303
												62								70				-	*1304, 1310
												62		64						70				-	*1306
																				70				Null	*1307N
												62									71			-	*1309
												62					67			70				-	*1311
												62								70				-	*1312, 1313
												62								70				-	*1314
									59			62								70				-	*1316
												62								70				-	*1321
																					71			B64	*1401, 1407N
																					71			B65	*140201-1404
																					71			-	*1405-140602
												62								69	71			B62, Null, B15	*15010101- 15010102N, 150103, 1504, 1526N, 152701, 1532, 1534, 1535, 1538, 1550, 1556, 1557, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509-9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529
												62								69	71			B62	*150102
												62	63							69	71			B62	*150104
												62								69	71			B62	*150106, 1507
								57													71			B75	*1502, 1588, 9512, 9521
												62									71			B72	*1503, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532
												62									71			B62, B15	*1505, 1506, 1533, 1539, 1540, 1565, 1578, 1584, 9507
								57												69	71			B75	*1508, 151101-151103, 1515

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																						71		B70	*1509
																						71		B71	*1510, 1537, 1590, 1599, 9533
													62							69		71		B76	*1512, 1514, 1519
								57													70			B77	*1513
									59												70			B63	*1516, 1567, 1595
																					70			B63	*15170101-151702
																						71		B71	*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534
													62									71		B62	*1520, 1525
								57														71		B75	*1521, 1544
																					70			-	*1523
													62							69	70			B62	*1524, 1587
													62							69		71		-	*152702
																				69		71		B15	*1528
																						71		B15	*1529
													62							69		71		B62, B15	*1530, 1558, 1563
								57														71		B75	*1531
													62								70			-	*1536
									59				62		64							71		-	*1542
													62							69	70			-	*1543
													62							69		71		B62	*1545
												61	62							69		71		B72	*1546
													62									71		B62	*1548
												61	62							69		71		-	*1553
													62							69		71		-	*1554
								57														71		-	*1555
													62									71		-	*1562
																						71		-	*1564
													62							69		71		-	*1566
													62							69		71		-	*1568
													62									71		-	*1569
													62			65		67		69		71		-	*1573
													62									71		-	*1574
			52																	69		71		-	*1576, 9501
													62							69		71		-	*1577
																						71		-	*1580
					54								62									71		-	*1583
													62							69		71		-	*1585
													62		64							71		-	*1586
								57													70			-	*1589
													62									71		-	*1591, 9531

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
												61	62								71			-	*9506
																					70			-	*9515
49												62									71			-	*9516
																				69	71			-	*9520
49																					71			-	*9524
												62								69	71			-	*9526, 9535
												62									71			-	*9536
										59		62								69	71			-	*9537
												62									71			-	*9538
								57													71			-	*9539
																		68			71			B18	*180101-1803, 1805-1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825, 1827
													64					68			71			-	*1804
																		68			70			-	*1809
												62									71			-	*1812
																		68	69		71			-	*1819
						55												68			71			-	*1822
																		68			71			-	*1826
			52									62									70			B27	*2701
			52				56			60		62									70			B27	*2702
			52				56					62									70			B27	*2703-270402, 270502-2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737
			52				56					62									70			B27	*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738
			52				56					62									71			B2708	*2708, 2726
												62									71			-	*2712
												62									70			-	*2716
												62									71			-	*2718
			52				56					62									70			-	*2719
																					70			-	*2723
			52				56	57				62									70			-	*2724
			52				56					62								69	70			-	*2725
												62									70			-	*2729
			52				56			60		62									70			-	*2730
			52				56					62										71		-	*2733
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																			68			71		B35	*350101-350106, 3507-350803, 3511, 3515, 352001-352002, 3523, 3529, 3540N-3542, 3548, 3550, 3552, 3554, 3557, 3561, 3577, 3578, 3582
																			68			71		B35	*350201-350401, 3506, 3512, 3533, 3534, 3536, 3539, 3555, 3559, 3565Q, 3570, 3574, 3583
																			68			71		-	*350402
																			68			71		B35	*3505, 3551, 3558, 3572
																			68			71		B35	*350901-350902, 3518, 3531, 3575
												62										71		-	*3510, 3528, 3569
												62										71		B35	*3513
																			68	69		71		B35	*351401-351402, 3562
												62										71		-	*3516
																			68			71		B35	*3517, 3530
												61										71		B35	*3519
						55													68			71		-	*3521, 3524
																			68			71		-	*3522
																						71		-	*3525, 3549
																						71		-	*3526
																			68			71		B35	*3527
																			68			71		-	*3532, 3537, 3553N, 3564, 3568
																			68			71		-	*3535
				53															68			71		-	*3538
																			68	69		71		B35	*3543, 3567, 3579
																			68	69		71		-	*3544
				53															68			71		-	*3545, 3571
								57														71		-	*3546
												61										71		-	*3547
																			68			71		-	*3556
															64				68			71		-	*3560
												61	62									71		-	*3563
																			68			71	72	-	*3566
			52				56												68			71		-	3576
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
				53								62									70			B37, Null	*370101-370103, 3703N, 3706, 3713
				53								62	63								70			-	*370104
												62									70			-	*3702
				53		55						62									70			-	*3704
				53								62										71		-	*3705
												62									70	72		-	*3707
				53														68			70			-	*3708
												62									70			-	*3709
				53								62									70			-	*3710
				53								62										71		-	*3711
				53								62									70			-	*3712
																					70			B38	*380101-380102, 3809, 3811-3814, 3816
																					70			B38	*380201-380202, 3808, 3815
												62									70			B16	*3803
												62									70			-	*3804
																					70			-	*3805
																		68			70			-	*3806
																		68			70			-	*3807
																					70			-	*3810
																					70			-	*3817
																						71		B3901, B39	*39010101-39010102L, 390103-390104, 3903, 3905-390602, 3909, 3911, 3914, 3915, 3924, 3926-3929, 3931, 3935, 3937, 3938Q, 3940N, 3941
												62									71			B3902, B39	*390201, 3908
												62										71		B3902, B39	*390202, 391301- 391302, 3923
																					71			B39	*3904
																						71		-	*3907
																					71			B39	*3910, 3916, 3920
																						71		-	*3912
													64				67				71			-	*3917
																			69			71		-	*3918
																	68				71			-	*3919
												62										71		-	*3922
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																					71			-	*3925N, 3930, 3932-3934
																				69	71			-	*3936
																					71			-	*3939
																					71			-	*3942
																					71			-	*3943
49												61	62									71		B60, B40	*400101-400105, 4007
	50											61	62									71		B61, B40	*400201-400203, 4004, 40060101-400602, 4039, 4075, 4078
	50											61	62									71		B40	*4003, 4020
	50											61	62									71		B4005	*4005
	50											61										71		-	*4008
	50											61	62									71		B61, B40	*4009, 4011, 4018, 4024, 4027, 4029, 4035, 4037, 4040, 4044, 4050, 4056, 4057, 4064
49												61	62									71		Null, B40, B60	*4010, 4022N, 4033, 4036, 4042, 4043, 4049, 4054, 4055, 4061-4063, 4065-4067, 4073
49												62										71		-	*4012
	50							58	60			62									70			-	*4013, 4019
												61	62									71		-	*401401-401403, 4053, 4072
												61	62									71	72	B61	*4015, 4016, 4032
49												62										71		-	*4021
49												61	62									71		-	*4023
49												61										71		-	*4025
	50											61	62									71		B21	*4026
	50				55							61	62									71		-	*4028
												61	62									71	72	-	*4030, 4034
								57				61	62									71	72	-	*4031, 4045
49												61	62									71		-	*4038, 4052
49												62										71		-	*4046
49												62									70			-	*4047
49												61	62				67					71		-	*4048
49												61	62									71		-	*4051
												61	62		64							71		-	*4058
												61	62									71	72	-	*4059
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49												61	62								71			-	*4060
	50											61									71			-	*4068
49												61									71			-	*4069
	50											61	62								71			-	*4070
	50											61	62			65	67				71			-	*4071
49	50											61	62								71			-	*4074
49												62												-	*4076
		51			54							61	62								71			B41	*4101, 4106, 4107
		51										61	62								71	72		B41	*4102
		51										61	62								71			B41	*410301-410302
		51										61	62								71	72		-	*4104
		51			54							? 62									71			-	*4105
		51										61	62								71			-	*4108
		51	52																		71	72		B42	*4201, 4202, 4206
		51	52		54																71			-	*4204
		51	52																		71	72		-	*420501-420502
			52																		71			-	*4208
		51	52																		71	72		-	*4209
				53								62	63								70			B44, B47, Null	*44020101-440204, 4411, 4416, 4419N, 4421-4424, 4427, 4433, 4434, 4448, 4451, 4452N, 4455
				53								62									70			B44	*440301-4405, 4413, 4426, 4428-4430, 4432, 4435, 4436, 4438, 4439, 4442, 4445, 4456N
				53	54					60		63				68				70			B44	*4406	
				53								62								70				*4407	
				53								62	63							70			B44	*4408	
				53								61	62	63							71		B12	*4409	
				53								62				65				70			-	*4410	
				53									63							70			-	*4412	
				53								62								70			-	*4414	
					54							62				65				70			-	*4415	
				53								62								70			-	*4417, 4443	
					54				58			62				65				70			-	*4418	
				53	54							62	63							70			-	*4420	
				53					58	60		62								70			-	*4425	
49				53								62								70			-	*4431	
												62								70			-	*4437	
				53								62								70			-	*4440	
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
				53								62	63								70			-	*4441
				53								62	63								70			-	*4444
				53								61	62									71		-	*4446
				53								62									70			-	*4447
				53								62	63								70			-	*4449
				53				58		60		62									70			-	*4450
				53								62	63								70			-	*4453
				53								62									70			-	*4454
				53								62									70			-	*4457
				54								61	62			65					71			B45	*4501, 4505, 4507
				54								61	62								71			-	*4502, 4503
				54								61	62			65					71			-	*4504
			52	54								61				65					71			-	*4506
			52			55															69	71		B46	*460101-460102, 4604, 4605, 4607N, 4609
			w			55															69	71		-	*4602
			52			55															71			-	*4603, 4608
49			52			55																71		-	*4606
			52			55						62									69	71		-	*4610
			52			55						62				65						71		-	*4611
			52			55						62									69	71		-	*4612
							56					62									70			B47	*47010101-47010102
							56					61	62								71			-	*4702
							56					62									70	w		-	*4703
												62									70			-	*4704, 4705
								57				62										71	72	B48	*480101-480102, 4807, 4809, 4811, 4819
												62									71			-	*4802
49												62									71			B48	*480301-480302
								57				62									71			-	*4804
												62									71	72		B48	*4805, 4808
								57													71	72		-	*4806
												62									71	72		-	*4810, 4812
												62									71			-	*4813
												62									71	72		-	*4814
								57				62									71	72		-	*4815
								57													71	72		-	*4816
49												62									71	72		-	*4817
								57				62									70		72	-	*4818
								58	59			62				65					70			B49	*4901
									59			62				65					70			-	*4902
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
								59				62				65				70				-	*4903
								58	59			62								70				-	*4904, 4905
								59				61	62			65				71				B50	*5001, 5004
									59			61	62			65					71			B45	*5002
					54					59	60						68				70			B51, B5102, B5103, Null	*510101, 510103, 510105-510201, 510203, 5103, 5107, 510901, 5111N, 5112, 511302, 5114, 5117- 5119, 5122, 5123, 5126-5130, 5132, 5133, 5138-5141N, 5143, 5148, 5149, 5151
										59	60							68			70			B51	*510102, 510202, 5105, 510902
					54					59	60						68				70			B51, B52	*510104, 511301, 5116, 5121, 5124, 5131, 5134, 5135
						54	55				60							68			70			B51	*5104, 5146
					54						60							68			70			B51	*5106
					54					59	60							68			70			B51	*5108, 5120
					54						60							68			70			-	*5110
											60					65		68			70			-	*5115
					54					59	60							68			70			-	*5136
					54					59	60							68			70			-	*5137
				53	54						60							68			70			-	*5142
					54					59	60							68			70			Null	*5144N
					54						60							68			70			-	*5145
										59								68			70				*5150
										59	60		62								70			B52	*520101, 520103, 5204, 5205, 5207, 5210, 5211
					54					59	60		62								70			B52	*520102, 520104-5203, 520601-520602, 5209
										59	60		62								70			-	*5208
											60							68			70			B53	*530101-530104, 5305, 5308, 5310
					55						60							68			70			B53	*5302
																		68			70			-	*5303, 5309, 5311- 5313
											60							68			70			-	*5304, 5307
					54	55					60							68			70			B53	*5306

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72			Well No.
----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	----	--	--	----------

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
			52				56							63	64							71		B54	*5401, 5402, 5405N, 5407, 5408N, 5410, 5413
			52				56							63		65						71		-	*5403
			52				56							63								71		-	*5404, 5411
			52				56							63						69		71		-	*5406
			52				56							63								71		-	*5409
			52				56							63	64						70			-	*5412
			52				56			59					64							71		B55	*550101-550104, 5503, 5505, 5511, 5515, 5525
			52				56								64							71		B54, B55	*550201-550202, 5507, 5510, 5512, 5516, 5519, 5526
			52				56															71		B55	*5504, 5508, 5513, 5527, 5615, 5619N
			52				56			59												71		-	*5509, 5517, 5528
			52				56										66					71		-	*5514
													62		64							71		-	*5518
		51	52		54		56															71		-	*5520
			52				56			59					64					69		71		-	*5521
			52				56															71		-	*5522
			52				56															71		-	*5523, 5618
			52				56			59												71		-	*5524
			52				56									65						71		B56	*5601, 5608, 5614, 5616, 5620
			52				56									65		67				71		B56	*5602, 5604
			52				56													69		71		B22	*5603
			52				56			59					64							71		-	*560501
			52				56			59												71		-	*560502
			52				56			59					64							71		B78	*5606
			52				56									65					70			-	*5607
			52				56															71		-	*5609
			52				56								64			67				71		-	*5610
			52				56															71		-	*5611, 5612
			52		54		56									65						71		-	*5613
																65						71		-	*5617
			52				56			59											70			-	*5621
										60							66					70		B57	*570101-570103, 5706, 5708, 5710
										60							66				70			B57	*5702-5705
				53						60							66				70			-	*5707
				53						60							66				70			-	*5709
										60							66				70			-	*5711

49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
												62				66					71			-	*5712
										60		62				66					70				*5713
						55				60		62				66					70				*5714
										60						67					70			B58	*5801, 5804, 5805, 5810N-5814
										60						67					70			B58	*5802, 5806
										60						67					70			-	*5807
					54					59	60					67					70			-	*5808
						55				60						67					70			-	*5809
										60		62				67					70			-	*5815
												64									70			B59	*5901
										60											70			-	*5902
			52																		71			B67	*670101
			52																		71			B67	*670102
			w																		71			-	*6702
			52																	69	71			B73	*7301
										59			64					68			71			B78	*7801, 780202, 7803
										59								68			71			B78	*780201, 7804
										59		62									71			-	*7805
			52				56	57													71	72		B81	*8101
			52					57													71	72		-	*8102
			52					57													71	72		-	*8103
			52				56									65					71			-	*8201, 8202
			52	53			56						63								71			-	*8301
																								ser. ⁵	HLA-B allele ⁴
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.
																67									Cw*0206
																									Cw*0305, 0325, 0327
																									Cw*0746
49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72		Well No.