

*0838	-	26		32				48	51			62		71 72	-
*130101, 130102, 1317, 1320, 132201-1323, 1325	B13	27 28			34		41					62		70	B13
*130201-130205, 1308Q, 1315, 1318, 1319	B13	27 28			34							62		70	B13
*1303	-	27 28										62	65	70	-
*1304, 1310	-	27 28			36							62		70	-
*1306	-	27 28					41					62	64	70	-
*1307N	Null	27 28			34		41							70	Null
*1309	-	28			34							62		71	-
*1311	-	27 28			34							62	67	70	-
*1312, 1313	-	27 28					41					62		70	-
*1314	-	27			34							62		70	-
*1316	-	27 28			34					59		62		70	-
*1321	-	27 28										62		70	-
*1326	-	27 28			36		41					62		70	-
*1401, 1407N	B64	28 29						47 48						71	B64
*140201-1404, 1409	B65	28	30					47 48						71	B65
*1405-140602	-		30					47 48						71	-
*1408, 3925N, 3930, 3932-3934, 3947, 3950	-		29					47 48						71	-
*1410	-		29				41	47 48		55				71	-
*15010101, 15010102N, 150103, 150106-150108, 1526N-152703, 1532, 1534, 1535, 1538, 1550, 1556, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509-9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529, 9535, 9540, 9545-9547, 9552, 9559, 9560, 9563, 9564	B62, Null, B15			31		36		43				62		69 71	B62, Null, B15
*150102	B62			31	33	36		43				62		69 71	B62
*150104	B62					36		43				62 63		69 71	B62
*1502, 1588, 9512, 9521, 9539	B75			31		36		41 43			57			71	B75
*150301, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532, 9551, 9556, 9558	B72				32	35 36		43				62		71	B72
*1504				31		36		43			w	62		69 71	
*1505, 1506, 1533, 1539, 1540, 1565, 157801, 1584, 9507, 9536, 9541, 9555	B62, B15			31		36		43				62		71	B62, B15
*1507, 9526	B62	27		31		36		43				62		69 71	B62

*1508, 151101-151103, 1515, 9548	B75			31		36			43					57				69	71	B75		
*1509	B70			32	33	34			43										71	B70		
*1510, 1537, 1590, 1599, 9533	B71			32		34			43										71	B71		
*1512, 1514, 1519	B76			31			36	37	43						62			69	71	B76		
*1513	B77			31			36	38	41	43				57				70		B77		
*1516, 1567, 1595	B63				33		36	38		43					59			70		B63		
*15170101-151702	B63			31	33			38		43								70		B63		
*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534, 9553	B71			32		34	36			43									71	B71		
*1520, 152501	B62			31			36		41	43					62				71	B62		
*1521, 1544	B75			31		34	36		41	43				57					71	B75		
*1523	-			32		34	36	38		43								70		-		
*1524, 1587, 9557	B62			31			36	38		43					62			69	70	B62		
*1528, 9520, 9549N	B15			31			36			43								69	71	B15		
*1529	B15			32			36			43									71	B15		
*1530, 1558, 1563	B62, B15			31						43					62			69	71	B62, B15		
*1531	B75			31			36			43				57					71	B75		
*1536	-		w	31			36		41	43								70		-		
*1542	-			31						43					59		64		71	-		
*1543	-			31			36			43								69	70	-		
*1545	B62		27	31						43								69	71	B62		
*1546	B72			28	31	32		36		43					61	62		69	71	B72		
*1548, 9550	B62				31					43						62			71	B62		
*1553				28		32		36		43						61	62	69	71			
*1554	-				32		35	36		43						62		69	71	-		
*1555	-		27		31			36		43				57					71	-		
*1557	-				31			36		43			w					69	71	-		
*1562	-				32		35	36	41	43						62			71	-		
*1564	-				32		35	36		43									71	-		
*1566	-				31		34	36		43						62		69	71	-		
*1568	-		27				35	36								62		69	71	-		
*1569	-				32		35	36		43	45					62			71	-		
*1573	-				31					43						62		65	67	69	71	-
*1574	-				32			36		43						62			71	-		
*1576	-				31			36		w		52		w				69	71	-		
*1577	-				31				41	43						62		69	71	-		
*1580	-				32		34	36	41	43									71	-		
*1583	-				31	32				43			54			62			71	-		
*1585, 9554	-				31			36	41	43						62		69	71	-		
*1586	-				31					43						62	64		71	-		
*1589	-		w		31			36	41	43				57				70		-		
*1591, 9531	-				32		35	36	37	43						62			71	-		
*9501	-				31			36		43		52				62		69	71	-		

*9506	-	28		31 32	36			41 43					61 62			71	-
*9515	-			32	34 36			43							70	-	
*9516	-			31	36			43	49				62			71	-
*9524	-			32	34 36			43	49						71		-
*9537	-			31				43			w		62			69 71	-
*9538	-	25		31	36			43					62			71	-
*9542	-	26		31	36			43					62			69 71	-
*9543	-			31 32	36			43					61			69 71	-
*9544	-			31	36			41 43			57					69 71	-
*9561	-			32	34 35 36	37		43							71		-
*180101-1803, 1805-1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825, 1827, 1828, 1830-1833, 1835	B18			36		39		42							68	71	B18
*1804	-			36		39		42					64		68	71	-
*1809	-			36		39									68	70	-
*1812	-			36		39		42					62			71	-
*1819	-			36		39		42							68 69	71	-
*1822	-			36		39		41 42	55						68	71	-
*1826	-					39		42							68	71	-
*1829, 3532, 3537, 3553N, 3564, 356801, 356802, 3599	-			36				42							68	71	-
*2701	B27	29				40				52			62			70	B27
*2702	B27	29				40				52	56	60	62			70	B27
*2703-2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737, 2745, 2746, 2751	B27	29				40				52	56		62			70	B27
*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738, 2743	B27					40				52	56		62			70	B27
*2708, 2726, 2740, 2742, 2744	B2708	29				40				52	56		62			71	B2708
*2712	-	29				40							62			71	-
*2716, 2739	-	29				40							62			70	-
*2718	-	29		35		40							62			71	-
*2719	-					40		41	52		56		62			70	-
*2723	-	29				40										70	-
*2724	-					40				52	56	57	62			70	-
*2725	-					40				52	56		62			69 70	-
*2729	-	29		35		40							62			70	-
*2730	-					40		41	52		56	60	62			70	-
*2733	-					40				52	56		62			71	-
*2741	-	29		36		40				52	56		62			70	-
*2750	-	29		35		40				52	56		62			70	-
*2752	-	29								52	56		62			70	-

[illegible]

*3705	-							37			41		44					53								62					71		-		
*3707	-	25						37					44														62				70	72	-		
*3708	-							37			41		44					53											68		70		-		
*3709	-							37					44																		70		-		
*3710	-										41		44					53													70		-		
*3711	-										41	42	44					53													71		-		
*3712	-					32			37				44					53													70		-		
*3714	-										41		44					53													70	w	-		
*380101-380202, 3808, 3809, 3811-3816, 3818, 3819	B38				29									45	46	47															70		B38		
*3803	B16				29				35					45	46																70		B16		
*3804	-													45		47															70		-		
*3805	-					30								45	46	47															70		-		
*3806	-													45																	68	70	-		
*3807	-				29									45																	68	70	-		
*3810	-				29											46	47														70		-		
*3817	-				29				37				44	45	46	47															70		-		
*39010101, 39010102L, 390103-390106, 3903, 3905-390602, 3909, 3911, 3914, 3915, 3924, 3926-3929, 3931, 3935, 3937, 3938Q, 3940N, 3941, 3944, 3946, 3948	B3901, B39				29									45		47	48															71		B3901, B39	
*390201, 3908	B3902, B39				29				35					45			48														71		B3902, B39		
*390202, 391301, 391302, 3923, 3949	B3902, B39								35					45			48															71		B3902, B39	
*3904	B39					30								45		47	48															71		B39	
*3907	-				? ?				36					45		47	48															71		-	
*3910, 3916, 3920	B39													45			48															71		B39	
*3912, 3945	-													45		47	48															71		-	
*3917	-													45			48															64	67	71	-
*3918	-				29									45		47	48																69	71	-
*391901	-											42		45		47	48															68	71	-	
*391902	-				29							42		45		47	48															68	71	-	
*3922	-				29									45		47	48															62	71	-	
*3936	-				29											47	48															69	71	-	
*3939	-				29				35					45			48															71		-	
*3942	-				29							41		45		47	48															71		-	
*3943	-				29				36							47	48															71		-	
*400101-400106, 4007, 4051, 4066, 4073, 4079	B60, B40				28												48	49														71		B60, B40	
*400201-400205, 4004, 40060101-400602, 4039, 4070, 4089, 4097	B61, B40																		50													71		B61, B40	

*4003, 4020	B40						36										61 62				71	B40
*4005	B4005		27														61 62				71	B4005
*4008	-																61				71	-
*4009, 4011, 4018, 4024, 4027, 4029, 4035, 4037, 4040, 4044, 4050, 4056, 4057, 4064, 4075, 4078, 4082, 4085, 4086, 4090, 4091	B61, B40												?	50			61 62				71	B61, B40
*4010, 4022N, 4033, 4036, 4042, 4043, 4049, 4054, 4055, 4061-4063, 4065, 4067, 4081, 4084, 4087, 4088, 4092	Null, B40, B60			28									?	49			61 62				71	Null, B40, B60
*4012	-				32		35				43			49			62				71	-
*4013, 4019	-													50			58 60	62			70	-
*401401-401403, 4053	-		28										?				61 62				71	-
*4015, 4032	B61	25	28										?				61 62				71 72	B61
*4016	-	25	28										48				61 62				71 72	-
*4021	-												?	49			62				71	-
*4023	-	25	28										48	49			61 62				71	-
*4025, 4069	-		28										?	49			61				71	-
*4026	B21					33							?	50			61 62				71	B21
*4028	-					33				41			?	50		55	61 62				71	-
*4030, 4034	-		28										?				61 62				71 72	-
*4031, 4045, 4080	-		28										?			57	61 62				71 72	-
*4038, 4052	-		28					36					?	49			61 62				71	-
*4046	-						35			40			?	49			62				71	-
*4047	-		28											49			62				70	-
*4048	-		28				34						?	49			61 62		67		71	-
*4058	-		28										?				61 62	64			71	-
*4059	-		28					36					48				61 62				71 72	-
*4060	-		28					36					48	49			61 62				71	-
*4068	-												?	50			61				71	-
*4071	-		27										?	50			61 62		65 67		71	-
*407201, 407202, 4077	-		28										48				61 62				71	-
*4074	-												?	49 50			61 62				71	-
*4076	-		28											49			62					-
*4083	-												?				61 62				71	-
*4093	-						35			40			?				62				71	-
*4094	-												?	50			61 62	63			71	-
*4095	-												?	50			61 62				71	-
*4096	-												?	50			62				70	-
*4098	-	25											?	50			61 62				71 72	-
*4101, 4106, 4107	B41		28		32								48		51	54	61 62				71	B41
*410201	B41		28		32								48		51		61 62				71 72	B41
*410301, 410302	B41		28		32								48		51		61 62				71	B41

*4104	-		28						48	51				61 62			71 72	-
*4105	-		28		32				48	51	54			? 62			71	-
*4108	-	25	28		32			38	48	51				61 62			71	-
*4201, 4202, 4206, 4210, 4212	B42				32				48	51 52							71 72	B42
*4204	-				32				48	51 52	54						71	-
*420501, 420502	-	25			32			38	48	51 52							71 72	-
*4208	-				32				48	52							71	-
*4209	-						36		48	51 52							71 72	-
*4211	-				32	35			48	51				62			71 72	-
*44020101-440205, 440208, 4411, 4416, 4419N, 4421-4424, 4427, 4433, 4434, 4448, 4452N, 4455, 4459, 4463, 4466, 4467	B44, B47, Null		28						41		53			62 63			70	B44, B47, Null
*440301-4405, 4413, 4426, 442801-4430, 4432, 4435, 4436, 4438, 4439, 4442, 4445, 4451, 4456N, 4458N, 4461N, 4469	B44		28						41		53			62			70	B44
*4406	B44								41		53 54		60	63		68	70	B44
*4407									41		53			62			70	
*4408	B44		w						41		53			62 63			70	B44
*4409	B12		28						41		53			61 62 63			71	B12
*4410	-		28			35			41	43				62		65	70	-
*4412	-		28						41		53			63			70	-
*4414	-		28		32				41		53			62			70	-
*4415	-		28		32	35					54			62		65	70	-
*4417, 4443	-		28			36 37			41		53			62			70	-
*4418	-		28		32	35					54		58	62		65	70	-
*4420	-		28		32						53 54			62 63			70	-
*4425, 4450	-		28						41		53		58 60	62			70	-
*4431	-		28							49	53			62			70	-
*4437, 446401, 446402	-		28						41					62			70	-
*4440	-					35		40	41		53			62			70	-
*4441, 4453	-		28								53			62 63			70	-
*4444	-					35		40	41		53			62 63			70	-
*4446	-		28						41		53			61 62			71	-
*4447, 4465	-		28								53			62			70	-
*4449	-	26	28						41		53			62 63			70	-
*4454	-		27 28								53			62			70	-
*4457	-		w						41		53			62			70	-
*4460	-		27 28						41		53			62			70	-
*4462	-		28		32	33			41		53			62			70	-
*4501, 4505, 4507	B45		28		32	35			48		54			61 62		65	71	B45
*4502, 4503	-		28		32				48		54			61 62			71	-
*4504	-		28		32				48		54			61 62		65	71	-

*4506	-		28		32		35					48		52		54				61	62		65				71	-	
*4508	-		28		32							48				54				61	62	64					71	-	
*4509	-		28		32		36	37				48				54				61	62						71	-	
*460101, 460102, 4604, 4605, 4607N, 4609, 4610, 4614-4616	B46				31		36							52		55										69	71	B46	
*4602	-				31		36							w		55										69	71	-	
*4603, 4608, 461301, 461302, 4619, 4621	-				31		36							52		55											71	-	
*4606	-				31		36					49		52		55											71	-	
*4611	-				31									52		55						65					71	-	
*4612	-		27		31		36							52		55										69	71	-	
*4617	-				31		36	37						52		55										69	71	-	
*4618	-				31									52		55						64					71	-	
*47010101, 47010102	B47		28													56				62						70		B47	
*4702	-		28													56				61	62						71	-	
*4703	-		28													56				w	62					70	w	-	
*4704, 4705	-		28	29																62						70		-	
*480101, 480102, 4807, 4809, 4811, 4819	B48						35					48						57		62							71	72	B48
*4802	-						35	36		41		48								62							71	-	
*480301, 480302	B48						35					48	49							62							71	B48	
*4804	B48						35					48						57		62							71	B48	
*4805, 4808	B48	25					35					48								62							71	72	B48
*4806	-											48						57									71	72	-
*4810, 4812, 4817	-						35					48								62							71	72	-
*4813	-						35					48								62							71	-	-
*4814	-						35	36				48								62							71	72	-
*4815	-	25					35					48						57		62							71	72	-
*4816	-						35					48						57									71	72	-
*4818	-						35											57		62						70	72	-	
*490101	B49		28															58	59	62		65				70		B49	
*490102	-		28															58		62		65				70		-	
*4902	-		28																59	62		65				70		-	
*4903	-																		59	62		65				70		-	
*4904, 4905	-		28				36											58	59	62						70		-	
*5001, 5004	B50		28									48							59	61	62		65				71	B50	
*5002	B45		28				35					48							59	61	62		65				71	B45	
*510101, 510103, 510105-510108, 510201, 510203, 5103, 5107, 510901, 5111N, 5112, 511302, 5114, 5117-5119, 5122, 5123, 5126-5130, 5132, 5133, 5138-5141N, 5143, 5148, 5149, 5151-5153, 5155, 5157, 5158, 5160, 5165	B51, B5102, B5103, Null					33										54			59	60					68	70		B51, B5102, B5103, Null	

*510102, 510109, 510202, 5105, 510902	B51					33									59	60				68	70		B51
*510104, 510204, 511301, 5116, 5121, 512401, 512402, 5131, 5134, 5135	B51, B52												54		59	60				68	70		B51, B52
*5104, 5146, 5156	B51					33			41				54	55		60				68	70		B51
*5106, 5159	B51					33							54			60				68	70		B51
*5108, 5120, 5144N	B51				32	33							54		59	60				68	70		B51
*5110	-												54			60				68	70		-
*5115	-															60			65	68	70		-
*5136	-				32								54		59	60				68	70		-
*5137, 5163	-					33		36					54		59	60				68	70		-
*5142	-								41				53	54		60				68	70		-
*5145	-							36					54			60				68	70		-
*5150, 5154	-					33									59					68	70		-
*5161	-					33							54		w	60				68	69	70	-
*5162	-												54			60			65	67	68	70	-
*5164	-		27			33							54			60				68	70		-
*520101, 520103, 5204, 5205, 5207, 5210-5213	B52					33									59	60	62				70		B52
*520102, 520104-5203, 520601, 520602, 5209	B52					33							54		59	60	62				70		B52
*5208, 5214	-														59	60	62				70		-
*530101-530104, 5305, 5308, 5310, 5316, 5318	B53						36		41							60				68	70		B53
*5302	B53						36		41					55		60				68	70		B53
*5303, 5309, 5311-5313	-						36		41											68	70		-
*5304, 5307	-								41							60				68	70		-
*5306	B53						36		41				54	55		60				68	70		B53
*5314	-		27				36									60				68	70		-
*5315	-						36		41							60				68	70	72	-
*5317							36		41							60	62				70		
*5401, 5402, 5405N, 5407, 5408N, 5410, 5413, 5416-5419	B54											52		56				63	64			71	B54
*5403	-											52		56				63	65			71	-
*5404, 5411, 5415	-											52		56				63				71	-
*5406	-						36					52		56				63			69	71	-
*5409, 5414	-						36					52		56				63				71	-
*5412	-											52		56	58			63	64			70	-
*550101-550104, 5503, 5505, 5511, 5515, 5525, 5529, 5531, 5533	B55											52		56	59			64				71	B55
*550201-550203, 5507, 5510, 5512, 5516, 5519, 5526, 5530	B54, B55											52		56				64				71	B54, B55

*5504, 5508, 5513, 5527, 5615, 5619N, 5622	B55										52		56						71	B55		
*5509, 5524	-				34						52		56		59				71	-		
*5514	-					36			41		52		56				66		71	-		
*5517, 5528	-										52		56		59				71	-		
*5518	-					35							56			62	64		71	-		
*5520	-			32							51	52	54	56					71	-		
*5521	-										52		56		w		64		69	71	-	
*5522	-				34						52		56						71	-		
*5523, 5532, 5618	-					36					52		56						71	-		
*5534	-										52		56			62	64		71	-		
*5535	-										52		56	w			64		71	-		
*5601, 5608, 5614, 5616, 5620, 5624	B56										52		56					65		71	B56	
*5602, 5604	B56										52		56					65	67	71	B56	
*5603	B22					36					52		56						69	71	B22	
*560501	-				33						52		56		59		64			71	-	
*560502	-				33						52		56		59					71	-	
*5606	B78				33				42		52		56		59		64		w	71	B78	
*5607	-										52		56					65		70	-	
*5609	-					36			41		52		56							71	-	
*5610	-										52		56				64		67	71	-	
*5611, 5612	-								41		52		56							71	-	
*5613	-			32							52		54	56				65		71	-	
*5617	-																	65		71	-	
*5621	-				33						52		56		58	59				70	-	
*5623	-																64			71	-	
*570101-570103, 5706, 5708, 5710, 5715, 5718, 5719	B57					36			41							60		66		70	B57	
*5702-5705, 5717	B57								41							60		66		70	B57	
*5707	-								41				53			60		66		70	-	
*5709	-			32					41				53			60		66		70	-	
*5711, 5713	-					36										60		66		70	-	
*5712	-								41									66		71	-	
*5714	-					36			41				55			60		66		70	-	
*5716	-					36			41							60				70	-	
*580101, 580102, 5804, 5805, 5810N-5815, 5819, 5821	B58					36			41							60			67		70	B58
*5802, 5806, 5816	-					36										60			67		70	-
*5807	B58					36	37									60			67		70	B58
*5808	-				33								54		59	60			67		70	-
*5809	-					36			41				55			60			67		70	-
*5817N	-					36			41										67		70	-
*5818			27			36										60			67		70	
*5820										45						60			67		70	

[illegible]

Well No.
HLA-B allele ¹
*070201-070207, 070209, 070501-0706, 0710, 0712, 0714, 071801, 071802, 0721- 0723, 0726, 0728-0731, 0734, 0735, 0739, 0741- 0747, 0749N, 0752, 0755- 0759, 0761-0764, 0766-0768, 0770-0772, 0775, 0776
*0703, 0708, 0716, 0732, 0737
*0704, 0725
*0707, 0777
*0709, 0711, 0717
*0713, 0715
*0719, 0733, 0740, 0753
*0720, 0724
*0727
*0736
*0738
*0748, 0751
*0750
*0754
*0760
*0765
*0769
*080101-080106, 0805, 0808N, 0810, 0811, 0815, 0818, 0819N, 0822-0824, 0827, 0829-0831, 0833, 0834
*0802, 0803
*0804
*0806, 0813, 0820, 0840
*0807, 0814
*0809
*081201, 0816
*0817
*0821
*0825
*0826, 4207
*0828, 0837
*0832
*0835
*0836

*0838
*130101, 130102, 1317, 1320, 132201-1323, 1325 *130201-130205, 1308Q, 1315, 1318, 1319
*1303
*1304, 1310
*1306
*1307N
*1309
*1311
*1312, 1313
*1314
*1316
*1321
*1326
*1401, 1407N
*140201-1404, 1409
*1405-140602
*1408, 3925N, 3930, 3932- 3934, 3947, 3950
*1410
*15010101, 15010102N, 150103, 150106-150108, 1526N-152703, 1532, 1534, 1535, 1538, 1550, 1556, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509-9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529, 9535, 9540, 9545-9547, 9552, 9559, 9560, 9563, 9564
*150102
*150104
*1502, 1588, 9512, 9521, 9539
*150301, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532, 9551, 9556, 9558
*1504
*1505, 1506, 1533, 1539, 1540, 1565, 157801, 1584, 9507, 9536, 9541, 9555
*1507, 9526

*1508, 151101-151103, 1515, 9548
*1509
*1510, 1537, 1590, 1599, 9533
*1512, 1514, 1519
*1513
*1516, 1567, 1595
*15170101-151702
*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534, 9553
*1520, 152501
*1521, 1544
*1523
*1524, 1587, 9557
*1528, 9520, 9549N
*1529
*1530, 1558, 1563
*1531
*1536
*1542
*1543
*1545
*1546
*1548, 9550
*1553
*1554
*1555
*1557
*1562
*1564
*1566
*1568
*1569
*1573
*1574
*1576
*1577
*1580
*1583
*1585, 9554
*1586
*1589
*1591, 9531
*9501

	*9506
	*9515
	*9516
	*9524
	*9537
	*9538
	*9542
	*9543
	*9544
	*9561
*180101-1803, 1805-1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825, 1827, 1828, 1830-1833, 1835	
	*1804
	*1809
	*1812
	*1819
	*1822
	*1826
*1829, 3532, 3537, 3553N, 3564, 356801, 356802, 3599	
	*2701
	*2702
*2703-2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737, 2745, 2746, 2751	
*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738, 2743	
*2708, 2726, 2740, 2742, 2744	
	*2712
	*2716, 2739
	*2718
	*2719
	*2723
	*2724
	*2725
	*2729
	*2730
	*2733
	*2741
	*2750
	*2752

*350101-350107, 3507-350803, 3511, 3515, 352001, 352002, 3523, 3529, 3540N-3542, 3548, 3550, 3552, 3554, 3557, 3561, 3577, 3578, 3582, 3590-3594
*350201-350401, 350403, 3506, 3512, 3533, 3534, 3536, 3539, 3555, 3559, 3565Q, 3570, 3574, 3583-3585, 3595
*350402
*3505, 3551, 3558, 3572, 3589, 3597
*350901, 350902, 3518, 3531, 3575, 3588
*3510, 3528, 3569, 3580
*3513
*351401, 351402, 3562
*3516
*3517, 3530
*3519
*3521, 3524
*3522
*3525, 3549
*3526
*3527
*3535
*3538
*3543, 3567, 3579, 3586
*3544
*3545, 3571
*3546
*3547
*3556
*3560
*3563
*3566
*3576
*3581
*3587
*370101-370103, 370105, 3703N, 3706, 3713
*370104
*3702
*3704

*3705
*3707
*3708
*3709
*3710
*3711
*3712
*3714
*380101-380202, 3808, 3809, 3811-3816, 3818, 3819
*3803
*3804
*3805
*3806
*3807
*3810
*3817
*39010101, 39010102L, 390103-390106, 3903, 3905- 390602, 3909, 3911, 3914, 3915, 3924, 3926-3929, 3931, 3935, 3937, 3938Q, 3940N, 3941, 3944, 3946, 3948
*390201, 3908
*390202, 391301, 391302, 3923, 3949
*3904
*3907
*3910, 3916, 3920
*3912, 3945
*3917
*3918
*391901
*391902
*3922
*3936
*3939
*3942
*3943
*400101-400106, 4007, 4051, 4066, 4073, 4079
*400201-400205, 4004, 40060101-400602, 4039, 4070, 4089, 4097

*4003, 4020
*4005
*4008
*4009, 4011, 4018, 4024, 4027, 4029, 4035, 4037, 4040, 4044, 4050, 4056, 4057, 4064, 4075, 4078, 4082, 4085, 4086, 4090, 4091
*4010, 4022N, 4033, 4036, 4042, 4043, 4049, 4054, 4055, 4061-4063, 4065, 4067, 4081, 4084, 4087, 4088, 4092
*4012
*4013, 4019
*401401-401403, 4053
*4015, 4032
*4016
*4021
*4023
*4025, 4069
*4026
*4028
*4030, 4034
*4031, 4045, 4080
*4038, 4052
*4046
*4047
*4048
*4058
*4059
*4060
*4068
*4071
*407201, 407202, 4077
*4074
*4076
*4083
*4093
*4094
*4095
*4096
*4098
*4101, 4106, 4107
*410201
*410301, 410302

*4104
*4105
*4108
*4201, 4202, 4206, 4210, 4212
*4204
*420501, 420502
*4208
*4209
*4211
*44020101-440205, 440208, 4411, 4416, 4419N, 4421- 4424, 4427, 4433, 4434, 4448, 4452N, 4455, 4459, 4463, 4466, 4467
*440301-4405, 4413, 4426, 442801-4430, 4432, 4435, 4436, 4438, 4439, 4442, 4445, 4451, 4456N, 4458N, 4461N, 4469
*4406
*4407
*4408
*4409
*4410
*4412
*4414
*4415
*4417, 4443
*4418
*4420
*4425, 4450
*4431
*4437, 446401, 446402
*4440
*4441, 4453
*4444
*4446
*4447, 4465
*4449
*4454
*4457
*4460
*4462
*4501, 4505, 4507
*4502, 4503
*4504

*4506
*4508
*4509
*460101, 460102, 4604, 4605, 4607N, 4609, 4610, 4614-4616
*4602
*4603, 4608, 461301, 461302, 4619, 4621
*4606
*4611
*4612
*4617
*4618
*47010101, 47010102
*4702
*4703
*4704, 4705
*480101, 480102, 4807, 4809, 4811, 4819
*4802
*480301, 480302
*4804
*4805, 4808
*4806
*4810, 4812, 4817
*4813
*4814
*4815
*4816
*4818
*490101
*490102
*4902
*4903
*4904, 4905
*5001, 5004
*5002
*510101, 510103, 510105- 510108, 510201, 510203, 5103, 5107, 510901, 5111N, 5112, 511302, 5114, 5117- 5119, 5122, 5123, 5126- 5130, 5132, 5133, 5138- 5141N, 5143, 5148, 5149, 5151-5153, 5155, 5157, 5158, 5160, 5165

*510102, 510109, 510202, 5105, 510902
*510104, 510204, 511301, 5116, 5121, 512401, 512402, 5131, 5134, 5135
*5104, 5146, 5156
*5106, 5159
*5108, 5120, 5144N
*5110
*5115
*5136
*5137, 5163
*5142
*5145
*5150, 5154
*5161
*5162
*5164
*520101, 520103, 5204, 5205, 5207, 5210-5213
*520102, 520104-5203, 520601, 520602, 5209
*5208, 5214
*530101-530104, 5305, 5308, 5310, 5316, 5318
*5302
*5303, 5309, 5311-5313
*5304, 5307
*5306
*5314
*5315
*5317
*5401, 5402, 5405N, 5407, 5408N, 5410, 5413, 5416- 5419
*5403
*5404, 5411, 5415
*5406
*5409, 5414
*5412
*550101-550104, 5503, 5505, 5511, 5515, 5525, 5529, 5531, 5533
*550201-550203, 5507, 5510, 5512, 5516, 5519, 5526, 5530

*5504, 5508, 5513, 5527, 5615, 5619N, 5622
*5509, 5524
*5514
*5517, 5528
*5518
*5520
*5521
*5522
*5523, 5532, 5618
*5534
*5535
*5601, 5608, 5614, 5616, 5620, 5624
*5602, 5604
*5603
*560501
*560502
*5606
*5607
*5609
*5610
*5611, 5612
*5613
*5617
*5621
*5623
*570101-570103, 5706, 5708, 5710, 5715, 5718, 5719
*5702-5705, 5717
*5707
*5709
*5711, 5713
*5712
*5714
*5716
*580101, 580102, 5804, 5805, 5810N-5815, 5819, 5821
*5802, 5806, 5816
*5807
*5808
*5809
*5817N
*5818
*5820

*5901, 5905
*5902, 5903
*5904
*670101
*670102
*6702
*7301
*7801, 780202, 7803
*780201, 7804
*7805
*7806
*8101
*8102
*8103, 8104N
*8201, 8202
*8301
Cw*0206
Cw*0223
Cw*0305, 0325, 0327
Cw*0620
Cw*0746
Cw*150204
Length of spec.
PCR product(s)
Well No.