

Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84
----------	----	----	----	----	----	----	----	----	----	----	----	----	----

[illegible]

[illegible]

*1216	-										82		
Well No.	ser ⁵	73	74	75	76	77	78	79	80	81	82	83	84
Well No.	ser ⁵	73	74	75	76	77	78	79	80	81	82	83	84
*130101-130201, 130203, 1304, 1308, 1316, 1320, 1322-1324, 1327-1329, 1331, 1332, 1334-1336, 1338-1341, 1348, 1351, 1352, 1354, 1359, 1361, 1363-1365, 1368-1376, 1378-1380, 1383, 1384, 1387, 1391-1393, 1396, 1397	11, 13, 14						78						
*130202, 130301-130304, 1310, 133301-133303, 1337, 1366, 1381, 1388-1390, 1394, 1395	13						78						
*130501, 130502, 130701, 130702, 131101, 131102, 131401-131403, 132101, 132102, 1342, 1346, 1349-135002, 1362	6, 11, 13												
*1306, 1312, 1325, 1330, 1356, 1358, 1360, 1377, 1382	6, 11, 13												
*1309	13												
*1313, 1484	13, -												
*1315, 1319, 1353, 1357	13						78	79					
*1317	13						78			81			
*1326	14							79					
*1343	13						78						
*1344, 1386	-							79					
*1345	13						78						
*1367	13												
*1385, 1419, 1421	-, 14						78	79					
*140101, 140102, 1404, 140701, 140702, 1410, 1426, 1428, 1431, 1435, 1438, 1439, 1454, 1455, 1457, 1460-1462, 1470, 1471, 1475, 1476, 1479, 1486-1488, 1490	4, 6, 14, 1404												
*140103, 1408, 142302, 1434, 1472, 1492N	14												
Well No.	ser ⁵	73	74	75	76	77	78	79	80	81	82	83	84

Well No.	ser ⁵	73	74	75	76	77	78	79	80	81	82	83	84
*1402, 140601, 140602, 1409, 1413, 1417, 1420, 1429, 1430, 1433, 1441, 1447, 1448, 1451, 1480, 1483, 1494	6, 14							79					
*140301, 140302, 1412, 1440, 1463, 1467, 1477, 1478, 1485	6, 14, 1403							79					
*140501-140503, 1414, 142301, 142303, 1436, 1443-1445, 1456, 1459, 1464, 1491, 1496	14												
*1411	14										82		
*1416	6						78						
*1418, 1481	14							79					
*1422	14												
*1424	14							79					
*1425, 1453	6, 13, 14												
*1427	14							79					
*143201, 143202	14							w					
*1437	14												
*1442	-												
*1446, 1452	14												
*1449	14							79					
*1450	14									81			
*1458	14												
*1465	6							w					
*1468, 1493	14										82		
*1469	-												
*1473	-												
*1474	-												
*1482	-						78						
*1489	-							79					
*1495	-						78						
*15010101-1520, 1522-1524, 1526, 1528-1533, 1535-1543	2, 15, Null			75									
*1521	2			75									
*1525	-			75		77							
*1527, 1534	-			75									
*160101-1603, 160501, 160502, 1607-1615	16				76								
*1604	16				76								

Well No.	ser ⁵	73	74	75	76	77	78	79	80	81	82	83	84
Well No.	ser ⁵	73	74	75	76	77	78	79	80	81	82	83	84
DRB3*01010201-0114, DRB3*0201-0225, DRB3*030101-0303	52												
DRB4*01010101-01030101, 010302-0108	51												
DRB4*01030102N	Null												
DRB5*010101-0114, DRB5*0202-0205	51												
Well No.	ser ⁵	73	74	75	76	77	78	79	80	81	82	83	84

DR low resolution SSP typing

Well	
------	--

Negative Control

85	86	87	88	89	90	91	92	93	94	95	96	DR	Well No.
----	----	----	----	----	----	----	----	----	----	----	----	----	----------

85	86	87	88	89	90	91	92	93	94	95	96	DR	Well No.
												ser ⁵	DRB1 allele ⁴
												1	*010101-010205, 0104-0131
												1/103	*0103
				89								3, 17	*03010101-030108, 0304-0306, 0309, 031101-0316, 0318-0320, 0322, 0323, 0325, 0326, 0328, 0330, 0331, 0333, 0334, 0336, 0337, 0343-0345, 0347, 0348, 0350-0352
				89								3, 18	*030201-0303, 0327, 0329, 0338
				89								3	*0307, 0317, 0321, 0324, 0332, 0335, 0339-0341, 0349
85												3	*0308
						91	92					3	*0310
												-	*0342
												-	*0346
												3, 4	*040101-0461, 0463-0468, 0470-047202, 0474-0489
					90							4	*0462, 0469, 0473
												7, null	*07010101-070103, 0703-0717
			88									8	*080101-080203, 080401-0807, 0811, 0816, 0817, 0822, 0824, 0826, 0828, 0839
												8	*080302, 0810, 0812-0815, 0818, 0819, 0823, 0825, 0827, 0829-083002, 0833, 0834, 0836-0838
			88		90							8	*0808
			88				92					8	*0809, 0821, 1415
		87	88	89			92					-/13	*0820, 1318, 1347, 1355
85			88									8, 11	*0831, 1167
	86						92					-	*0832

							92					-	*0835	
						91						9	*090102-090105, 090201, 090202, 0904-0909	
												9	*090106, 0903	
												10	*100101-1003	
85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.	
85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.	
85		87	88								Negative Control	11	*110101-110111, 110401-110405, 110601, 110602, 1109-111002, 111201, 111202, 1115, 1124, 112701 1130, 1132, 1133, 1135, 1137-1139, 1143, 1144, 1146, 1147, 1149-1151, 115401, 115402, 1156, 115801, 115802, 1160-1162, 1166, 1172, 1174, 1175, 1177, 1178, 1181, 1184, 1188	
													11, 13	*110201-1103, 111101, 111102, 111401, 111402, 1116, 1120, 1121, 1136, 1140, 1141, 1148, 1159, 1163, 116501, 116502, 1168, 1170, 1176, 1180, 1183, 1185-1187
														11
11	*1107, 1153													
85			88									11	*110801, 110802, 1118-111902, 1142, 1157	
85												11	*111301, 111302	
85						91	92					11, 14	*1117, 1152	
85												-	*1122	
85		87	88				92					11	*1123, 1125	
85												11	*1126, 1134	
85		87					92					11	*1131, 1145, 1164	
85			88				92					11	*1155	
85			88		90							-	*1169, 1182	
85		87										-	*1173, 1179	
85							92					-	*1189	
	86											12	*120101-120103, 120302, 1205-1212, 1214, 1217	
	86		88								12	*120201-120204, 1213, 1215, 1218-1220		
	86										12	*1204		

	86		88									-	*1216
85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.
85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.
		87	88	89								11, 13, 14	*130101-130201, 130203, 1304, 1308, 1316, 1320, 1322-1324, 1327-1329, 1331, 1332, 1334-1336, 1338-1341, 1348, 1351, 1352, 1354, 1359, 1361, 1363-1365, 1368-1376, 1378 1380, 1383, 1384, 1387, 1391-1393, 1396, 1397
		87		89								13	*130202, 130301-130304, 1310, 133301-133303, 1337, 1366, 1381, 1388-1390, 1394, 1395
		87	88	89								6, 11, 13	*130501, 130502, 130701, 130702, 131101, 131102, 131401-131403, 132101, 132102, 1342, 1346, 1349- 135002, 1362
		87		89								6, 11, 13	*1306, 1312, 1325, 1330, 1356, 1358, 1360, 1377, 1382
			88	89								13	*1309
		87		89			92					13, -	*1313, 1484
		87	88	89								13	*1315, 1319, 1353, 1357
			88									13	*1317
		87	88	89								14	*1326
		87	88			91	92					13	*1343
				89								-	*1344, 1386
		87	88		90		92					13	*1345
		87	88									13	*1367
		87		89								-, 14	*1385, 1419, 1421
					90	91	92					4, 6, 14, 1404	*140101, 140102, 1404, 140701, 140702, 1410, 1426, 1428, 1431, 1435, 1438, 1439, 1454, 1455, 1457, 1460-1462, 1470, 1471, 1475, 1476, 1479, 1486-1488, 1490
						91	92					14	*140103, 1408, 142302, 1434, 1472, 1492N
85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.

85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.
				89		91						6, 14	*1402, 140601, 140602, 1409, 1413, 1417, 1420, 1429, 1430, 1433, 1441, 1447, 1448, 1451, 1480, 1483, 1494
		87		89			92					6, 14, 1403	*140301, 140302, 1412, 1440, 1463, 1467, 1477, 1478, 1485
				89		91	92					14	*140501-140503, 1414, 142301, 142303, 1436, 1443, 1445, 1456, 1459, 1464, 1491, 1496
						91	92					14	*1411
		87	88		90	91	92					6	*1416
				89		91	92					14	*1418, 1481
		87	88		90	91	92					14	*1422
			88	89								14	*1424
		87	88		90		92					6, 13, 14	*1425, 1453
		87	88	89			92					14	*1427
					90	91	92					14	*143201, 143202
			88	89	90							14	*1437
				89			92					-	*1442
						91						14	*1446, 1452
					90	91	92					14	*1449
					90	91	92					14	*1450
					90		92					14	*1458
				89		91	92					6	*1465
					90	91	92					14	*1468, 1493
		87			90		92					-	*1469
			88		90	91	92					-	*1473
		87			90	91	92					-	*1474
					90	91	92					-	*1482
				89			92					-	*1489
				89		91	92					-	*1495
												2, 15, Null	*15010101-1520, 1522-1524, 1526, 1528-1533, 1535-1543
							w					2	*1521
												-	*1525
						91						-	*1527, 1534
												16	*160101-1603, 160501, 160502, 1607-1615
							w					16	*1604

85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.
85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.
								93			Negative Control	52	DRB3*01010201-0114, DRB3*0201-0225, DRB3*030101-0303
								94				51	DRB4*01010101-01030101, 010302-0108
					90				94			Null	DRB4*01030102N
								95				51	DRB5*010101-0114, DRB5*0202-0205
85	86	87	88	89	90	91	92	93	94	95	96	ser ⁵	Well No.