

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01-01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:04N, 01:06-01:07, 01:09-01:11N, 01:16N-01:18N, 01:20, 01:22N-01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:08, 01:14-01:15N, 01:30, 01:79, 01:99, 01:145	A1, Null, -	1																							
*01:01:56	-	1			4									w											
*01:12, 01:19	A1, -	1		3																					
*01:13	A1	1			4							11		13		15									
*01:21, 01:126	A1, -	1		3	4																				
*01:28	-	1			4							11				15									
*01:43	-	1			4						10														
*01:51	-	1			4				8		10														
*01:95	-	1			4													17			19				
*01:121	-	1			4															19					
*01:136, 11:94, 11:112 ⁴	-	1			4							11													
*02:01:01:01-02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24-01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64, 02:66-02:69, 02:71-02:77, 02:79-01-02:97:02, 02:99, 02:101-01-02:102, 02:104-02:128, 02:130-02:134, 02:136-02:148, 02:150-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:478	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																							
*02:17:01-02:17:03	A2, -		2					w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2		2											13						20		22			
*02:35:02	A2													13						20		22			
*02:46, 02:70	A2		2																	20		22			
*02:48, 02:129	A2, -																			20		22			
*02:55	A2/A28	2							8													21		23	
*02:65, 02:407, 02:449	A2/A19, -	2																	19						
*02:78	-		w		4															20		22			
*02:135	-		2											13										24	
*02:149	-		2																			21			
*02:169	-		2		4																				
*02:237	-		2												14		16							23	
*02:243:01-02:243:02	-		2																18			21			
*02:309	-		2												14							21			24
*02:338	-		2	3									12		14							21			24
*02:454	-		2										12		14										24
*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:07, 03:09-03:11N, 03:13-03:14, 03:16-03:17:02, 03:20-03:23:02, 03:25-03:29, 03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44-03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:171, 03:174-03:175, 03:179-03:182	A3, A3/blank, Null, -			3										13											
*03:01:19	-			3									12	13											24
*03:01:23, 03:08, 03:32, 03:36N, 03:57, 03:59, 03:72, 03:89, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N	A3, Null, -			3																					
*03:12	A3			3	4																				
*03:15, 03:19, 03:30	A3			3	5									13											
*03:18, 03:135	-	1			4									13											
*03:24, 03:50	A3, -			3					8					13											
*03:41	-			3																			23		
*03:43, 03:82	-			3										13		15		17							
*03:63	-			3								11		13									23		
*03:75, 03:177	-													13									23		
*03:88, 11:130 ⁵	-			3	4							11											23		
*03:95	-													13	14		16						23		
*03:152, 24:92 ⁵	-			3	5																				
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*11:01:01:11-02:06, 11:04-11:09, 11:12-11:16, 11:18-11:19, 11:21N-11:24:02, 11:26-11:27, 11:29-11:39, 11:41-11:52Q, 11:54-11:59, 11:61-11:93, 11:95-11:111, 11:114-11:129, 11:131-11:138, 11:140-11:154, 11:156-11:157, 11:159-11:161, 11:163-11:165, 11:167-11:174, 11:176	A11, Null, -				4							11													
*11:03, 11:20, 11:25, 11:60, 11:158, 11:175	A11, -			3	4								11												
*11:10	A11				4				8			11													

*11:11	-				4							11	12													
*11:17, 11:40	-				4						10	11														
*11:113, 11:162	-				4							11			15											
*11:139	-				4	5		7				11														
*11:155	-				4							11								22						
*11:166	-				4		6					11														
*23:01:01-23:01:12, 23:02-23:13, 23:15-23:56, 23:58-23:60	A23, Null, -					5	6																			
*23:01:13	-					5	6										20		22							
*23:14:01-23:14:02, 24:24, 24:71 ⁵	A23, A9, A24, -					5	6	7																		
*23:57	-					5													21							
*24:02:01:01-24:11N, 24:13:01-24:13:02, 24:17, 24:20-24:23, 24:25-24:43, 24:45N-24:50, 24:54-24:56, 24:58-24:63, 24:66-24:70, 24:72-24:81, 24:83N-24:91, 24:93, 24:95-24:113, 24:115-24:137, 24:139-24:187, 24:189-24:203, 24:205-24:210, 24:212, 24:214-24:221, 24:223-24:227, 24:229-24:242, 24:244-24:270	A24, Low A24, A2403/A24, A9, A24/A9, Null, -					5		7																		
*24:14-24:15, 24:51-24:53, 24:57, 24:64, 24:94, 24:114, 24:138, 24:188, 24:222N, 24:228, C*04:01:03 ⁴	A24, Cw4, Null, -					5																				
*24:18, 24:204, 24:213	A24/A3, -					5		7												23						
*24:19, 24:44	A9					5		7			11															
*24:82	-					5		7											21							
*24:243	-					5		7								17										
*25:01:01-25:04, 25:07-25:12N, 25:14-25:16, 25:18-25:24	A25, Null, -							8	9				13													
*25:05	A25							8	9			12	13					20								
*25:06	A25							8	9			12														
*25:13	-							8	9				13							24						
*25:17	-								9				13													
*26:01:01-26:01:20, 26:01:22-26:01:31, 26:08, 26:10-26:15, 26:17-26:18, 26:20, 26:23-26:28, 26:31-26:33, 26:35-26:43:02, 26:45-26:48, 26:50-26:53, 26:55-26:63, 26:66-26:71N, 26:74-26:77, 26:79-26:88, 26:90, 26:93-26:98	A26, A10, Null, -							8		10			13													
*26:01:21, 26:04, 26:34, 26:73	A26							8		10																
*26:02:01-26:02:02, 26:29, 26:49	A26, -							8		10			w													
*26:03:01-26:03:02, 26:06, 26:21	A26, -							8			11		13							23						
*26:05	A26							8					13							23						
*26:07:01-26:07:02, 26:64, 26:89	A26, -									10			13													
*26:09, 26:91	A26, -							8		10		12														
*26:16	A26							7		10			13													
*26:19	-			4						10			13	14												
*26:22	A26							8		10			13	14					21							
*26:30	A26							8					13							23						
*26:54	-							8		10		12	13					20								
*26:65	-							8		10			13							24						
*26:72	-			4						10			13													
*26:78	-							8			11									23						
*26:92	-										11		13													
*29:01:01:01-29:06, 29:08N-29:12, 29:15-29:27, 29:29-29:38, 29:40-29:47, 29:50-29:58	A29, Null, -													14												
*29:07, 29:49	A29, -					5	6							14												
*29:13, 29:39	-													14			17									
*29:14	-													14		16										
*29:28	-							8						14												
*29:48	-													14				18								
*30:01:01-30:04:02, 30:06-30:07, 30:09-30:20, 30:22-30:54, 30:56-30:80	A30, Null, -														15											
*30:08	A30				4											15										
*30:55	-													13		15										
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.			1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*31:01:02-31:02, 31:05-31:07, 31:09-31:28, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:82	A31, Null, -																	16								
*31:03	A31													12		14		16								
*31:04	A31													12		14		16							24	
*31:08, 33:53 ⁵	A31/A24, -						5											16								
*31:29	-						5	6										16								
*31:35	-																15	16	17							
*31:41	-																	16				21				
*31:62	-																	16					22			
*32:01:01-32:01:13, 32:01:15-32:02, 32:07-32:14, 32:16-32:37, 32:39-32:41, 32:43-32:59, 32:61	A32, Null, -									9									17							
*32:01:14, 32:03, 32:38, 32:60	A32, -																		17							

[illegible]