

Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB1 allele ^{1,2}	ser ³																								
*01:01:01-01:02:09, 01:04-01:38, 01:40N-01:41, 01:43-01:60	DR1, Null, -	73																							
*01:03, 01:39N	DR103/DR1, Null		74																						
*01:42	-	73	74																						
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:22, 03:04:01-03:06, 03:09, 03:12-03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93	DR17, DR3, Null, -					77	78											89							
*03:01:06, 03:01:09, 03:46, 03:64, 03:92	DR17, -					77	78																		
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90	DR18, DR3, -					77		79										89							
*03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41, 03:49, 03:94	DR3, -					77												89							
*03:08, 03:65	DR3, -					77	78				83			85											
*03:10	DR3					77	78												91	92					
*03:11:01	DR3					78												89							
*03:42, 03:69, 03:87	-					77																			
*03:76	-					78					84			87				89							
*04:01:01-04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:172	DR4, DR4/DR3, Null, -								80																
*04:62, 04:69, 04:73, 04:105:01-04:105:02, 04:122, 04:146	DR4, -								80										90						
*07:01:01-01:07:01:07, 07:03-07:29	DR7, Null, -									81															
*08:01:01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54	DR8, -										82						88								
*08:03:02-08:03:04, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45-08:49, 08:51	DR8, -										82														
*08:08	DR8										82						88		90						
*08:09	DR8										82						88			92					
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164 ⁴	DR13, -															87	88	89		92					
*08:21	DR8										82						87	88		92					
*08:31, 08:41, 11:67 ⁴	DR8/DR11, -										82			85			88								
*08:32, 08:53	-										82				86					92					
*08:35, 08:36:02	-										82									92					
*08:40	-					78					82														
*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04-09:22	DR9, -										83								91						
*09:01:06, 09:03	DR9										83														
*10:01:01-10:07	DR10, -												84												
*11:01:01-11:01:17, 11:01:20-11:01:21, 11:04:01-11:04:11, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:02, 11:15, 11:24, 11:27:01-11:30, 11:32-11:33, 11:35, 11:37:01-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:51, 11:54:01-11:54:02, 11:56, 11:58:01-11:58:02, 11:60-11:62, 11:66, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01-11:84:03, 11:88, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144-11:147, 11:150, 11:152	DR11, -													85		87	88								
*11:01:18-11:01:19, 11:08:01-11:08:02, 11:18-11:19:03, 11:42, 11:57, 11:130	DR11, -													85		87									
*11:02:01-11:03, 11:11:01-11:11:02, 11:14:01-11:14:02, 11:16, 11:20-11:21, 11:36, 11:40-11:41, 11:48, 11:63, 11:65:01-11:65:02, 11:68, 11:70, 11:76, 11:85-11:86, 11:93, 11:118, 11:122, 11:124, 11:127, 11:138-11:139, 11:151	DR11, DR11/DR13, -					78								85		87	88								
Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*11:05	DR11													85			88								
*11:07, 11:103, 11:105, 11:107, 11:125	DR11, -					77					83			85											
*11:13:01-11:13:02	DR11							w						85				89		91	92				
*11:17, 11:52	DR14, DR11													85						91	92				
*11:22, 11:98, 11:104, 11:136, 11:143	-													85											
*11:23:01-11:23:02, 11:25, 11:96, 11:148	DR11, -													85		87	88			92					
*11:26, 11:34	DR11								79					85											
*11:31, 11:45, 11:64, 11:119	DR11, -													85		87				92					
*11:53	-												83	85											
*11:55	DR11													85			88			92					
*11:59, 11:80, 11:83, 11:87, 11:135, 11:142	DR11, -					78						84		85		87	88								
*11:69, 11:82	-													85			88			90					
*11:73, 11:79, 11:131-11:132, 11:153	-					78								85		87									

Negative Control

*11:89	-														85						92				
*11:149	-														85	87	88	89							
*12:01:01-12:01:05, 12:03:02-12:03:03, 12:05-12:12, 12:14, 12:17, 12:24N-12:25, 12:28-12:30, 12:34-12:36, 12:38, 12:40-12:41	DR12, Null, -															86									
*12:02:01-12:02:05, 12:13, 12:15, 12:18-12:21, 12:23, 12:26-12:27, 12:31N-12:33, 12:37, 12:42-12:44	DR12, Null, -															86	88								
*12:04, 12:39	DR12, -										82				86										
*12:16:01-12:16:03	-										82				86	88									
*12:22	-										81	82			86										
*13:01:01-13:02:01, 13:02:03-13:02:09, 13:04, 13:08, 13:16, 13:20, 13:22-13:24, 13:28-13:29, 13:31-13:32, 13:34-13:36, 13:38-13:40, 13:48, 13:51-13:52, 13:54, 13:59, 13:61:01-13:61:02, 13:63-13:65, 13:68-13:70, 13:72-13:76, 13:78-13:80, 13:83-13:84, 13:87, 13:91-13:93, 13:96:01-13:99, 13:102, 13:105-13:107, 13:109, 13:111-13:114, 13:117, 13:121, 13:123-13:128, 13:130-13:131, 13:135, 13:138-13:143, 13:145, 13:147-13:149, 13:153, 13:155, 13:162, 13:166, 13:168	DR13, DR11, DR14, Null, -										78						87	88	89						
*13:02:02, 13:167	DR13, -										78								89						
*13:03:01-13:03:06, 13:10, 13:33:01-13:33:03, 13:37, 13:66:01-13:66:02, 13:81, 13:88-13:90, 13:94:01-13:95, 13:101, 13:115, 13:120, 13:133, 13:137N, 13:151-13:152	DR13, Null, -										78						87		89						
*13:05:01-13:05:02, 13:07:01-13:07:02, 13:11:01-13:11:02, 13:14:01-13:14:03, 13:21:01-13:21:02, 13:42, 13:46, 13:49-13:50:02, 13:62, 13:100, 13:108, 13:132, 13:136, 13:150, 13:169	DR13, DR13/DR11, DR6, -																87	88	89						
*13:06, 13:12:01-13:12:03, 13:25, 13:30, 13:56, 13:58, 13:60, 13:77, 13:82, 13:110, 13:118, 13:134, 13:157, 13:161, 13:163	DR13, DR11, DR6, -																87		89						
*13:09	DR13																	88	89						
*13:13, 13:119, 13:154, 13:156, 14:84, 14:116, 14:144 ⁴	DR13, -																87		89		92				
*13:15, 13:19, 13:53, 13:57, 13:104	DR13, -										78	79					87	88	89						
*13:17, 13:116	DR13, -										78		81					88							
*13:26:01-13:26:02	DR14, -										79							87	88	89					
*13:27, 13:41, 13:71	DR13										78				84			87	88	89					
*13:43, 13:159	DR13, -										78							87	88		91	92			
*13:44, 13:86	-										79								89						
*13:45	DR13										78							87	88		90	92			
*13:67, 13:103, 13:160	DR13, -																	87	88						
*13:85	-										78	79						87		89					
*13:122	-										78							87							
*13:129	-										78				84				88	89					
*13:144	-										78							87	88	89		92			
*13:146	-																		88	89		92			
*13:165	-										78							87	88						
*14:01:01-14:01:02, 14:01:04, 14:04, 14:07:01-14:07:02, 14:10, 14:54:01, 14:54:03, 14:57	DR14, DR1404/DR14, DR4/DR6, -																			89	90	91	92		
Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*14:01:03, 14:05:01-14:05:04, 14:08, 14:14, 14:23:01, 14:23:03-14:23:04, 14:36, 14:43-14:45, 14:54:02, 14:56, 14:59, 14:64, 14:91, 14:96, 14:100, 14:103, 14:123, 14:127:01-14:127:02, 14:134	DR14, -																	89		91	92				
*14:02, 14:06:01-14:06:03, 14:09, 14:13, 14:17, 14:20, 14:29-14:30, 14:33, 14:41, 14:47-14:48, 14:51, 14:80, 14:83, 14:94, 14:106, 14:108, 14:121	DR14, -, DR6							79										89		91					
*14:03:01-14:03:02, 14:12:01-14:12:02, 14:40, 14:63, 14:67, 14:77-14:78, 14:85, 14:102, 14:115, 14:135	DR1403/DR14, DR6, -							79								87		89		92					
*14:11	DR14										82							89		91	92				
*14:15	DR8										82						88	?	90	91	92				
*14:16	DR6						78									87	88	?	90	91	92				
*14:18, 14:81	DR14, -						79											89		91	92				
*14:19, 14:21, 14:109	DR14, -						78	79								87		89		w					
*14:22, 14:105	DR14, -															87	88	?	90	91	92				
*14:23:02, 14:34, 14:72, 14:92N, 14:97, 14:126, 14:130, 14:133	DR14, Null, -																	?		91	92				
*14:24	DR14						79										88	89							
*14:25, 14:53, 14:128	DR6, DR13/14, -															87	88	?	90		92				
*14:26, 14:28, 14:31, 14:32:03, 14:35, 14:38-14:39, 14:55, 14:60-14:62, 14:70-14:71, 14:75-14:76, 14:79, 14:86-14:88, 14:90, 14:99, 14:101, 14:104, 14:107, 14:110-14:114, 14:117-14:118, 14:120, 14:122, 14:124-14:125, 14:129, 14:131, 14:138-14:140, 14:142-14:143	DR14, -																	?	90	91	92				
*14:27	DR14						79									87	88	89			92				
*14:32:01-14:32:02	DR14						w											?	90	91	92				
*14:37	DR14																88	89	90						

*14:42, 14:136	-																	89	92						
*14:46	DR14																		91						
*14:49, 14:119	DR14, -							79										?	90	91	92				
*14:50	DR14									81								?	90	91	92				
*14:52	DR14																	?		91					
*14:58	DR14																	?	90		92				
*14:65	DR6							w										89		91	92				
*14:68, 14:93	DR14, -									82								?	90	91	92				
*14:69	-																	?	90		92				
*14:73	-																	88	?	90	91	92			
*14:74	-																	87	?	90	91	92			
*14:82	-							78											89	90	91	92			
*14:89	-							79											89			92			
*14:95	-							78											89		91	92			
*14:98	-							79										87	89						
*14:132	-							78											?		91	92			
*14:137N	Null							78										87	89	90	91	92			
*14:141	-																		89				93		
*14:145	-																			90	91	92			
*14:146	-							79												90	91	92			
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:65, 15:67-15:104	DR15, DR2, Null, -							75																	
*15:21	DR2							75														w			
*15:25	-							75	77																
*15:27, 15:34, 15:66	-							75												91					
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:24	DR16, Null, -							76																	
*16:04, 16:18	DR16, -							76													w				
DRB3*01:02:01-01:15, DRB3*02:01-02:26, DRB3*02:28-02:29N, DRB3*03:01:01-03:03	DR52, Null, -																						93		
DRB3*02:27	-																	87					93		
DRB4*01:01:01-01:01:03:01:01, DRB4*01:03:01:03-01:08	DR53, -																						94		
DRB4*01:03:01:02N	Null																		90				94		
DRB5*01:01:01-01:14, DRB5*02:02-02:06	DR51, Null, -																						95		
Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96