

Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB1 allele ^{1,2}	ser ³																								
*01:01:01-01:02:11, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:71	DR1, Null, -	73																							
*01:03, 01:39N	DR103/DR1, Null		74																						
*01:42, 01:61	-	73	74																						
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25, 03:04:01-03:06, 03:09, 03:11:01-03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:113-03:114, 03:116-03:118, 03:121	DR17, DR3, Null, -					77	78											89							
*03:01:06, 03:01:09, 03:46, 03:64, 03:92	DR17, -					77	78																		
*03:01:24, 03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120	DR3, -					77												89							
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115	DR18, DR3, -					77	79											89							
*03:08, 03:65	DR3, -					77	78				83			85											
*03:10	DR3					77	78												91	92					
*03:11:01, 13:02:02 ⁴	DR3, DR13					78												89							
*03:42, 03:69, 03:87, 03:111	-					77																			
*03:76, 13:176 ⁴	-					78					84				87			89							
*03:97	-																	89							
*03:112	-					77	78				83							89							
*04:01:01-04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:194	DR4, DR4/DR3, Null, -								80																
*04:62, 04:69, 04:73, 04:105:01-04:105:02, 04:122, 04:146	DR4, -								80										90						
*07:01:01-07:01:18, 07:03-07:61	DR7, Null, -									81															
*08:01:01-08:01:05, 08:02:01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73	DR8, -										82						88								
*08:01:06, 08:03:02-08:03:05, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:69, 08:71, 08:74, 08:76	DR8, Null, -										82														
*08:08	DR8										82						88		90						
*08:09	DR8										82						88			92					
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164 ⁴	DR13, -															87	88	89		92					
*08:21	DR8										82						87	88		92					
*08:31, 08:41, 08:75, 11:67 ⁴	DR8/DR11, -										82			85			88								
*08:32, 08:53	-										82				86					92					
*08:35, 08:36:02	-										82									92					
*08:40	-					78					82														
*08:57	-																88								
*08:63	-										82			85											
*08:67	-									81							88								
*08:68	-										82									w					
*08:70	-										82						88			w					
*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04-09:22, 09:24, 09:26	DR9, -										83								91						
*09:01:06, 09:03, 09:23	DR9, -										83														
*10:01:01-10:15	DR10, -											84													
Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.	DR	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*11:01:01-11:01:17, 11:01:20-11:01:27, 11:04:01-11:04:12, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:02, 11:15, 11:24, 11:27:01-11:29:01, 11:32-11:33, 11:35, 11:37:01-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:51, 11:54:01-11:54:02, 11:56, 11:58:01-11:58:02, 11:60-11:62:02, 11:66, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01-11:84:03, 11:88, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144-11:147:02, 11:150, 11:152, 11:154-11:158, 11:160, 11:162-11:167, 11:169N-11:170, 11:172, 11:174-11:175, 11:178-11:181, 11:183	DR11, Null, -													85		87	88								
*11:01:18-11:01:19, 11:08:01-11:08:03, 11:18-11:19:03, 11:42, 11:57, 11:130	DR11, -													85		87									

Negative Control

