

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB1 allele ^{1,2}	ser ³																								
*01:01:01-01:02:12, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:83	DR1, Null, -	73																							
*01:03:01-01:03:02, 01:39N	DR103/DR1, Null, -		74																						
*01:42, 01:61	-	73	74																						
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:26, 03:04:01-03:06, 03:09, 03:11-01:03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139	DR17, DR3, Null, -					77	78											89							
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:130	DR17, -					77	78																		
*03:01:24, 03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120	DR3, -					77												89							
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131	DR18, DR3, -					77	79											89							
*03:08, 03:65, 03:140	DR3, -					77	78				83			85											
*03:10	DR3					77	78												91	92					
*03:42, 03:69, 03:87, 03:111	-					77																			
*03:76	-					77	78				84			87				89							
*03:125	-					77	78				84			87	88			89							
*03:126, 14:179 ⁴	-					78	79				84							89	91						
*04:01:01-01:04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:233	DR4, DR4/DR3, Null, -								80																
*04:62, 04:69, 04:73, 04:105-01:04:105:02, 04:122, 04:146	DR4, -								80										90						
*04:211	-								80										91						
*07:01:01-01:07:01:21, 07:03-07:80	DR7, Null, -									81															
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:02:01-01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24-01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82	DR8, Null, -									82							88								
*08:01:06, 08:03:02-01-08:03:06, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36-01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:69, 08:71, 08:74, 08:76, 08:81	DR8, Null, -									82															
*08:08	DR8									82							88	90							
*08:09	DR8									82							88			92					
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164, 13:232 ⁴	DR13, -															87	88	89		92					
*08:21	DR8									82							87	88		92					
*08:31, 08:41, 08:75, 11:67, 11:193:01-11:193:02, 11:209 ⁴	DR8/DR1 1, -									82			85				88								
*08:32, 08:53	-									82				86						92					
*08:35, 08:36:02	-									82										92					
*08:40	-					78				82															
*08:57, 08:80	-																88								
*08:63	-									82			85												
*08:67	-								81								88								
*08:68	-									82										w					
*08:70	-									82							88			w					
*09:01:02-01-09:01:05, 09:01:07-09:02:02, 09:04, 09:06-09:22, 09:24-09:28, 09:30-09:31	DR9, -										83								91						
*09:01:06, 09:03, 09:23, 09:29	DR9, -										83														
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*09:05	DR9										83	84							91						
*10:01:01-01-10:10, 10:12-10:18, 10:20-10:22	DR10, -											84													
*10:11	-									81		84													
*10:19	-											84												94	

Negative Control

[illegible]

[illegible]

*14:65	DR6							w						89	90	91	92			Negative Contr					
*14:68:01-14:68:02, 14:93	DR14, -									82				?	90	91	92								
*14:69	-												87	?	90		92								
*14:73	-													88	?	90	91	92							
*14:74, 14:173	-												87	?	90	91	92								
*14:82	-						78				84				89	90	91	92							
*14:89	-							79							89			92							
*14:95	-						78				84				89		91	92							
*14:120, 14:122, 14:178	-						78								?	90	91	92							
*14:132	-						78				84				?		91	92							
*14:141	-														89				93						
*14:148	-										82				?		91	92							
*14:168	-														?	w	91	92							
*14:171	-						77								89	90	91	92							
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:53, 15:55-15:65, 15:67-15:99, 15:101-15:103, 15:104:02-15:121, 15:123-15:140, 15:142-15:147	DR15, DR2, Null, -			75																					
*15:21	DR2			75														w							
*15:25, 15:37:01, 15:100, 15:104:01	-			75		77																			
*15:27, 15:34, 15:54, 15:66:01-15:66:02, 15:122	-			75													91								
*15:141	-			75																94					
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:47	DR16, Null, -			76																					
*16:04:01-16:04:02, 16:18	DR16, -			76														w							
DRB3*01:01:02-01:01:41, DRB3*01:43-01:45, DRB3*02:01-02:26, DRB3*02:28-02:65, DRB3*03:01-01-03:13	DR52, Null, -																			93					
DRB3*01:42	-										83									93					
DRB3*02:27	-													87						93					
DRB4*01:01:01-01:01:03:01:01, DRB4*01:03:01:03-01:30, DRB4*01:32-01:56N, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																			94					
DRB4*01:03:01:02N	Null															90				94					
DRB4*01:31	-																	92		94					
DRB5*01:01:01-01:35, DRB5*02:02:01-02:17	DR51, Null, -																			95					
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96