

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB1 allele ^{1,2}	ser ³																								
*01:01:01-01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:84	DR1, Null, -	73																							
*01:03:01-01:03:02, 01:39N	DR1, DR103, Null, -		74																						
*01:42, 01:61	-	73	74																						
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:26, 03:04:01-03:06, 03:09, 03:11:01-03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139, 03:141-03:142	DR3, DR17, Null, -						77	78										89							
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:130	DR17, -						77	78																	
*03:01:24, 03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120	DR3, -						77											89							
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131	DR3, DR18, -						77	79										89							
*03:08, 03:65, 03:140	DR3, -						77	78				83		85											
*03:10	DR3						77	78												91	92				
*03:42, 03:69, 03:87, 03:111	-						77																		
*03:76	-						77	78					84			87		89							
*03:125	-						77	78					84			87	88	89							
*03:126, 14:179 ^d	-						78	79					84					89		91					
*04:01:01-01:04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:237	DR3, DR4, Null, -								80																
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146	DR4, -								80										90						
*04:211	-								80										91						
*07:01:01-01:07:01:22, 07:03-07:05, 07:07-07:84	DR7, Null, -									81															
*07:06	DR7									81	82														
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:02:01-01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24-01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82	DR8, Null, -									82							88								
*08:01:06, 08:03:02-01-08:03:07, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81	DR8, Null, -									82															
*08:08	DR8									82							88		90						
*08:09	DR8									82							88				92				
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164, 13:232 ^d	DR13, -															87	88	89		92					
*08:21	DR8									82						87	88			92					
*08:31, 08:41, 08:75, 11:67, 11:193:01-11:193:02, 11:209 ^d	DR8, DR11, -									82			85				88								
*08:32, 08:53	-									82				86						92					
*08:35, 08:36:02	-									82										92					
*08:40	-						78			82															
*08:57, 08:80	-																88								
*08:63	-									82			85												
*08:67	-									81							88								
*08:68:01	-									82										w					
*08:70	-									82							88			w					
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
*09:01:02-01-09:01:05, 09:01:07-09:02:02, 09:04, 09:06-09:22, 09:24-09:28, 09:30-09:32	DR9, -											83							91						
*09:01:06, 09:03, 09:23, 09:29	DR9, -											83													
*09:05	DR9											83	84						91						
*10:01:01-01-10:10, 10:12-10:18, 10:20-10:24	DR10, -												84												
*10:11	-									81			84												
*10:19	-												84										94		
*11:01:01-01-11:01:17, 11:01:20-11:01:27, 11:01:29, 11:04-01-11:04:15, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:03, 11:15, 11:24, 11:27:01, 11:28:01-11:29:02, 11:32-11:33, 11:35, 11:37:01-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:51, 11:54:01-11:54:02, 11:56, 11:58:01-11:58:02, 11:60-11:62:02, 11:66, 11:72, 11:74-01-11:75, 11:77-11:78, 11:81, 11:84:01, 11:88, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144-11:147:02, 11:150, 11:152, 11:154-11:158, 11:160, 11:162-11:167, 11:169N-11:170, 11:172, 11:174-11:175, 11:177-11:181, 11:183, 11:185-11:190, 11:192, 11:194-11:201, 11:203-11:208, 11:210-11:212, 11:214-11:220	DR11, Null, -													85		87	88								

Negative Control

[illegible]

[illegible]

*16:04:01-16:04:02, 16:18	DR16, -		76															w								
DRB3*01:01:02:01-01:41, DRB3*01:43-01:47,	DR52,																									
DRB3*02:01-02:26, DRB3*02:28-02:68, DRB3*03:01:01-03:15	Null, -																		93							
DRB3*01:42	-									83									93							
DRB3*02:27	-												87						93							
DRB4*01:01:01-01:01:03:01:01, DRB4*01:03:01:03-01:30, DRB4*01:32-01:63, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																			94						
DRB4*01:03:01:02N	Null																90			94						
DRB4*01:31	-																	92		94						
DRB5*01:01:01-01:35, DRB5*02:02:01-02:18	DR51, Null, -																				95					
Well No.			73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96