

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04:01:02N, 01:06, 01:09:01:01:01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-01:331N	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314	A1, Null, -	1																							
*01:01:56	-	1			4								12												
*01:07, 01:95, 01:289	A1, -	1			4													17							
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																					
*01:13, 01:176	A1, -	1			4							11		13		15									
*01:15N	Null	1			w																				
*01:21, 01:126	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211, 11:290, 11:326 ⁴	A1, -	1			4							11													
*01:28, 01:229, 01:299, 11:282 ⁴	-	1			4							11				15									
*01:43, 11:271 ⁴	-	1			4							10													
*01:51	-	1			4				8			10													
*01:121	-	1			4															19					
*01:194	-	1			4									13		15									
*01:298	-	1			4					9								17							
*01:301Q	-	1			4	5		7																	
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:17:02:02, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830-02:841, 02:843-02:899	A2, Low A2, A203, A210, Null, -	2																							
*02:17:02:01, 02:17:03-02:17:04	A2, -	2						w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -	2												13						20		22			
*02:35:02	A2													13						20		22			
*02:46, 02:70	A2	2																		20		22			
*02:48, 02:129, 02:571	A2, -																			20		22			
*02:55	A2, A28	2						8													21		23		
*02:65, 02:407, 02:449	A2, A19, -	2																	19						
*02:78	-		w		4															20		22			
*02:81, 02:124	A2	2							9																
*02:135	-	2												13											24
*02:149, 02:842	-	2																			21				
*02:169	-	2			4																				
*02:237, 02:809	-	2													14		16							23	
*02:243:01-02:243:03	-	2																18			21				
*02:309	-	2											12		14						21			24	
*02:338	-	2	3																						
*02:454	-	2											12		14										24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*02:527, 02:582	-	2						8																	
*02:576, 02:682	-	2																				22			
*02:644	-							8		10												21			
*02:651	-	2																			20				
*02:671	-	2																						23	
*02:741	-	2						8						13							20	21	22		
*02:804	-	2					7																		
*02:815	-	2						8														21			

*68:01:01:01-68:04:02, 68:06-68:08:01:02, 68:09-68:10, 68:12:01-68:12:02, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:40, 68:42-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:102, 68:104:01-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:177, 68:179-68:217, 68:219-68:226	A28, A68, Null, -								8											20					
*68:05, 68:15, 68:20, 68:109, 68:136	A68, -								8											20			23		
*68:08:02, 68:105, 68:157	A68, -								8											20	21				
*68:11N, 68:41	Null, -								8											20		22			
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4				8											20					
*68:19, 68:130:02	A68, -								8				14							20					
*68:25	A68								8									19	20						
*68:29	A68								8									18		20	21				
*68:45, 68:117	-							7	8											20					
*68:71	-								8				13							20					
*68:84, 68:227	-								8		10									20					
*68:103:01-68:103:02	-				3				8											20					
*68:130:01	-								8				12							20					
*68:178	-					4														20					
*68:218	-								8																
*69:01:01:01-69:01:03, 69:03-69:06, 69:08	A69, -								8													21			
*69:02	-								8			11										21			
*69:07	-								8				13									21			
*74:01:01:01-74:01:02, 74:01:04-74:03:01:02, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36	A74, Null, -																			19					
*74:01:03	-											12								19				24	
*74:04, 74:21	A74, -																	18	19						
*74:07, 74:33	A74, -																	17		19					
*74:13	-												13			16				19					
*74:19	-					4														19					
*74:23	-					3														19					
*74:30	-											11								19					
*74:35	-																			19		21			
*74:37	-													13						19					
*80:01:01:01-80:01:02, 80:03, 80:05-80:06	A80, -						6																23		
*80:02	-						6					11													
*80:04	-						6											17					23		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -								9																
B*18:27	-						6																		
B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569	-																						22		
C*03:365, C*12:131	-				4																				
C*06:187	-																							23	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24