

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04:01:02N, 01:06, 01:09:01:01:01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-01:343, 01:345-01:348	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314	A1, Null, -	1																							
*01:01:56	-	1			4							12													
*01:07, 01:95, 01:289	A1, -	1			4													17							
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																					
*01:13, 01:176	A1, -	1			4						11		13		15										
*01:15N	Null	1			w																				
*01:21, 01:126	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211, 11:290, 11:326 ⁴	A1, -	1			4						11														
*01:28, 01:229, 01:299, 11:282 ⁴	-	1			4						11				15										
*01:43, 11:271 ⁴	-	1			4						10														
*01:51	-	1			4				8		10														
*01:121	-	1			4															19					
*01:194	-	1			4								13		15										
*01:298	-	1			4					9								17							
*01:301Q	-	1			4	5		7																	
*01:344	-	1			4				8																
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:17:02:02, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:939	A2, A203, A210, Null, -	2																							
*02:17:02:01, 02:17:03-02:17:04	A2, -	2						w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907	A2, -	2											13							20		22			
*02:35:02	A2												13							20		22			
*02:46, 02:70	A2	2																		20		22			
*02:48, 02:129, 02:571	A2, -																			20		22			
*02:55	A2, A28	2						8													21		23		
*02:65, 02:407, 02:449, 02:932	A2, A19, -	2																	19						
*02:78	-	w		4																20		22			
*02:81, 02:124	A2	2							9																
*02:135	-	2											13											24	
*02:149, 02:842	-	2																			21				
*02:169	-	2		4																					
*02:237, 02:809, 02:927	-	2												14		16							23		
*02:243:01-02:243:03	-	2																	18		21				
*02:309	-	2											12		14						21			24	
*02:338	-	2	3																						
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*02:454	-	2											12		14										24
*02:527, 02:582	-	2						8																	
*02:576, 02:682	-	2																					22		
*02:644	-							8		10												21			
*02:651	-	2																		20					
*02:671	-	2																					23		
*02:741	-	2						8					13							20	21	22			
*02:804	-	2					7																		
*02:815	-	2						8														21			
*02:829	-			4					9																
*02:929	-	2																18							

*03:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:01:42, 03:01:44-03:01:46, 03:01:48-03:01:50, 03:01:52-03:01:54, 03:01:56-03:01:58, 03:01:59-03:02:01, 03:02:03-03:02:05, 03:02:07-03:02:09, 03:02:11-03:02:13, 03:02:15-03:02:17, 03:02:19-03:02:21, 03:02:23-03:02:25, 03:02:27-03:02:29, 03:02:31-03:02:33, 03:02:35-03:02:37, 03:02:39-03:02:41, 03:02:43-03:02:45, 03:02:47-03:02:49, 03:02:51-03:02:53, 03:02:55-03:02:57, 03:02:59-03:03:01, 03:03:03-03:03:05, 03:03:07-03:03:09, 03:03:11-03:03:13, 03:03:15-03:03:17, 03:03:19-03:03:21, 03:03:23-03:03:25, 03:03:27-03:03:29, 03:03:31-03:03:33, 03:03:35-03:03:37, 03:03:39-03:03:41, 03:03:43-03:03:45, 03:03:47-03:03:49, 03:03:51-03:03:53, 03:03:55-03:03:57, 03:03:59-03:04:01, 03:04:03-03:04:05, 03:04:07-03:04:09, 03:04:11-03:04:13, 03:04:15-03:04:17, 03:04:19-03:04:21, 03:04:23-03:04:25, 03:04:27-03:04:29, 03:04:31-03:04:33, 03:04:35-03:04:37, 03:04:39-03:04:41, 03:04:43-03:04:45, 03:04:47-03:04:49, 03:04:51-03:04:53, 03:04:55-03:04:57, 03:04:59-03:05:01, 03:05:03-03:05:05, 03:05:07-03:05:09, 03:05:11-03:05:13, 03:05:15-03:05:17, 03:05:19-03:05:21, 03:05:23-03:05:25, 03:05:27-03:05:29, 03:05:31-03:05:33, 03:05:35-03:05:37, 03:05:39-03:05:41, 03:05:43-03:05:45, 03:05:47-03:05:49, 03:05:51-03:05:53, 03:05:55-03:05:57, 03:05:59-03:06:01, 03:06:03-03:06:05, 03:06:07-03:06:09, 03:06:11-03:06:13, 03:06:15-03:06:17, 03:06:19-03:06:21, 03:06:23-03:06:25, 03:06:27-03:06:29, 03:06:31-03:06:33, 03:06:35-03:06:37, 03:06:39-03:06:41, 03:06:43-03:06:45, 03:06:47-03:06:49, 03:06:51-03:06:53, 03:06:55-03:06:57, 03:06:59-03:07:01, 03:07:03-03:07:05, 03:07:07-03:07:09, 03:07:11-03:07:13, 03:07:15-03:07:17, 03:07:19-03:07:21, 03:07:23-03:07:25, 03:07:27-03:07:29, 03:07:31-03:07:33, 03:07:35-03:07:37, 03:07:39-03:07:41, 03:07:43-03:07:45, 03:07:47-03:07:49, 03:07:51-03:07:53, 03:07:55-03:07:57, 03:07:59-03:08:01, 03:08:03-03:08:05, 03:08:07-03:08:09, 03:08:11-03:08:13, 03:08:15-03:08:17, 03:08:19-03:08:21, 03:08:23-03:08:25, 03:08:27-03:08:29, 03:08:31-03:08:33, 03:08:35-03:08:37, 03:08:39-03:08:41, 03:08:43-03:08:45, 03:08:47-03:08:49, 03:08:51-03:08:53, 03:08:55-03:08:57, 03:08:59-03:09:01, 03:09:03-03:09:05, 03:09:07-03:09:09, 03:09:11-03:09:13, 03:09:15-03:09:17, 03:09:19-03:09:21, 03:09:23-03:09:25, 03:09:27-03:09:29, 03:09:31-03:09:33, 03:09:35-03:09:37, 03:09:39-03:09:41, 03:09:43-03:09:45, 03:09:47-03:09:49, 03:09:51-03:09:53, 03:09:55-03:09:57, 03:09:59-03:10:01, 03:10:03-03:10:05, 03:10:07-03:10:09, 03:10:11-03:10:13, 03:10:15-03:10:17, 03:10:19-03:10:21, 03:10:23-03:10:25, 03:10:27-03:10:29, 03:10:31-03:10:33, 03:10:35-03:10:37, 03:10:39-03:10:41, 03:10:43-03:10:45, 03:10:47-03:10:49, 03:10:51-03:10:53, 03:10:55-03:10:57, 03:10:59-03:11:01, 03:11:03-03:11:05, 03:11:07-03:11:09, 03:11:11-03:11:13, 03:11:15-03:11:17, 03:11:19-03:11:21, 03:11:23-03:11:25, 03:11:27-03:11:29, 03:11:31-03:11:33, 03:11:35-03:11:37, 03:11:39-03:11:41, 03:11:43-03:11:45, 03:11:47-03:11:49, 03:11:51-03:11:53, 03:11:55-03:11:57, 03:11:59-03:12:01, 03:12:03-03:12:05, 03:12:07-03:12:09, 03:12:11-03:12:13, 03:12:15-03:12:17, 03:12:19-03:12:21, 03:12:23-03:12:25, 03:12:27-03:12:29, 03:12:31-03:12:33, 03:12:35-03:12:37, 03:12:39-03:12:41, 03:12:43-03:12:45, 03:12:47-03:12:49, 03:12:51-03:12:53, 03:12:55-03:12:57, 03:12:59-03:13:01, 03:13:03-03:13:05, 03:13:07-03:13:09, 03:13:11-03:13:13, 03:13:15-03:13:17, 03:13:19-03:13:21, 03:13:23-03:13:25, 03:13:27-03:13:29, 03:13:31-03:13:33, 03:13:35-03:13:37, 03:13:39-03:13:41, 03:13:43-03:13:45, 03:13:47-03:13:49, 03:13:51-03:13:53, 03:13:55-03:13:57, 03:13:59-03:14:01, 03:14:03-03:14:05, 03:14:07-03:14:09, 03:14:11-03:14:13, 03:14:15-03:14:17, 03:14:19-03:14:21, 03:14:23-03:14:25, 03:14:27-03:14:29, 03:14:31-03:14:33, 03:14:35-03:14:37, 03:14:39-03:14:41, 03:14:43-03:14:45, 03:14:47-03:14:49, 03:14:51-03:14:53, 03:14:55-03:14:57, 03:14:59-03:15:01, 03:15:03-03:15:05, 03:15:07-03:15:09, 03:15:11-03:15:13, 03:15:15-03:15:17, 03:15:19-03:15:21, 03:15:23-03:15:25, 03:15:27-03:15:29, 03:15:31-03:15:33, 03:15:35-03:15:37, 03:15:39-03:15:41, 03:15:43-03:15:45,

[illegible]

[illegible]

*68:05, 68:15, 68:20, 68:109, 68:136	A68, -								8										20		23				
*68:08:02, 68:105, 68:157	A68, -								8										20	21					
*68:11N, 68:41, 68:230	Null, -								8										20		22				
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4				8										20						
*68:19, 68:130:02	A68, -								8				14						20						
*68:25	A68								8									19	20						
*68:29	A68								8									18	20	21					
*68:45, 68:117	-							7	8										20						
*68:71	-								8				13						20						
*68:84, 68:227	-								8		10								20						
*68:103:01-68:103:02	-				3				8										20						
*68:130:01	-								8				12						20						
*68:178	-				4														20						
*68:218	-								8																
*69:01:01:01-69:01:03, 69:03-69:06, 69:08	A69, -								8											21					
*69:02	-								8			11								21					
*69:07	-								8				13							21					
*74:01:01:01-74:01:02, 74:01:04-74:03:02, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:29, 74:31-74:32N, 74:34, 74:36, 74:38-74:39	A74, Null, -																		19						
*74:01:03	-											12							19		24				
*74:04, 74:21	A74, -																	18	19						
*74:07, 74:33	A74, -																	17	19						
*74:13	-												13			16			19						
*74:19	-				4														19						
*74:23	-				3														19						
*74:30	-											11							19						
*74:35	-																		19	21					
*74:37	-												13						19						
*80:01:01:01-80:01:02, 80:03, 80:05-80:07	A80, -							6													23				
*80:02	-							6				11													
*80:04	-							6										17			23				
B*07:81, B*08:52, B*18:67, B*38:41, B*38:67, B*49:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:01:38, B*57:60	B53, -									9															
B*18:27	-							6																	
B*40:359, B*55:96, B*57:65, C*04:31, C*06:137, C*07:569	-																				22				
C*03:365, C*12:131	-				4																				
C*06:187	-																				23				
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24