

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-A allele ^{1,2}	ser ³																								
*01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04:01:02N, 01:06, 01:09:01:01:01:11N, 01:16N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:266:02, 01:268-01:270, 01:272-01:278, 01:280-01:288, 01:290N-01:297, 01:300, 01:302-01:313, 01:315-01:343, 01:345-01:359	A1, Null, -	1			4																				
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159Q, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 01:267, 01:271, 01:279, 01:314	A1, Null, -	1																							
*01:01:56	-	1			4							12													
*01:07, 01:95, 01:289	A1, -	1			4													17							
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																					
*01:13, 01:176	A1, -	1			4						11		13		15										
*01:15N	Null	1			w																				
*01:21, 01:126	A1, -	1		3	4																				
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211, 11:290, 11:326 ⁴	A1, -	1			4						11														
*01:28, 01:229, 01:299, 11:282 ⁴	-	1			4						11				15										
*01:43, 11:271 ⁴	-	1			4						10														
*01:51	-	1			4				8		10														
*01:121	-	1			4														19						
*01:194	-	1			4								13		15										
*01:298	-	1			4					9								17							
*01:301Q	-	1			4	5		7																	
*01:344	-	1			4				8																
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49:01-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99:01-02:99:02, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150:01-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570:02, 02:572-02:575, 02:577-02:579, 02:581Q, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:681, 02:683-02:740, 02:742-02:803N, 02:805Q-02:808, 02:810-02:814, 02:816-02:828, 02:830-02:841, 02:843-02:906, 02:908-02:926, 02:928, 02:930-02:931, 02:933-02:946N, 02:948-02:951, 02:953-02:954	A2, A203, A210, Null, -	2																							
*02:17:02:01-02:17:04	A2, -	2						w																	
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580, 02:907	A2, -	2											13						20		22				
*02:35:02	A2												13							20		22			
*02:46, 02:70	A2	2																		20		22			
*02:48, 02:129, 02:571	A2, -																			20		22			
*02:55	A2, A28	2						8													21		23		
*02:65, 02:407, 02:449, 02:932	A2, A19, -	2																	19						
*02:78	-	w			4															20		22			
*02:81, 02:124	A2	2							9																
*02:135	-	2											13											24	
*02:149, 02:842, 02:947	-	2																			21				
*02:169	-	2			4																				
*02:237, 02:809, 02:927	-	2												14		16							23		
*02:243:01-02:243:03	-	2																			21				
*02:309	-	2										12		14					18			21			24
*02:338, 02:952	-	2	3																		21				
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*02:454	-	2											12		14										24
*02:527, 02:582	-	2						8															22		
*02:576, 02:682	-	2																							
*02:644	-							8		10												21			
*02:651	-	2																		20					
*02:671	-	2																						23	
*02:741	-	2						8					13							20	21	22			
*02:804	-	2					7																		
*02:815	-	2						8														21			
*02:829	-				4					9															
*02:929	-	2																	18						

*03:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:01:42, 03:01:44-03:01:46, 03:01:48-03:01:50, 03:01:52- 03:01:54, 03:01:56-03:01:58, 03:01:59-03:02:01, 03:02:03-03:02:05, 03:02:07-03:02:09, 03:02:11-03:02:13, 03:02:15-03:02:17, 03:02:19-03:02:21, 03:02:23-03:02:25, 03:02:27-03:02:29, 03:02:31-03:02:33, 03:02:35-03:02:37, 03:02:39-03:02:41, 03:02:43-03:02:45, 03:02:47-03:02:49, 03:02:51-03:02:53, 03:02:55-03:02:57, 03:02:59-03:03:01, 03:03:03-03:03:05, 03:03:07-03:03:09, 03:03:11-03:03:13, 03:03:15-03:03:17, 03:03:19-03:03:21, 03:03:23-03:03:25, 03:03:27-03:03:29, 03:03:31-03:03:33, 03:03:35-03:03:37, 03:03:39-03:03:41, 03:03:43-03:03:45, 03:03:47-03:03:49, 03:03:51-03:03:53, 03:03:55-03:03:57, 03:03:59-03:04:01, 03:04:03-03:04:05, 03:04:07-03:04:09, 03:04:11-03:04:13, 03:04:15-03:04:17, 03:04:19-03:04:21, 03:04:23-03:04:25, 03:04:27-03:04:29, 03:04:31-03:04:33, 03:04:35-03:04:37, 03:04:39-03:04:41, 03:04:43-03:04:45, 03:04:47-03:04:49, 03:04:51-03:04:53, 03:04:55-03:04:57, 03:04:59-03:05:01, 03:05:03-03:05:05, 03:05:07-03:05:09, 03:05:11-03:05:13, 03:05:15-03:05:17, 03:05:19-03:05:21, 03:05:23-03:05:25, 03:05:27-03:05:29, 03:05:31-03:05:33, 03:05:35-03:05:37, 03:05:39-03:05:41, 03:05:43-03:05:45, 03:05:47-03:05:49, 03:05:51-03:05:53, 03:05:55-03:05:57, 03:05:59-03:06:01, 03:06:03-03:06:05, 03:06:07-03:06:09, 03:06:11-03:06:13, 03:06:15-03:06:17, 03:06:19-03:06:21, 03:06:23-03:06:25, 03:06:27-03:06:29, 03:06:31-03:06:33, 03:06:35-03:06:37, 03:06:39-03:06:41, 03:06:43-03:06:45, 03:06:47-03:06:49, 03:06:51-03:06:53, 03:06:55-03:06:57, 03:06:59-03:07:01, 03:07:03-03:07:05, 03:07:07-03:07:09, 03:07:11-03:07:13, 03:07:15-03:07:17, 03:07:19-03:07:21, 03:07:23-03:07:25, 03:07:27-03:07:29, 03:07:31-03:07:33, 03:07:35-03:07:37, 03:07:39-03:07:41, 03:07:43-03:07:45, 03:07:47-03:07:49, 03:07:51-03:07:53, 03:07:55-03:07:57, 03:07:59-03:08:01, 03:08:03-03:08:05, 03:08:07-03:08:09, 03:08:11-03:08:13, 03:08:15-03:08:17, 03:08:19-03:08:21, 03:08:23-03:08:25, 03:08:27-03:08:29, 03:08:31-03:08:33, 03:08:35-03:08:37, 03:08:39-03:08:41, 03:08:43-03:08:45, 03:08:47-03:08:49, 03:08:51-03:08:53, 03:08:55-03:08:57, 03:08:59-03:09:01, 03:09:03-03:09:05, 03:09:07-03:09:09, 03:09:11-03:09:13, 03:09:15-03:09:17, 03:09:19-03:09:21, 03:09:23-03:09:25, 03:09:27-03:09:29, 03:09:31-03:09:33, 03:09:35-03:09:37, 03:09:39-03:09:41, 03:09:43-03:09:45, 03:09:47-03:09:49, 03:09:51-03:09:53, 03:09:55-03:09:57, 03:09:59-03:10:01, 03:10:03-03:10:05, 03:10:07-03:10:09, 03:10:11-03:10:13, 03:10:15-03:10:17, 03:10:19-03:10:21, 03:10:23-03:10:25, 03:10:27-03:10:29, 03:10:31-03:10:33, 03:10:35-03:10:37, 03:10:39-03:10:41, 03:10:43-03:10:45, 03:10:47-03:10:49, 03:10:51-03:10:53, 03:10:55-03:10:57, 03:10:59-03:11:01, 03:11:03-03:11:05, 03:11:07-03:11:09, 03:11:11-03:11:13, 03:11:15-03:11:17, 03:11:19-03:11:21, 03:11:23-03:11:25, 03:11:27-03:11:29, 03:11:31-03:11:33, 03:11:35-03:11:37, 03:11:39-03:11:41, 03:11:43-03:11:45, 03:11:47-03:11:49, 03:11:51-03:11:53, 03:11:55-03:11:57, 03:11:59-03:12:01, 03:12:03-03:12:05, 03:12:07-03:12:09, 03:12:11-03:12:13, 03:12:15-03:12:17, 03:12:19-03:12:21, 03:12:23-03:12:25, 03:12:27-03:12:29, 03:12:31-03:12:33, 03:12:35-03:12:37, 03:12:39-03:12:41, 03:12:43-03:12:45, 03:12:47-03:12:49, 03:12:51-03:12:53, 03:12:55-03:12:57, 03:12:59-03:13:01, 03:13:03-03:13:05, 03:13:07-03:13:09, 03:13:11-03:13:13, 03:13:15-03:13:17, 03:13:19-03:13:21, 03:13:23-03:13:25, 03:13:27-03:13:29, 03:13:31-03:13:33, 03:13:35-03:13:37, 03:13:39-03:13:41, 03:13:43-03:13:45, 03:13:47-03:13:49, 03:13:51-03:13:53, 03:13:55-03:13:57, 03:13:59-03:14:01, 03:14:03-03:14:05, 03:14:07-03:14:09, 03:14:11-03:14:13, 03:14:15-03:14:17, 03:14:19-03:14:21, 03:14:23-03:14:25, 03:14:27-03:14:29, 03:14:31-03:14:33, 03:14:35-03:14:37, 03:14:39-03:14:41, 03:14:43-03:14:45, 03:14:47-03:14:49, 03:14:51-03:14:53, 03:14:55-03:14:57, 03:14:59-03:15:01, 03:15:03-03:15:05, 03:15:07-03:15:09, 03:15:11-03:15:13, 03:15:15-03:15:17, 03:15:19-03:15:21, 03:15:23-03:15:25, 03:15:27-03:15:29, 03:15:31-03:15:33, 03:15:35-03:15:37, 03:15:39
--

[illegible]

*30:163	-								7							15									
*31:01:02:01-31:02:02, 31:05-31:07, 31:09-31:20, 31:22-31:23, 31:26-31:28, 31:30-31:34, 31:36-31:40, 31:42-31:61, 31:63-31:84, 31:86-31:108, 31:110-31:122, 31:124-31:175, 31:177-31:178, 31:180-31:186	A31, Null, -															16									
*31:03	A31		w									12		14	16										
*31:04:01:01	A31		w									12		14	16					24					
*31:04:01:02-31:04:02	-		3									12		14	16					24					
*31:08, 33:53 ⁴	A31, A24, -					5									16		18								
*31:21	-														16	17									
*31:24-31:25	-												13		16										
*31:29	-					5	6								16										
*31:35	-														15	16	17								
*31:41	-														16				21						
*31:62	-														16				22						
*31:85	-					6									16										
*31:109, 33:125, 33:131 ⁴	-														16		18								
*31:123	-											12		14	16					24					
*31:176, 31:179	-			3											16										
*32:01:01:01-32:02, 32:07-32:14, 32:16-32:25, 32:27N-32:37, 32:39-32:41, 32:43:01-32:59, 32:61, 32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:106:01:02, 32:108-32:127, 32:129-32:141, 32:143-32:147	A32, Null, -									9							17								
*32:03:01:01-32:03:01:02, 32:38, 32:60, 32:71, 32:78, 32:80, 32:107	A32, -																17								
*32:04, 32:142	A32, A3, -			3						9							17								
*32:05, 32:79	A32, -					5									16	17	18								
*32:06	A32									9							17		21						
*32:15	A32								8	9							17	18							
*32:26:01-32:26:02	-									9			13				17								
*32:42	-									9				14			17								
*32:62	-									9							17	19							
*32:64	-				4					9							17								
*32:72	-						6			9							17								
*32:93	-									9							17	18							
*32:128	-									9						16	17								
*33:01:01:01-33:01:15, 33:03:01:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:130, 33:132-33:143N, 33:145-33:178, 33:180-33:183, 33:185-33:186, 33:188-33:189, 33:191-33:200, 33:202-33:208	A33, Null, -																	18							
*33:13, 33:48	-									10							18								
*33:19	-					5	7										18								
*33:22	-																18		21						
*33:24	-																18			23					
*33:49, 33:144, 33:179, 33:201	-			w													18								
*33:51	-								8								18								
*33:61, 33:190	-												13				18								
*33:119	-						7										18								
*33:184	-											11					18			23					
*33:187	-					5	6										18								
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	ser. ³	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*34:01:01:01-34:01:05, 34:05-34:06, 34:11-34:12, 34:14, 34:16-34:18, 34:23, 66:40 ⁴	A34, A10								8			11	12												
*34:02:01:01-34:03, 34:07, 34:10N, 34:13, 34:15, 34:20-34:21, 34:24	A34, Null, -			3					8			11	12												
*34:04	A34			3					8			11	12		14										
*34:08	A34			3					8			11	12	13											
*34:09	-			3					8				12												24
*34:22	-			3					8			11	12												
*36:01:01:01-36:01:01:02, 36:03, 36:05-36:11	A36, -	1																					22		
*36:02	A36	1		3											14							22	23		
*36:04	A36	1			4																	22			
*43:01-43:02N	A43, Null										10	12	13												
*66:01:01:01-66:01:05, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N, 66:29-66:33, 66:35-66:38, 66:41	A66, A26, Null, -								8			11		13											
*66:02-66:03:01:03, 66:16, 66:21, 66:25-66:26Q, 66:28N, 66:34, 66:39N, 66:43	A66, A10, Null, -								8				12												24
*66:06	A66								8			11		13							21				
*66:09	-								8			11		13	14						21				
*66:10	-								8			11		w											
*66:12	-								8					13											24
*66:19	-								8			11	12	13											

[illegible]