

Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB alleles ^{1,2,5}	ser ³																								
*01:01:01:01:01:02:14, 01:04:01:01:38, 01:40N-01:41, 01:43-01:60, 01:62:01N-01:88, 01:90-01:98, 01:100-01:101, 01:103-01:111, 01:113-01:126, 01:128-01:138, 01:140-01:154, 01:156	DR1, Null, -	73																							
*01:03:01:01:01:03:05, 01:39N, 01:99, 01:102, 01:112, 01:127, 01:155	DR103, DR1, Null, -	74																							
*01:42, 01:61, 01:89	-	73	74																						
*01:139	-	73																					94		
*03:01:01:01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:41, 03:04:01-03:06, 03:09, 03:11:01, 03:11:03-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52:02, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139, 03:141-03:149, 03:151-03:153, 03:158-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175, 03:180-03:184, 03:186-03:187, 03:191-03:205, 03:207-03:210, 03:214, 03:216-03:217, 03:219-03:222	DR17, DR3, Null, -					77	78											89							
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:130, 03:156N, 03:189N	DR17, Null, -					77	78																		
*03:01:24, 03:07:01:01:03:07:02, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150, 03:155, 03:171, 03:178, 03:185, 03:188, 03:213	DR3, -					77												89							
*03:02:01:01:03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131, 03:154, 03:167, 03:176, 03:179, 03:206, 03:212, 03:215	DR18, DR3, -					77	79											89							
*03:08, 03:65, 03:140	DR3, -					77	78				83		85												
*03:10	DR3					77	78													91	92				
*03:42, 03:69, 03:87, 03:111, 03:162, 03:173, 03:190	-					77																			
*03:76, 03:174N	Null, -					77	78				84				87		89								
*03:125	-					77	78				84				87	88	89								
*03:126, 13:193 ⁴	-					78	79										89	91							
*03:157	-					77	78										89		92						
*03:177	-					77	78	79									89	91							
*03:211	-					77	79										89							94	
*03:218N, 13:176 ⁴	Null, -					78					84				87		89								
*04:01:01:01:04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74:01-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:161, 04:163-04:210, 04:212N-04:250, 04:252-04:283, 04:285-04:381, 04:383-04:393, 04:395-04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:162, 04:251, 04:284:01-04:284:02, 04:394	DR4, DR4, DR3, Null, -								80											90					
*04:211	-								80											91					
*07:01:01:01:07:01:01:02, 07:01:01:04-07:01:35, 07:03-07:05, 07:07-07:127, 07:129N-07:168	DR7, Null, -									81															
*07:06	DR7									81	82														
*07:128	-									81										90					
Well No.		73	74	75	76	77	78	79	80	81	82	83	84	85	86	87	88	89	90	91	92	93	94	95	96
DRB alleles ^{1,2,5}	ser ³																								
*08:01:01:01:08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:07, 08:04:01:01:08:07, 08:11:01:01:08:11:01:02, 08:16-08:17, 08:22, 08:24:01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82-08:86, 08:88-08:90, 08:93-08:94, 08:96, 08:99-08:101, 08:103, 08:105-08:107, 08:109, 08:111-08:114, 08:116-08:118, 08:120, 08:122, 08:127, 08:130, 08:132, 08:135	DR8, Null, -										82						88								
*08:01:06, 08:03:02:01-08:03:10, 08:03:12-08:03:16, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81, 08:91, 08:95, 08:97, 08:102, 08:104, 08:110, 08:115, 08:119, 08:121, 08:128, 08:131, 08:136	DR8, Null, -										82														
*08:03:11, 08:68:01	-										82														
*08:08	DR8										82						88		90						
*08:09:01	DR8										82						88				92				
*08:09:02, 14:15	DR8, -										82						88	89		92					
*08:20, 08:129, 13:18, 13:47, 13:55, 13:158-01-13:158:02, 13:164, 13:232, 13:286, 13:340 ⁴	undefined, DR13, -																87	88	89		92				
*08:21	DR8										82						87	88			92				
*08:31, 11:05, 11:209, 11:309 ⁴	DR8, DR11, -													85			88								

Negative control

gative control

