

INTERPRETATION TABLE																											
HLA-B low resolution SSP																											
Amplification patterns of the B*0702 to B*8301 alleles ⁶																											
		Well																									
		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45		
Length of spec.		110	215	140	130	185	185	290	165	165	180	105	195	105	115	80	150	135	245	205	170	110	400	430	175		
PCR product(s)				235	265	235			220	390	395	195		435					400								
									330																		
Length of int.																											
pos. control ¹		800	1070	1070	1070	1070	800	1070	1070	800	800	1070	1070	1070	1070	1070	1070	1070	1070	800	800	1070	1070	1070	1070		
5'-primer(s) ²		527	97	209	103	103	103	45	45	45	45	206	142	45	161	167	355	206	45	142	409	246	44	44	363		
		5'-TgA ^{3'}	5'-Tcg ^{3'}	5'-ggC ^{3'}	5'-CCg ^{3'}	5'-CCT ^{3'}	5'-CCg ^{3'}	5'-ggA ^{3'}	5'-ggA ^{3'}	5'-ggA ^{3'}	5'-ggA ^{3'}	5'-AgA ^{3'}	5'-TCT ^{3'}	5'-ggA ^{3'}	5'-Cgg ^{3'}	5'-gCT ^{3'}	5'-TCA ^{3'}	5'-gAC ^{3'}	5'-ggA ^{3'}	5'-TCT ^{3'}	5'-ggC ^{3'}	5'-gAA ^{3'}	5'-ggC ^{3'}	5'-ggC ^{3'}	5'-Agg ^{3'}		
		540		363	361	363			357	435	420	420	419	540					368		420						
		5'-gAC ^{3'}		5'-AgC ^{3'}	5'-AgT ^{3'}	5'-AAT ^{3'}			5'-Tgg ^{3'}	5'-AAA ^{3'}	5'-TTA ^{3'}	5'-TTA ^{3'}	5'-gTC ^{3'}	5'-gAC ^{3'}					5'-gTT ^{3'}		5'-TTA ^{3'}						
									412																		
									5'-ATA ^{3'}																		
3'-primer(s) ³		605	272	309	193	246	246	165	206	266	272	272	301	309	234	204	463	302	259	301	544	317	272	301	499		
		5'-gCT ^{3'}	5'-Tgg ^{3'}	5'-gTg ^{3'}	5'-CgT ^{3'}	5'-TAT ^{3'}	5'-TAT ^{3'}	5'-Tgg ^{3'}	5'-CCT ^{3'}	5'-TCC ^{3'}	5'-TgC ^{3'}	5'-Tgg ^{3'}	5'-gTC ^{3'}	5'-ATC ^{3'}	5'-TCT ^{3'}	5'-TCT ^{3'}	5'-gCT ^{3'}	5'-ggC ^{3'}	5'-gTT ^{3'}	5'-gTC ^{3'}	5'-ggT ^{3'}	5'-ggA ^{3'}	5'-TgC ^{3'}	5'-ggC ^{3'}	5'-ggA ^{3'}		
			272	559	583	559			538	559	559	572	570	605					266	309				302			
			5'-TgA ^{3'}	5'-CAG ^{3'}	5'-gTg ^{3'}	5'-CTC ^{3'}			5'-gTC ^{3'}	5'-CAG ^{3'}	5'-CTC ^{3'}	5'-gCg ^{3'}	5'-CCg ^{3'}	5'-gCT ^{3'}					5'-TCC ^{3'}	5'-gTg ^{3'}				5'-ggT ^{3'}			
													572						272								
																			292								
																			5'-gTA ^{3'}								
																			572								
																			5'-gCg ^{3'}								
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45		
HLA-B allele ⁴	ser. ⁵																										
*070201-070207, 070501-0706, 0710, 0712, 0714, 071801- 071802, 0721-0723, 0726, 0728-0731, 0734, 0735, 0739, 0741, 0743-0747, 0749N, 0752, 0755- 0759, 0761-0764, 0766	B7	22																						44			

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45		
*0703, 0708, 0716, 0732, 0737	B7, B703	22																							44		
*0704, 0725	B7	22												34											44		
*0707, 0742	B7	22																							44		
*0709, 0711, 0717	B7	22																							44		
*0713, 0715	-	22																							44		
*0719	-																								44		
*0720, 0724	-	22		24																					44		
*0727	-	22										32							40								
*0733, 0753	-																								44		
*0736	-	22																									
*0738	-	22																									
*0740, 0748, 0751	-																								44		
*0750	-	22										32													44		
*0754	-	22																							44		
*0760	-			24																					44		
*0765	-	22																38									
*080101-080105, 0805, 0808N, 0810, 0811, 0815, 0818, 0819N, 0822-0824, 0827, 0829, 0830N, 0833, 0834	B8, Null		23						29																44		
*0802	B8		23						29										40								
*0803	B8		23						29																		
*0804	-		23						29			32													44		
*0806, 0813, 0820	B8		23																						44		
*0807, 0814	-		23																						44		
*0809	-		23						29																	44	
*0812, 0816, 0831	-		23						29																	44	
*0817			23						29			32														44	
*0821	-		23	24					29																	44	
*0825	-		23	24																						44	
*0826, 4207	-								29																	44	
*0828	-		23																							44	
*0832	-		23						29									38								44	
*0835	-		23						29																	44	
*0836	-		23						29																		
*1301, 1317, 1320, 1322	B13			24	25						31							37									
*130201-130204, 1308Q, 1315, 1318, 1319	B13				24	25					31																
*1303	-			24	25																						
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45		

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*1304, 1310	-			24	25																				
*1306	-			24	25												37								
*1307N	Null			24	25					31							37								
*1309	-				25					31															
*1311	-			24	25					31															
*1312, 1313	-			24	25												37								
*1314	-			24						31															
*1316	-			24	25					31															
*1321	-			24	25																				
*1401, 1407N	B64				25	26																	43	44	
*140201-1404, 1409	B65				25		27																43	44	
*1405-140602	-						27																43	44	
*1408, 3925N, 3930, 3932-3934	-					26																	43	44	
*15010101-15010102N, 150103, 1526N, 152701, 1532, 1534, 1535, 1538, 1550, 1556, 1557, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509-9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529	B62, Null, B15							28										39							
*150102	B62							28		30								39							
*150104	B62																	39							
*150106-150107, 1507	B62			24				28										39							
*150108				24														39							
*1502, 1588, 9512, 9521	B75							28									37	39							
*1503, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532	B72							29			32							39							
*1504								28										39							
*1505, 1506, 1533, 1539, 1540, 1565, 1578, 1584, 9507	B62, B15							28										39							
*1508, 151101-151103, 1515	B75							28										39							
*1509	B70							29		30	31							39							
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*1510, 1537, 1590, 1599, 9533	B71								29		31								39						
*1512, 1514, 1519	B76							28				33							39						
*1513	B77							28						34			37		39						
*1516, 1567, 1595	B63									30				34					39						
*15170101-151702	B63							28		30				34					39						
*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534	B71								29		31								39						
*1520, 1525	B62							28									37		39						
*1521, 1544	B75							28			31						37		39						
*1523	-								29	31				34					39						
*1524, 1587	B62							28						34					39						
*152702	-			24															39						
*152703	-			24				28											39						
*1528	B15							28											39						
*1529	B15								29										39						
*1530, 1558, 1563	B62, B15							28											39						
*1531	B75							28											39						
*1536	-			24				28									37		39						
*1542	-							28											39						
*1543	-							28											39						
*1545	B62			24				28											39						
*1546	B72				25			28	29										39						
*1548	B62							28											39						
*1553	-				25				29										39						
*1554	-								29			32							39						
*1555	-			24				28											39						
*1562	-								29			32					37		39						
*1564	-								29			32							39						
*1566	-							28			31								39						
*1568	-			24				28	29		32								39						
*1569	-								29		32								39		41				
*1573	-							28											39						
*1574	-								29										39						
*1576, 9501	-							28											39						
*1577	-							28									37		39						
*1580	-								29	31							37		39						
*1583	-							28	29										39						
*1585	-							28									37		39						
*1586	-							28											39						
*1589	-			24				28									37		39						
*1591, 9531	-								29		32	33							39						
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*9506	-				25			28	29								37		39						
*9515	-								29		31								39	40					
*9516	-							28											39						45
*9520	-							28											39						
*9524	-								29		31								39						45
*9526, 9535, 9540, 9545-9547	-			24				28											39						
*9536	-			24				28											39		41				
*9537	-			24				28											39						
*9538	-	22						28											39						
*9539	-			24				28									37		39						
*9541	-			24				28											39						
*9542	-			23	24			28											39						
*9543	-							28	29										39						
*9544	-			24				28									37		39						
*9548	-			24				28											39						
*9549N	-			24				28											39						
*180101-1803, 1805- 1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825, 1827,	B18														35			38							
*1804	-														35			38							
*1809	-														35				40						
*1812	-														35			38							
*1819	-														35			38							
*1822	-														35		37	38							
*1826	-														35			38							
*1829, 3532, 3537, 3553N, 3564, 356801- 356802																			38						
*2701	B27					26										36									
*2702	B27					26										36									
*2703-270402, 270502- 2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737	B27					26										36									
*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738,	B27															36									
*2708, 2726, 2740, 2742	B2708					26										36									
*2712	-					26										36									
*2716, 2739	-					26										36									
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*2718	-					26						32				36									
*2719	-															36	37								
*2723	-					26										36									
*2724	-															36									
*2725	-															36									
*2729	-					26						32				36									
*2730	-															36	37								
*2733	-															36									
*2741	-					26										36									
*350101-350107, 3507-350803, 3511, 3515, 352001-352002, 3523, 3529, 3540N-3542, 3548, 3550, 3552, 3554, 3557, 3561, 3577, 3578, 3582, 3590-3594	B35																37	38							
*350201-350401, 350403, 3506, 3512, 3533, 3534, 3536, 3539, 3555, 3559, 3565Q, 3570, 3574, 3583-3585	B35																37	38							
*350402	-									30							37	38							
*3505, 3551, 3558, 3572	B35			24														38							
*350901-350902, 3518, 3531, 3575, 3588	B35																	38							
*3510, 3528, 3569, 3580	-																37	38							
*3513	B35																37	38							
*351401-351402, 3562	B35																37	38							
*3516	-			24													37	38							
*3517, 3530	B35			24													37	38							
*3519	B35																37								
*3521, 3524	-																37	38							
*3522	-			24														38							
*3525, 3549	-																37								
*3526	-						27										37								
*3527	B35																37								
*3535	-																37	38			41				
*3538	-																37	38							
*3543, 3567, 3579, 3586	B35																	38							
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*3544	-																38								
*3545, 3571	-											33					37	38							
*3546	-																37								
*3547	-				25												37								
*3556	-																37								
*3560	-																	38							
*3563	-				25												37								
*3566	-																	38							
3576	-																37	38							
*3581	-																37	38							
*3587	-								29								38								
*370101-370103, 3703N, 3706, 3713	B37, Null											33					37			40					
*370104	-											33					37			40					
*3702	-					26						33								40					
*3704	-											33					37			40					
*3705	-											33					37			w					
*3707	-	22										33								40					
*3708	-											33					37			40					
*3709	-											33								40					
*3710	-																37			40					
*3711	-																37	38							
*3712	-								29			33								40					
*3714	-																38								
*380101-380102, 3809, 3811-3814, 3816	B38					26															41	42	43		
*380201-380202, 3808, 3815	B38					26													40		41	42	43		
*3803	B16					26						32							40		41	42			
*3804	-																		40		41		43		
*3805	-						27														41	42	43		
*3806	-																				41				
*3807	-					26															41				
*3810	-					26																42	43		
*3817	-					26						33							40		41	42	43		
*39010101-39010102L, 390103-390105, 3903, 3905-390602, 3909, 3911, 3914, 3915, 3924, 3926-3929, 3931, 3935, 3937, 3938Q, 3940N, 3941, 3944	B3901, B39					26															41		43	44	
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*390201, 3908	B3902, B39					26						32									41			44	
*390202, 391301-391302, 3923	B3902, B39											32									41			44	
*3904	B39						27														41		43	44	
*3907	-					?	?														41		43	44	
*3910, 3916, 3920	B39																				41			44	
*3912, 3945	-																				41		43	44	
*3917	-																				41			44	
*3918	-					26															41		43	44	
*391901	-																	38			41		43	44	
*391902	-					26												38			41		43	44	
*3922	-					26															41		43	44	
*3936	-					26																	43	44	
*3939	-					26						32									41			44	
*3942	-					26											37				41			44	
*3943	-					26																		44	
*400101-400106, 4007	B60, B40				25																			44	45
*400201-400203, 4004, 40060101-400602, 4039	B61, B40																								
*4003, 4020	B40																								
*4005	B4005			24																					
*4008	-																								
*4009, 4011, 4018, 4024, 4027, 4029, 4035, 4037, 4040, 4044, 4050, 4056, 4057, 4064, 4075, 4078, 4082, 4085,	B61, B40																							?	
*4010, 4022N, 4033, 4036, 4042, 4043, 4049, 4054, 4055, 4061-4063, 4065-4067, 4073, 4081, 4084, 4087, 4088	Null, B40, B60				25																			?	45
*4012	-								29			32							39						45
*4013, 4019	-																								
*401401-401403, 4053, 407201	-				25																			?	
*4015, 4016, 4032	B61	22			25																			?	
*4021	-																							?	45
*4023	-	22			25																			44	45
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*4025	-				25																			?	45
*4026	B21									30														?	
*4028	-									30							37							?	
*4030, 4034	-				25																			?	
*4031, 4045, 4080	-				25																			?	
*4038, 4052	-				25																			?	45
*4046	-										32					36								?	45
*4047	-				25																				45
*4048	-				25					31														?	45
*4051, 4079	-				25																			44	45
*4058	-				25																			?	
*4059	-				25																			44	
*4060	-				25																			44	45
*4068	-																							?	
*4069	-				25																			?	45
*4070, 4089-4091	-																								
*4071	-			24																				?	
*407202, 4077	-				25																			44	
*4074	-																							?	45
*4076	-				25																			?	45
*4083	-																							?	
*4092	-				25																				45
*4101, 4106, 4107	B41				25				29															44	
*4102	B41				25				29															44	
*410301-410302	B41				25				29															44	
*4104	-				25																			44	
*4105	-				25				29															44	
*4108	-	22			25				29					34										44	
*4201, 4202, 4206	B42								29															44	
*4204	-								29															44	
*420501-420502	-	22							29					34										44	
*4208	-								29															44	
*4209	-																							44	
*44020101-440205, 4411, 4416, 4419N, 4421-4424, 4427, 4433, 4434, 4448, 4451, 4452N, 4455, 4459, 4463	B44, B47, Null				25												37								
*440301-4405, 4413, 4426, 442801-4430, 4432, 4435, 4436, 4438, 4439, 4442, 4445, 4456N, 4458N, 4461N	B44				25												37								

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*4406	B44																37								
*4407																	37								
*4408	B44			24													37								
*4409	B12				25												37								
*4410	-				25							32					37		39						
*4412	-				25												37								
*4414	-				25				29								37								
*4415	-				25				29			32													
*4417, 4443	-				25							33					37								
*4418	-				25				29			32													
*4420	-				25				29																
*4425, 4450	-				25												37								
*4431	-				25																				45
*4437	-				25												37								
*4440	-											32				36	37								
*4441	-				25																				
*4444	-											32				36	37								
*4446	-				25												37								
*4447	-				25																				
*4449	-			23	25												37								
*4450	-				25												37								
*4453	-				25																				
*4454	-				24	25																			
*4457	-				24												37								
*4460	-				24	25											37								
*4462	-				25				29	30							37								
*446401-446402	-				25												37								
*4501, 4505, 4507	B45				25				29			32													44
*4502, 4503	-				25				29																44
*4504	-				25				29																44
*4506	-				25				29			32													44
*4508	-				25				29																44
*4509	-				25				29			33													44
*460101-460102, 4604, 4605, 4607N, 4609	B46								28																
*4602	-								28																
*4603, 4608	-								28																
*4606	-								28																45
*4610	-								28																
*4611	-				24				28																
*4612, 4614-4616	-				24				28																
*4613	-				24				28																
*47010101-47010102	B47				25																				
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*4702	-				25																				
*4703	-				25																				
*4704, 4705	-				25	26																			
*480101-480102, 4807, 4809, 4811, 4819	B48											32												44	
*4802	-											32					37							44	
*480301-480302	B48											32												44	45
*4804	B48											32												44	
*4805, 4808	B48	22										32												44	
*4806	-																							44	
*4810, 4812	-											32												44	
*4813	-											32												44	
*4814	-											32												44	
*4815	-	22										32												44	
*4816	-											32												44	
*4817	-											32												44	45
*4818	-											32													
*4901	B49				25																				
*4902	-				25																				
*4903	-																								
*4904, 4905	-				25																				
*5001, 5004	B50				25																			44	
*5002	B45				25							32												44	
*510101, 510103, 510105-510108, 510201, 510203, 5103, 5107, 510901, 5111N, 5112, 511302, 5114, 5117-5119, 5122, 5123, 5126-5130, 5132, 5133, 5138-5141N, 5143, 5148, 5149, 5151-5153, 5155	B51, B5102, B5103, Null											30													
*510102, 510109, 510202, 5105, 510902	B51											30													
*510104, 511301, 5116, 5121, 512401-512402, 5131, 5134, 5135	B51, B52																								
*5104, 5146	B51											30					37								
*5106	B51											30													
*5108, 5120, 5144N	B51								29			30													
*5110	-																								
*5115	-																								
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*5136	-								29																
*5137	-									30															
*5142	-																37								
*5145	-																								
*5150, 5154	-									30															
*5156	-									30							37								
*5157	-									30															
*520101, 520103, 5204, 5205, 5207, 5210-5212	B52									30															
*520102, 520104-5203, 520601-520602, 5209	B52									30															
*5208	-																								
*5213										30															
*5214	-																								
*530101-530104, 5305, 5308, 5310, 5316	B53																37								
*5302	B53																37								
*5303, 5309, 5311- 5313	-																37								
*5304, 5307	-																37								
*5306	B53																37								
*5314	-			24																					
*5315	-																37								
*5401, 5402, 5405N, 5407, 5408N, 5410, 5413	B54																								
*5403	-																								
*5404, 5411, 5415	-																								
*5406	-																								
*5409, 5414	-																								
*5412	-																								
*550101-550104, 5503, 5505, 5511, 5515, 5525, 5529, 5531	B55																								
*550201-550202, 5507, 5510, 5512, 5516, 5519, 5526, 5530	B54, B55																								
*5504, 5508, 5513, 5527, 5615, 5619N, 5622	B55																								
*5509, 5517, 5528	-																								
*5514	-																37								
*5518	-																								
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*5520	-																								
*5521	-																								
*5522	-									31															
*5523, 5532, 5618	-																								
*5524	-									31															
*5601, 5608, 5614, 5616, 5620	B56																								
*5602, 5604	B56																								
*5603	B22																								
*560501	-									30															
*560502	-									30															
*5606	B78									30							38								
*5607	-																								
*5609	-																37								
*5610	-																								
*5611, 5612	-																37								
*5613	-								29																
*5617	-																								
*5621	-									30															
*5623	-																								
*570101-570103, 5706, 5708, 5710	B57																37								
*5702-5705	B57																37								
*5707	-																37								
*5709	-								29								37								
*5711	-																								
*5712	-																37								
*5713	-																								
*5714	-			24													37								
*5715	-			24													37								
*5716	-																37								
*5717	-			24													37								
*580101, 5804, 5805, 5810N-5814	B58																37								
*580102, 5815	-																37								
*5802, 5806	B58																								
*5807	-												33												
*5808	-									30															
*5809	-																37								
*5816																									
*5817N, 5819																	37								
*5818				24																					
*5901	B59																								
*5902	-																								
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*5903	-																								
*670101	B67					26															41			44	
*670102	B67																				41			44	
*6702	-																				41			44	
*7301	B73																							44	
*7801, 780202, 7803	B78									30								38							
*780201, 7804	B78									30								38							
*7805	-									30								38							
*7806	-									30															
*8101	B81																								
*8102	-																							44	
*8103, 8104N	-																								
*8201, 8202	-								29									39							
*8301	-																	37							
HLA-B allele ⁴	ser. ⁵																								
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
Cw*0206																									
Cw*0305, 0325, 0327				24																					
Cw*0746									29																
Cw*150204																									
Well No.		22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
																	64		B7, B703	*0703, 0708, 0716, 0732, 0737
46	47																64		B7	*0704, 0725
	47																64		B7	*0707, 0742
	47																64		B7	*0709, 0711, 0717
	w																64		-	*0713, 0715
46	47																64		-	*0719
	47																64		-	*0720, 0724
																63			-	*0727
	47																64		-	*0733, 0753
	47															63			-	*0736
																63			-	*0738
	47																64		-	*0740, 0748, 0751
																	64		-	*0750
	47									56							64		-	*0754
	47																64		-	*0760
	47			50								58			61		64		-	*0765
46																	64		B8, Null	*080101-080105, 0805, 0808N, 0810, 0811, 0815, 0818, 0819N, 0822-0824, 0827, 0829, 0830N, 0833, 0834
46																63			B8	*0802
46																63			B8	*0803
46																	64		-	*0804
																	64		B8	*0806, 0813, 0820
46																	64		-	*0807, 0814
46																	64		-	*0809
46																	64		-	*0812, 0816, 0831
46										56							64			*0817
46																	64		-	*0821
																	64		-	*0825
46																	64		-	*0826, 4207
46																	64		-	*0828
46															61		64		-	*0832
46																	64		-	*0835
46																	64		-	*0836
										56							63		B13	*1301, 1317, 1320, 1322
										56							63		B13	*130201-130204, 1308Q, 1315, 1318, 1319
										56			59				63		-	*1303
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
										56						63			-	*1304, 1310
										56		58				63			-	*1306
																63			Null	*1307N
										56							64		-	*1309
										56						63			-	*1311
										56						63			-	*1312, 1313
										56						63			-	*1314
							53			56						63			-	*1316
										56						63			-	*1321
																	64		B64	*1401, 1407N
																	64		B65	*140201-1404, 1409
																	64		-	*1405-140602
																	64		-	*1408, 3925N, 3930, 3932-3934
										56						62	64		B62, Null, B15	*15010101-15010102N, 150103, 1526N, 152701, 1532, 1534, 1535, 1538, 1550, 1556, 1557, 1560, 1570, 1571, 1575, 1579N, 1581, 1582, 1592, 1594N, 1596, 1597, 9502, 9504, 9505, 9509-9511N, 9513, 9517, 9518, 9522, 9525, 9528, 9529
										56						62	64		B62	*150102
										56	57					62	64		B62	*150104
										56						62	64		B62	*150106-150107, 1507
										56						62	64			*150108
					51												64		B75	*1502, 1588, 9512, 9521
										56							64		B72	*1503, 1547, 1549, 1561, 1598, 9503, 9523, 9527, 9532
							w			56						62	64			*1504
										56							64		B62, B15	*1505, 1506, 1533, 1539, 1540, 1565, 1578, 1584, 9507
					51											62	64		B75	*1508, 151101-151103, 1515
																	64		B70	*1509
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
																		64	B71	*1510, 1537, 1590, 1599, 9533
										56						62		64	B76	*1512, 1514, 1519
					51												63		B77	*1513
							53										63		B63	*1516, 1567, 1595
																	63		B63	*15170101-151702
																		64	B71	*1518, 1551, 1552, 1572, 1593, 9508, 9514, 9519, 9534
										56								64	B62	*1520, 1525
					51													64	B75	*1521, 1544
																	63		-	*1523
										56						62	63		B62	*1524, 1587
										56						62		64	-	*152702
										56						62		64	-	*152703
																62		64	B15	*1528
																		64	B15	*1529
										56						62		64	B62, B15	*1530, 1558, 1563
					51													64	B75	*1531
										56							63		-	*1536
							53			56		58						64	-	*1542
										56						62	63		-	*1543
										56						62		64	B62	*1545
									55	56						62		64	B72	*1546
										56								64	B62	*1548
									55	56						62		64	-	*1553
										56						62		64	-	*1554
					51													64	-	*1555
										56								64	-	*1562
																		64	-	*1564
										56						62		64	-	*1566
										56						62		64	-	*1568
										56								64	-	*1569
										56			59			62		64	-	*1573
										56								64	-	*1574
	47															62		64	-	*1576, 9501
										56						62		64	-	*1577
																		64	-	*1580
										56								64	-	*1583
										56						62		64	-	*1585
										56		58						64	-	*1586
					51												63		-	*1589
										56								64	-	*1591, 9531
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
									55	56								64	-	*9506
																	63		-	*9515
										56								64	-	*9516
																62		64	-	*9520
																		64	-	*9524
										56						62		64	-	*9526, 9535, 9540, 9545-9547
										56								64	-	*9536
							53			56						62		64	-	*9537
										56								64	-	*9538
					51													64	-	*9539
										56								64	-	*9541
										56						62		64	-	*9542
									55							62		64	-	*9543
					51											62		64	-	*9544
					51											62		64	-	*9548
																62		64	-	*9549N
															61				B18	*180101-1803, 1805-1808, 1810, 1811, 1813-1815, 1817N, 1818, 1820, 1821, 1823N-1825, 1827,
												58			61			64	-	*1804
															61		63		-	*1809
										56								64	-	*1812
															61	62		64	-	*1819
			49												61			64	-	*1822
															61			64	-	*1826
															61			64		*1829, 3532, 3537, 3553N, 3564, 356801-356802
	47									56							63		B27	*2701
	47			50				54		56							63		B27	*2702
	47			50						56								63	B27	*2703-270402, 270502-2706, 2709, 2710, 2713, 2717, 2720, 2727, 2731, 2735-2737
	47			50						56								63	B27	*2707, 2711, 2714, 2715, 2721, 2728, 2732, 2734, 2738,
	47			50						56									B2708	*2708, 2726, 2740, 2742
										56								64	-	*2712
										56							63		-	*2716, 2739
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
									56							64			-	*2718
	47			50					56							63			-	*2719
																63			-	*2723
	47			50	51				56							63			-	*2724
	47			50					56							62	63		-	*2725
									56							63			-	*2729
	47			50				54	56							63			-	*2730
	47			50					56								64		-	*2733
	47			50					56							63			-	*2741
															61			64	B35	*350101-350107, 3507-350803, 3511, 3515, 352001-352002, 3523, 3529, 3540N-3542, 3548, 3550, 3552, 3554, 3557, 3561, 3577, 3578, 3582, 3590-3594
															61			64	B35	*350201-350401, 350403, 3506, 3512, 3533, 3534, 3536, 3539, 3555, 3559, 3565Q, 3570, 3574, 3583-3585
															61			64	-	*350402
															61			64	B35	*3505, 3551, 3558, 3572
															61			64	B35	*350901-350902, 3518, 3531, 3575, 3588
									56									64	-	*3510, 3528, 3569, 3580
									56									64	B35	*3513
															61	62		64	B35	*351401-351402, 3562
									56									64	-	*3516
															61			64	B35	*3517, 3530
									55									64	B35	*3519
		49													61			64	-	*3521, 3524
															61			64	-	*3522
																		64	-	*3525, 3549
																		64	-	*3526
															61			64	B35	*3527
															61			64	-	*3535
		48													61			64	-	*3538
															61	62		64	B35	*3543, 3567, 3579, 3586
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
															61	62	64		-	*3544
		48													61		64		-	*3545, 3571
					51												64		-	*3546
									55								64		-	*3547
															61		64		-	*3556
												58			61		64		-	*3560
									55	56							64		-	*3563
															61		64		-	*3566
	47			50											61		64		-	3576
			49												61		64		-	*3581
46															61		64		-	*3587
		48								56							63		B37, Null	*370101-370103, 3703N, 3706, 3713
		48								56	57						63		-	*370104
										56							63		-	*3702
		48	49							56							63		-	*3704
		48								56							64		-	*3705
										56							63		-	*3707
		48													61		63		-	*3708
										56							63		-	*3709
		48								56							63		-	*3710
		48								56							64		-	*3711
		48								56							63		-	*3712
										56							63	64	-	*3714
																	63		B38	*380101-380102, 3809, 3811-3814, 3816
																	63		B38	*380201-380202, 3808, 3815
										56							63		B16	*3803
										56							63		-	*3804
																	63		-	*3805
															61		63		-	*3806
															61		63		-	*3807
																	63		-	*3810
																	63		-	*3817
																	64		B3901, B39	*39010101-39010102L, 390103-390105, 3903, 3905-390602, 3909, 3911, 3914, 3915, 3924, 3926-3929, 3931, 3935, 3937, 3938Q, 3940N, 3941, 3944
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
									56									64	B3902, B39	*390201, 3908
									56									64	B3902, B39	*390202, 391301-391302, 3923
																		64	B39	*3904
																		64	-	*3907
																		64	B39	*3910, 3916, 3920
																		64	-	*3912, 3945
											58							64	-	*3917
																62		64	-	*3918
															61			64	-	*391901
															61			64	-	*391902
									56									64	-	*3922
																62		64	-	*3936
																		64	-	*3939
																		64	-	*3942
																		64	-	*3943
									55	56								64	B60, B40	*400101-400106, 4007
									55	56								64	B61, B40	*400201-400203, 4004, 40060101-400602, 4039
									55	56								64	B40	*4003, 4020
									55	56								64	B4005	*4005
									55									64	-	*4008
									55	56								64	B61, B40	*4009, 4011, 4018, 4024, 4027, 4029, 4035, 4037, 4040, 4044, 4050, 4056, 4057, 4064, 4075, 4078, 4082, 4085,
									55	56								64	Null, B40, B60	*4010, 4022N, 4033, 4036, 4042, 4043, 4049, 4054, 4055, 4061-4063, 4065-4067, 4073, 4081, 4084, 4087, 4088
									56									64	-	*4012
					52		54		56							63			-	*4013, 4019
									55	56								64	-	*401401-401403, 4053, 407201
									55	56								64	B61	*4015, 4016, 4032
									56									64	-	*4021
									55	56								64	-	*4023
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
									55									64	-	*4025
									55	56								64	B21	*4026
			49						55	56								64	-	*4028
									55	56								64	-	*4030, 4034
					51				55	56								64	-	*4031, 4045, 4080
									55	56								64	-	*4038, 4052
										56								64	-	*4046
										56						63		-	*4047	
									55	56								64	-	*4048
									55	56								64	-	*4051, 4079
									55	56		58						64	-	*4058
									55	56								64	-	*4059
									55	56								64	-	*4060
									55									64	-	*4068
									55									64	-	*4069
									55	56								64	-	*4070, 4089-4091
									55	56			59					64	-	*4071
									55	56								64	-	*407202, 4077
									55	56								64	-	*4074
										56								-	-	*4076
									55	56								64	-	*4083
									55	56								64	-	*4092
46									55	56								64	B41	*4101, 4106, 4107
46									55	56								64	B41	*4102
46									55	56								64	B41	*410301-410302
46									55	56								64	-	*4104
46									?	56								64	-	*4105
46									55	56								64	-	*4108
46	47																	64	B42	*4201, 4202, 4206
46	47																	64	-	*4204
46	47																	64	-	*420501-420502
	47																	64	-	*4208
46	47																	64	-	*4209
		48								56	57						63		B44, B47, Null	*44020101-440205, 4411, 4416, 4419N, 4421-4424, 4427, 4433, 4434, 4448, 4451, 4452N, 4455, 4459, 4463
		48								56							63		B44	*440301-4405, 4413, 4426, 442801-4430, 4432, 4435, 4436, 4438, 4439, 4442, 4445, 4456N, 4458N, 4461N

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
		48						54			57				61		63		B44	*4406
		48								56							63			*4407
		48								56	57						63		B44	*4408
		48							55	56	57							64	B12	*4409
		48								56			59				63		-	*4410
		48									57						63		-	*4412
		48								56							63		-	*4414
										56			59				63		-	*4415
		48								56							63		-	*4417, 4443
					52					56			59				63		-	*4418
		48								56	57						63		-	*4420
		48			52		54			56							63		-	*4425, 4450
		48								56							63		-	*4431
										56							63		-	*4437
		48								56							63		-	*4440
		48								56	57						63		-	*4441
		48								56	57						63		-	*4444
		48							55	56								64	-	*4446
		48								56							63		-	*4447
		48								56	57						63		-	*4449
		48			52		54			56							63		-	*4450
		48								56	57						63		-	*4453
		48								56							63		-	*4454
		48								56							63		-	*4457
		48								56							63		-	*4460
		48								56							63		-	*4462
										56							63		-	*446401-446402
									55	56			59					64	B45	*4501, 4505, 4507
									55	56								64	-	*4502, 4503
									55	56			59					64	-	*4504
47									55				59					64	-	*4506
									55	56		58						64	-	*4508
									55	56								64	-	*4509
47		49														62		64	B46	*460101-460102, 4604, 4605, 4607N, 4609
w		49														62		64	-	*4602
47		49																64	-	*4603, 4608
47		49																64	-	*4606
47		49								56						62		64	-	*4610
47		49								56			59					64	-	*4611
47		49								56						62		64	-	*4612, 4614-4616
47		49								56								64	-	*4613
				50						56							63		B47	*47010101-47010102
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
				50					55	56							64		-	*4702
				50					56								63	w	-	*4703
									56								63		-	*4704, 4705
					51				56									64	B48	*480101-480102, 4807, 4809, 4811, 4819
									56									64	-	*4802
									56									64	B48	*480301-480302
					51				56									64	B48	*4804
									56									64	B48	*4805, 4808
					51													64	-	*4806
									56									64	-	*4810, 4812
									56									64	-	*4813
									56									64	-	*4814
					51				56									64	-	*4815
					51													64	-	*4816
									56									64	-	*4817
					51				56								63		-	*4818
					52	53			56				59				63		B49	*4901
						53			56				59				63		-	*4902
						53			56				59				63		-	*4903
					52	53			56								63		-	*4904, 4905
					53				55	56			59					64	B50	*5001, 5004
						53			55	56			59					64	B45	*5002
								53 54						61			63		B51, B5102, B5103, Null	*510101, 510103, 510105-510108, 510201, 510203, 5103, 5107, 510901, 5111N, 5112, 511302, 5114, 5117-5119, 5122, 5123, 5126-5130, 5132, 5133, 5138-5141N, 5143, 5148, 5149, 5151-5153, 5155
								53 54						61			63		B51	*510102, 510109, 510202, 5105, 510902
								53 54						61			63		B51, B52	*510104, 511301, 5116, 5121, 512401-512402, 5131, 5134, 5135
			49					54						61			63		B51	*5104, 5146
								54						61			63		B51	*5106
								53 54						61			63		B51	*5108, 5120, 5144N
								54						61			63		-	*5110
								54					59	61			63		-	*5115
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
							53	54							61		63		-	*5136
							53	54							61		63		-	*5137
		48						54							61		63		-	*5142
								54							61		63		-	*5145
							53								61		63		-	*5150, 5154
		49													61		63		-	*5156
							53								61		63		-	*5157
							53	54		56							63		B52	*520101, 520103, 5204, 5205, 5207, 5210-5212
							53	54		56							63		B52	*520102, 520104-5203, 520601-520602, 5209
							53	54		56							63		-	*5208
							53			56							63			*5213
							53			56							63		-	*5214
							54								61		63		B53	*530101-530104, 5305, 5308, 5310, 5316
		49					54								61		63		B53	*5302
															61		63		-	*5303, 5309, 5311- 5313
								54							61		63		-	*5304, 5307
		49					54								61		63		B53	*5306
							54								61		63		-	*5314
							54								61		63		-	*5315
47				50							57	58					64		B54	*5401, 5402, 5405N, 5407, 5408N, 5410, 5413
47				50							57		59				64		-	*5403
47				50							57						64		-	*5404, 5411, 5415
47				50							57					62	64		-	*5406
47				50							57						64		-	*5409, 5414
47				50							57	58					63		-	*5412
47				50			53					58					64		B55	*550101-550104, 5503, 5505, 5511, 5515, 5525, 5529, 5531
47				50								58					64		B54, B55	*550201-550202, 5507, 5510, 5512, 5516, 5519, 5526, 5530
47				50													64		B55	*5504, 5508, 5513, 5527, 5615, 5619N, 5622
47				50			53										64		-	*5509, 5517, 5528
47				50										60			64		-	*5514
										56		58					64		-	*5518
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
46	47			50														64	-	*5520
	47			50			53					58				62		64	-	*5521
	47			50														64	-	*5522
	47			50														64	-	*5523, 5532, 5618
	47			50			53											64	-	*5524
	47			50									59					64	B56	*5601, 5608, 5614, 5616, 5620
	47			50									59					64	B56	*5602, 5604
	47			50												62		64	B22	*5603
	47			50			53					58						64	-	*560501
	47			50			53											64	-	*560502
	47			50			53					58						64	B78	*5606
	47			50									59				63		-	*5607
	47			50														64	-	*5609
	47			50								58						64	-	*5610
	47			50														64	-	*5611, 5612
	47			50									59					64	-	*5613
													59					64	-	*5617
	47			50			53										63		-	*5621
												58						64	-	*5623
								54						60			63		B57	*570101-570103, 5706, 5708, 5710
								54						60			63		B57	*5702-5705
		48						54						60			63		-	*5707
		48						54						60			63		-	*5709
								54						60			63		-	*5711
										56				60			64		-	*5712
								54		56				60			63		-	*5713
			49					54		56				60			63		-	*5714
								54		56				60			63		-	*5715
								54		56							63		-	*5716
										56				60			63		-	*5717
								54									63		B58	*580101, 5804, 5805, 5810N-5814
								54		56							63		-	*580102, 5815
								54									63		B58	*5802, 5806
								54									63		-	*5807
							53	54									63		-	*5808
			49					54									63		-	*5809
								54		56							63			*5816
										56							63			*5817N, 5819
										56							63			*5818
												58					63		B59	*5901
								54									63		-	*5902
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.

46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
																63			-	*5903
	47																64		B67	*670101
	47																64		B67	*670102
	w																64		-	*6702
	47															62	64		B73	*7301
							53					58			61		64		B78	*7801, 780202, 7803
							53								61		64		B78	*780201, 7804
							53			56							64		-	*7805
							53			56							64		-	*7806
	47			50	51												64		B81	*8101
	47				51												64		-	*8102
	47				51												64		-	*8103, 8104N
	47			50									59				64		-	*8201, 8202
	47	48		50							57						64		-	*8301
																			ser. ⁵	HLA-B allele ⁴
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.
																				Cw*0206
																				Cw*0305, 0325, 0327
																				Cw*0746
												58								Cw*150204
46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64		Well No.