

[illegible]

Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ⁴	ser ⁵																							
*01:01:01-01:02:05, 01:04-01:34	DR1, Null, –	65																						
*01:03	DR103		66																					
*03:01:01:01-03:01:09, 03:04-03:06, 03:09, 03:11:01-03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:58	DR17, DR3, –					69	70											81						
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53	DR18, DR3, –					69		71										81						
*03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41, 03:49	DR3, –					69												81						
*03:08	DR3					69	70				75		77											
*03:10	DR3					69	70												83	84				
*03:42	–					69																		
*03:46	–					69	70																	
*04:01:01-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:92	DR4, DR3, Null, –								72															
*04:62, 04:69, 04:73	DR4, –								72									82						
*07:01:01-01:07:01:03, 07:03-07:19	DR7, Null, –									73														
*08:01:01-08:02:03, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24, 08:26, 08:28, 08:39	DR8, –										74						80							
*08:03:02, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:02, 08:33-08:34, 08:36-08:38	DR8, –										74													
*08:08	DR8										74						80		82					
*08:09, 14:15	DR8										74						80			84				
*08:20, 13:18, 13:47, 13:55	DR13, –															79	80	81		84				
*08:21	DR8										74					79	80			84				
*08:31, 08:41, 11:67	DR8, DR11, –										74			77			80							
*08:32	–										74				78					84				
*08:35	–										74									84				
*08:40	–						70				74													
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87

*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04- 09:09	DR9, –										75						83							
*09:01:06, 09:03	DR9										75													
*10:01:01-10:03	DR10, –										76													
*11:01:01-11:01:12, 11:04:01-11:04:06, 11:06:01- 11:06:02, 11:09-11:10:02, 11:12:01-11:12:02, 11:15, 11:24, 11:27:01-11:30, 11:32- 11:33, 11:35, 11:37-11:39, 11:43-11:44, 11:46-11:47, 11:49-11:51, 11:54:01- 11:54:02, 11:56, 11:58:01- 11:58:02, 11:60-11:62, 11:66, 11:72, 11:74-11:75, 11:77-11:78, 11:81, 11:84, 11:88, 11:90-11:92, 11:94- 11:95, 11:97	DR11, –												77		79	80								
*11:02:01-11:03, 11:11:01- 11:11:02, 11:14:01- 11:14:02, 11:16, 11:20- 11:21, 11:36, 11:40-11:41, 11:48, 11:59, 11:63, 11:65:01-11:65:02, 11:68, 11:70, 11:76, 11:80, 11:83, 11:85-11:87, 11:93	DR11, DR13, –						70						77		79	80								
*11:05	DR11												77		80									
*11:07, 11:53	DR11					69					75		77											
*11:08:01-11:08:02, 11:18- 11:19:02, 11:42, 11:57	DR11												77		79									
*11:13:01-11:13:02	DR11						w						77				83	84						
*11:17, 11:52	DR14, DR11												77				83	84						
*11:22, 11:98	–												77											
*11:23, 11:25, 11:96	DR11, –												77		79	80		84						
*11:26, 11:34	DR11						71						77											
*11:31, 11:45, 11:64	DR11, –												77		79			84						
*11:55	DR11												77			80		84						
*11:69, 11:82	–												77			80	82							
*11:73, 11:79	–						70						77		79									
*11:89	–												77					84						
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*12:01:01-12:01:03, 12:03:02, 12:05-12:12, 12:14, 12:17, 12:24N-12:25	DR12, –, Null													78										

[illegible]

*14:01:01-14:01:02, 14:04, 14:07:01-14:07:02, 14:10, 14:26, 14:28, 14:31, 14:35, 14:38-14:39, 14:54-14:55, 14:57, 14:60-14:62, 14:70-14:71, 14:75-14:76, 14:79, 14:86-14:88, 14:90, 14:99	DR14, DR 1404, DR4, DR6, –																	82	83	84				
*14:01:03, 14:08, 14:23:02, 14:34, 14:72, 14:92N, 14:97	DR14, Null, –																		83	84				
*14:02, 14:06:01-14:06:02, 14:09, 14:13, 14:17, 14:20, 14:29-14:30, 14:33, 14:41, 14:47-14:48, 14:51, 14:80, 14:83, 14:94	DR14, DR6, –							71										81		83				
*14:03:01-14:03:02, 14:12:01-14:12:02, 14:40, 14:63, 14:67, 14:77-14:78, 14:85	DR 1403, DR14, DR6, –							71						79				81			84			
*14:05:01-14:05:03, 14:14, 14:23:01, 14:23:03, 14:36, 14:43-14:45, 14:56, 14:59, 14:64, 14:91, 14:96, 14:100	DR14, –																	81		83	84			
*14:11	DR14									74										83	84			
*14:16	DR6						70							79	80				82	83	84			
*14:18, 14:81	DR14, –							71										81		83	84			
*14:19, 14:21	DR14						70	71						79				81		w				
*14:22	DR14													79	80				82	83	84			
*14:24	DR14							71							80			81						
*14:25, 14:53	DR6, DR14													79	80				82		84			
*14:27	DR14							71						79	80			81			84			
*14:32:01-14:32:02	DR14							w											82	83	84			
*14:37	DR14														80			81	82					
*14:42	–																	81			84			
*14:46, 14:52	DR14																			83				
*14:49	DR14							71											82	83	84			
*14:50	DR14									73									82	83	84			
*14:58	DR14																		82		84			
*14:65	DR6							w										81		83	84			
*14:68, 14:93	DR14, –									74									82	83	84			
*14:69	–													79					82		84			
*14:73	–														80				82	83	84			
*14:74	–													79					82	83	84			
*14:82	–							70											82	83	84			
*14:89	–							71										81			84			
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*14:95	–						70											81		83	84			

*14:98	–						71							79		81								
*15:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:49, 15:51	DR15, DR2, Null, –			67																				
*15:21	DR2			67														w						
*15:25	–			67		69																		
*15:27, 15:34	–			67														83						
*15:50N	Null			w																				
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17	DR16, Null, –				68																			
*16:04	DR16				68														w					
Well No.		65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ⁴	ser ⁵																							
DRB3*01:01:02:01-01:14, DRB3*02:01:02:25, DRB3*03:01:01-03:03	DR52, –																					85		
DRB4*01:01:01-01:03:01:01, DRB4*01:03:02-01:08	DR53, –																						86	
DRB4*01:03:01:02N	Null																	82				86		
DRB5*01:01:01-01:14, DRB5*02:02-02:05	DR51, Null, –																						87	
Well No.	ser ⁵	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87