

[illegible]

*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103	A2	2								12								19							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21			
HLA-A allele ⁴	ser ⁵																								
*02:35:02	A2									12								19							
*02:46, 02:70	A2	2																19							
*02:48, 02:129	A2, -																	19							
*02:55	A2/A28	2				7																20	21		
*02:65, 02:407	A2/A19, -	2																18							
*02:78	-	w				4																19			
*02:135	-	2												12											
*02:149	-	2																				20			
*02:169	-	2				4																			
*02:237	-	2												13				15							
*02:243	-	2																17				20			
*02:309	-	2												11				13				20			
*02:338	-	2				3																			
*03:01:01:01-03:01:18, 03:01:20-03:01:22, 03:01:24-03:07, 03:09-03:11N, 03:13-03:17:02, 03:19-03:23:02, 03:25-03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44-03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:166	A3, A3/blank, Null, -	3												12											
*03:01:19	-	3												11				12							
*03:01:23, 03:08, 03:32, 03:36N, 03:57, 03:59, 03:72, 03:89, 03:107-03:108, 03:111, 03:142, 03:152, 24:92 ⁷	A3, Null, -	3																							
*03:12	A3	3				4																			
*03:18, 03:135	-	1	4												12										
*03:24, 03:50	A3, -	3				7								12											
*03:41	-	3																				21			
*03:43, 03:82	-	3												12				14				16			
*03:63	-	3												10				12				21			
*03:75	-													12								21			
*03:88, 11:130 ⁸	-	3				4								10								21			
*03:95	-													12				13				15			
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21			

[illegible]

*32:01:14, 32:03, 32:38, 74:07 ¹³	A32, A74, -															16						
*32:04	A32/A3			3				8														
*32:05	A32													15	16							
*32:06	A32							8							16				20			
*32:15	A32						7	8							16	17						
*32:42	-							8					13		16							
*33:01:01-33:01:07, 33:03:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:49-33:50, 33:52, 33:54-33:68	A33, -																17					
*33:13, 33:48	-								9								17					
*33:19	-					6											17					
*33:22	-																17		20			
*33:24	-																17			21		
*33:51	-						7										17					
*34:01:01-34:01:02, 34:05-34:06, 34:11	A34, A10, -						7			10	11											
*34:02:01-34:03, 34:07, 34:10N	A34, Null, -			3			7			10	11											
*34:04	A34			3			7			10	11		13									
*34:08	A34			3			7			10	11	12										
*34:09	-			3			7				11											
*36:02	A36	1		3																21		
*43:01	A43								9		11	12										
*66:01, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:19	A66, A26, -						7			10		12										
*66:02-66:03, 66:16	A66, A10, -						7				11											
*66:06	A66						7			10		12							20			
*66:09	-						7			10		12	13						20			
*66:10	-						7			10		w										
*66:12, 66:15	-						7					12										
*68:01:01:01-68:04, 68:06-68:12, 68:13:02-68:14, 68:16-68:19, 68:21:01-68:24, 68:26-68:28, 68:30-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:100	A68, A28, Null, -						7											19				
*68:05, 68:15, 68:20	A68						7											19		21		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-A allele ⁴	ser ⁵																					
*68:13:01, 68:66	A68, -				4			7											19			
*68:25	A68							7											18	19		
*68:29	A68							7										17		19	20	
*68:45	-						6	7											19			

[illegible]