

[illegible]

*14:25, 14:53, 14:128	DR6, DR13, 14, -															79	80		82		84			
*14:27	DR14						71									79	80	81			84			
*14:32:01-14:32:02	DR14						w												82	83	84			
*14:37	DR14																80	81	82					
*14:42, 14:136	-																		81			84		
*14:46, 14:52	DR14																				83			
*14:49, 14:119	DR14, -						71													82	83	84		
*14:50	DR14									73										82	83	84		
*14:58	DR14																			82		84		
*14:65	DR6						w												81		83	84		
*14:68, 14:93	DR14, -									74										82	83	84		
*14:69	-															79				82		84		
*14:73	-																80			82	83	84		
*14:74	-															79				82	83	84		
*14:82	-						70													82	83	84		
*14:89	-							71											81			84		
*14:95	-						70												81		83	84		
*14:98	-							71								79		81						
*14:132	-						70														83	84		
*14:137N	Null						70									79				82	83	84		
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:65, 15:67-15:94	DR15, DR2, Null, -			67																				
*15:21	DR2			67																		w		
*15:25	-			67		69																		
*15:27, 15:34, 15:66	-			67																	83			
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:23	DR16, Null, -				68																			
*16:04, 16:18	DR16, -				68																	w		
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ⁴	ser ⁹																							
DRB3*01:01:02:01-01:15, DRB3*02:01-02:26, DRB3*02:28-02:29N, DRB3*03:01:01-03:03	DR52, Null, -																						85	
DRB3*02:27	-															79						85		
DRB4*01:01:01:01-01:03:01:01, DRB4*01:03:01:03-01:08	DR53, -																						86	
DRB4*01:03:01:02N	Null																		82			86		
DRB5*01:01:01-01:14, DRB5*02:02-02:06	DR51, -, Null, -																							87
Well No.	ser ⁹	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87