

Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ^{1,2}	ser ³																							
*01:01:01-01:02:12, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:72	DR1, Null, -	65																						
*01:03, 01:39N	DR103/DR1, Null		66																					
*01:42, 01:61	-	65	66																					
*03:01:01-03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25, 03:04:01-03:06, 03:09, 03:11:01-03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:113-03:114, 03:116-03:118, 03:121-03:124	DR17, DR3, Null, -					69	70											81						
*03:01:06, 03:01:09, 03:46, 03:64, 03:92	DR17, -					69	70																	
*03:01:24, 03:07, 03:21, 03:24, 03:32, 03:39-03:41:02, 03:49, 03:94, 03:101, 03:105, 03:120	DR3, -					69												81						
*03:02:01-03:03, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119	DR18, DR3, -					69		71										81						
*03:08, 03:65	DR3, -					69	70				75		77											
*03:10	DR3					69	70													83	84			
*03:17, 03:35, 03:97	DR3																	81						
*03:27, 13:44, 13:86, 13:206 ⁴	DR3, -							71										81						
*03:42, 03:69, 03:87, 03:111	-					69																		
*03:76	-					69	70					76				79		81						
*03:112	-					69	70					75						81						
*03:125	-					69	70					76				79	80	81						
*04:01:01-04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:202	DR4, DR4/DR3, Null, -							72																
*04:62, 04:69, 04:73, 04:105:01-04:105:02, 04:122, 04:146	DR4, -							72											82					
*07:01:01-07:01:18, 07:03-07:66	DR7, Null, -									73														
*08:01:01-08:01:05, 08:02:01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77	DR8, -									74							80							
*08:01:06, 08:03:02-08:03:05, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:69, 08:71, 08:74, 08:76	DR8, Null, -									74														
*08:08	DR8									74							80		82					
*08:09	DR8									74							80				84			
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164 ⁴	DR13, -															79	80	81			84			
*08:21	DR8									74						79	80				84			
*08:31, 08:41, 08:75, 11:67, 11:193 ⁴	DR8/DR11, -									74			77				80							
*08:32, 08:53	-									74					78						84			
*08:35, 08:36:02	-									74											84			
*08:40	-					70				74														
*08:57	-																80							
*08:63	-									74				77										
*08:67	-								73								80							
*08:68	-									74										w				
*08:70	-									74							80			w				
*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04-09:22, 09:24-09:27	DR9, -										75								83					
*09:01:06, 09:03, 09:23	DR9, -										75													
*10:01:01-10:16	DR10, -											76												
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*11:01:01-11:01:17, 11:01:20-11:01:27, 11:04:01-11:04:12, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:03, 11:15, 11:24, 11:27:01, 11:28:01-11:29:02, 11:32-11:33, 11:35, 11:37:01-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:51, 11:54:01-11:54:02, 11:56, 11:58:01-11:58:02, 11:60-11:62:02, 11:66, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01, 11:88, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144-11:147:02, 11:150, 11:152, 11:154-11:158, 11:160, 11:162-11:167, 11:169N-11:170, 11:172, 11:174-11:175, 11:177-11:181, 11:183, 11:185-11:190, 11:194	DR11, Null, -													77		79	80							
*11:01:18-11:01:19, 11:08:01-11:08:03, 11:18-11:19:03, 11:42, 11:57, 11:130, 11:191	DR11, -													77		79								

[illegible]

*13:193	-					70	71									81	83							
*13:197	-														79	80	82	84						
*14:01:01, 14:04:01, 14:07:01, 14:10, 14:54:01, 14:54:03, 14:54:05, 14:57, 14:137N, 14:152N	DR14, DR1404/DR14, DR4/DR6, Null, -																81	82	83	84				
*14:01:02, 14:01:04, 14:04:02-14:04:03, 14:07:02, 14:26, 14:28, 14:31, 14:32:03, 14:35, 14:38:01, 14:39, 14:55, 14:60-14:62, 14:70-14:71, 14:75-14:76, 14:79, 14:86-14:88, 14:90, 14:99, 14:101, 14:104, 14:107, 14:110-14:114, 14:117-14:118, 14:124-14:125, 14:129, 14:131, 14:138-14:140, 14:142-14:143, 14:145, 14:147, 14:149-14:151, 14:153, 14:157-14:158, 14:163-14:164, 14:166N	DR14, Null, -																?	82	83	84				
*14:01:03, 14:23:02, 14:34, 14:72, 14:92N, 14:97, 14:126:01-14:126:02, 14:130, 14:133, 14:162	DR14, Null, -																?		83	84				
*14:02:01-14:02:02, 14:06:01-14:06:03, 14:09, 14:13, 14:17, 14:20, 14:29-14:30, 14:33, 14:41, 14:47-14:48, 14:51, 14:80, 14:83, 14:94, 14:106, 14:108, 14:121, 14:159, 14:165	DR14, DR6, -						71										81		83					
*14:03:01-14:03:02, 14:12:01-14:12:02, 14:40, 14:63, 14:67, 14:77-14:78, 14:85, 14:102, 14:115, 14:135, 14:156	DR1403/DR14,DR6, -						71								79		81			84				
*14:05:01-14:05:04, 14:08, 14:14, 14:23:01, 14:23:03-14:23:04, 14:36, 14:43-14:44:03, 14:54:02, 14:56, 14:59, 14:64, 14:91, 14:96, 14:100, 14:103, 14:123, 14:127:01-14:127:02, 14:134, 14:155, 14:167	DR14, -																81		83	84				
*14:11	DR14										74						81		83	84				
*14:15	DR8										74					80	?		84					
*14:16	DR6						70									79	80	?	82	83	84			
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*14:18, 14:81, 14:154, 14:160	DR14, -							71										81		83	84			
*14:19, 14:21, 14:109	DR14, -						70	71								79		81		w				
*14:22, 14:105	DR14, -															79	80	?	82	83	84			
*14:24	DR14							71									80	81						
*14:25, 14:53, 14:128	DR6, DR13/14, -															79	80	?	82		84			
*14:27	DR14							71								79	80	81			84			
*14:32:01-14:32:02	DR14							w										?	82	83	84			
*14:37	DR14																80	81	82					
*14:38:02	-				69													?	82	83	84			
*14:42, 14:136, 14:161	-																	81			84			
*14:45, 14:95	DR14, -						70											81		83	84			
*14:46	DR14																			83				
*14:49, 14:119, 14:146	DR14, -							71										?	82	83	84			
*14:50	DR14									73								?	82	83	84			
*14:52	DR14														78			?		83				
*14:54:04	-																	81	82	w	84			
*14:58	DR14																	?	82		84			
*14:65	DR6						w											81		83	84			
*14:68:01-14:68:02, 14:93	DR14, -									74								?	82	83	84			
*14:69	-															79		?	82		84			
*14:73	-																80	?	82	83	84			
*14:74	-															79		?	82	83	84			
*14:82	-					70												81	82	83	84			
*14:89	-						71											81			84			
*14:98	-						71									79		81						
*14:120, 14:122	-					70												?	82	83	84			
*14:132	-					70												?		83	84			
*14:141	-																	81				85		
*14:148	-									74								?		83	84			
*14:168	-																	?	w		84			
*15:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:65, 15:67-15:99, 15:101-15:103, 15:105-15:121, 15:123-15:127	DR15, DR2, Null, -			67																				
*15:21	DR2			67																	w			
*15:25, 15:37:01, 15:100, 15:104	-			67	69																			
*15:27, 15:34, 15:66:01-15:66:02	-			67																83				
*15:122	-			67																w				
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:37	DR16, Null, -			68																				
*16:04, 16:18	DR16, -			68																	w			
DRB3*01:01:02:01-01:17, DRB3*02:01:02:26, DRB3*02:28-02:29N, DRB3*03:01:01-03:03	DR52, Null, -																					85		
DRB3*02:27	-															79						85		
DRB4*01:01:01:01-01:03:01:01, DRB4*01:03:01:03-01:10	DR53, -																						86	
DRB4*01:03:01:02N	Null																		82				86	

DRB5*01:01:01-01:16, DRB5*02:02-02:06	DR51, Null, -																		87					
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87