

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-A allele ^{1,2}	ser ³																					
*01:01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04N, 01:06-01:07, 01:09-01:01:11N, 01:15N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52-01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 36:04 ⁴	A1, A36, Null, -	1			4																	
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 36:01, 36:03, 36:05 ⁴	A1, A36, Null, -	1																				
*01:01:56	-	1			4						11											
*01:12, 01:19, 01:200	A1, -	1		3																		
*01:13, 01:176	A1, -	1			4					10	12			14								
*01:21, 01:126	A1, -	1		3	4																	
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211, 11:226 ⁵	A1, -	1			4					10												
*01:28, 01:229	-	1			4					10				14								
*01:43	-	1			4					9												
*01:51	-	1			4			7		9												
*01:95	-	1			4												16					
*01:121	-	1			4														18			
*01:194	-	1			4								12		14							
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79-01:02:80, 02:82N-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528-01:02:570, 02:572-02:579, 02:581, 02:583-02:643N, 02:645-02:650, 02:652-02:661, 02:663-02:670, 02:672Q-02:683	A2, Low A2, A203/A2, A210/A2, A2/blank, Null, -	2																				
*02:17:01-02:17:04	A2, -	2				w																
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -	2										12							19			
*02:35:02	A2											12							19			
*02:46, 02:70, 02:651	A2, -	2																	19			
*02:48, 02:129, 02:571	A2, -																		19			
*02:55	A2/A28	2				7														20	21	
*02:65, 02:407, 02:449	A2/A19, -	2																	18			
*02:78	-	w		4															19			
*02:81, 02:124	A2	2						8														
*02:135	-	2										12										
*02:149	-	2																		20		
*02:169	-	2		4																		
*02:237	-	2												13		15						21
*02:243:01-02:243:02	-	2																17		20		
*02:309	-	2									11		13							20		
*02:338	-	2	3																			
*02:454	-	2									11		13									
*02:527, 02:582	-	2				7																
*02:644	-					7			9											20		
*02:662	-	2								10												
*02:671	-	2																				21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*03:01:01:01:03:01:18, 03:01:20-03:01:22, 03:01:24-03:01:37, 03:01:39-03:07:02, 03:09-03:11N, 03:13-03:17:02, 03:19-03:23:02, 03:25-03:31, 03:33-03:35, 03:37-03:40, 03:42, 03:44-01:03:49, 03:51-03:56, 03:58, 03:60-03:62, 03:64-03:71, 03:73-03:74, 03:76-03:81, 03:83-03:87, 03:90-03:94, 03:96-03:106, 03:109-03:110, 03:112-03:123:01, 03:124-03:134, 03:136-03:141, 03:143-03:151, 03:153-03:170, 03:174-03:175, 03:179-03:185, 03:188-03:193, 03:195-03:197N, 03:201-03:202, 03:204, 03:206-03:210, 03:212-03:214, 03:216-03:218, 03:220-03:224, 03:226-03:230, 03:231-02:03:245, 03:247-03:248, 03:250-03:251, 03:253-03:259, 03:261-03:266N, 03:268-03:272, 03:274-03:278	A3, A3/blank, Null, -			3									12									
*03:01:19	-			3								11	12									
*03:01:23, 03:08, 03:36N, 03:57, 03:59, 03:72, 03:89:01-03:89:02, 03:107-03:108, 03:111, 03:142, 03:172-03:173, 03:176, 03:178N, 03:198-03:200Q, 03:203, 03:205, 03:211, 03:252, 03:267, 03:273, 24:92 ⁵	A3, Null, -			3																		
*03:01:38, 03:123:02, 03:171, 03:231:01	-			3									12	13								
*03:12	A3			3	4																	
*03:18, 03:135	-	1			4							12										
*03:24, 03:50	A3, -			3			7					12										
*03:32, 03:152, 03:219	-			3													16					
*03:41	-			3																		21
*03:43, 03:82, 03:186	-			3								12		14		16						
*03:63	-			3							10	12										21
*03:75, 03:177	-											12										21
*03:88	-			3	4					10												21
*03:95	-											12	13		15							21
*03:187, 03:215, 03:225, 03:249	-											12										
*03:246	-			3								12							18			
*03:260	-	1		3											14							

*31:35	-														14	15	16						
*31:41	-														15						20		
*32:01:01:01-32:02, 32:07-32:14, 32:16-32:25, 32:27N-32:37, 32:39-32:41, 32:43-32:59, 32:61, 32:63, 32:65-32:70, 32:73-32:77, 32:81-32:92N, 32:94-32:101Q	A32, Null, -							8									16						
*32:03, 32:38, 32:60, 32:71, 32:78, 32:80	A32, -																16						
*32:04	A32/A3			3				8									16						
*32:05, 32:79	A32, -															15	16	17					
*32:06	A32							8									16				20		
*32:15	A32						7	8									16	17					
*32:26:01-32:26:02	-							8				12					16						
*32:42	-							8					13				16						
*32:62	-							8									16		18				
*32:64	-				4			8									16						
*32:72	-					5		8									16						
*32:93	-							8									16	17					
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	
*33:01:01:01-33:01:10, 33:03:01-33:12, 33:14-33:18:02, 33:20-33:21, 33:23, 33:25-33:37, 33:39-33:47, 33:49-33:50, 33:52, 33:54-33:60, 33:62-33:118, 33:120-33:124, 33:126-33:128	A33, Null, -																	17					
*33:13, 33:48	-								9									17					
*33:19, 33:119	-					6												17					
*33:22	-																	17			20		
*33:24	-																	17				21	
*33:51	-						7											17					
*33:61	-											12						17					
*34:01:01-34:01:02, 34:05-34:06, 34:11-34:12, 34:14, 34:16-34:17	A34, A10, -						7			10	11												
*34:02:01-34:02:03, 34:03, 34:07, 34:10N, 34:13, 34:15	A34, Null, -			3			7			10	11												
*34:02:04	-			3			7			10	11									19			
*34:04	A34			3			7			10	11		13										
*34:08	A34			3			7			10	11	12											
*34:09	-			3			7					11											
*36:02	A36	1		3										13								21	
*43:01	A43								9		11	12											
*66:01:01:01-66:01:03, 66:04-66:05, 66:07-66:08, 66:11, 66:13-66:14, 66:17-66:18, 66:20, 66:22, 66:24, 66:27N	A66, A26, Null, -						7			10	12												
*66:02-66:03, 66:16, 66:21, 66:25-66:26Q	A66, A10, -						7				11												
*66:06	A66						7			10	12										20		
*66:09	-						7			10	12	13									20		
*66:10	-						7			10	w												
*66:19	-						7			10	11	12											
*68:01:01:01-68:04:02, 68:06-68:08:01, 68:09-68:12, 68:13:02-68:14, 68:16-68:18N, 68:21:01-68:24, 68:26-68:28, 68:30-68:44, 68:46-68:65, 68:67-68:70, 68:72-68:83, 68:85-68:104:02, 68:106-68:108, 68:110-68:116, 68:118-68:129, 68:131-68:133, 68:135, 68:137-68:156, 68:158-68:166	A68, A28, Null, -						7													19			
*68:05, 68:15, 68:20, 68:109, 68:136	A68, -						7													19		21	
*68:08:02, 68:105, 68:157	A68, -						7													19	20		
*68:13:01, 68:66:01-68:66:02, 68:134	A68, -				4		7													19			
*68:19, 68:130:02	A68, -						7					13								19			
*68:25	A68						7													18	19		
*68:29	A68						7											17		19	20		
*68:45, 68:117	-					6	7													19			
*68:71	-						7					12								19			
*68:84	-						7		9											19			
*68:130:01	-						7				11									19			
*69:01:01:01-69:01:02, 69:03	A69, -						7														20		
*69:02	-						7			10											20		
*74:01:01-74:01:02, 74:01:04-74:03, 74:05-74:06, 74:08-74:12N, 74:14N-74:18, 74:20, 74:22, 74:24-74:28	A74, Null, -																			18			
*74:01:03	-										11									18			
*74:04, 74:21	A74, -																			17	18		
*74:07	A74																16			18			
*74:13	-											12			15					18			
*74:19	-				4															18			
*74:23	-			3																18			
*80:01:01:01-80:01:01:02, 80:03	A80, -					5																21	
*80:02	-					5				10													
B*07:81, B*08:52, B*18:67, B*38:41, 38:67, B*51:185, B*53:05:01-53:05:02, B*53:16, B*53:33, B*57:60	B53, -							8															
C*06:187	-																					21	
C*12:131	-				4																		
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	