

Well No. DRB1 allele ^{1,2}	DR ser ³	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*01:01:01-01:02:12, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:81	DR1, Null, -	65																						
*01:03:01-01:03:02, 01:39N	DR103/DR1, Null, -		66																					
*01:42, 01:61	-	65	66																					
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:26, 03:04:01-03:06, 03:09, 03:11:01-03:15:01, 03:16, 03:18-03:20, 03:22-03:23, 03:25-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139	DR17, DR3, Null, -					69	70											81						
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:130	DR17, -					69	70																	
*03:01:24, 03:07, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120	DR3, -					69													81					
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131	DR18, DR3, -					69		71											81					
*03:08, 03:65, 03:140	DR3, -					69	70				75		77											
*03:10	DR3					69	70													83	84			
*03:15:02, 13:02:02 ⁴	DR13, -						70												81					
*03:42, 03:69, 03:87, 03:111	-					69																		
*03:76	-					69	70					76			79				81					
*03:125	-					69	70					76			79	80			81					
*03:126, 14:179 ⁴	-						70	71				76							81		83			
*04:01:01-01:04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:230	DR4, DR4/DR3, Null, -								72															
*04:62, 04:69, 04:73, 04:105:01-04:105:02, 04:122, 04:146	DR4, -								72											82				
*04:211	-								72											83				
*07:01:01-01:07:01:21, 07:03-07:77	DR7, Null, -									73														
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:02:01-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24:01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82	DR8, Null, -										74						80							
*08:01:06, 08:03:02-08:03:06, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:																								

