

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-A allele ^{1,2,6}	ser ³																					
*01:01:01:01:01:01:22, 01:01:24-01:01:47, 01:01:49-01:01:55, 01:01:57-01:01:64, 01:01:67-01:01:78, 01:01:80-01:04:01:02N, 01:06, 01:09:01:01-01:11N, 01:15N-01:18N, 01:20, 01:22N-01:25, 01:27N, 01:29, 01:31N-01:33, 01:35-01:42, 01:44-01:50, 01:52:01N-01:78, 01:80-01:94, 01:96-01:98, 01:100-01:120, 01:122-01:125, 01:127-01:135, 01:137-01:144, 01:146, 01:148, 01:150-01:158, 01:160N-01:166, 01:168-01:175, 01:177, 01:179N-01:191, 01:193, 01:195-01:199, 01:201-01:204, 01:206-01:207, 01:209-01:213, 01:215-01:227, 01:230-01:235, 01:237-01:243, 01:245-01:261, 36:04 ⁴	A1, A36, Null, -	1			4																	
*01:01:23, 01:01:48, 01:01:65-01:01:66, 01:01:79, 01:08, 01:14, 01:30, 01:79, 01:99, 01:145, 01:147Q, 01:149, 01:159, 01:167, 01:178N, 01:205, 01:208Q, 01:214, 01:228Q, 01:236, 36:01, 36:03, 36:05-36:06 ⁴	A1, A36, Null, -	1																				
*01:01:56	-	1			4						11											
*01:07, 01:95	A1, -	1			4												16					
*01:12, 01:19, 01:200, 01:244	A1, -	1		3																		
*01:13, 01:176	A1, -	1			4						10		12		14							
*01:21, 01:126	A1, -	1		3	4																	
*01:26, 01:136, 01:192, 11:94, 11:112, 11:211, 11:226 ⁵	A1, -	1			4						10											
*01:28, 01:229, 11:282 ⁵	-	1			4						10				14							
*01:43, 11:271 ⁵	-	1			4					9												
*01:51	-	1			4			7		9												
*01:121	-	1			4														18			
*01:194	-	1			4								12		14							
*02:01:01:01:02:01:15, 02:01:17-02:01:19, 02:01:21-02:01:81, 02:01:83-02:16, 02:18-02:22:02, 02:24:01-02:33, 02:36-02:45, 02:47, 02:49-02:54, 02:57-02:61, 02:63-02:64:02, 02:66-02:69, 02:71-02:77, 02:79:01-02:80, 02:82N-02:97:02, 02:99, 02:101:01-02:102, 02:104-02:123, 02:125N-02:128, 02:130-02:134, 02:136-02:148, 02:150-02:168, 02:170-02:236, 02:238-02:242, 02:244-02:308, 02:310-02:337, 02:339-02:406, 02:408-02:448, 02:450-02:453, 02:455-02:526, 02:528:01-02:570, 02:572-02:579, 02:581, 02:583-02:643N, 02:645-02:650, 02:652-02:670, 02:672Q-02:740	A2, Low A203, A210, Null, -	2																				
*02:17:01-02:17:04	A2, -	2				w																
*02:34-02:35:01, 02:35:03, 02:56:01-02:56:02, 02:62, 02:103, 02:580	A2, -	2											12							19		
*02:35:02	A2												12							19		
*02:46, 02:70, 02:651	A2, -	2																		19		
*02:48, 02:129, 02:571	A2, -																			19		
*02:55	A2, A28	2						7													20	21
*02:65, 02:407, 02:449	A2, A19, -	2																	18			
*02:78	w		4																	19		
*02:81, 02:124	A2	2						8														
*02:135	-	2											12									
*02:149	-	2																			20	
*02:169	-	2			4																	
*02:237	-	2												13		15						21
*02:243:01-02:243:03	-	2																17			20	
*02:309	-	2										11		13							20	
*02:338	-	2		3																		
*02:454	-	2										11		13								
*02:527, 02:582	-	2						7														
*02:644	-							7		9											20	
*02:671	-	2																				21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21

[illegible]