

Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
DRB1 allele ^{1,2}	ser ³																							
*01:01:01-01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:88	DR1, Null, -	65																						
*01:03:01-01:03:02, 01:39N	DR1, DR103, Null, -		66																					
*01:42, 01:61, 01:89	-	65	66																					
*03:01:01-01:03:01:05, 03:01:07-03:01:08, 03:01:10-03:01:23, 03:01:25-03:01:28, 03:04:01-03:06, 03:09, 03:11:01-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:26, 03:28, 03:30-03:31, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:63, 03:66-03:68N, 03:70-03:73, 03:75, 03:77-03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121-03:124, 03:127-03:129, 03:132-03:139, 03:141-03:146	DR3, DR17, Null, -					69	70											81						
*03:01:06, 03:01:09, 03:46, 03:64, 03:92, 03:130	DR17, -					69	70																	
*03:01:24, 03:07:01-03:07:02, 03:17, 03:21, 03:24, 03:32, 03:35, 03:39-03:41:02, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120	DR3, -					69												81						
*03:02:01-03:03, 03:27, 03:29, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:119, 03:131	DR3, DR18, -					69		71										81						
*03:08, 03:65, 03:140	DR3, -					69	70				75			77										
*03:10	DR3					69	70												83	84				
*03:42, 03:69, 03:87, 03:111	-					69																		
*03:76	-					69	70					76			79			81						
*03:125	-					69	70					76			79	80		81						
*03:126, 14:179 ⁴	-						70	71				76						81	83					
*04:01:01-01:04:05:11, 04:05:13-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:250, 04:252	DR4, DR3, Null, -							72																
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:251	DR4, -							72											82					
*04:211	-							72											83					
*07:01:01-01:07:01:22, 07:03-07:05, 07:07-07:90	DR7, -,									73														
*07:06	DR7									73	74													
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:04, 08:04:01-08:07, 08:11, 08:16-08:17, 08:22, 08:24:01-08:24:02, 08:26, 08:28, 08:39, 08:42-08:44, 08:50, 08:52, 08:54-08:55, 08:59, 08:64, 08:72-08:73, 08:77-08:79, 08:82-08:86	DR8, Null, -									74							80							
*08:01:06, 08:03:02-01-08:03:08, 08:10, 08:12-08:15, 08:18-08:19, 08:23, 08:25, 08:27, 08:29-08:30:03, 08:33-08:34, 08:36:01, 08:37-08:38, 08:45:01-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81	DR8, Null, -									74														
*08:08	DR8									74							80		82					
*08:09	DR8									74							80				84			
*08:20, 13:18, 13:47, 13:55, 13:158, 13:164, 13:232 ⁴	DR13, -															79	80	81			84			
*08:21	DR8									74							79	80			84			
*08:31, 11:05, 11:209 ⁴	DR8, DR11, -													77			80							
*08:32, 08:53	-									74											84			
*08:35, 08:36:02	-									74											84			
*08:40	-					70				74														
*08:41, 08:75, 11:67, 11:193:01-11:193:02 ⁴	-									74				77			80							
*08:57, 08:80, 13:248 ⁴	-																80							
*08:63	-									74				77										
*08:67	-									73							80							
*08:68:01	-									74										w				
*08:70	-									74							80			w				
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*09:01:02-01-09:01:05, 09:01:07-09:02:02, 09:04, 09:06-09:22, 09:24-09:28, 09:30-09:34	DR9, -										75								83					
*09:01:06, 09:03, 09:23, 09:29	DR9, -										75													
*09:05	DR9										75	76							83					
*10:01:01-01-10:10, 10:12-10:27	DR10, -											76												
*10:11	-									73		76												

[illegible]

[illegible]

*14:54:04	-																	81	82	w	84			
*14:58, 14:186	DR14, -																	?	82		84			
*14:65	DR6						w											81		83	84			
*14:68:01-14:68:02, 14:93	DR14, -								74									?	82	83	84			
*14:69	-															79		?	82		84			
*14:73	-																80	?	82	83	84			
*14:74, 14:173	-															79		?	82	83	84			
*14:82	-					70					76							81	82	83	84			
*14:89	-						71											81			84			
*14:95	-					70					76							81		83	84			
*14:120, 14:122, 14:178	-					70												?	82	83	84			
*14:132	-					70					76							?		83	84			
*14:141	-																	81					85	
*14:168	-																	?	w	83	84			
*14:171	-						69											81	82	83	84			
*15:01:01:01-15:20, 15:22-15:24, 15:26, 15:28-15:33, 15:35-15:36, 15:37:02-15:53, 15:55-15:65, 15:67-15:99, 15:101-15:103, 15:104:02-15:121, 15:123-15:157	DR15, DR2, Null, -			67																				
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87
*15:21	DR2			67																	w			
*15:25, 15:37:01, 15:100, 15:104:01	-			67		69																		
*15:27, 15:34, 15:54, 15:66:01-15:66:02, 15:122	-			67																83				
*15:158	-			67																83	w			
*16:01:01-16:03, 16:05:01-16:05:02, 16:07-16:17, 16:19-16:48	DR16, Null, -			68																				
*16:04:01-16:04:02, 16:18	DR16, -			68																	w			
DRB3*01:01:02:01-01:41, DRB3*01:43-01:48, DRB3*02:01:02:26, DRB3*02:28-02:79, DRB3*03:01:01:03:18	DR52, Null, -																						85	
DRB3*01:42	-										75												85	
DRB3*02:27	-														79								85	
DRB4*01:01:01:01-01:03:01:01, DRB4*01:03:01:03-01:30, DRB4*01:32-01:69, DRB4*02:01N, DRB4*03:01N	DR53, Null, -																						86	
DRB4*01:03:01:02N	Null																	82					86	
DRB4*01:31	-																			84			86	
DRB5*01:01:01-01:39, DRB5*02:02:01-02:20	DR51, Null, -																							87
Well No.	DR	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	81	82	83	84	85	86	87