

DPB1 SSP typing

Well⁵[illegible]

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

Length of spec.	245	220	100	140	235	165	160	210	165	220	190	80	180	105	200	80	260	180	200	235	100	185	145	195
PCR product														175		165								265
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
DPB1 allele ⁴																								
*01:01:01-01:01:03	1	2	3	4	5	6	7																	
*02:01:02, 02:01:04-02:01:05, 02:01:07								8	9	10	11	12												
*02:01:03								8	9	10	11	w												
*02:01:06								8	w	10	11	12												
*02:02								8	9	10			13	14	15									
*03:01:01-03:01:02	1	2														16	17	18	19	20	21			
*04:01:01-04:01:02			3	4				8			11											22		
*04:02								8	9		11					16								
*05:01:01-05:01:02								8	9				13	14									23	24
*06:01									9			12					17	18	19					
*08:01					5			8		10		12												24
*09:01												12					17	18		20	21			
*10:01												12					17			20				
*11:01:01-11:01:02			3														17		19					
*13:01	1		3			6	7										17			20				
*14:01																16	17	18	19	20	21			
*15:01			3																					
*16:01								8	9	10		12												24
*17:01									9			12					17	18						
*18:01		2							9							16								
*19:01					5			8	9	10			13											24
*20:01:01-20:01:02		2							9							16	17	18	19					
*21:01									9				14				17							
*22:01								8	9	10			13	14										24
*23:01								8	9		11													
*24:01				4				8			11		13									22		
*25:01	1	2														16	17		19	20				
*26:01:01	1	2	3	4		6	7										17			20				
*26:01:02	1	2	3	4		6	7										17			20				
*27:01		2	3	4													17							
*28:01				4												16								
*29:01	1											12					17	18	19	20	21			
*30:01									9				14				17							
*31:01			3	4																				24
*32:01								8	9	10	11	12												
*33:01			3					8		10	11											22		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Length of spec.	245	220	100	140	235	165	160	210	165	220	190	80	180	105	200	80	260	180	200	235	100	185	145	195
PCR product														175		165								265
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*34:01									9														23	
*35:01:01																16	17	18		20	21			
*35:01:02																16	17	18		20	21			
*36:01		2							9					14			17						23	
*37:01	1											12					17			20				
*38:01								8	9					13	14								23	24
*39:01			3	4				8													22			
*40:01			3	4				8																
*41:01:01-41:01:02									9	10	11	12												
*44:01	1											12					17	18	19	20	21			
*45:01																16	17		19	20				
*46:01								8	9	10	11	12												
*47:01								8	9	10	11		13											
*48:01								8	9	10		12			15									
*49:01				4				8								16						22		
*50:01	1	2			5											16					21			
*51:01				4				8			11					16						22		
*52:01	1	2															17		19	20				
*53:01				4				8								16								
*54:01														14			17			20				
*55:01									9								17							
*56:01	1	2	3	4			7										17		19	20				
*57:01					5											16					21			24
*58:01									9								17							
*59:01									9		11					16								
*60:01								8	9		11					16								
*61:01N	1	2														16	17	18	19	20	21			
*62:01								8	9														23	
*63:01								8	9														23	24
*64:01N									9			12					17	18	19					
*65:01			3	4	5	6	7	8																24
*66:01			3	4							11										22			
*67:01																	17		19	20				
*68:01					5			8								16								24
*69:01									9								17	18	19					
*70:01																16	17	18	19	20	21			
*71:01								8	9	10	11													
*72:01			3	4							11											22		
*73:01					5						11					16								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Length of spec.	245	220	100	140	235	165	160	210	165	220	190	80	180	105	200	80	260	180	200	235	100	185	145	195
PCR product														175		165								265
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*74:01			3															19						
*75:01					5			8			11					16								
*76:01				4			7									16	17	18	19	20	21			
*77:01								8	9		11					16								
*78:01	1	2														16	17	18	19	20	21			
*79:01	1	2														16	17			20				
*80:01								8	9		11					16								
*81:01								8		10	11	12										22		
*82:01								8	9		11					16								
*83:01									9		11					16								
*84:01					5			8																24
*85:01		2	3	4													17							
*86:01									9		11	12						18						
*87:01		2							9								17		19					
*88:01	1											12					17	18		20	21			
*89:01		2	3	4																				
*90:01	1	2	3	4	5		7																	
*91:01									9							16	17	18	19					
*92:01	1	2		4			7									16	17	18	19	20	21			
*93:01									9			12					17							
*94:01								8	9		11													
*95:01									9	10					15									
*96:01			3	4				8														22		
*97:01								8	9				13	14									23	24
*98:01									9							16	17	18						
*99:01			3	4				8			11				15	16						22		
*100:01								8	9				13	14	15								23	
*101:01								8	9	10			13	14	15									24
*102:01			3	4																				24
*103:01								8	9	10														24
*104:01	1	2														16	17	18	19	20	21			
*105:01								8	9		11					16								
*106:01	1				5			8	9	10			13											24
*107:01	1		3			6	7										17			20				
*108:01									9		11					16								
*109:01									9		11	12												
*110:01	1		3				7										17			20				
*111:01	1	2														16	17	18	19	20	21			
*112:01								8	9						15								23	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Length of spec.	245	220	100	140	235	165	160	210	165	220	190	80	180	105	200	80	260	180	200	235	100	185	145	195
PCR product														175		165								265
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*113:01									9		11	12												
*114:01	1	2												14			17		19	20			23	
*115:01								8	9	10	11	12												
*116:01								8	9						15	16							23	
*117:01			3																			22		
*118:01	1	2	3	4	5	6	7																	
*119:01											11					16	17	18	19	20	21			
*120:01N			3	4				8			11											22		
*121:01			3	4				8			11											22		
*122:01	1	2		4	5	6	7									16								
*123:01								8	9	10	11	12												
*124:01	1	2														16	17	18	19	20	21			
*125:01			3	4			7										17			20				
DPB1 allele ⁴																								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

INTERPRETATION TABLE																												
DPB1 SSP typing																												
Amplification patterns of the DPB1*01:01 to DPB1*125:01 alleles																												
Well ⁵																												
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48					
190	75	95	200	220	185	120	75	140	210	195	260	185	100	210	95	75	140	65	110	120	70	165	115	Length of spec.				
265	205		260				195	205				255				180	230	185	190	185	190	210	255	210	PCR product			
																		285		245	245			280				
																			270									
515	515	430	430	515	430	430	430	430	515	430	515	515	515	515	430	515	430	430	430	430	515	515	515	515	Length of int. pos. control ¹			
9(112)	15(130)	57(258)	11(119)	9(112)	9(112)	57(258)	33(184)	15(130)	9(112)	33(184)	11(119)	36(194)	35(190)	9(113)	-22(24)	1 st I	9(113)	35(190)	7(108)	17(136)	9(113)	35(190)	11(119)	5'-primer(s) ²				
5'-Tgg ^{3'}	5'-AAg ^{3'}	5'-gAC ^{3'}	5'-gTT ^{3'}	5'-TgC ^{3'}	5'-TgC ^{3'}	5'-gAg ^{3'}	5'-ggC ^{3'}	5'-AAg ^{3'}	5'-TgC ^{3'}	5'-ggC ^{3'}	5'-ggg ^{3'}	5'-CgC ^{3'}	5'-AgC ^{3'}	5'-TTT ^{3'}	5'-gCA ^{3'}	5'-CAT ^{3'}	5'-TTT ^{3'}	5'-AgC ^{3'}	5'-TAA ^{3'}	5'-ACA ^{3'}	5'-TTT ^{3'}	5'-AgC ^{3'}	5'-gTT ^{3'}					
35(190)	57(258)		11(119)				33(184)	36(194)		33(184)	11(119)	36(194)	36(194)	36(194)	9(112)	9(113)		196(675)	10(115)	35(191)	56(254)	107(406)	11(119)					
5'-AgC ^{3'}	5'-gAC ^{3'}		5'-gTT ^{3'}				5'-ggC ^{3'}	5'-CgC ^{3'}		5'-ggC ^{3'}	5'-ggg ^{3'}	5'-CgC ^{3'}	5'-CgT ^{3'}		5'-TgC ^{3'}	5'-TTT ^{3'}		5'-AAG ^{3'}	5'-TCT ^{3'}	5'-gTA ^{3'}	5'-TgA ^{3'}	5'-CCT ^{3'}	5'-gTT ^{3'}					
	60(265)						73(305)	36(194)				107(406)	39(202)						17(136)	57(258)								
	5'-ggC ^{3'}						5'-gCT ^{3'}	5'-CgC ^{3'}				5'-CCT ^{3'}	5'-TCT ^{3'}						5'-ACC ^{3'}	5'-gAC ^{3'}								
																			32(182)									
																			5'-CCC ^{3'}									
5'-CgT ^{3'}	5'-CTC ^{3'}	5'-TAT ^{3'}	5'-gAT ^{3'}	5'-CTC ^{3'}	5'-TAG ^{3'}	5'-CgT ^{3'}	5'-CgT ^{3'}	5'-CTC ^{3'}	5'-gAg ^{3'}	5'-CgA ^{3'}	5'-CgA ^{3'}	5'-CgA ^{3'}	5'-CgA ^{3'}	5'-CAG ^{3'}	5'-gAg ^{3'}	5'-CAC ^{3'}	5'-gCA ^{3'}	5'-CCA ^{3'}	5'-CgA ^{3'}	5'-CAG ^{3'}	5'-CgC ^{3'}	5'-gTg ^{3'}	5'-CAT ^{3'}	5'-CgA ^{3'}	3'-primer(s) ³			
84(338)	69(292)	76(313)	65(280)	69(292)	57(258)	84(338)	84(338)	69(292)	65(280)	84(338)	84(338)	84(338)	55(251)	65(280)	-4(76)	12(122)	43(214)	84(338)	55(251)	84(338)	64(277)	76(315)	35(191)					
		76(313)	84(337)									178(619)				55(252)	70(295)	57(257)	205(700)	84(338)	65(280)	178(619)	67(286)					
		5'-CAT ^{3'}	5'-gTT ^{3'}									5'-CAT ^{3'}			5'-gCC ^{3'}	5'-CCA ^{3'}	5'-ACA ^{3'}	5'-CAT ^{3'}	5'-CgT ^{3'}		5'-gAA ^{3'}	5'-CAG ^{3'}	91(359)					
72(301)	91(359)																											
5'-CAA ^{3'}	5'-ggT ^{3'}																											

[illegible]

190	75	95	200	220	185	120	75	140	210	195	260	185	100	210	95	75	140	65	110	120	70	165	115	Length of spec.
265	205		260				195	205				255			180	230	185	190	185	190	210	255	210	PCR product
																	285		245	245			280	
																			270					
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
																								DPB1 allele ⁴
						31																		*01:01:01-01:01:03
																						47		*02:01:02, 02:01:04-02:01:05, 02:01:07
																						47		*02:01:03
																						47		*02:01:06
																						47		*02:02
																								*03:01:01-03:01:02
																						47		*04:01:01-04:01:02
												37												*04:02
25						31											43							*05:01:01-05:01:02
	26	27													?		?							*06:01
						31						?			?		?				?			*08:01
	26		28	29	30																			*09:01
			28	29		31									?		?							*10:01
						31	32								?		?							*11:01:01-11:01:02
			28			31		33																*13:01
					30			34																*14:01
									35	36	37													*15:01
						31									?		?							*16:01
	26	27	28	29	30										?		?				47			*17:01
									36						?		?							*18:01
						31											43							*19:01
		27													?		?							*20:01:01-20:01:02
25			28			31						?			?		?				?			*21:01
25						31						?			?		?				?			*22:01
												38			?		?				47			*23:01
												?			?		?				?			*24:01
						31						?			?		?				?			*25:01
			28			31									?		?				?			*26:01:01
			28			31									?		?							*26:01:02
			28			31						?			?		?				?			*27:01
									36	37		39	?				?				47			*28:01
	26											?			?		?				?			*29:01
			28	29		31									40		?				47			*30:01
						31						?		39	?	41	?				?			*31:01
												?			?		42	?			?			*32:01
								33				?			?		?				?			*33:01
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.

190	75	95	200	220	185	120	75	140	210	195	260	185	100	210	95	75	140	65	110	120	70	165	115	Length of spec.
265	205		260				195	205				255			180	230	185	190	185	190	210	255	210	PCR product
																	285		245	245			280	
																			270					
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
											36	?	38	39	?	41		43				?		*34:01
			28		30							?			?			?				?		*35:01:01
			28												?			?						*35:01:02
25			28			31						?			?			?				?		*36:01
			28			31						?			?			?				?		*37:01
25						31						?			?			?	44			?		*38:01
															?			?		45		47		*39:01
											36	37			?			?				?		*40:01
												?			?			?			46	?		*41:01:01-41:01:02
25	26											?			?			?				47		*44:01
						31			34						?			?						*45:01
	26	27										?			?			?		45		?		*46:01
												?			?			?				?		*47:01
												?			?			?				?		*48:01
												?			?			?		45		?		*49:01
												?			?			?				?		*50:01
												?			?			?				?		*51:01
						31						?	38		?			?				?		*52:01
											36	37			?			?				?		*53:01
			28	29		31						?			40			?				?		*54:01
			28	29		31							38		?			?			47			*55:01
						31						?			?			?				?	48	*56:01
												?		39	?			?				?		*57:01
25			28	29		31						?	38		?			?				?		*58:01
												37		39	?			?						*59:01
												?			?			?		46	?			*60:01
												?			?			?				?	48	*61:01N
											36	?	38		?			43				?		*62:01
25						31						?	38		?			?				?		*63:01
	26	27										?			?			?	44			?		*64:01N
						31						?			?			?				?		*65:01
			28									?			?			?				?	48	*66:01
						31			34			?	38		?			?				?		*67:01
						31						?			?			?				?		*68:01
		27										?			?			?		46	?			*69:01
25												?			?			?				?		*70:01
												?	38		?			?				?		*71:01
												?		39	?			?				?		*72:01
												?		39	?			?				?		*73:01
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.

190	75	95	200	220	185	120	75	140	210	195	260	185	100	210	95	75	140	65	110	120	70	165	115	Length of spec.
265	205		260				195	205				255			180	230	185	190	185	190	210	255	210	PCR product
																	285		245	245			280	
																			270					
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
										35		37			?			?				?		*74:01
												?			?			?				?		*75:01
					30				34			?			?			?				?	48	*76:01
												?			?	41		?				?		*77:01
							32					?			?			?				?		*78:01
			28			31						?			?			?				?		*79:01
		27										?			?			?		45		?		*80:01
								33				?			?			?				?		*81:01
												?			?		42	?				?		*82:01
												?			?			?			46	?		*83:01
						31						?			?			?				?		*84:01
			28			31									?			?					48	*85:01
	26	27	28	29	30							?			?			?		45		?		*86:01
						31						?	38		?			?				?		*87:01
	26		28												?			?						*88:01
						31						?			?			?				?		*89:01
						31						?			?			?				?		*90:01
		27			30				34			?			?			?				?		*91:01
												?			?			?				?	48	*92:01
			28			31						?			?			?				?		*93:01
												?			?	41		?				?		*94:01
												?	38	39	?			?				?		*95:01
												?			?			?		45	46	?		*96:01
25						31						?			?		42	?				?		*97:01
		27	28		30							?			?			?				?		*98:01
												?			?			?				?		*99:01
												?			?			?				?		*100:01
26												?			?			?				?		*101:01
						31						?		39	?			?				?		*102:01
						31						?	38		?			?				?		*103:01
																		43						*104:01
																					47			*105:01
						31																47		*106:01
			28			31		33							40									*107:01
		27										?		39	?			?		45		?		*108:01
			28									?			?			?				?		*109:01
			28			31		33				?			?			?				?	48	*110:01
												?			?			?	44			?		*111:01
												?	38		?			?				?		*112:01
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.

190	75	95	200	220	185	120	75	140	210	195	260	185	100	210	95	75	140	65	110	120	70	165	115	Length of spec.
265	205		260				195	205				255			180	230	185	190	185	190	210	255	210	PCR product
																	285		245	245			280	
																			270					
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
			28	29								?			?			?				?		*113:01
25						31						?			?			?				47		*114:01
												?			?			?		45		?		*115:01
												?			?			?				?		*116:01
			28					33				?			?			?		45		?		*117:01
						31						?			?			?	44			?		*118:01
			28		30				34			?			?			?				?		*119:01
												?			?			?	44			?		*120:01N
													38		?			?				47		*121:01
						31						?			?			?				?		*122:01
26								33							?			?				47		*123:01
															?			?				47		*124:01
			28			31						?			?			?				?	48	*125:01
																								DPB1 allele ⁴
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.