

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13
DQB1 allele ^{1,2}	ser. ³													
*02:01:01-02:02:02, 02:04-02:48	DQ2, Null, -			3										
*02:03	DQ2			3			6							
*03:01:01:01-03:01:01:03, 03:01:03-03:01:06, 03:01:09-03:01:25, 03:04:01-03:04:02, 03:09-03:10:02, 03:13-03:14:02, 03:16, 03:19, 03:21-03:22, 03:24, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:57, 03:59-03:60, 03:69, 03:71, 03:76-03:77, 03:80, 03:82-03:84N, 03:92-03:94, 03:101-03:103, 03:108, 03:114-03:116, 03:119-03:122, 03:127-03:131, 03:133-03:135, 03:138-03:140, 03:142-03:144, 03:147-03:148, 03:150, 03:152, 03:157-03:160, 03:163-03:165	DQ7, DQ3, Null, -				4			7					12	
*03:01:02, 03:01:08, 03:73, 03:75, 03:118N, 03:162	DQ7, Null, -				4			7						
*03:01:07, 03:58	-				4								12	
*03:02:01-03:02:02, 03:02:04-03:02:12, 03:02:14-03:02:17, 03:05:04, 03:07, 03:11, 03:18, 03:32, 03:45, 03:63-03:64, 03:66N-03:68, 03:70, 03:85, 03:106, 03:125, 03:146, 03:153, 03:161	DQ8, Null, -						5	7					12	
*03:02:03, 03:02:13, 03:05:02, 03:37	DQ8, -					5		7						
*03:03:02:01-03:03:02:03, 03:03:04-03:03:10, 03:03:12-03:03:13, 03:12, 03:15, 03:26, 03:30-03:31, 03:33-03:34, 03:38-03:41, 03:43, 03:74, 03:79, 03:86-03:91Q, 03:95N-03:98, 03:111, 03:113, 03:117, 03:123-03:124, 03:126, 03:136, 03:141, 03:145, 03:149, 03:155	DQ9, Null, -						6	7					12	
*03:03:03, 03:20, 03:99Q, 03:105, 03:112, 03:156	DQ9, -						6	7						
*03:03:11	-						6	7				11	12	
*03:05:01, 03:61	DQ8, -					5		7				11		
*03:05:03	DQ8					5		7				11	12	
*03:06, 03:25, 03:65, 03:137	DQ3, -						6						12	
*03:08, 03:107	DQ8, -					5							12	
*03:17:01	DQ9							7				11	12	
*03:17:02, 03:23, 03:109, 03:154	-							7						
*03:62, 03:78, 03:81, 03:110, 03:151	-							7					12	
*03:72, 03:100	-							7				11		
*03:104	-					5	6							
*03:132	-					5		7	8					
*04:01:01-04:01:02, 04:01:04-04:02:07, 04:06-04:09, 04:11-04:21, 04:23, 04:25N	DQ4, Null, -								8	9				
*04:01:03	-								8	9			12	
*04:02:08	-									9				
*04:03:01-04:03:02	-						6		8	9				
*04:04-04:05, 04:24	-								8					
*04:10	-								8	9				13
*04:22	-								8	w				
*05:01:01:01-05:03:09, 05:03:11-05:03:16, 05:05:01-05:33, 05:35-05:43, 05:45-05:51, 05:53, 05:55-05:59, 05:61-05:71, 05:74-05:76, 05:78-05:81, 05:83-05:86	DQ5, Null, -	1									10			
*05:03:10, 05:04, 05:34, 05:44, 05:52, 05:54, 05:72-05:73, 05:77	DQ5, -	1												
*05:60, 05:82	-										10			
*06:01:01-06:01:14, 06:03:01-06:03:09, 06:03:11-06:13:02, 06:17-06:18:02, 06:21-06:22:03, 06:25-06:28, 06:30-06:32, 06:34-06:36, 06:38-06:45, 06:52-06:65, 06:67, 06:69, 06:85-06:94, 06:98-06:106, 06:108, 06:110, 06:118:01-06:121, 06:128-06:135, 06:140-06:145, 06:148-06:149, 06:153-06:155, 06:157-06:158N, 06:160, 06:164-06:165	DQ6, DQ5, DQ1, Null, -		2											

[illegible]