

Well No.	DR	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
DRB1 allele ^{1,2}	ser ³																																
*01:01:01:01:02:10, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:65	DR1, Null, -	14																															
*01:03, 01:39N	DR103/DR1, Null		15																														
*01:42, 01:61	-	14	15																														
*03:01:01:01-03:01:22, 03:04:01-03:05:03, 03:09, 03:13:01-03:16, 03:18-03:20, 03:22-03:23, 03:25, 03:28, 03:30, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:59, 03:61-03:63, 03:66-03:68N, 03:70-03:73, 03:77-03:79, 03:81, 03:83-03:84, 03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:113-03:114	DR17, DR3, Null, -					18	19											30								39							
*03:01:23, 03:06, 03:26, 03:31, 03:60, 03:75, 03:80, 03:82, 03:85	DR3, -					18	19											30															
*03:02:01-03:03, 03:27, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115	DR18, DR3, -					18		20										30															
*03:07, 03:21, 03:24, 03:32, 03:39-03:40, 03:49, 03:94, 03:101, 03:105	DR3, -					18												30								39							
*03:08, 03:65	DR3, -					18	19				24		26													39							
*03:10	DR3					18	19												32	33						39							
*03:11:01, 13:02:02 ⁴	DR3, DR13						19											30								39							
*03:12	DR3					18	19											30						37		39							
*03:17, 03:35, 03:41	DR3, -					18												30															
*03:29	DR3					18		20										30								39							
*03:42, 03:69, 03:87, 03:111	-					18																											
*03:46, 03:64	-					18	19																				39						
*03:76, 13:176 ⁴	-						19				25		28					30								39		41					

[illegible]

*08:32, 08:53	-										23			27				33	34														
*08:35	-										23							33	34					40									
*08:36:02	-										23							33	34														
*08:37, 08:45	-										23								34		36												
*08:40	-						19				23								34														
*08:57	-														29				34														
Well No.	DR	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
Well No.	DR	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04-09:22, 09:24	DR9, -										24							32															
*09:01:06, 09:03, 09:23	DR9, -										24																						
*10:01:01-10:07	DR10, -										25																						
*11:01:01-11:01:06, 11:01:08-11:01:17, 11:01:20-11:01:24, 11:04:01-11:04:11, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:02, 11:15, 11:24, 11:27:01-11:29, 11:32-11:33, 11:35, 11:38-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:49:02, 11:51, 11:54:01-11:54:02, 11:58:01-11:58:02, 11:60-11:62:02, 11:66, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01-11:84:03, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144, 11:146-11:147:02, 11:150, 11:152, 11:154-11:158, 11:160, 11:162-11:167, 11:169N	DR11, Null, -												26		28	29									39								
*11:01:07, 11:30, 11:37:01-11:37:02, 11:50, 11:56, 11:88, 11:145, 11:170	DR11, -													26		28	29																
*11:01:18-11:01:19, 11:08:01-11:08:03, 11:18-11:19:03, 11:42, 11:57, 11:130	DR11, -													26		28										39							

[illegible]

[illegible]

*13:66:01-13:66:02	DR13, -							19								28		30							37		39						
*13:67, 13:103, 13:160	DR13, -															28	29									38							
*13:70, 13:149	DR13, -							19								28	29	30															
*13:85	-							19	20							28		30											41				
*13:93, 13:139	-							19								28	29	30							37	38	39						
*13:116	-							19									29					34											
*13:119	-															28		30			33						39	40					
*13:120	-							19								28		30							37				41				
*13:122	-							19								28									37								
*13:129	-							19					25				29	30									39						
*13:137N, 13:172	Null, -							19								28		30									39						
*13:144	-							19								28	29	30			33		35				39	40					
*13:146	-																29	30			33						39	40					
*13:154	-															28		30			33			37		39	40						
*13:156	-															28		30			33		35				39	40					
*13:158	-															28	29	30			33						39	40					
*13:159, 13:191	-							19								28	29			32	33					38							
*13:164	-															28	29	30			33		35		37			40					
*13:165	-							19								28	29									38	39						
*13:167	-							19										30							37								
Well No.	DR	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
Well No.	DR	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
*13:175	-						19									29					34		36		38								
*13:183	-																								38								
*13:193	-						19	20										30		32							39		41				
*14:01:01-14:01:02, 14:01:04, 14:04:01, 14:07:01-14:07:02, 14:10, 14:26, 14:28, 14:31, 14:38:01-14:39, 14:50, 14:54:01, 14:54:03, 14:54:05-14:55, 14:60-14:62, 14:70-14:71, 14:75, 14:86-14:88, 14:90, 14:99, 14:101, 14:110, 14:112-14:114, 14:117-14:118, 14:124-14:125, 14:129, 14:131, 14:139-14:140, 14:142-14:143, 14:145, 14:147, 14:149-14:151, 14:153, 14:157-14:158	DR14, DR140 4/DR14, -																	31	32	33							40						

*14:01:03, 14:08, 14:23:02, 14:34, 14:54:02, 14:92N, 14:97, 14:130, 14:133	DR14, Null, -										32	33				40			
*14:02:01, 14:06:01-14:06:03	DR14, -				20						30	32				41			
*14:02:02, 14:09, 14:29, 14:47-14:48, 14:51, 14:80, 14:83, 14:106, 14:108, 14:121	DR14, -				20						30	32				? 41			
*14:03:01-14:03:02, 14:40, 14:67, 14:77, 14:102, 14:115, 14:135	DR140 3/DR14, DR6, -				20				28		30		33			40			
*14:05:01-14:05:04, 14:14, 14:23:01, 14:23:03-14:23:04, 14:36, 14:43-14:44:03, 14:56, 14:59, 14:91, 14:96, 14:100, 14:103, 14:123, 14:127:01-14:127:02, 14:155	DR14, -										30	32	33			40			
*14:11, 14:148	DR14, -					23						32	33			40			
*14:12:01-14:12:02, 14:156	DR14, -				20				28		30		33	35		40			
*14:13	DR14				20						30	32			37	41			
*14:15	DR8					23			29				33	34	35	40			
*14:16	DR6			19					28	29	31	32	33			38	?		
*14:17	DR6				20						30	32				39	41		
*14:18, 14:81, 14:154	DR14, -				20						30	32	33			40			
*14:19, 14:109	DR14, -			19	20				28		30	w				?	41		
*14:20, 14:41, 14:94, 14:159	DR14, -				20						30	32				?			
*14:21	DR14			19	20				28		30	w				39	41		
*14:22, 14:105	DR14, -								28	29	31	32	33			?			
*14:24	DR14				20					29	30								
*14:25	DR6								28	29	31		33			?			
*14:27	DR14				20				28	29	30		33			40			
*14:30, 14:33	DR6				20						30	32				39	?	41	
*14:32:01-14:32:02	DR14				w							31	32	33		40			
*14:32:03	-											31	32	33					
*14:35	DR14											31	32	33		39	40		
*14:37	DR14									29	30	31				?			
*14:42	-										30		33			39	40		
*14:45	DR14			19							30	32	33			40			
*14:46	DR14											32					41		

[illegible]

[illegible]