

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14
DQB1 allele ^{1,2}	ser. ³														
*02:01:01-02:02:02, 02:04-02:64	DQ2, Null, -			3											
*02:03	DQ2			3			6								
*03:01:01:01-03:01:01:03, 03:01:03-03:01:06, 03:01:09-03:01:31, 03:04:01-03:04:02, 03:09-03:10:02, 03:13-03:14:02, 03:16, 03:19, 03:21-03:22, 03:24, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:57, 03:59-03:60, 03:69, 03:71, 03:76-03:77, 03:80, 03:82-03:84N, 03:92-03:94, 03:101-03:103, 03:108, 03:114-03:116, 03:119-03:122, 03:127-03:131, 03:133-03:135, 03:138-03:140, 03:142-03:144, 03:147-03:148, 03:150, 03:152, 03:157-03:160, 03:163-03:167, 03:169-03:173, 03:180, 03:182, 03:186-03:188, 03:191-03:193, 03:195-03:198, 03:201-03:202, 03:206-03:208	DQ7, DQ3, Null, -				4			7					12		
*03:01:02, 03:01:08, 03:73, 03:75, 03:109, 03:118N, 03:154, 03:162, 03:183	DQ7, Null, -				4			7							
*03:01:07, 03:58, 03:194	-				4								12		
*03:02:01-03:02:02, 03:02:04-03:02:12, 03:02:14-03:02:17, 03:02:19, 03:05:04, 03:07, 03:11, 03:18, 03:32, 03:45, 03:63-03:64, 03:66N-03:68, 03:70, 03:85, 03:106, 03:125, 03:146, 03:153, 03:161, 03:174-03:175, 03:178-03:179, 03:184-03:185, 03:190, 03:199, 03:203, 03:205, 03:210-03:211, 03:213N-03:215	DQ8, Null, -					5		7					12		
*03:02:03, 03:02:13, 03:05:02, 03:37, 03:189, 03:204	DQ8, -					5		7							
*03:02:18, 03:08, 03:107	DQ8, -					5							12		
*03:03:02:01-03:03:02:04, 03:03:04-03:03:10, 03:03:12-03:03:13, 03:12, 03:15, 03:26, 03:30-03:31, 03:33-03:34, 03:38-03:41, 03:43, 03:74, 03:79, 03:86-03:91Q, 03:95N-03:98, 03:111, 03:113, 03:117, 03:123-03:124, 03:126, 03:136, 03:141, 03:145, 03:149, 03:155, 03:168, 03:176-03:177, 03:200, 03:209, 03:212	DQ9, Null, -						6	7					12		
*03:03:03, 03:20, 03:99Q, 03:105, 03:112, 03:156	DQ9, -						6	7							
*03:03:11	-						6	7				11	12		
*03:05:01, 03:61, 03:181	DQ8, -					5		7				11			
*03:05:03	DQ8					5		7				11	12		
*03:06, 03:25, 03:65, 03:137	DQ3, -						6						12		
*03:17:01	DQ9							7				11	12		
*03:17:02, 03:23:01	-							7							
*03:23:02, 03:62, 03:78, 03:81, 03:110, 03:151	-							7					12		
*03:72, 03:100	-							7				11			
*03:104	-					5	6								
*03:132	-					5		7	8						
*04:01:01-04:01:02, 04:01:04-04:02:07, 04:02:09-04:02:10, 04:06-04:09, 04:11-04:21, 04:23, 04:25N-04:29, 04:31-04:32	DQ4, Null, -								8	9					
*04:01:03	-								8	9			12		
*04:02:08, 04:30	-								9						
*04:03:01-04:03:02	-						6		8	9					
*04:04-04:05, 04:24	-							8							
*04:10	-							8	9					13	
*04:22	-							8	w						
*05:01:01:01-05:01:15, 05:01:17-05:03:09, 05:03:11-05:03:17, 05:05:01-05:33, 05:35-05:43, 05:45-05:51, 05:53, 05:55-05:59, 05:61-05:71, 05:73-05:76, 05:78-05:81, 05:83-05:97, 05:99-05:104, 05:106-05:109	DQ5, Null, -	1									10				
*05:01:16, 05:60, 05:82, 05:98	-										10				
*05:03:10, 05:04, 05:34, 05:44, 05:52, 05:54, 05:72, 05:77, 05:105	DQ5, -	1													
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14

*06:01:01-06:01:15, 06:03:11, 06:04:01-06:05:01, 06:07:01-06:07:02, 06:09:01-06:09:06, 06:12, 06:17, 06:21-06:22:03, 06:25, 06:28, 06:34-06:36, 06:38-06:39, 06:42-06:43, 06:45, 06:52-06:58, 06:69, 06:85-06:86, 06:88-06:89, 06:92-06:94, 06:98, 06:100-06:105, 06:108, 06:118:01-06:121, 06:129, 06:132, 06:135, 06:140, 06:142, 06:144N, 06:149, 06:153, 06:155, 06:157-06:158N, 06:160, 06:164, 06:167, 06:171, 06:177, 06:180-06:181, 06:186, 06:189, 06:193N-06:194	DQ6, DQ1, Null, -		2																
*06:02:01-06:02:12, 06:02:14-06:02:25, 06:14:01-06:14:03, 06:16, 06:19:01-06:20, 06:23-06:24, 06:33, 06:37, 06:47-06:50, 06:51:02, 06:68, 06:70-06:78, 06:80-06:84, 06:95, 06:97, 06:107, 06:109, 06:111-06:117, 06:122, 06:124-06:127, 06:136-06:138, 06:147, 06:150-06:152, 06:159, 06:161, 06:163, 06:166, 06:173-06:175, 06:178-06:179N, 06:182-06:183, 06:188, 06:192, 06:197	DQ6, Null, -		2														13	14	
*06:02:13, 06:15:01-06:15:02, 06:46, 06:79:01-06:79:02	DQ6, -		2														13		
*06:03:01-06:03:09, 06:03:12-06:03:21, 06:08:01-06:08:03, 06:10-06:11:03, 06:13:01-06:13:02, 06:18:01-06:18:02, 06:26N-06:27:02, 06:30-06:32:02, 06:40-06:41, 06:44, 06:59-06:65, 06:67, 06:87, 06:90-06:91, 06:99:01-06:99:02, 06:106, 06:110, 06:128, 06:130-06:131, 06:133-06:134, 06:141, 06:143, 06:145, 06:148, 06:154, 06:165, 06:170, 06:176, 06:184-06:185, 06:187, 06:190-06:191, 06:195-06:196	DQ6, DQ1, Null, -		2															14	
*06:03:10, 06:51:01, 06:96	-		2				6											14	
*06:05:02-06:06	DQ6, DQ5		2															?	
*06:29, 06:123, 06:139	DQ6, -		2				5											14	
*06:66, 06:168, 06:172	-		2				6												
*06:146:01-06:146:02	-																13		
*06:156, 06:162	-		2									10					13	14	
*06:169	-											10						14	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14				