

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14
DQB1 allele ^{1,2}	ser. ³														
*02:01:01-02:02:04, 02:04-02:76, 02:78-02:88	DQ2, Null, -			3											
*02:03, 02:77	DQ2, -			3			6								
*03:01:01:01-03:01:01:05, 03:01:03-03:01:06, 03:01:09-03:01:37, 03:04:01-03:04:03, 03:09, 03:13, 03:16, 03:19:01-03:19:02, 03:21-03:22, 03:24, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:57, 03:59-03:60, 03:69, 03:71, 03:76-03:77, 03:80, 03:82-03:84N, 03:92-03:94, 03:101-03:103, 03:108, 03:114-03:116, 03:119-03:122, 03:127-03:131, 03:133-03:135, 03:138-03:140, 03:142-03:144, 03:147-03:148, 03:150, 03:152, 03:157-03:160, 03:163-03:167, 03:169-03:173, 03:180, 03:182, 03:186-03:188, 03:191-03:193, 03:196-03:198, 03:201-03:202, 03:206-03:208, 03:216, 03:218-03:219, 03:231, 03:235-03:236, 03:241-03:243, 03:246, 03:253-03:256	DQ7, Null, -				4			7				12			
*03:01:02, 03:01:08, 03:73, 03:75, 03:109, 03:118N, 03:162, 03:252	DQ7, Null, -				4			7							
*03:01:07, 03:58, 03:194, 03:232	-				4							12			
*03:02:01:01-03:02:02, 03:02:04-03:02:12, 03:02:14-03:02:17, 03:02:19-03:02:24, 03:05:04, 03:07, 03:11, 03:18, 03:32, 03:45, 03:63-03:64, 03:66N-03:68, 03:85, 03:106, 03:125, 03:146, 03:153, 03:161, 03:174-03:175, 03:178, 03:184-03:185, 03:190, 03:199, 03:203, 03:205, 03:210-03:211, 03:213N-03:215, 03:220-03:221, 03:224, 03:229, 03:240, 03:245, 03:247, 03:251	DQ8, Null, -					5		7				12			
*03:02:03, 03:02:13, 03:05:02, 03:37, 03:189, 03:204, 03:223, 03:226, 03:233, 03:237N	DQ8, Null, -					5		7							
*03:02:18, 03:08, 03:107, 03:228	DQ8, -					5						12			
*03:03:02:01-03:03:02:04, 03:03:04-03:03:05, 03:03:07-03:03:10, 03:03:12-03:03:13, 03:15, 03:26, 03:30-03:31, 03:33-03:34, 03:38-03:41, 03:43, 03:74, 03:79, 03:86-03:91Q, 03:95N-03:98, 03:111, 03:113, 03:117, 03:123-03:124, 03:126, 03:136, 03:141, 03:145, 03:149, 03:155, 03:168, 03:176-03:177, 03:200, 03:209, 03:212, 03:222, 03:227, 03:230, 03:234, 03:239, 03:248-03:249	DQ9, Null, -						6	7				12			
*03:03:03, 03:20, 03:99Q, 03:105, 03:112, 03:156, 03:238	DQ9, -						6	7							
*03:03:06, 03:62, 03:78, 03:81, 03:110, 03:151, 03:225, 03:244	-							7				12			
*03:03:11	-						6	7				11	12		
*03:05:01, 03:61, 03:181, 03:250	DQ8, -					5		7				11			
*03:05:03	DQ8					5		7				11	12		
*03:06, 03:25:01-03:25:02	DQ3, -												12		
*03:10:01-03:10:02, 03:14:01-03:14:02, 03:195	DQ3, -				4			7					12		
*03:12	DQ9						6	7					12		
*03:17:01	DQ9							7				11	12		
*03:17:02	-							7							
*03:23:01	-		2					7							
*03:23:02, 03:217	-		2					7					12		
*03:65, 03:137	DQ3, -						6						12		
*03:70, 03:179	-					5		7					12		
*03:72, 03:100	-							7				11			
*03:104	-					5	6								
*03:132	-					5		7	8						
*03:154	-				4			w							
*03:183	-				4			7							
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14

*04:01:01-04:01:02, 04:01:04-04:02:12, 04:03:01-04:03:02, 04:06-04:09, 04:11-04:21, 04:23, 04:25N-04:32, 04:34-04:37, 04:39	DQ4, Null, -								8	9						
*04:01:03	-								8	9			12			
*04:04-04:05, 04:33	-								8							
*04:10	-		2						8	9					13	
*04:22, 04:24, 04:38	-								8	w						
*05:01:01:01-05:01:15, 05:01:17-05:03:09, 05:03:11-05:03:17, 05:05:01-05:33, 05:35-05:43:02, 05:45-05:51, 05:53, 05:55-05:59, 05:61-05:71, 05:73-05:76, 05:78-05:81, 05:83-05:97, 05:99-05:104, 05:106-05:113, 05:115, 05:117-05:127, 05:129, 05:131, 05:133-05:143	DQ5, Null, -	1									10					
*05:01:16, 05:60, 05:82, 05:98, 05:116	-										10					
*05:03:10, 05:04, 05:34, 05:44, 05:52, 05:54, 05:72, 05:77, 05:105, 05:114, 05:128N, 05:130, 05:132Q, 06:212 ⁴	DQ5, Null, -	1														
*06:01:01-06:01:06, 06:01:08-06:01:15, 06:43, 06:54N-06:58, 06:98, 06:100-06:105, 06:108, 06:120, 06:132, 06:140, 06:153, 06:157, 06:167, 06:177, 06:181, 06:194, 06:205, 06:214, 06:229	DQ6, Null, -		2													
*06:01:07, 06:03:11, 06:04:01-06:05:01, 06:07:01-06:07:02, 06:09:01-06:09:06, 06:12, 06:17, 06:21-06:22:03, 06:25, 06:28, 06:34-06:36, 06:38-06:39, 06:42, 06:45, 06:52-06:53:02, 06:69, 06:85-06:86, 06:88-06:89, 06:92-06:94, 06:118:01-06:119, 06:121, 06:129, 06:135, 06:142, 06:144N, 06:149, 06:155, 06:158N, 06:160, 06:164, 06:171, 06:180, 06:186, 06:189, 06:193N, 06:199, 06:202-06:204, 06:207, 06:210, 06:217	DQ6, DQ1, Null, -		2													
*06:02:01:01-06:02:12, 06:02:14-06:02:26, 06:14:01-06:14:03, 06:16, 06:19:01-06:20, 06:23-06:24, 06:33, 06:37, 06:47-06:50, 06:51:02, 06:68, 06:70-06:78, 06:80-06:84, 06:95, 06:97, 06:107, 06:109, 06:111-06:117, 06:122, 06:124-06:127, 06:136-06:138, 06:147, 06:150-06:152, 06:159, 06:161, 06:163, 06:166, 06:173-06:175, 06:178-06:179N, 06:182-06:183, 06:188, 06:192, 06:197-06:198, 06:200-06:201, 06:208, 06:211, 06:213, 06:215-06:216N, 06:219, 06:224-06:228	DQ6, Null, -		2											13	14	
*06:02:13, 06:15:01-06:15:02, 06:46, 06:79:01-06:79:02, 06:146:01-06:146:02	DQ6, -		2											13		
*06:03:01-06:03:09, 06:03:12-06:03:26, 06:08:01-06:08:03, 06:10-06:11:03, 06:13:01-06:13:02, 06:18:01-06:18:02, 06:26N-06:27:02, 06:30-06:32:02, 06:40-06:41, 06:44, 06:59-06:65, 06:67, 06:87, 06:90-06:91, 06:99:01, 06:106, 06:110, 06:128, 06:130-06:131, 06:133-06:134, 06:141, 06:143, 06:145, 06:148, 06:154, 06:165, 06:170, 06:176, 06:184-06:185, 06:187, 06:190:01-06:191, 06:195-06:196, 06:206:01-06:206:02, 06:218, 06:221-06:223, 06:230	DQ6, DQ1, Null, -		2												14	
*06:03:10, 06:51:01, 06:96	-		2				6								14	
*06:05:02	DQ6		2												?	
*06:06	DQ5		2												?	
*06:29, 06:123, 06:139	DQ6, -		2			5									14	
*06:66, 06:172	-		2			6										
*06:99:02	-		2												14	
*06:156, 06:162	-		2								10			13	14	
*06:168	-		2			6										
*06:169	-										10				14	
*06:209	-		2		4											
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	