

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
DQB1 allele ^{1,2}	ser. ³															
*02:01:01-02:02:06, 02:04-02:76, 02:78-02:112	DQ2, Null, -			3												
*02:03, 02:77	DQ2, -			3			6									
*03:01:01:01-03:01:01:12, 03:01:01:14-03:01:01:20, 03:01:03-03:01:06, 03:01:09-03:01:44, 03:04:01-03:04:03, 03:09, 03:16, 03:19:01-03:19:02, 03:21-03:22, 03:24, 03:27-03:29, 03:35-03:36, 03:42, 03:44, 03:46-03:57, 03:59-03:60, 03:69, 03:71, 03:76-03:77, 03:80, 03:82-03:84N, 03:92-03:94, 03:101-03:103, 03:108, 03:114-03:116, 03:119-03:122, 03:127-03:131, 03:133-03:135, 03:138-03:140, 03:142-03:144, 03:148, 03:150, 03:152, 03:157-03:160, 03:163-03:167, 03:169-03:173, 03:180, 03:182, 03:186-03:188, 03:191-03:193, 03:196-03:198, 03:201-03:202, 03:206-03:208, 03:216, 03:218-03:219, 03:231, 03:235-03:236, 03:241-03:243, 03:246, 03:253-03:257, 03:260, 03:264, 03:266-03:268, 03:271, 03:275-03:276N, 03:281, 03:284-03:286, 03:288, 03:290, 03:292-03:294, 03:297	DQ7, Null, -				4			7				12				
*03:01:02, 03:01:08, 03:73, 03:75, 03:109, 03:118N, 03:154, 03:162, 03:252, 03:291	DQ7, Null, -				4			7								
*03:01:07, 03:13, 03:58, 03:194, 03:232	DQ7, -				4			w				12				
*03:02:01:01-03:02:02, 03:02:04-03:02:05, 03:02:07-03:02:12, 03:02:14-03:02:17, 03:02:19-03:02:29, 03:05:04, 03:07, 03:11, 03:18, 03:32, 03:45, 03:63-03:64, 03:66N-03:68, 03:85, 03:106, 03:146, 03:153, 03:161, 03:174-03:175, 03:178, 03:184-03:185, 03:190, 03:199, 03:203, 03:205, 03:210-03:211, 03:213N-03:215, 03:220-03:221, 03:224, 03:229, 03:240, 03:245, 03:247, 03:251, 03:261, 03:263, 03:265, 03:269N, 03:273-03:274, 03:277-03:279, 03:287, 03:289, 03:295-03:296, 03:298	DQ8, Null, -					5		7				12				
*03:02:03, 03:02:13, 03:05:02, 03:37, 03:189, 03:204, 03:223, 03:226, 03:233, 03:237N	DQ8, Null, -					5		7								
*03:02:06, 03:02:18, 03:02:30, 03:125, 03:228	-					5						12				
*03:03:02:01-03:03:02:05, 03:03:04-03:03:05, 03:03:07-03:03:10, 03:03:12-03:03:15, 03:03:17-03:03:18, 03:15, 03:23-03, 03:26, 03:30-03:31, 03:33-03:34, 03:38-03:41, 03:43, 03:74, 03:79, 03:86-03:91Q, 03:95N-03:98, 03:111, 03:113, 03:117, 03:123-03:124, 03:126, 03:136, 03:141, 03:145, 03:149, 03:155, 03:168, 03:176-03:177, 03:200, 03:209, 03:212, 03:222, 03:227, 03:230, 03:234, 03:239, 03:248-03:249, 03:270, 03:280, 03:282N	DQ9, Null, -						6	7				12				
*03:03:03, 03:03:16, 03:20, 03:99Q, 03:105, 03:112, 03:156, 03:238, 03:258	DQ9, -						6	7								
*03:03:06, 03:62, 03:78, 03:81, 03:110, 03:151, 03:225, 03:244, 03:272	-							7				12				
*03:03:11	-						6	7				11	12			
*03:05:01, 03:61, 03:181, 03:250	DQ8, -					5		7				11				
*03:05:03	DQ8					5		7				11	12			
*03:06, 03:25:01-03:25:02	DQ3, -											12				
*03:08, 03:107	DQ8, -					5		w				12				
*03:10:01-03:10:02:02, 03:14:01-03:14:02, 03:195	DQ3, -				4			7				12				15
*03:12	DQ9						6	7				12				15
*03:17:01	DQ9							7				11	12			
*03:17:02	-							7								
*03:23:01	-		2					7								
*03:23:02, 03:217, 03:259	-		2					7				12				
*03:65, 03:137, 03:283	-						6	w				12				
*03:70, 03:179	-					5		7				12				15
*03:72, 03:100	-							7				11				
*03:104	-					5	6	7								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15
*03:132	-					5		7	8							

*03:147	-				4									12			
*03:183	-				4			7									15
*03:262	-						5	w									
*04:01:01:01-04:01:02, 04:01:04-04:02:01:01, 04:02:01:04-04:02:15, 04:03:01-04:03:02, 04:06-04:09, 04:11-04:21, 04:23, 04:25N-04:32, 04:34-04:37, 04:39-04:48, 04:50-04:52	DQ4, Null, -									8	9						
*04:01:03, 04:02:16	-									8	9			12			
*04:03:03	-										9			12			
*04:04-04:05, 04:33, 04:49	-									8							
*04:10	-		2							8	9					13	
*04:22, 04:24, 04:38	-									8	w						
*05:01:01:01-05:01:15, 05:01:17-05:03:09, 05:03:11-05:03:19, 05:05:01-05:33, 05:35-05:43:02, 05:45-05:51, 05:53, 05:55-05:59, 05:61-05:71, 05:73-05:76, 05:78-05:81, 05:83-05:97, 05:99-05:104, 05:106-05:113, 05:115, 05:117-05:127, 05:129, 05:131, 05:133-05:145, 05:147-05:166	DQ5, Null, -	1										10					
*05:01:16, 05:60, 05:82, 05:98, 05:116	-											10					
*05:03:10, 05:04, 05:34, 05:44, 05:52, 05:54, 05:72, 05:77, 05:105, 05:114, 05:128N, 05:130, 05:132Q, 05:146	DQ5, Null, -	1															
*06:01:01:01-06:01:06, 06:01:08-06:01:19, 06:43, 06:54N-06:58, 06:98, 06:100-06:105, 06:108, 06:120, 06:132, 06:140, 06:153, 06:157, 06:167, 06:177, 06:181, 06:194, 06:205, 06:214, 06:229, 06:239, 06:243, 06:245-06:247, 06:251, 06:257-06:258, 06:260, 06:263, 06:268, 06:274, 06:277, 06:285	DQ6, Null, -		2														15
*06:01:07, 06:03:11, 06:04:01-06:05:01, 06:07:01-06:07:02, 06:09:01:01-06:09:09, 06:12, 06:17, 06:21-06:22:03, 06:25, 06:28, 06:34-06:36, 06:38-06:39, 06:42, 06:45, 06:52-06:53:02, 06:69:01-06:69:02, 06:85-06:86, 06:88-06:89, 06:92-06:94, 06:118:01-06:119, 06:121, 06:129, 06:135, 06:142, 06:144N, 06:149, 06:155, 06:158N, 06:160, 06:164, 06:171, 06:180, 06:186, 06:189, 06:193N, 06:199, 06:202-06:204, 06:207, 06:210, 06:212, 06:217, 06:231, 06:241, 06:252N, 06:254, 06:261, 06:265-06:267, 06:275, 06:280-06:283	DQ1, DQ6, Null, -		2														
*06:02:01:01-06:02:12, 06:02:14-06:02:36, 06:14:01-06:14:03, 06:16, 06:19:01-06:20, 06:23-06:24, 06:33, 06:37, 06:47-06:50, 06:51:02, 06:68, 06:70-06:78, 06:80-06:84, 06:95, 06:96:02-06:97, 06:107, 06:109, 06:111-06:117, 06:122, 06:124-06:127, 06:136-06:138, 06:147, 06:150-06:152, 06:159, 06:161, 06:163, 06:166, 06:173-06:175, 06:178-06:179N, 06:182-06:183, 06:188, 06:192, 06:197-06:198, 06:200-06:201, 06:208, 06:211, 06:213, 06:215-06:216N, 06:219, 06:224-06:228, 06:232, 06:236-06:237, 06:240, 06:242, 06:249, 06:255-06:256, 06:262, 06:264, 06:271, 06:273, 06:284, 06:286	DQ6, Null, -		2												13	14	
*06:02:13, 06:15:01-06:15:02, 06:46, 06:79:01-06:79:02, 06:146:01-06:146:02, 06:235	DQ6, -		2													13	
*06:03:01:01-06:03:09, 06:03:12-06:03:31, 06:08:01-06:08:03, 06:10-06:11:03, 06:13:01-06:13:03, 06:18:01-06:18:02, 06:26N-06:27:02, 06:30-06:32:02, 06:40-06:41, 06:44, 06:59-06:65, 06:67, 06:87, 06:90-06:91, 06:99:01, 06:106, 06:110, 06:128, 06:130-06:131, 06:133-06:134, 06:141, 06:143, 06:145, 06:148, 06:154, 06:165, 06:170, 06:176, 06:184-06:185, 06:187, 06:190:01-06:191, 06:195-06:196, 06:206:01-06:206:02, 06:218, 06:221-06:223, 06:230, 06:233-06:234, 06:238, 06:244, 06:248, 06:250, 06:253, 06:259, 06:269-06:270, 06:272, 06:276, 06:278-06:279	DQ1, DQ6, Null, -		2													14	
*06:03:10, 06:51:01, 06:96:01	-		2					6									14
*06:05:02	DQ6		?														?
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	
*06:06	DO5		2													?	?

*06:29, 06:123, 06:139	DQ6, -	2				5										14	
*06:66, 06:172	-	2				6											
*06:99:02	-	2														14	15
*06:156, 06:162	-									10					13	14	
*06:168	-	2				6											15
*06:169	-									10						14	
*06:209	-	2		4													15
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	