

Well No.		16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	
DRB1 allele ^{1,2}	ser ³																																	
*01:01:01-01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:88, 01:90-01:91Q		16																																
*01:03:01-01:03:03, 01:39N	DR1, DR103, Null, -		17																															
*01:42, 01:61, 01:89	-	16	17																															
*03:01:01-01:03:01:22, 03:01:25-03:01:28, 03:04:01-03:05:03, 03:09, 03:11:01, 03:13:01-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:25:02, 03:28, 03:30, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:59, 03:61-03:63, 03:66-03:68N, 03:70-03:73, 03:77-03:79, 03:81, 03:83-03:84, 03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121, 03:123-03:124, 03:127-03:129, 03:132-03:137, 03:139, 03:141-03:148	DR3, DR17, Null, -																																	
*03:01:23, 03:06, 03:26, 03:31, 03:60, 03:75, 03:80, 03:82, 03:85, 03:122, 03:138	DR3, -					20	21											32																
*03:01:24, 03:07:01-03:07:02, 03:21, 03:24, 03:32, 03:39-03:40, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120	DR3, -					20												32									41							
*03:02:01-03:03, 03:27, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:131	DR3, DR18, -					20		22										32																
*03:08, 03:65, 03:140	DR3, -					20	21				26		28														41							
*03:10	DR3					20	21													34	35						41							
*03:12	DR3					20	21											32							39		41							
*03:17, 03:35, 03:41:01-03:41:02	DR3, -					20												32																
*03:29, 03:119	DR3, -					20		22										32									41							
*03:42, 03:69, 03:87, 03:111	-					20																												
*03:46, 03:64, 03:130	-					20	21																					41						
*03:76	-					20	21					27				30		32										41		43				
*03:92	-					20	21															36		38										
*03:125	-					20	21						27			30	31	32									40	41						
*03:126	-					21	22					27						32		34								41		43				
*04:01:01-01:04:05:11, 04:05:13-04:11:05, 04:13-04:17:02, 04:19-04:24, 04:26-04:57, 04:59-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:229, 04:231-04:250, 04:252-04:257	DR3, DR4, Null, -								23																									
*04:12, 04:18, 04:25, 04:58	DR4								23														37											
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:251	DR4, -								23										33															
*04:211	-								23											34									43					
*04:230	-								23																				43					
*07:01:01-01:07:01:22, 07:03-07:05, 07:07-07:90	DR7, Null, -									24																								
*07:06	DR7									24	25																							
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:04, 08:05, 08:16, 08:24:01-08:24:02, 08:26, 08:39, 08:42-08:44, 08:50, 08:55, 08:64, 08:72-08:73, 08:77-08:79, 08:82, 08:84-08:86	DR8, Null, -									25							31					36												
Well No.		16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	
Well No.		16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	
*08:01:06, 08:03:02-01-08:03:08, 08:13, 08:18, 08:23, 08:27, 08:29-08:30:03, 08:33, 08:36:01, 08:38, 08:46-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81	DR8, Null, -									25												36												
*08:04:01-08:04:05, 08:04:07, 08:06, 08:22, 08:54, 08:59, 08:83	DR8, -									25							31					36	37											

Negative Control

Alternative Control

[illegible]

[illegible]

[illegible]