




[illegible]


*13:01:09, 13:35-13:36, 13:40, 13:72, 13:76, 13:84, 13:135	DR13, -							6								15	16	17									25						
*13:02:02	DR13							6										17									26						
*13:03:01-13:03:06, 13:33:01-13:33:03, 13:81, 13:88-13:90, 13:94:01-13:95, 13:101, 13:115, 13:133, 13:151-13:152	DR13, -							6								15		17								24							
*13:04, 13:75	DR13, -							6								15	16	17								24	26						
*13:05:01-13:05:02, 13:11:01-13:11:02, 13:14:01-13:14:03, 13:42, 13:46, 13:50:01- 13:50:02, 13:62, 13:100, 13:132, 13:136, 13:150	DR13, DR6, DR11, -															15	16	17									26						
*13:06, 13:25, 13:56, 13:77, 13:82, 13:110, 13:157, 13:163	DR13, DR11, DR6, -															15		17									26						
*13:07:01-13:07:02	DR13															15	16	17															
*13:08	DR13							6								15	16	17								25	27						
*13:09	DR13																16	17								26							
*13:10	DR13							6								15		17								26	28						
*13:12:01-13:12:03, 13:58, 13:118, 13:134, 13:161	DR13, -															15		17							24								
*13:13	DR13															15		17		20					24		27						
*13:15, 13:57, 13:104	DR13, -							6	7							15	16	17					21	23		25	26						
*13:17	DR13							6									16																
*13:18	DR13															15	16	17		20		22				26	27						
*13:19, 13:53	DR13							6	7							15	16	17								25							
*13:21:01-13:21:02, 13:169	DR13/DR11, -															15	16	17							24	26							
*13:22-13:24, 13:54, 13:97:02, 13:162	DR13, DR11, DR14, -							6								15	16	17								26							
*13:26:01-13:26:02	DR14, -								7							15	16	17															
*13:27, 13:41, 13:71	DR13							6					12			15	16	17							25	26							
*13:30	-															15		17							24	26							
*13:32, 13:65	DR13							6								15	16	17							24	25							
*13:37																																	


[illegible]

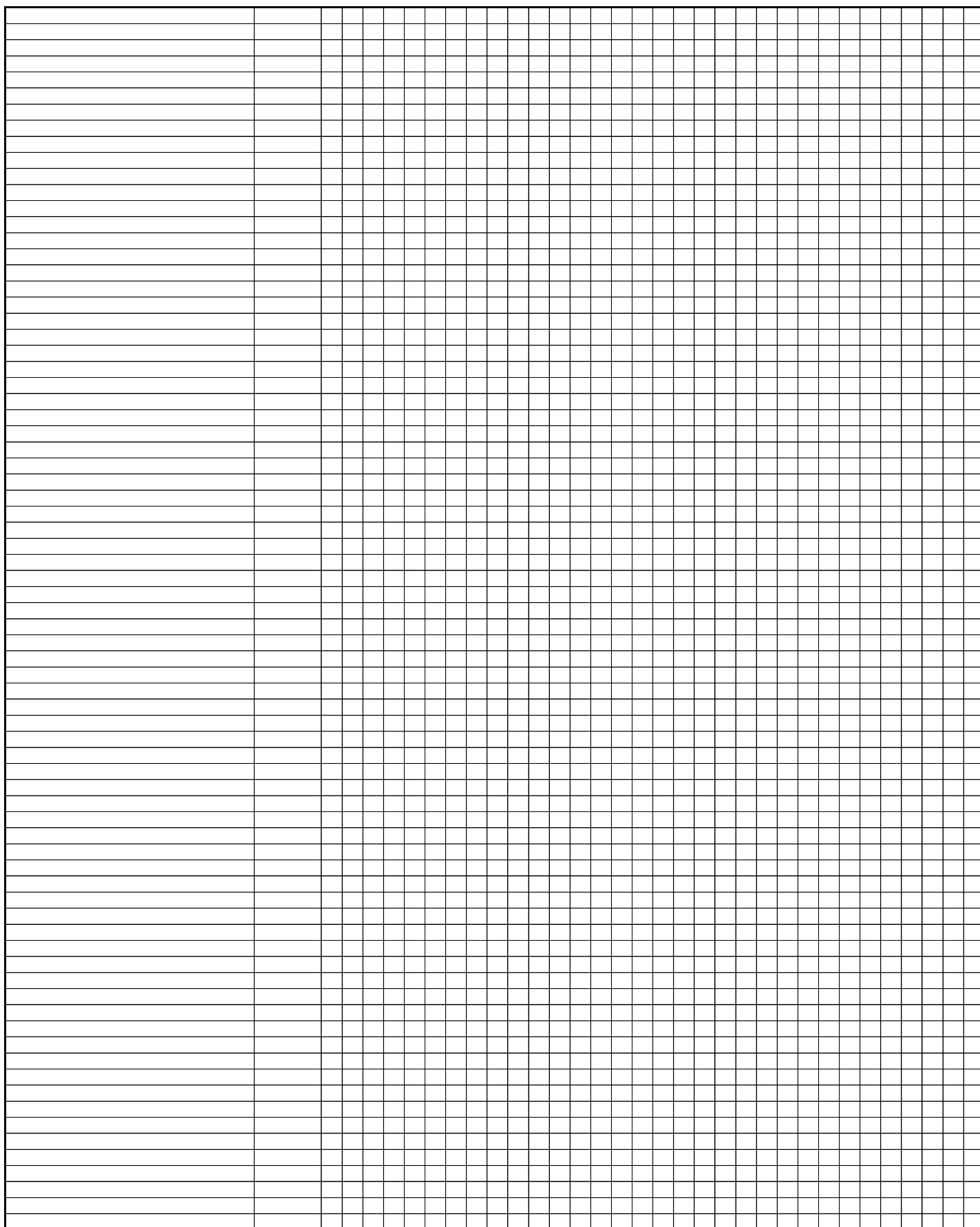

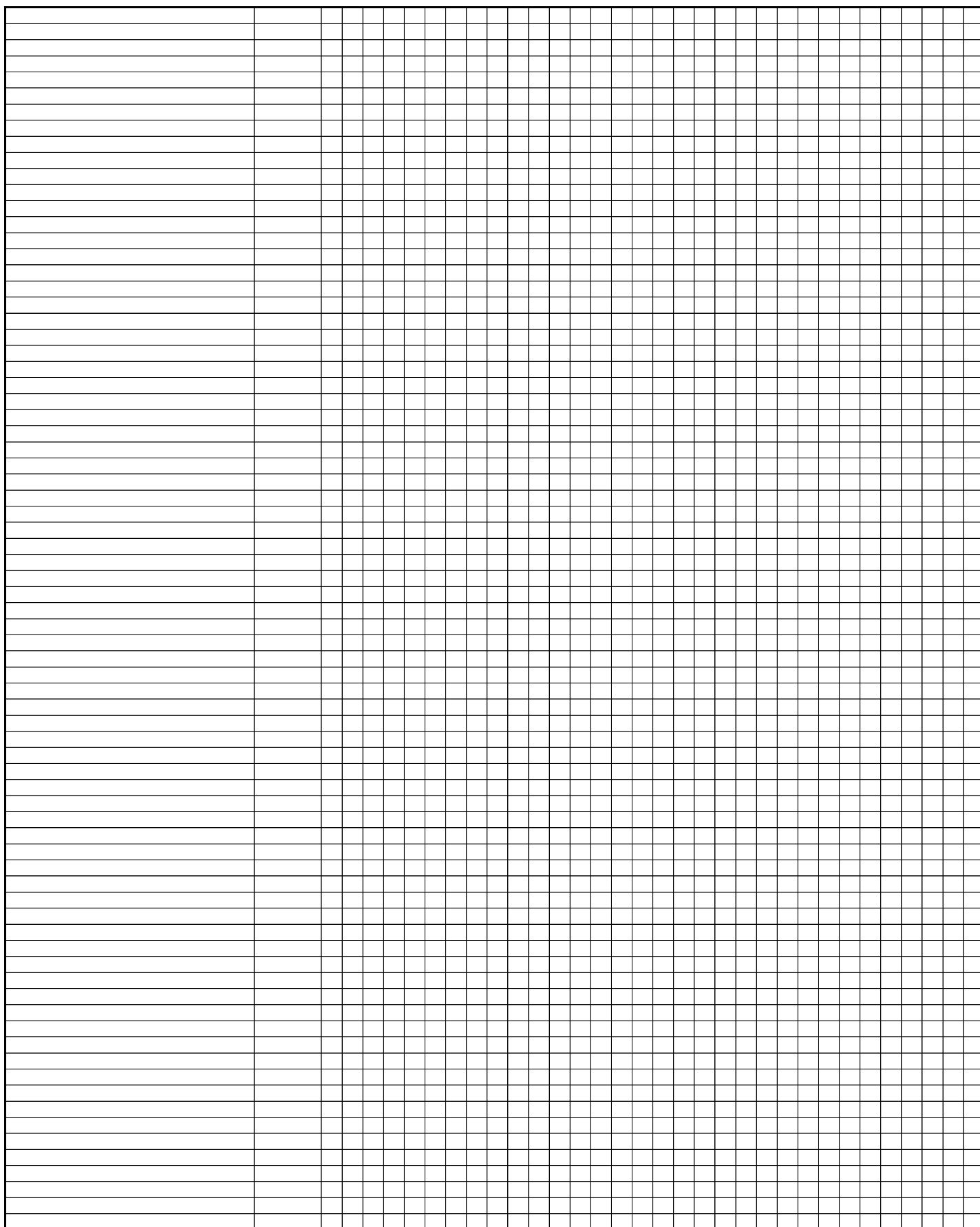

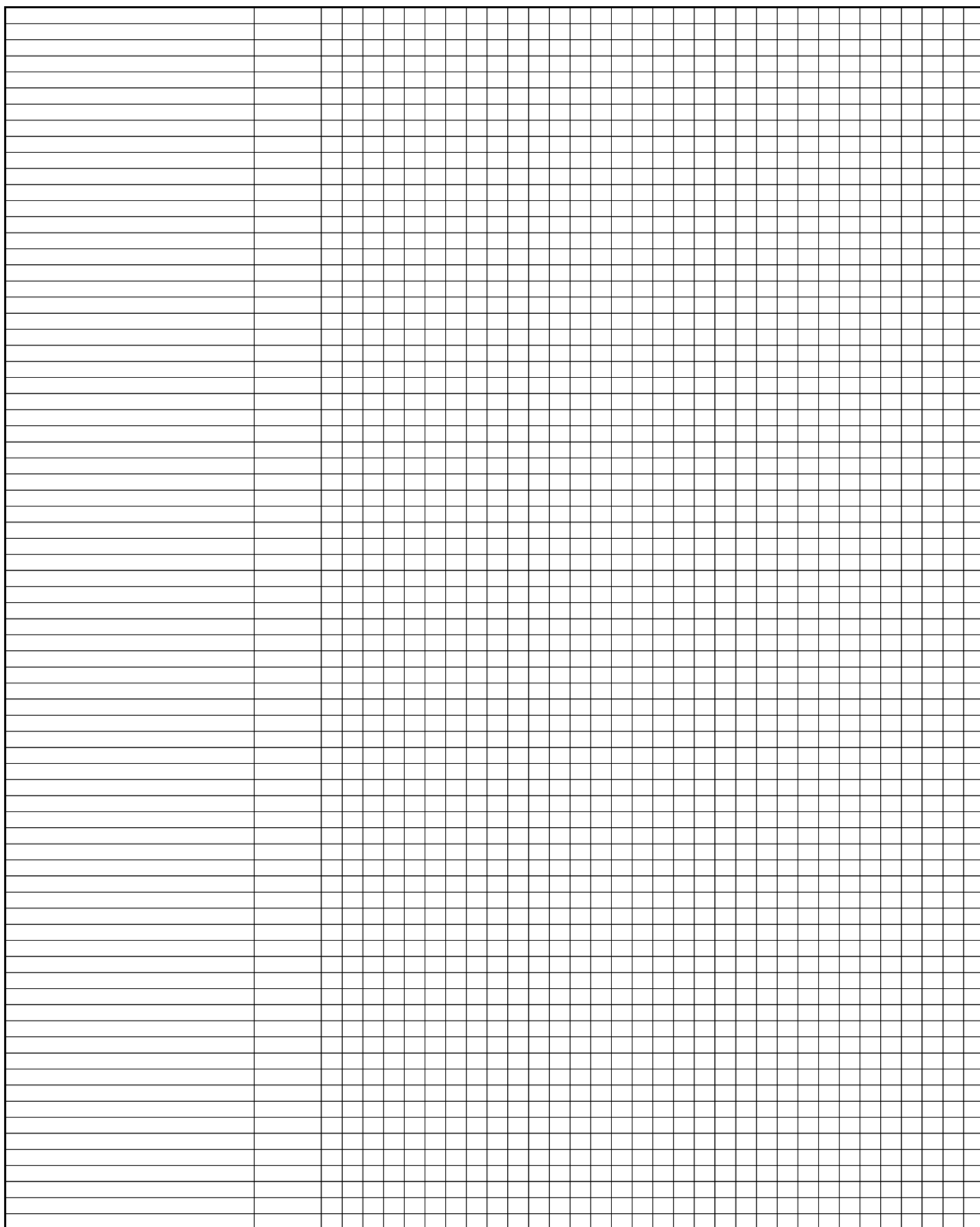


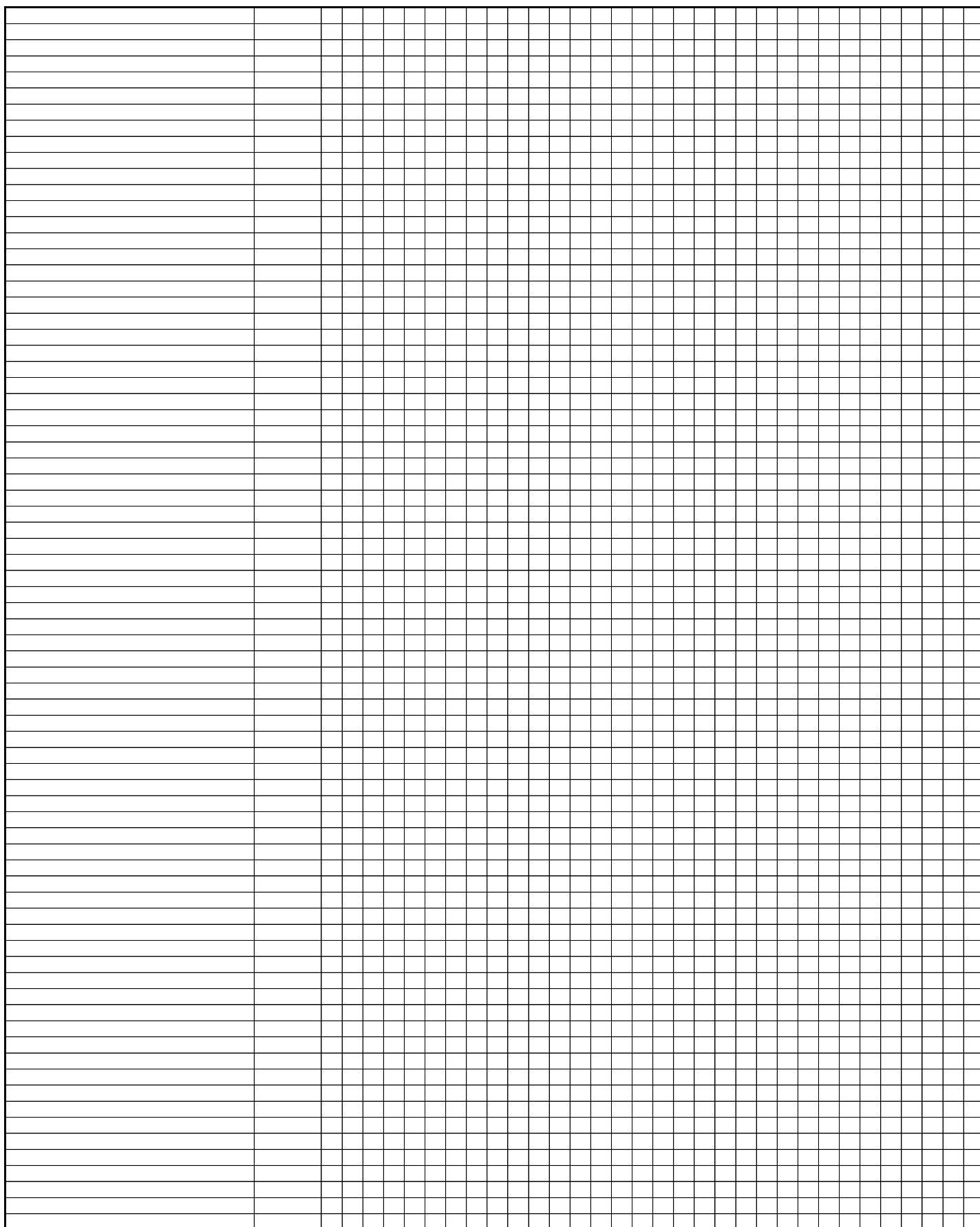


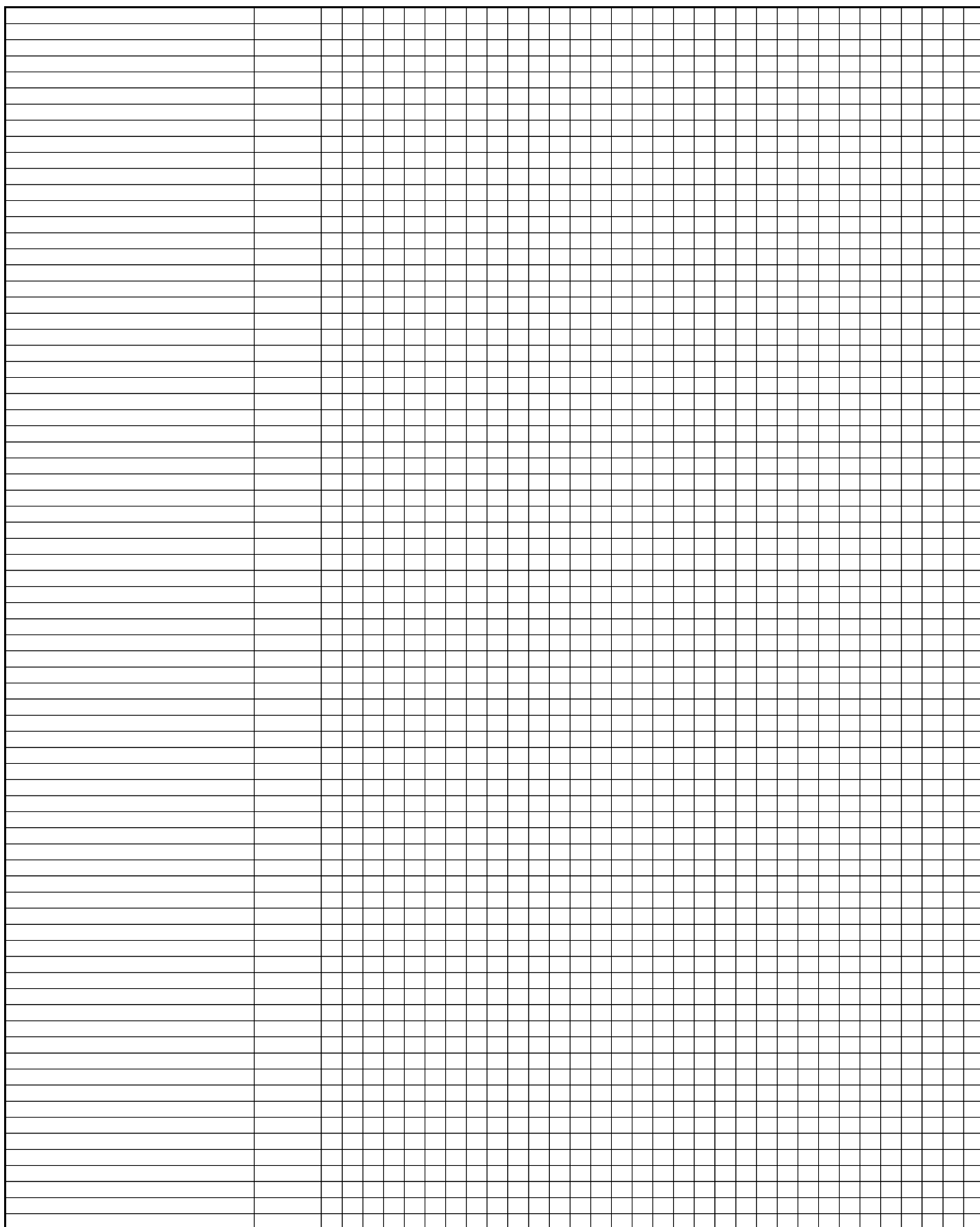


[illegible]

[illegible]

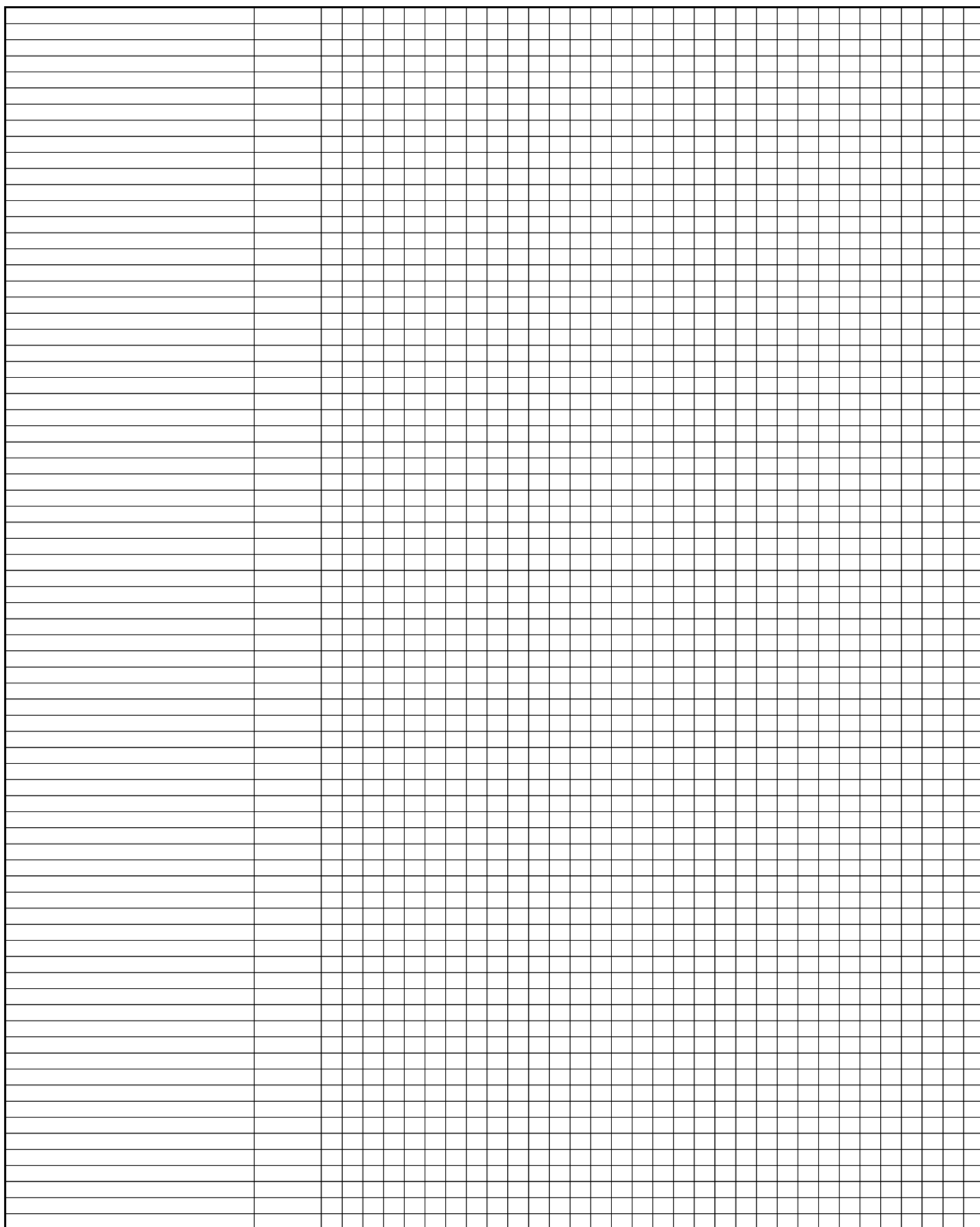

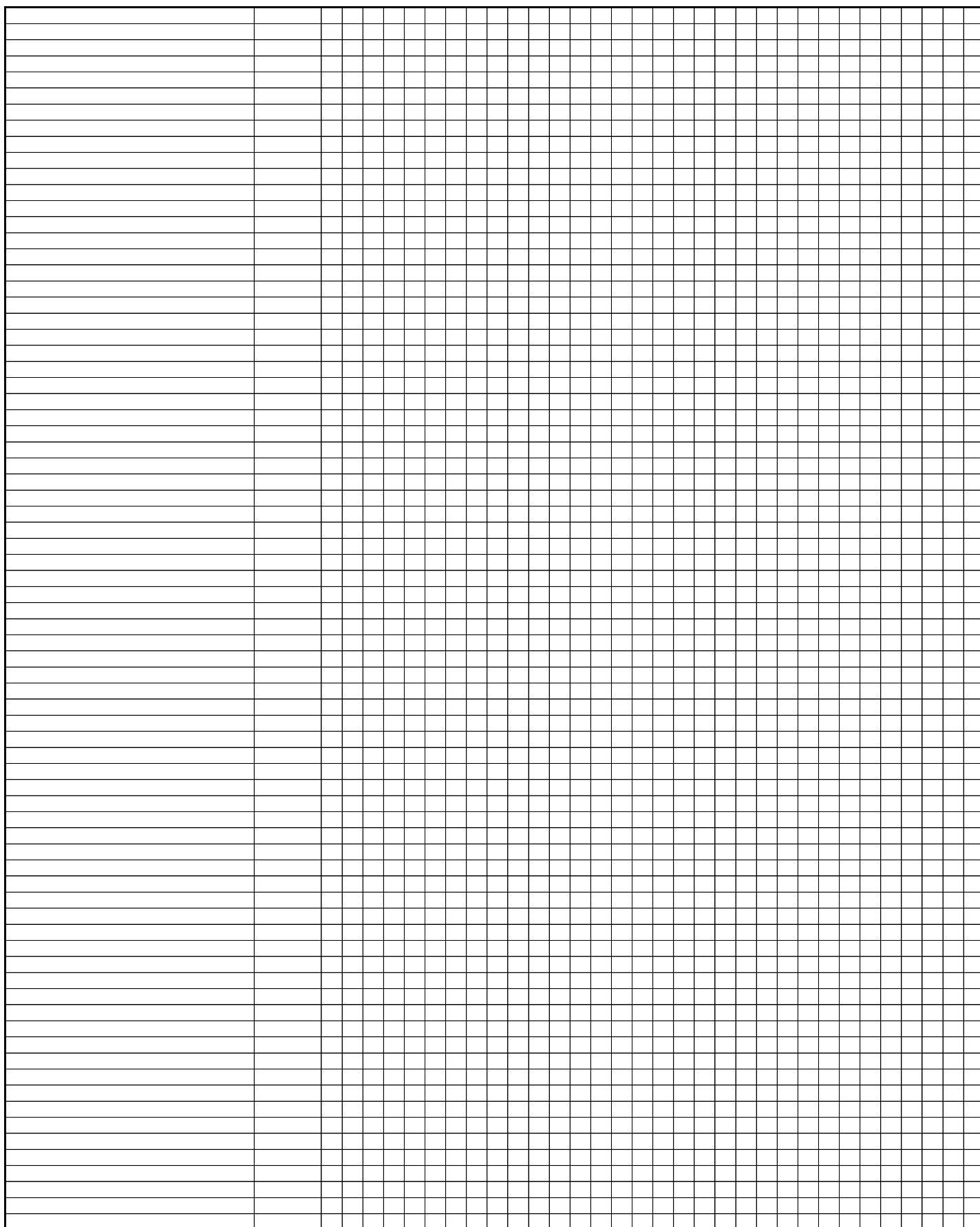

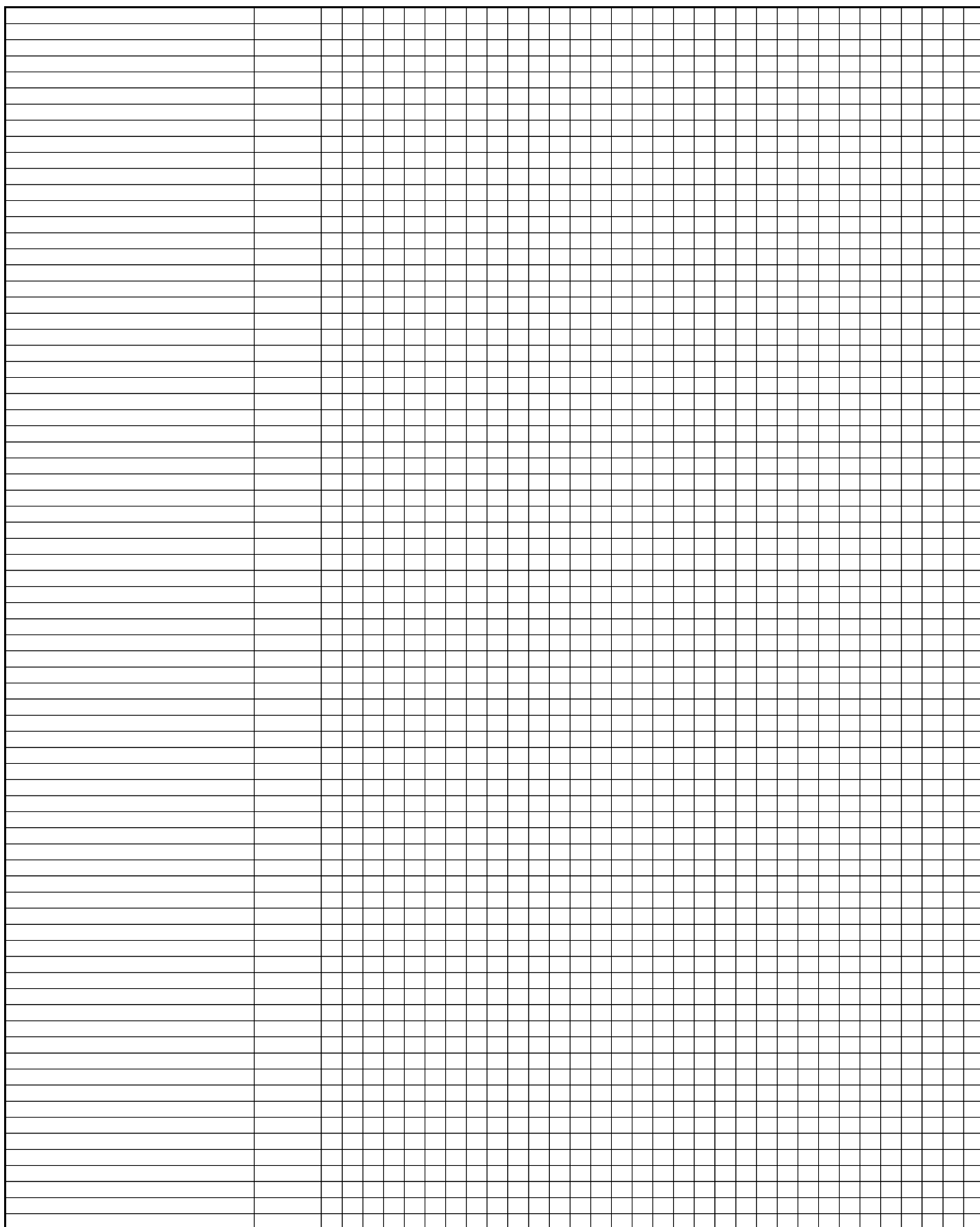

[illegible]

*08:09	DR8									10					16				20	21						27							
*08:10, 08:12, 08:14	DR8									10										21	22												
*08:15, 08:19, 08:25, 08:34	-									10																							
*08:17	DR8									10					16					21		23											
*08:20	-														15	16	17		20		22						27						
*08:21	-									10					15	16			20	21							27						
*08:28	-									10						16				21	22	23											
*08:31, 08:41	DR11(5), -									10			13		16																		
*08:32	-									10				14					20	21													
*08:35	-									10									20	21							27						
*08:37, 08:45	-									10										21		23											
*08:40	-				6					10										21													
*09:01:02-09:01:05, 09:01:07-09:02:02, 09:04-09:19	DR9, -										11							19															
*09:01:06, 09:03	DR9, -										11																						
*10:01:01-10:04	DR10, -											12																					
Well No.	DR	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	DR	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*11:01:01-11:01:06, 11:01:08-11:01:16, 11:04:01-11:04:10, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:02, 11:15, 11:24, 11:27:01-11:29, 11:32-11:33, 11:35, 11:38-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:49:02, 11:51, 11:54:01-11:54:02, 11:58:01-11:58:02, 11:60-11:62, 11:66, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01-11:84:02, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117, 11:120-11:121, 11:123, 11:126, 11:128-11:129	DR11(5), -													13		15	16									26							
*11:01:07, 11:30, 11:37:01-11:37:02, 11:50, 11:56, 11:88	DR11(5), -													13		15	16																
*11:02:01-11:03, 11:11:01-11:11:02, 11:14:01-11:14:02, 11:21, 11:36, 11:63, 11:65:01-11:65:02, 11:68, 11:70, 11:76, 11:85-11:86, 11:93, 11:124, 11:127	DR11(5), -					6								13		15	16										26						
*11:05	DR11(5)													13		16																	
*11:07, 11:53, 11:103, 11:105, 11:125	DR11(5), -					5					11			13													26						
*11:08:01-11:08:02, 11:18-11:19:03, 11:42, 11:57	DR11(5), -													13		15											26						
*11:13:01-11:13:02	DR11(5)						w							13					19	20						26	27						
*11:16, 11:20, 11:40, 11:118, 11:122	DR11(5), DR13 (6), -					6								13		15	16									25	26						
*11:17, 11:52	DR11(5), -													13					19	20							27						
*11:22, 11:104	-													13																			
*11:23	DR11(5)													13		15	16			20							26	27					
*11:25	DR11(5)													13		15	16			20		22					26	27					
*11:26, 11:34	DR11(5), -						7							13													26						
*11:31	-													13		15				20							26						
*11:41, 11:48	-					6								13		15	16																
*11:45, 11:64	-													13		15				20							26	27					
*11:55	-													13		16				20							26	27					
*11:59	-					6						12	13	13	15	16										25	26						
*11:67	-								10					13		16						22	23										
*11:69, 11:82	-													13		16			18								26						
*11:73, 11:79	-					6								13		15											26						
*11:80, 11:87	-					6						12	13	13	15	16											26						
*11:83	-					6						12	13	13	15	16																	
*11:89	-													13						20							26	27					
*11:96	-													13		15	16			20							26						
*11:98	-													13													26						
*11:107	-					5					11			13																			
*11:119	-													13		15				20		22					26	27					
*12:01:01-12:01:04, 12:03:02-12:03:03, 12:05-12:08, 12:10-12:12, 12:14, 12:17, 12:24N-12:25, 12:28-12:30, 12:34-12:36	DR12(5), Null, -														14								23										
*12:02:01-12:02:05, 12:13, 12:15, 12:18-12:20, 12:23, 12:26-12:27, 12:31N-12:33	DR12(5), Null, -														14		16						23										
*12:04, 12:22	-									10					14								23										
*12:09	-														14						21		23										
*12:16:01	-									10					14		16						23										
*12:21	-														14		16																

Neg. Control

Neg. Control

[illegible]

[illegible]

[illegible]