

Well No.	DR	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB1 allele ^{1,2}	ser ³																																	
*01:01:01-01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62N-01:88, 01:90-01:98, 01:100	DR1, Null, -	1																																
*01:03:01-01:03:04, 01:39N, 01:99	DR103, DR1, Null, -		2																															
*01:42, 01:61, 01:89	-	1	2																															
*03:01:01-01:03:01:22, 03:01:25-03:01:28, 03:04:01-03:05:03, 03:09, 03:11:01, 03:13:01-03:16, 03:18-03:20, 03:22-03:23, 03:25:01-03:25:02, 03:28, 03:30, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52, 03:54-03:59, 03:61-03:63, 03:66-03:68N, 03:70-03:73, 03:77-03:79, 03:81, 03:83-03:84, 03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121, 03:123-03:124, 03:127-03:129, 03:132-03:137, 03:139, 03:141-03:149, 03:151-03:153	DR3, DR17, Null, -					5	6										17									26								
*03:01:23, 03:06, 03:26, 03:31, 03:60, 03:75, 03:80, 03:82, 03:85, 03:122, 03:138	DR3, -					5	6											17																
*03:01:24, 03:07:01-03:07:02, 03:21, 03:24, 03:32, 03:39-03:40, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150	DR3, -					5												17									26							
*03:02:01-03:03, 03:27, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:131, 03:154	DR3, DR18, -					5		7										17																
*03:08, 03:65, 03:140	DR3, -					5	6				11		13														26							
*03:10	DR3					5	6													19	20						26							
*03:12	DR3					5	6											17							24	26								
*03:17, 03:35, 03:41:01-03:41:02	DR3, -					5												17																
*03:29, 03:119	DR3, -					5		7										17									26							
*03:42, 03:69, 03:87, 03:111	-					5																												
*03:46, 03:64, 03:130, 03:156N	Null, -					5	6																				26							
*03:76	-					5	6					12			15		17										26		28					
*03:92	-					5	6														21		23				26							
*03:125	-					5	6					12			15	16	17					21		23		25	26							
*03:126	-						6	7				12					17		19							26		28						
*03:155	-					5											17								24									
*03:157	-					5	6										17			20						26	27							
*03:158	-					5	6										17								24									
*04:01:01-01:04:05:11, 04:05:13-04:11:05, 04:13-04:17:02, 04:19-04:24, 04:26-04:57, 04:59-04:61, 04:63-04:68, 04:70-04:72:02, 04:74-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:229, 04:231-04:250, 04:252-04:264N, 04:266N-04:272	DR4, DR3, Null, -								8																									
*04:12, 04:18, 04:25, 04:58	DR4								8													22												
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:251	DR4, -								8										18															
*04:211	-								8											19										28				
*04:230	-								8																					28				
*04:265	-								8					13																				
*07:01:01-01:07:01:22, 07:03-07:05, 07:07-07:101N	DR7, Null, -									9																								
*07:06	DR7									9	10																							
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:04, 08:05, 08:16, 08:24:01-08:24:02, 08:26, 08:39, 08:42-08:44, 08:50, 08:55, 08:64, 08:72-08:73, 08:77-08:79, 08:82, 08:84-08:86, 08:88-08:90	DR8, Null, -										10					16						21												
Well No.	DR	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB1 allele ^{1,2}	ser ³																																	
Well No.	DR	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
DRB1 allele ^{1,2}	ser ³																																	
*08:01:06, 08:03:02:01-08:03:09, 08:13, 08:18, 08:23, 08:27, 08:29-08:30:03, 08:33, 08:36:01, 08:38, 08:46-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81	DR8, Null, -										10											21												
*08:04:01-08:04:05, 08:04:07, 08:06, 08:22, 08:54, 08:59, 08:83	DR8, -										10						16					21	22											
*08:04:06	-										10						16						22											
*08:07, 08:11, 08:52	DR8, -										10						16																	
*08:08	DR8										10						16		18															
*08:09	DR8										10						16				20	21						27						
*08:10, 08:12, 08:14	DR8										10											21	22											
*08:15, 08:19, 08:25, 08:34	DR8, -										10																							
*08:17	DR8										10						16					21		23										

Negative Control

*08:20	-														15	16	17		20		22				27			
*08:21	DR8									10					15	16			20	21					27			
*08:28	DR8									10						16					21	22	23					
*08:31, 11:05 ⁴	DR8, DR11													13		16												
*08:32, 08:53	-									10					14				20	21								
*08:35	-									10									20	21					27			
*08:36:02	-									10									20	21								
*08:37, 08:45:01-08:45:02	-									10										21		23						
*08:40	-					6				10										21								
*08:41	-									10					13		16											
*08:57	-									10						16				21								
*08:63	-									10					13							23						
*08:67, 08:80	-															16				21	22							
*08:68:01	-									10									w	21								
*08:70	-									10						16			w	21	22							
*08:75	-									10					13		16				22							
*08:87	-									10					14		16			w	21							
*09:01:02:01-09:01:05, 09:01:07-09:02:02, 09:04, 09:06-09:22, 09:24-09:28, 09:30-09:40	DR9, Null, -										11								19									
*09:01:06, 09:03, 09:23, 09:29	DR9, -										11																	
*09:05	DR9										11	12							19									
*10:01:01:01-10:18, 10:20-10:33	DR10, -											12																
*10:19	-											12															30	
*11:01:01:01-11:01:06, 11:01:08-11:01:17, 11:01:20-11:01:27, 11:01:29-11:01:33, 11:04:01-11:04:13, 11:04:15-11:04:16, 11:04:18, 11:06:01-11:06:03, 11:09-11:10:02, 11:12:01-11:12:03, 11:15, 11:24:01-11:24:02, 11:27:01, 11:28:01-11:29:02, 11:32-11:33, 11:35, 11:38-11:39, 11:43-11:44, 11:46:01-11:47, 11:49:01-11:49:02, 11:51, 11:54:01-11:54:02, 11:58:01-11:58:02, 11:60-11:62:02, 11:66:01-11:66:02, 11:72, 11:74:01-11:75, 11:77-11:78, 11:81, 11:84:01, 11:90-11:92, 11:94-11:95, 11:97, 11:99-11:102:02, 11:106, 11:108-11:117:02, 11:120-11:121, 11:123, 11:126, 11:128-11:129, 11:133-11:134, 11:137, 11:140-11:141, 11:144, 11:146-11:147:02, 11:150, 11:152, 11:154-11:158, 11:160, 11:162-11:167, 11:169N, 11:172, 11:174-11:175, 11:177-11:181, 11:185-11:188, 11:190, 11:192, 11:194-11:199, 11:201, 11:203-11:208, 11:210-11:212, 11:																												

[illegible]

[illegible]

[illegible]