

Well No.	DR	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
DRB1 allele ^{1,2}	ser ³																																		
*01:01:01:01-01:02:13, 01:04-01:38, 01:40N-01:41, 01:43-01:60, 01:62:01N-01:88, 01:90-01:98, 01:100-01:101, 01:103-01:111, 01:113-01:118	DR1, Null, -	1																																	
*01:03:01-01:03:05, 01:39N, 01:99, 01:102, 01:112	DR103, DR1, Null, -	2																																	
*01:42, 01:61, 01:89	-	1	2																																
*03:01:01:01-03:01:22, 03:01:25-03:01:31, 03:04:01-03:05:03, 03:09, 03:11:01, 03:11:03, 03:13:01-03:16, 03:18-03:20, 03:22-03:23, 03:25:01:03:25:02, 03:28, 03:30, 03:33-03:34, 03:36-03:37, 03:43-03:45, 03:47-03:48, 03:50-03:52:02, 03:54-03:59, 03:61-03:63, 03:66-03:68N, 03:70-03:73, 03:77-03:79, 03:81, 03:83-03:84, 03:86, 03:89, 03:91, 03:93, 03:95-03:96, 03:98-03:100:02, 03:104, 03:106-03:110, 03:112-03:114, 03:116-03:118, 03:121, 03:123-03:124, 03:127-03:129, 03:132-03:137, 03:139, 03:141-03:149, 03:151-03:153, 03:159-03:161, 03:163-03:166, 03:168-03:170, 03:172, 03:175	DR17, DR3, Null, -						5	6										17									26								
*03:01:23, 03:06, 03:26, 03:31, 03:60, 03:75, 03:80, 03:82, 03:85, 03:122, 03:138	DR3, -						5	6										17																	
*03:01:24, 03:07:01-03:07:02, 03:21, 03:24, 03:32, 03:39-03:40, 03:49, 03:94, 03:97, 03:101, 03:105, 03:120, 03:150, 03:171, 03:178	DR3, -						5											17									26								
*03:02:01-03:03, 03:27, 03:38, 03:53, 03:74, 03:88, 03:90, 03:102-03:103, 03:115, 03:131, 03:154, 03:167, 03:176, 03:179	DR18, DR3, -						5		7									17																	
*03:08, 03:65, 03:140	DR3, -						5	6				11		13													26								
*03:10	DR3						5	6												19	20						26								
*03:12	DR3						5	6										17							24		26								
*03:17, 03:35, 03:41:01-03:41:02	DR3, -						5											17																	
*03:29, 03:119	DR3, -						5		7									17									26								
*03:42, 03:69, 03:87, 03:173	-						5																												
*03:46, 03:64, 03:130, 03:156N	Null, -						5	6																			26								
*03:76, 03:174N	Null, -						5	6					12			15		17									26		28						
*03:92	-						5	6														21		23			26		28						
*03:111	-						5	6																											
*03:125	-						5	6					12			15	16	17									25	26							
*03:126	-						6	7					12					17	19								26		28						
*03:155	-						5											17							24										
*03:157	-						5	6										17		20							26	27							
*03:158	-						5	6										17						24											
*03:162	-						5	6													21														
*03:177	-						5	6	7									17	19								26		28						
*04:01:01:01-04:05:11, 04:05:13-04:11:06, 04:13-04:17:02, 04:19-04:24, 04:26-04:57, 04:59-04:61, 04:63-04:68, 04:70-04:72:02, 04:74:01-04:104, 04:106-04:121, 04:123-04:145, 04:147-04:210, 04:212N-04:229, 04:231-04:250, 04:252-04:283, 04:285-04:312N	DR4, DR3, Null, -								8																										
*04:12, 04:18, 04:25, 04:58	DR4								8														22												
*04:62, 04:69, 04:73:01-04:73:02, 04:105:01-04:105:02, 04:122, 04:146, 04:251, 04:284	DR4, -								8										18																
*04:211	-								8											19										28					
*04:230	-								8																					28					
*07:01:01:01-07:01:25, 07:03-07:05, 07:07-07:118N	DR7, Null, -									9																									
*07:06	DR7									9	10																								
*08:01:01-08:01:02, 08:01:04-08:01:05, 08:01:07-08:02:04, 08:05, 08:16, 08:24:01-08:24:02, 08:26, 08:39, 08:42-08:44, 08:50, 08:55, 08:64, 08:72-08:73, 08:77-08:79, 08:82, 08:84-08:86, 08:88-08:90, 08:93, 08:96, 08:100-08:101	DR8, Null, -										10						16					21													
*08:01:06, 08:03:02:01-08:03:10, 08:03:12, 08:13, 08:18, 08:23, 08:27, 08:29-08:30:03, 08:33, 08:36:01, 08:38, 08:46-08:49, 08:51, 08:56, 08:58, 08:60N-08:62, 08:65-08:66, 08:68:02-08:69, 08:71, 08:74, 08:76, 08:81, 08:91, 08:95, 08:97, 08:102, 08:104	DR8, Null, -										10											21													
Well No.	DR	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
DRB1 allele ^{1,2}	ser ³																																		
*08:03:11, 08:68:01	-									10											w	21													
*08:04:01-08:04:05, 08:04:07, 08:06, 08:22, 08:54, 08:59, 08:83, 08:94, 08:99	DR8, -									10							16					21	22												
*08:04:06	-									10							16						22												
*08:07, 08:11, 08:52	DR8, -									10							16																		
*08:08	DR8									10							16			18															
*08:09:01-08:09:02	DR8, -									10							16				20	21						27							
*08:10, 08:12, 08:14	DR8									10												21	22												
*08:15, 08:19, 08:25, 08:34	DR8, -									10																									
*08:17, 08:103	DR8, -									10							16					21		23											
*08:20	-															15	16	17			20		22					27							
*08:21	DR8									10							15	16			20	21						27							
*08:28	DR8									10							16					21	22	23											
*08:31, 11:05 ⁴	DR8, DR11													13		16																			
*08:32, 08:53	-									10					14						20	21													

Negative Control

Negative Control

[illegible]

Negative Control

[illegible]

[illegible]

[illegible]